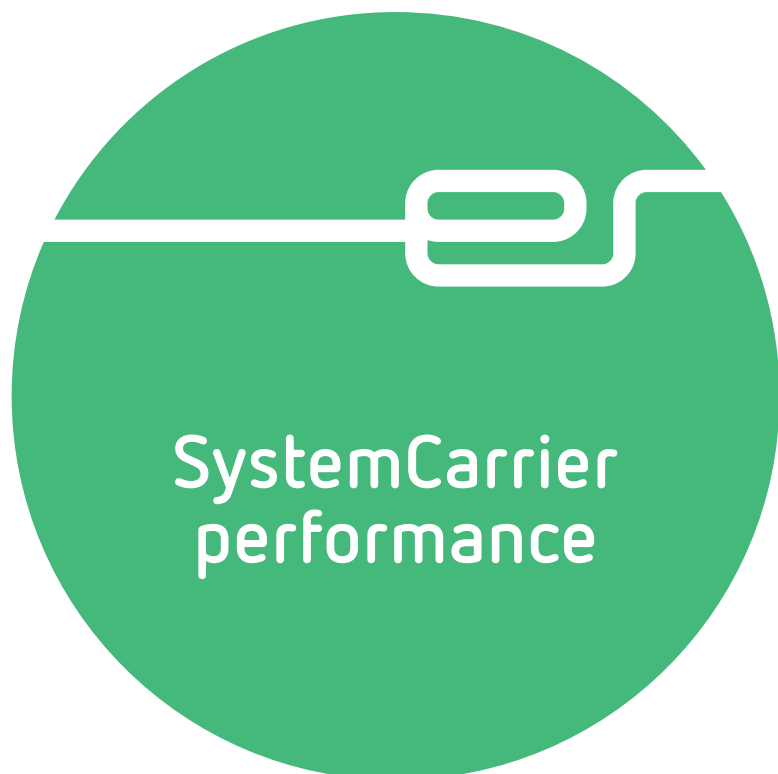
MOUNTING- INSTRUCTION

EN ISO 9001 EN ISO 13485

SystemCarrier performance Art. No. 20180-110

All rights to this Mounting Instruction, in particular rights of duplication, dissemination and translation, are reserved. No part of this Mounting Instruction may be reproduced in any form (by photocopying, microfilming or other methods) or processed, duplicated or disseminated by the use of electronic systems without the written consent of Erbe Elektromedizin GmbH.

The information contained in this Mounting Instruction can be changed or expanded without prior notice and without obligation on the part of Erbe Elektromedizin GmbH.

The illustrations in this Mounting Instruction may differ in some details from your product.

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
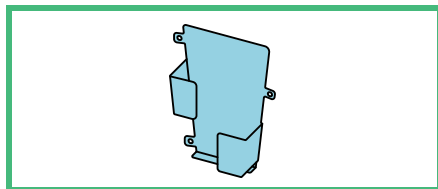
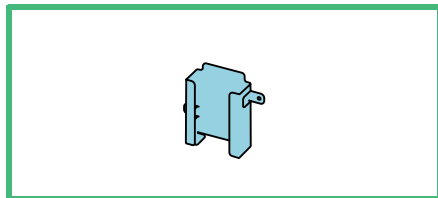
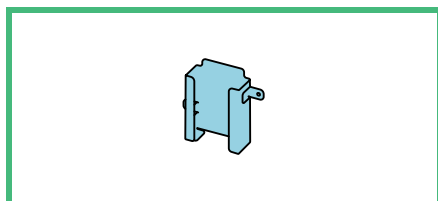
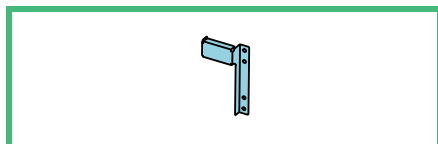
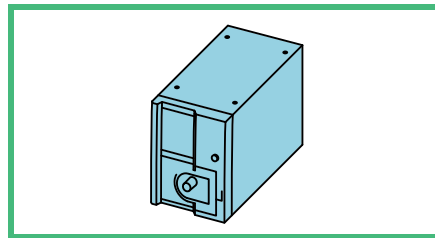
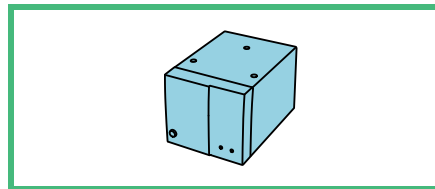
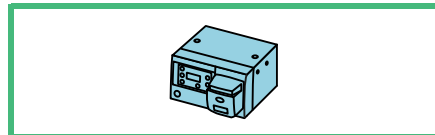
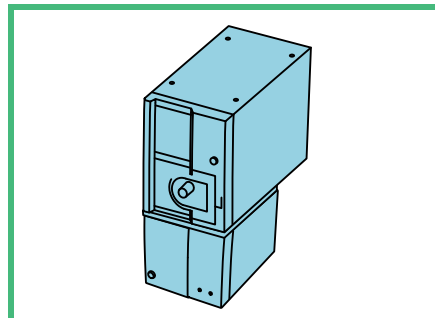
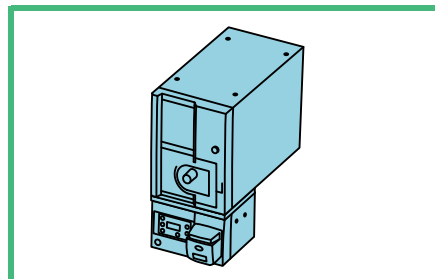
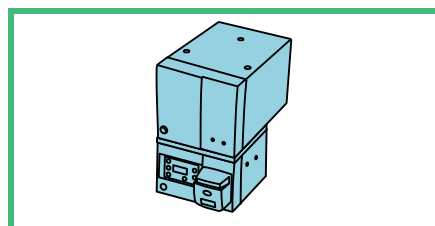
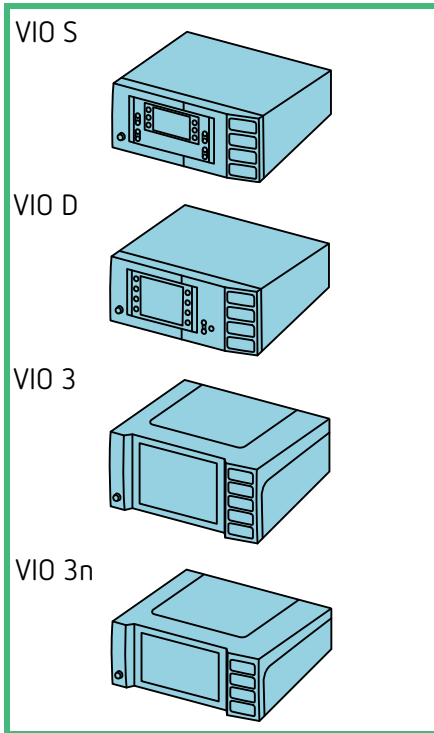
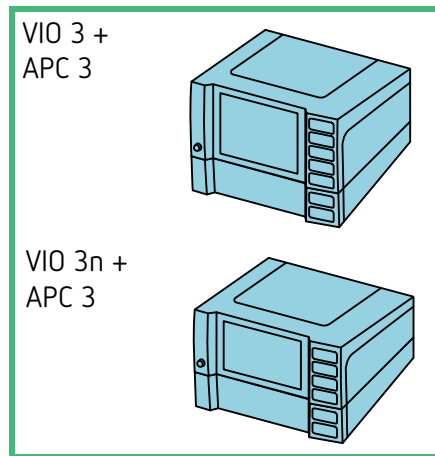
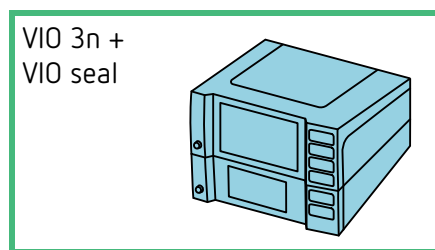
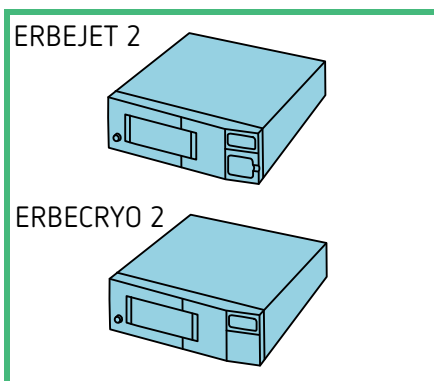
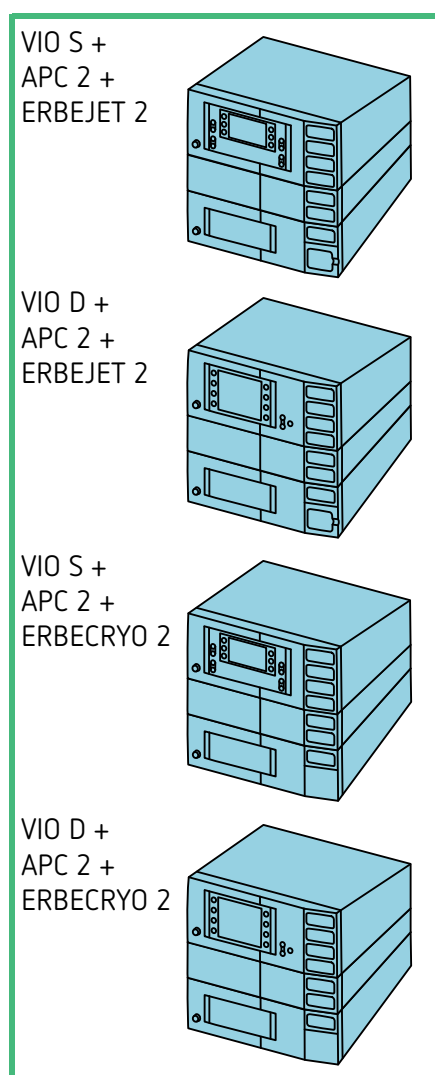
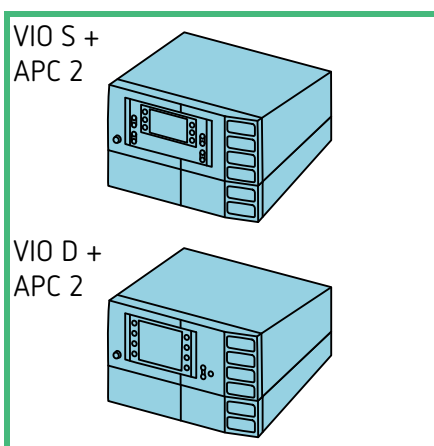
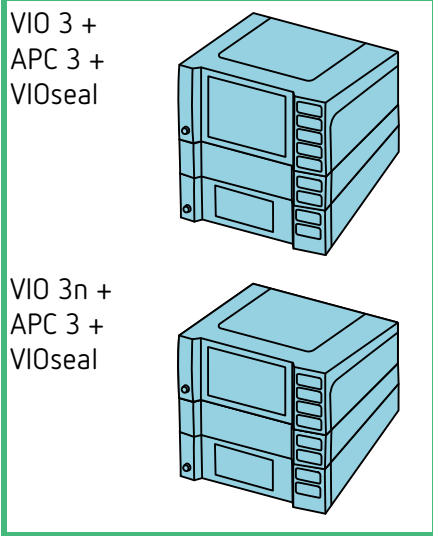
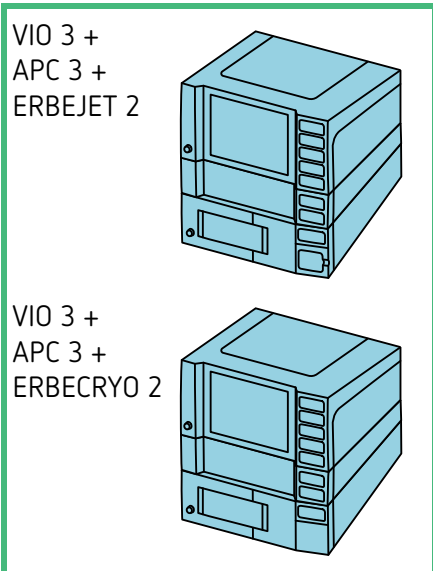
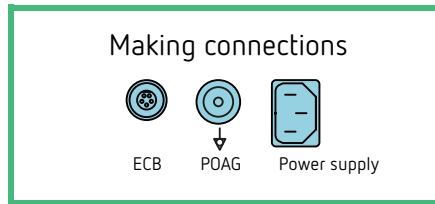
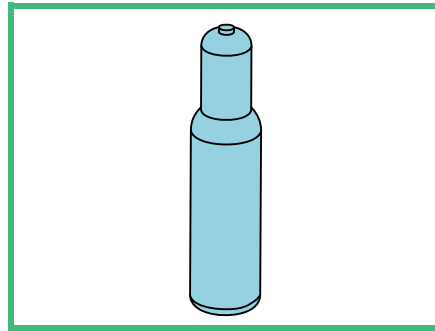
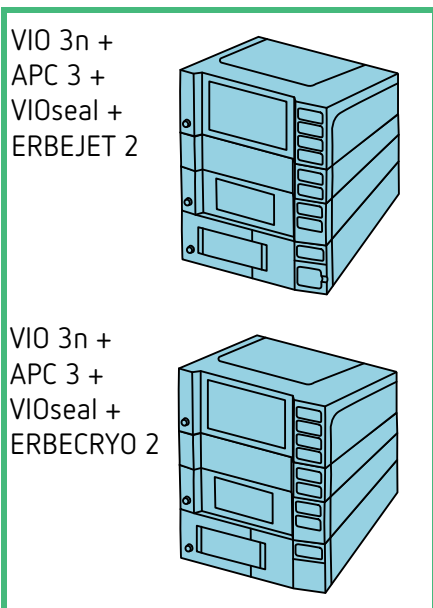
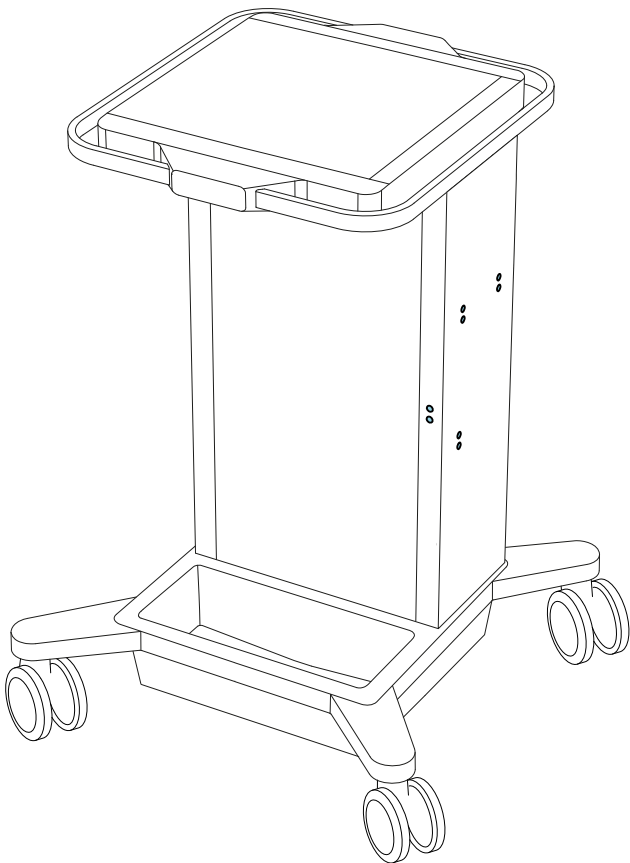
Installing the cable holders on the system carrier

Normal use

Permitted installation points


Maximum loading:
4 items

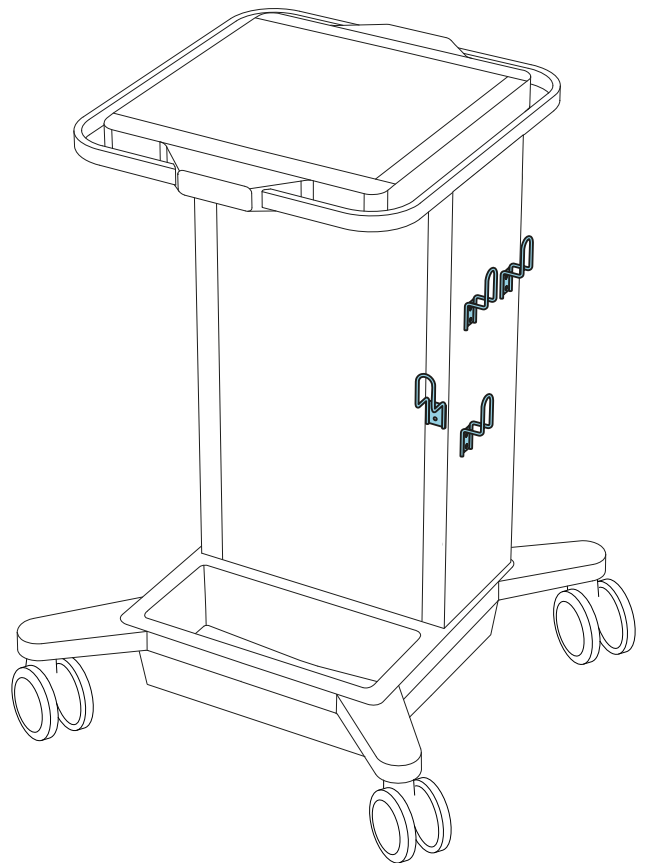Scope of delivery

IMPORTANT! Only use original Erbe accessories.

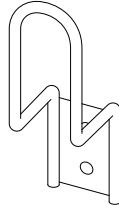Erbe REF 20180-460

M4 x 8

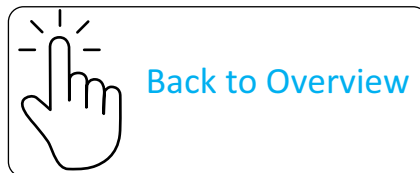
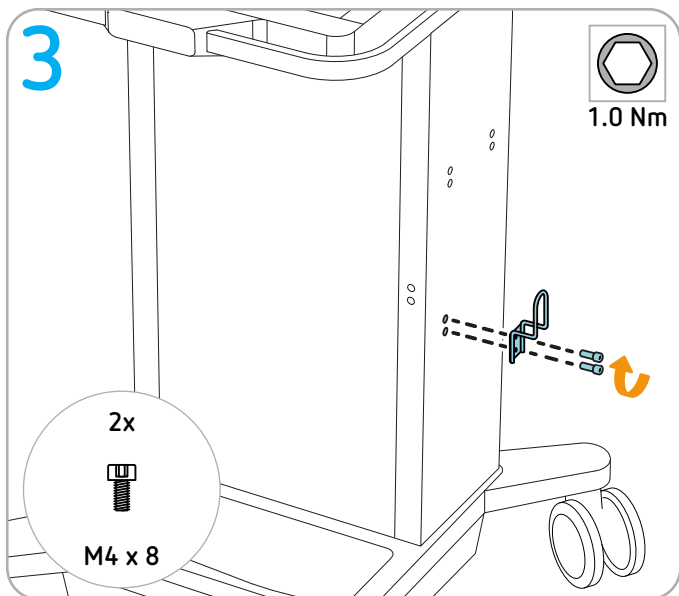
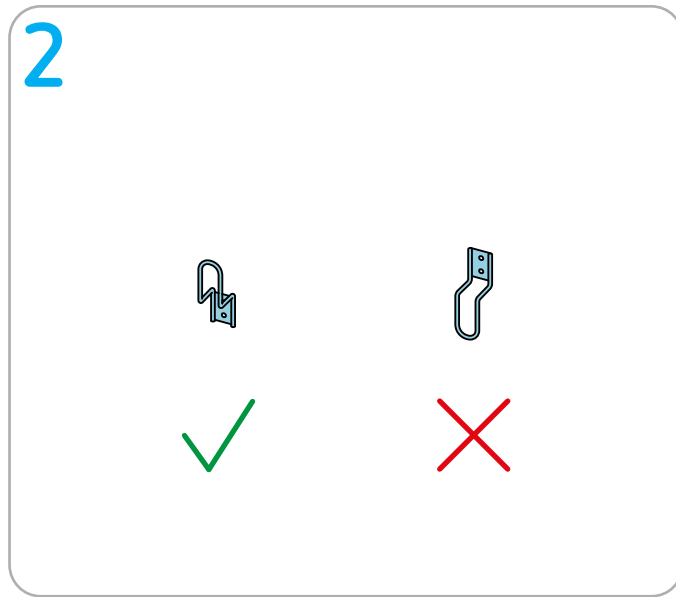
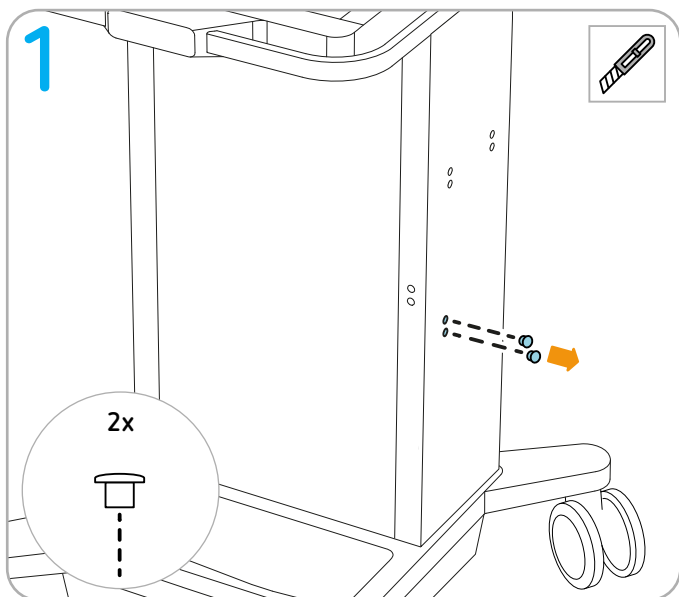
Tools required

Torque screwdriver (1 Nm) hex socket 2.5

Cutter



Installing the cable holders in the door

Normal use

Permitted installation points


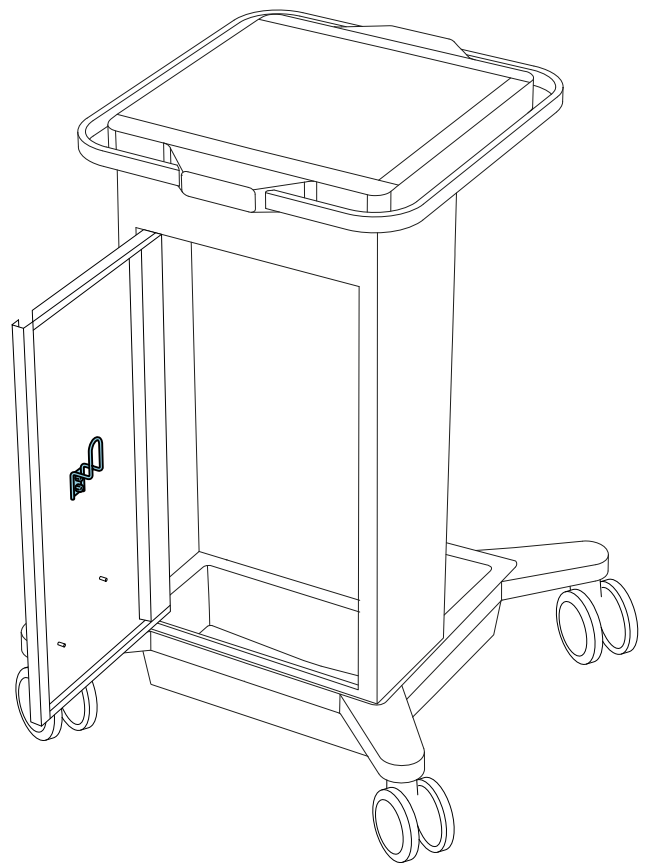
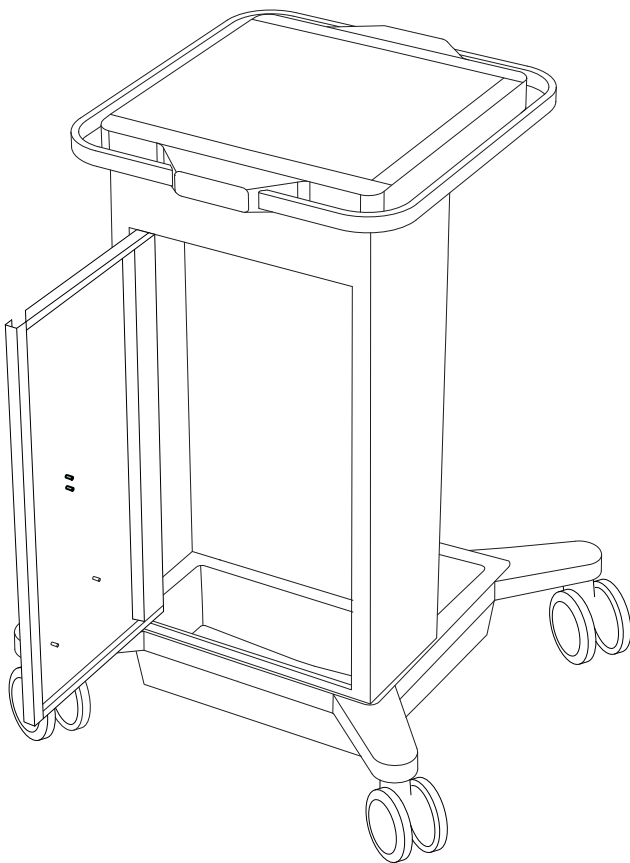
Maximum loading:
1 item

Scope of delivery

IMPORTANT! Only use original Erbe accessories.


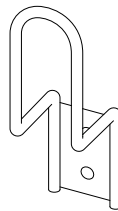
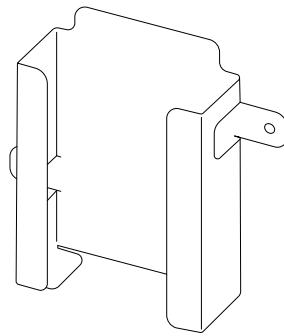
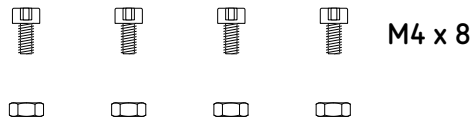
Erbe REF 20180-464

Universal fastening set: Not all parts are required for the following assembly steps.

Tools required

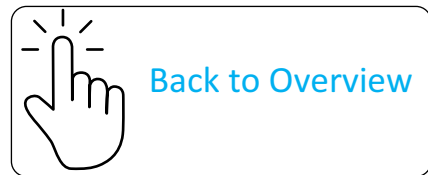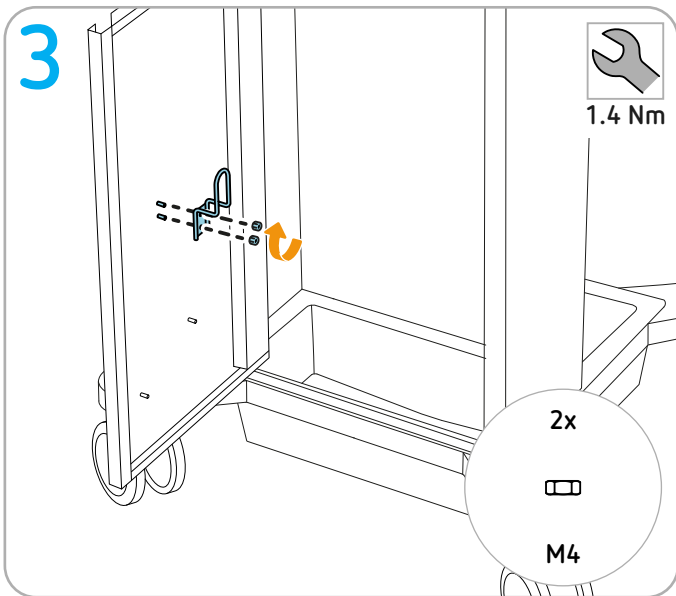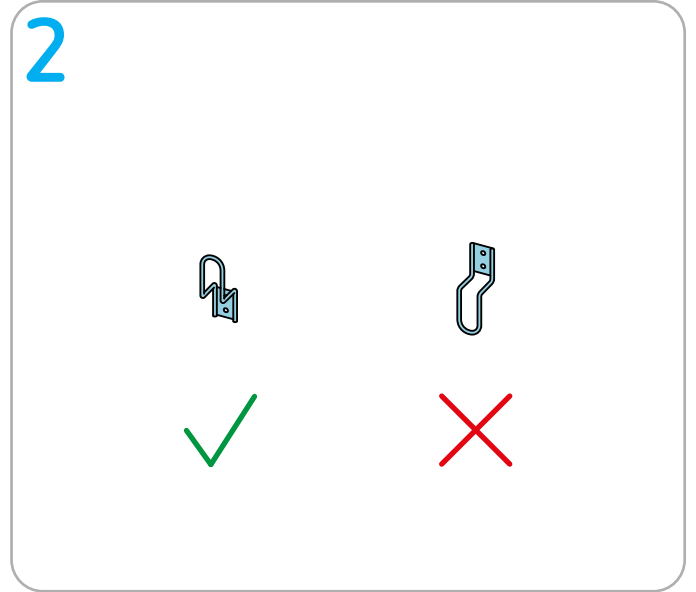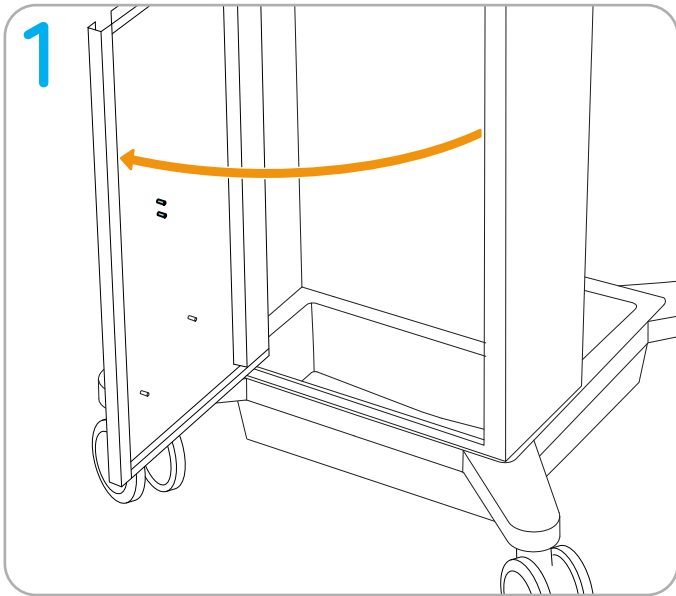 Torque screwdriver (1.4 Nm) Open-end wrench 7


Installing footswitch support for large one-pedal footswitch

Normal use


This footswitch support is compatible with the following Erbe one-pedal footswitches:

- VIO S, VIO D and VIO 3 one-pedal footswitch
- IES 3 one-pedal footswitch

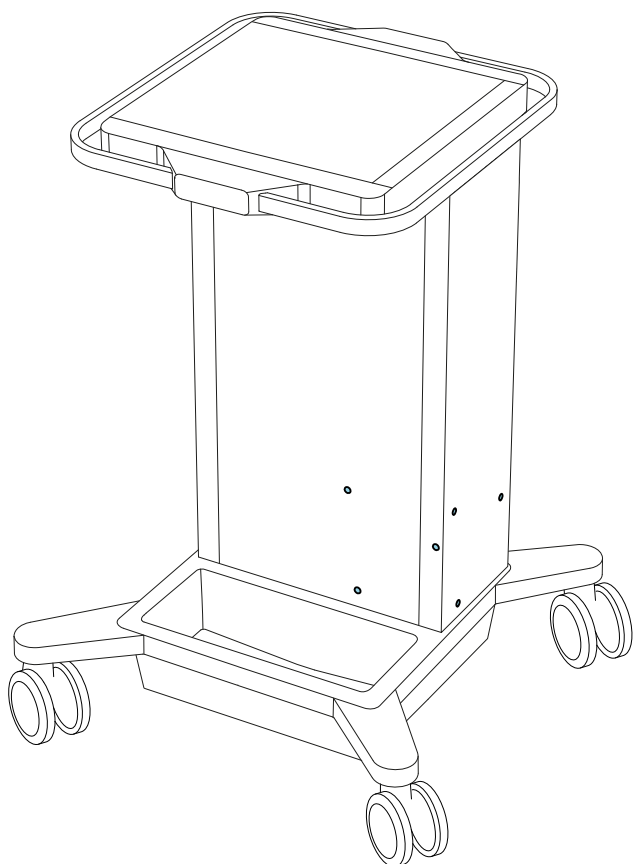Permitted installation points


Maximum loading:
2 items

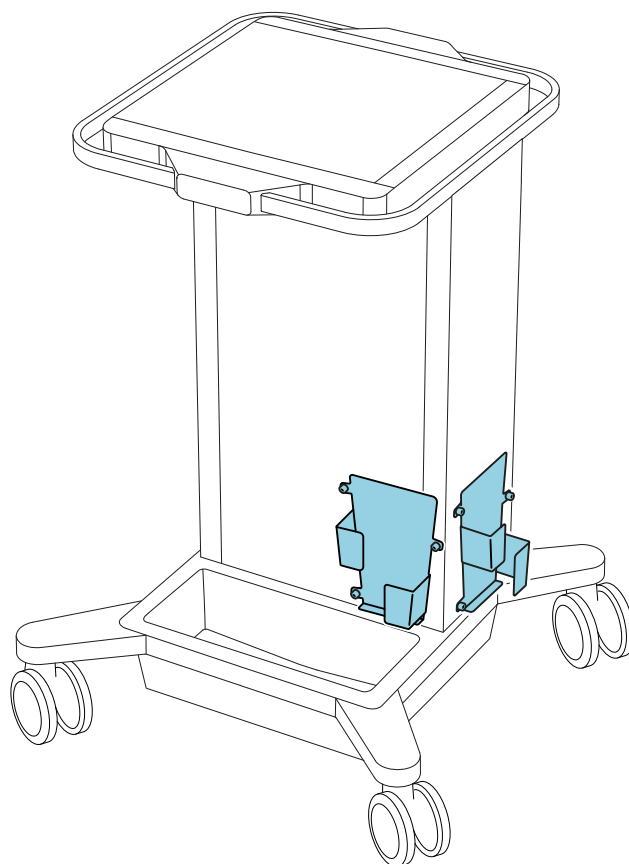Scope of delivery

IMPORTANT! Only use original Erbe accessories.




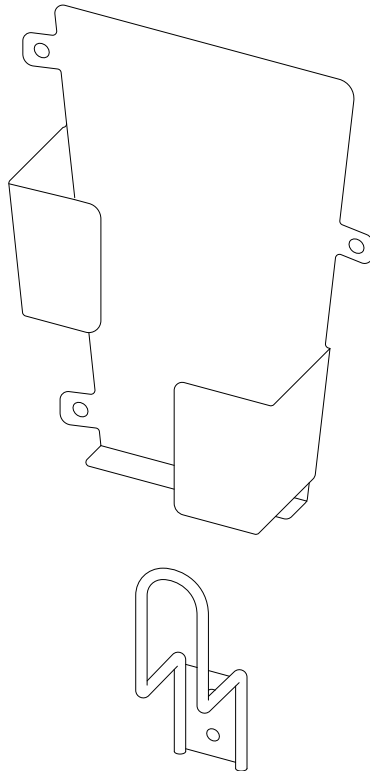
Erbe REF 20180-463

Tools required

 Torque screwdriver (1 Nm) hex socket 2.5

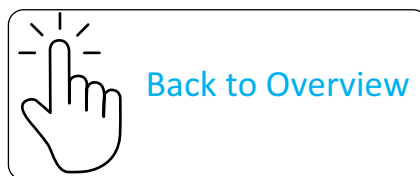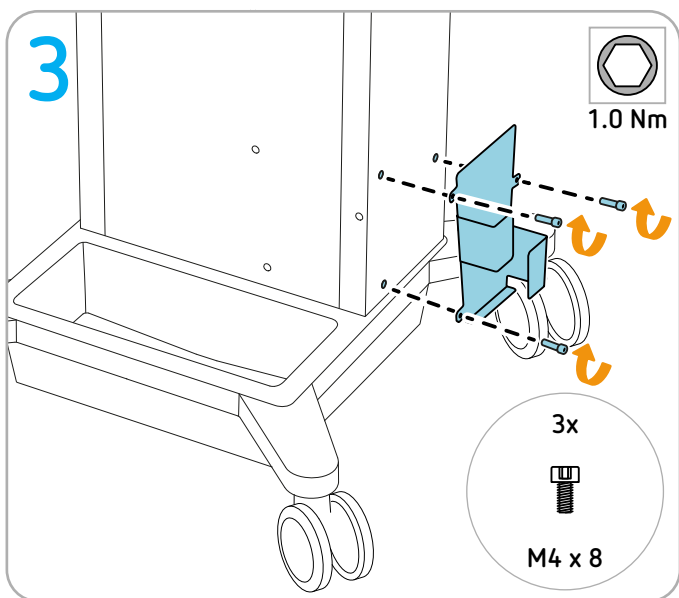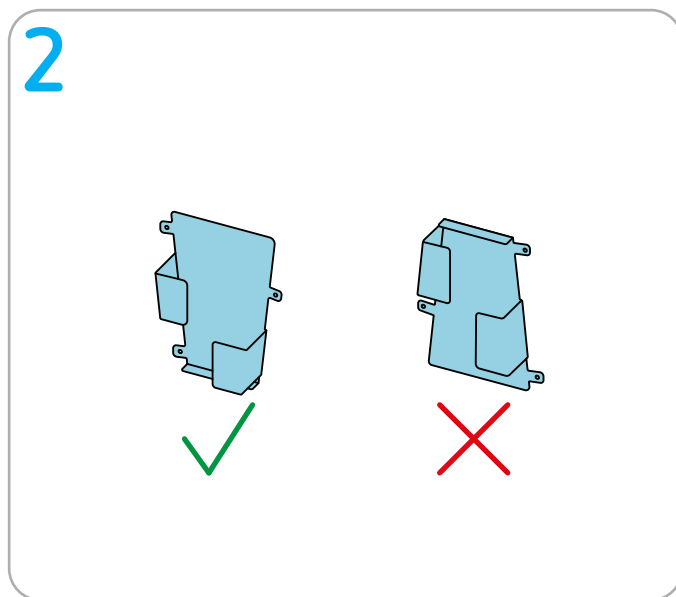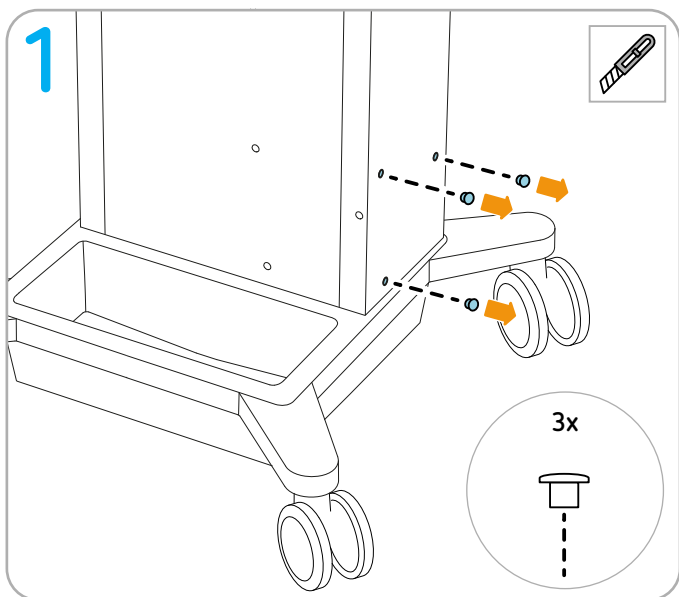 Cutter


Installing footswitch support for small one-pedal footswitch on the system carrier

Normal use


This footswitch support is compatible with the following Erbe one-pedal footswitches:

- EIP 2 one-pedal footswitch

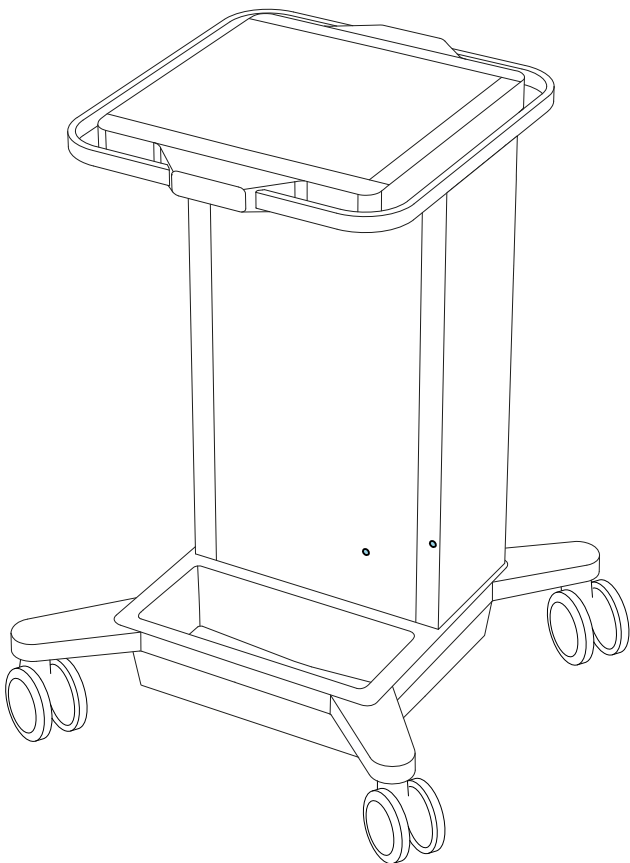Permitted installation points




Maximum loading:
1 item

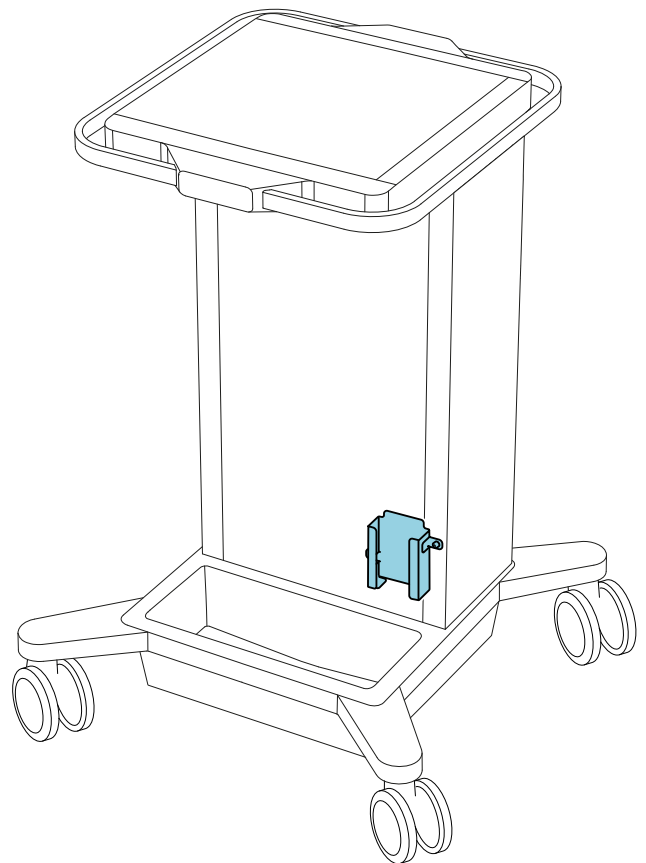Scope of delivery

IMPORTANT! Only use original Erbe accessories.

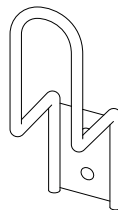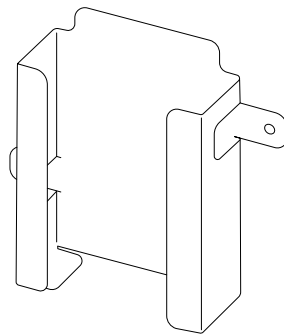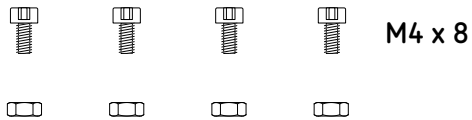Erbe REF 20180-464





Universal fastening set: Not all parts are required for the following assembly steps.

Tools required

 Torque screwdriver (1 Nm) hex socket 2.5

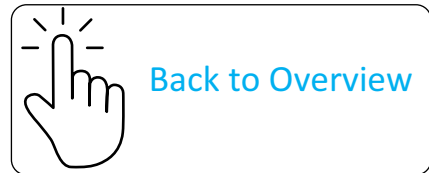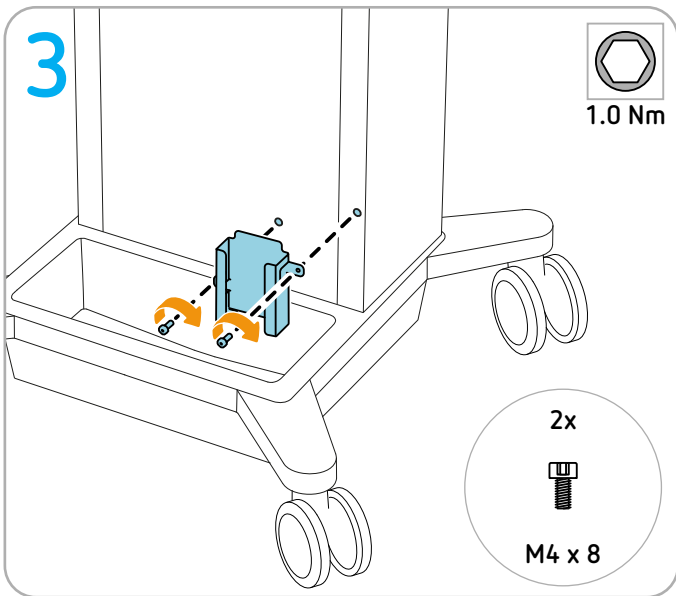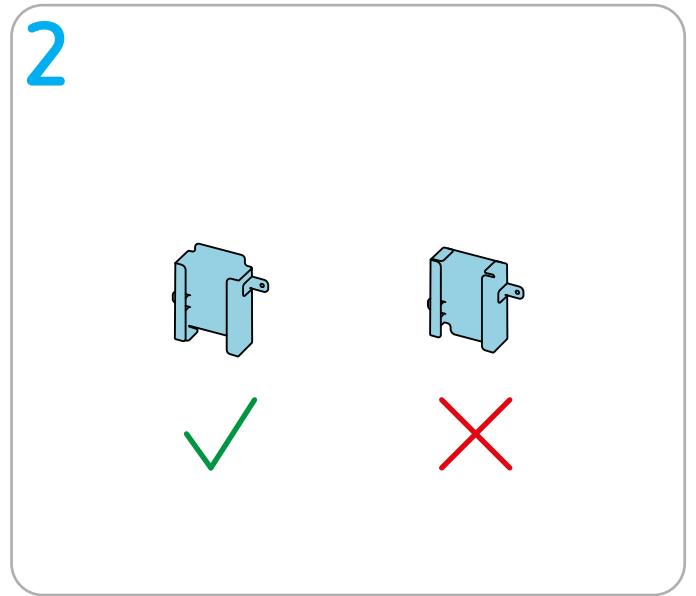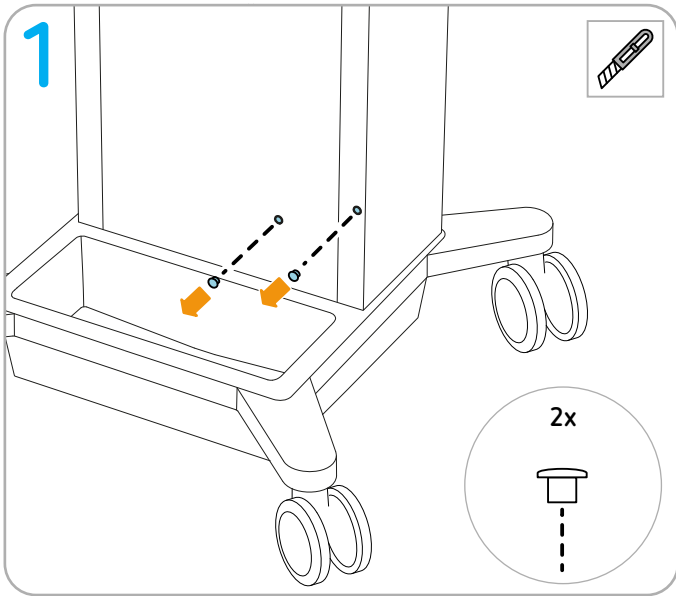 Cutter

Installing footswitch support for small one-pedal footswitch in the door

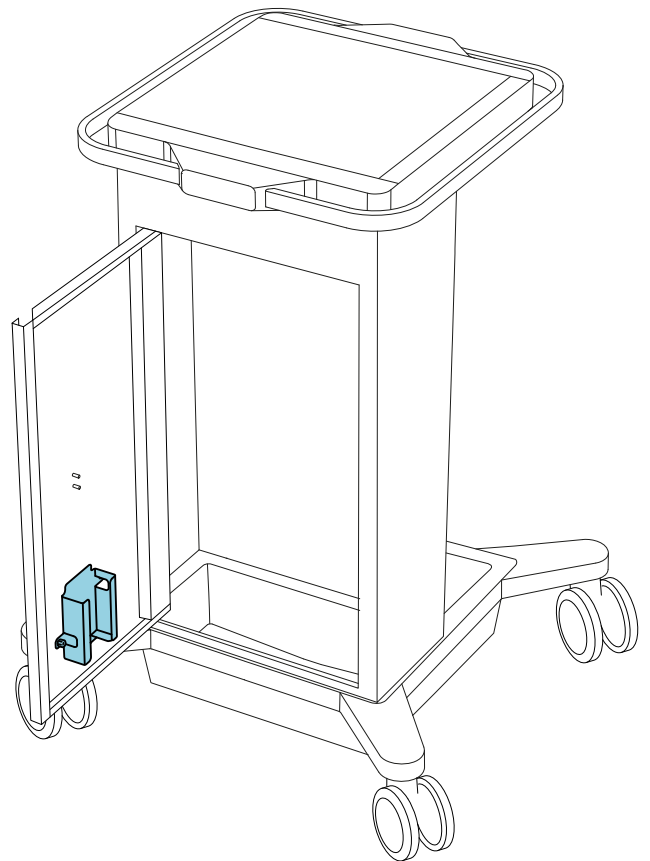
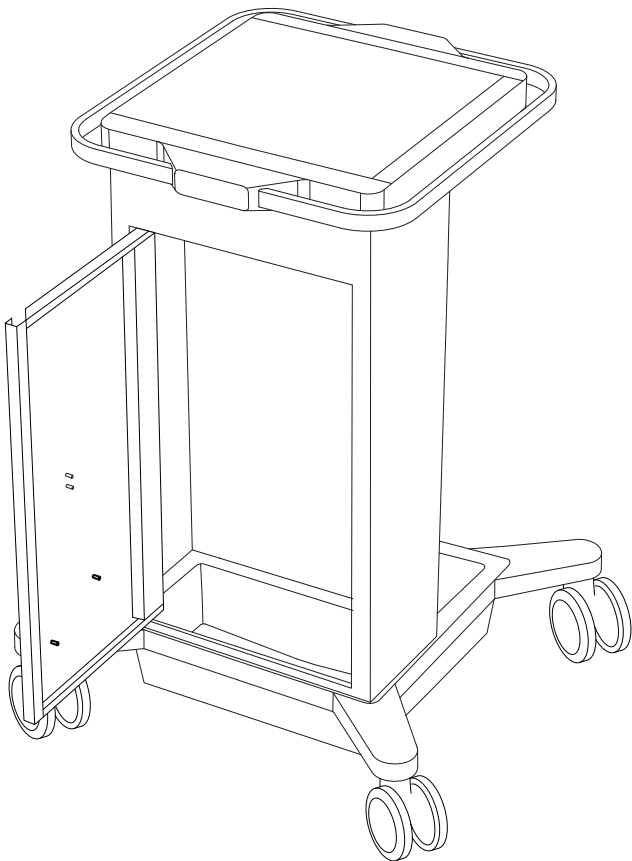
Normal use

This footswitch support is compatible with the following Erbe one-pedal footswitches:

- EIP 2 one-pedal footswitch

Permitted installation points




Maximum loading:
1 item

Scope of delivery

IMPORTANT! Only use original Erbe accessories.


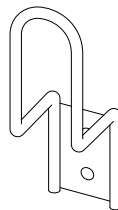
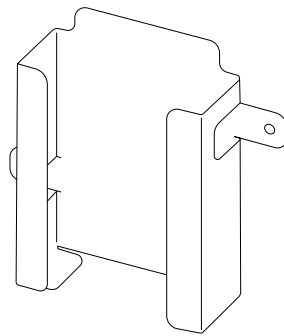
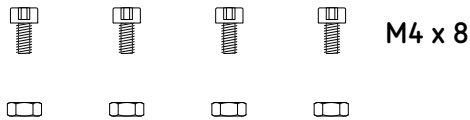
Erbe REF 20180-464

Universal fastening set: Not all parts are required for the following assembly steps.

Tools required

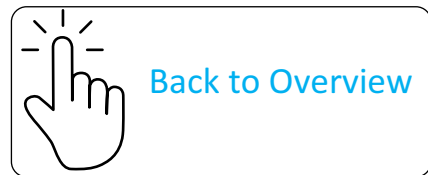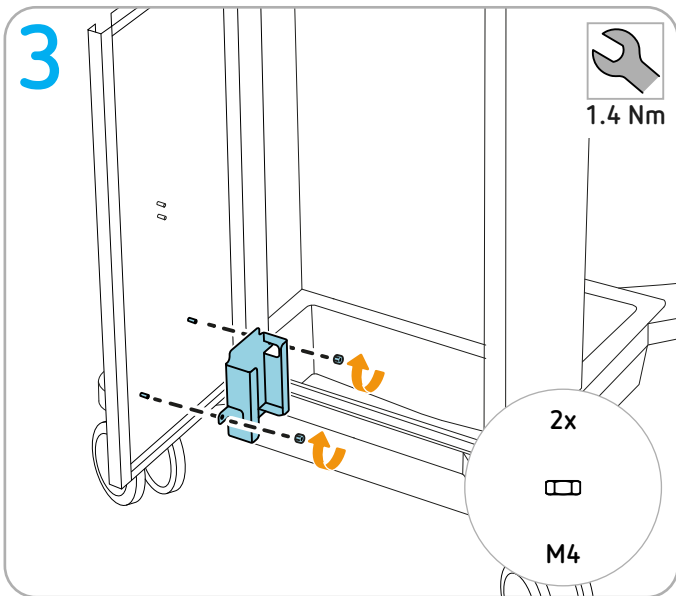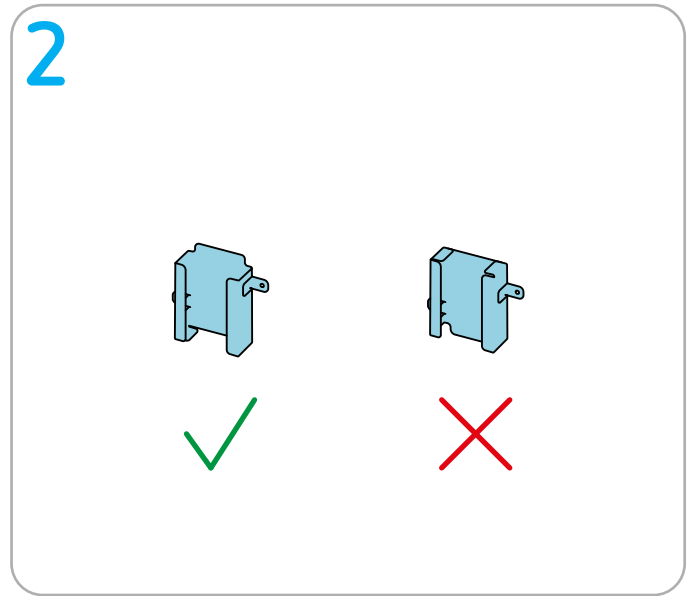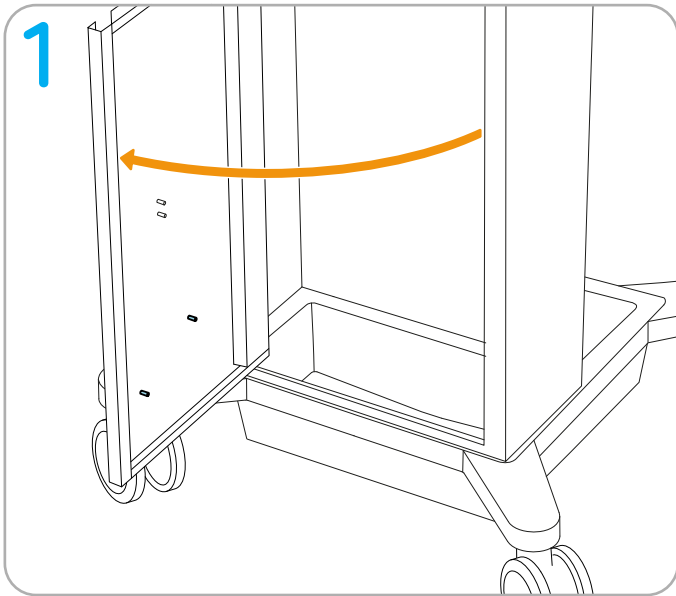
 Torque screwdriver (1.4 Nm) Open-end wrench 7


Installing attachment rail for ESM 2 suction container

Normal use

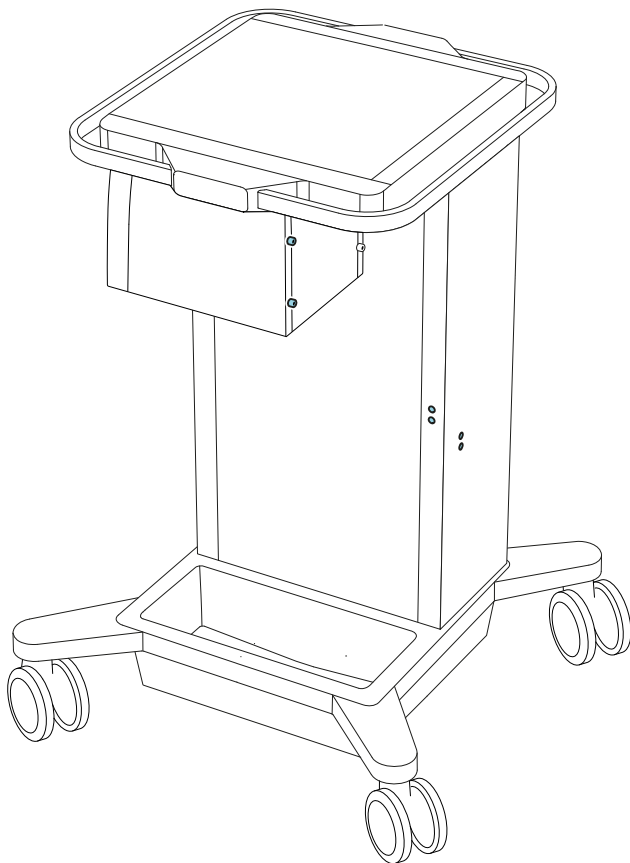Permitted installation points


Maximum loading:
3 items

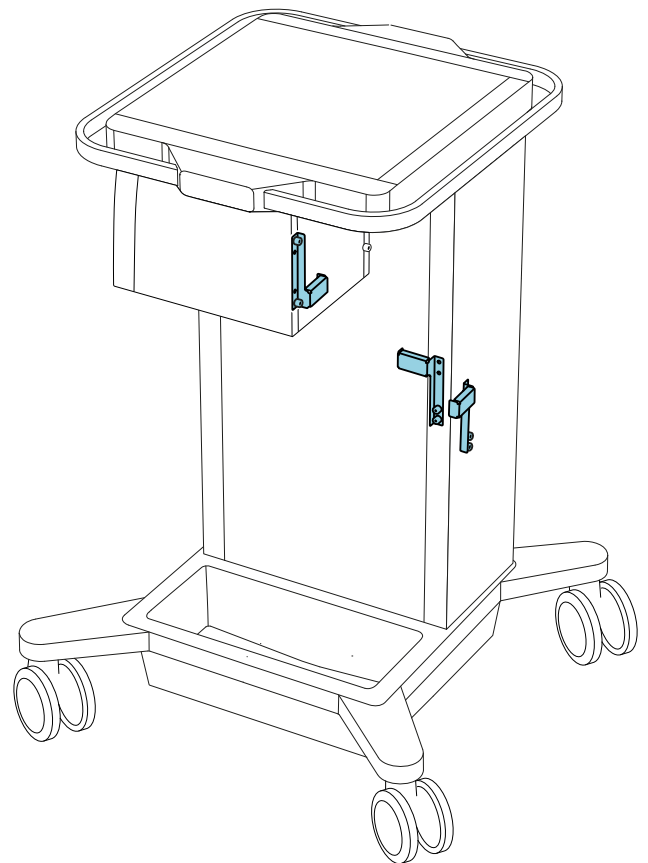Scope of delivery

IMPORTANT! Only use original Erbe accessories.




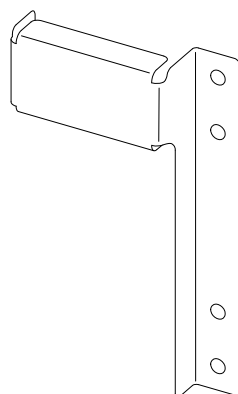
Erbe REF 20180-509





Tools required

 Torque screwdriver (1 Nm) hex socket 2.5

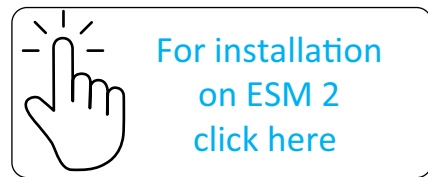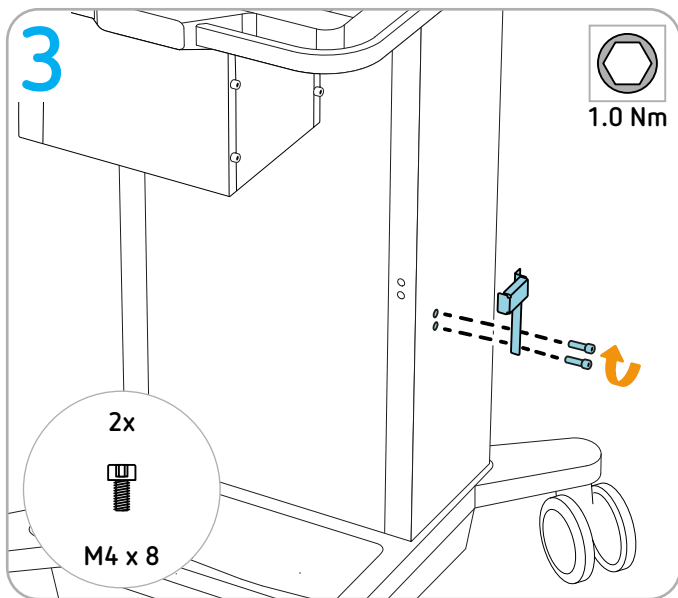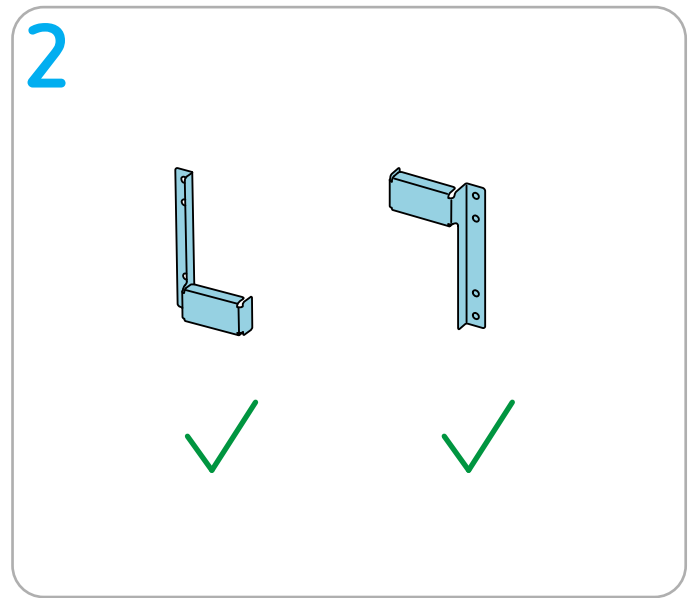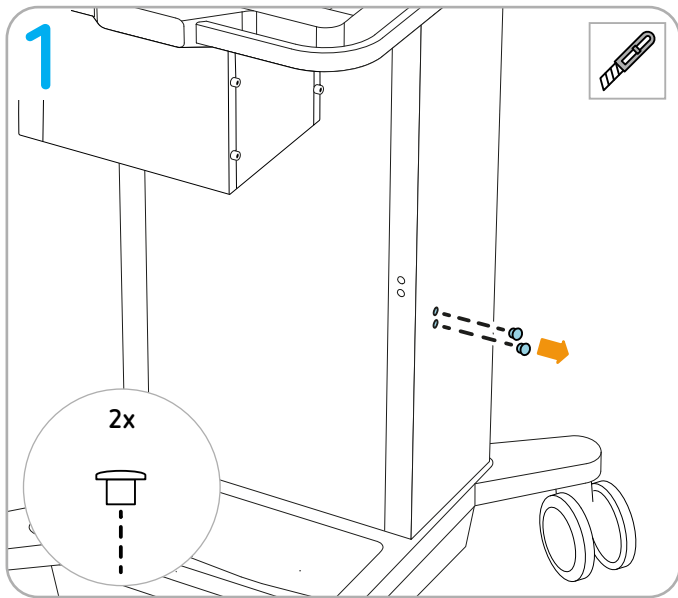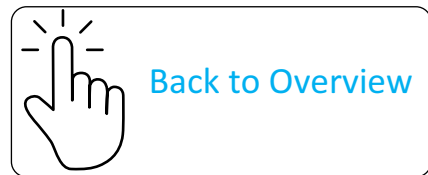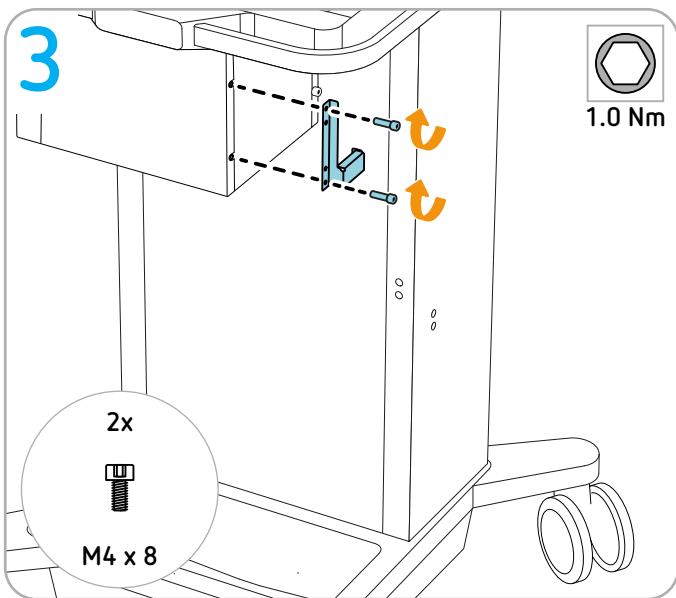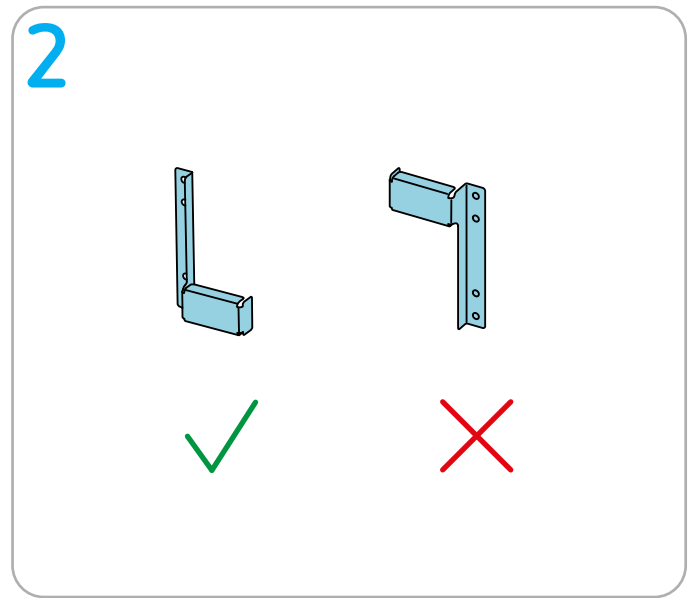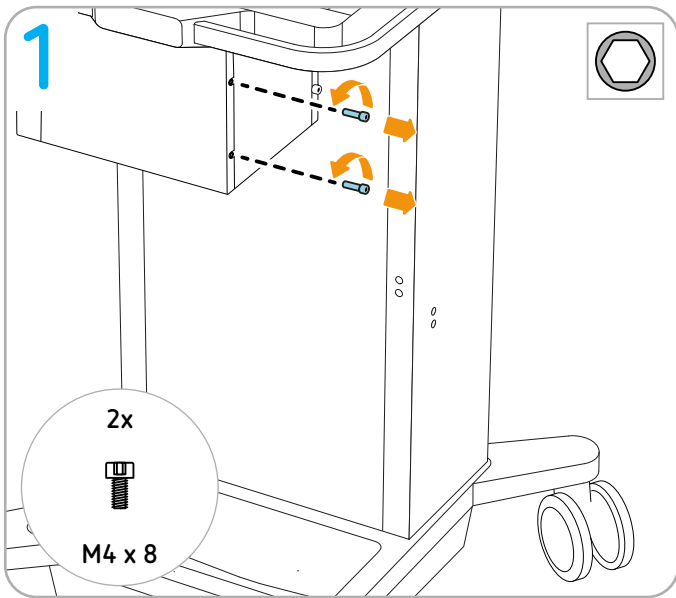
 Cutter

Installing IES 3

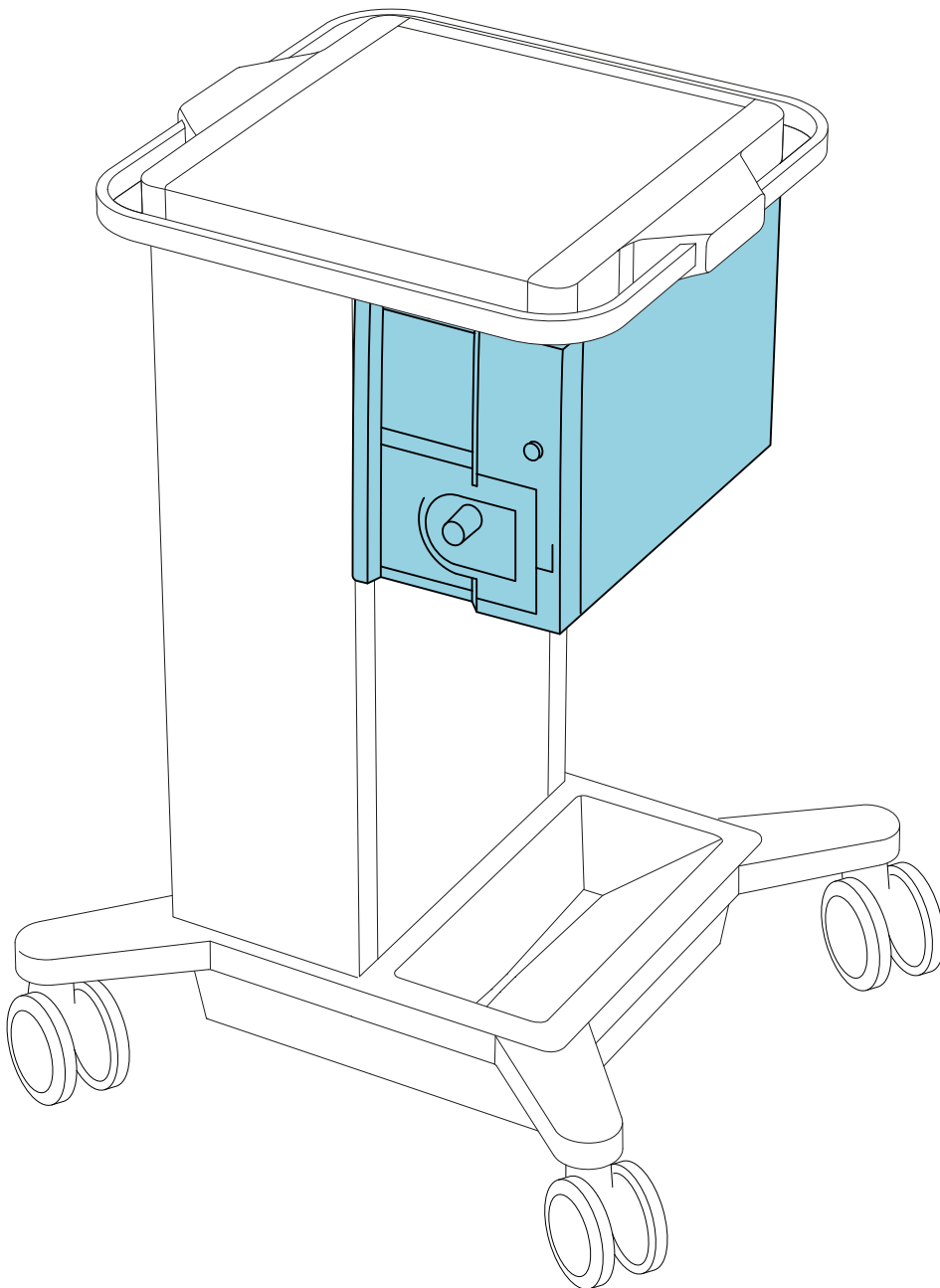
Permitted positioning

Erbe fastening set

IMPORTANT! Only use original Erbe accessories.

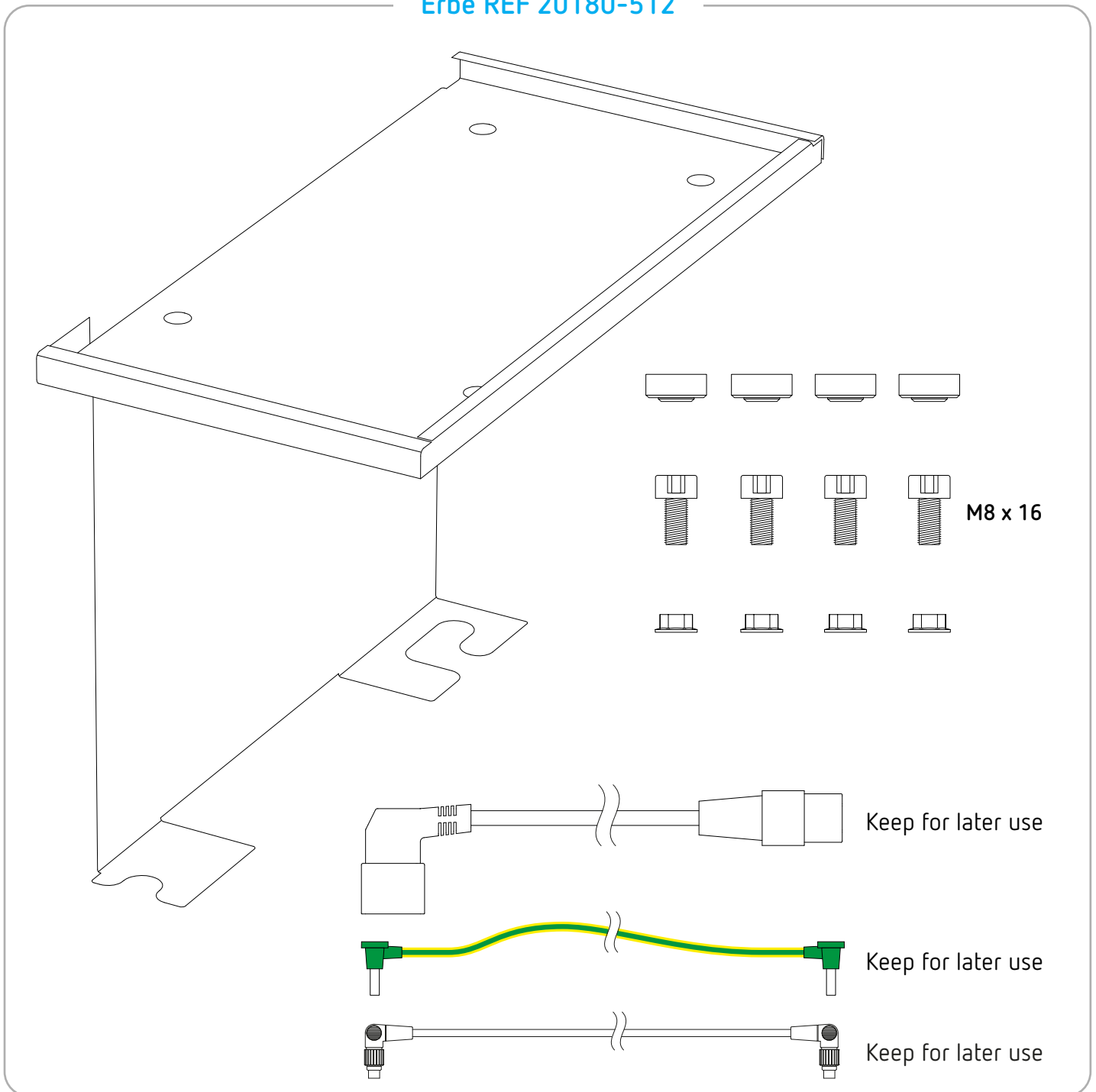Erbe REF 20180-512

Tools required

Torque screwdriver (1 Nm) hex socket 6

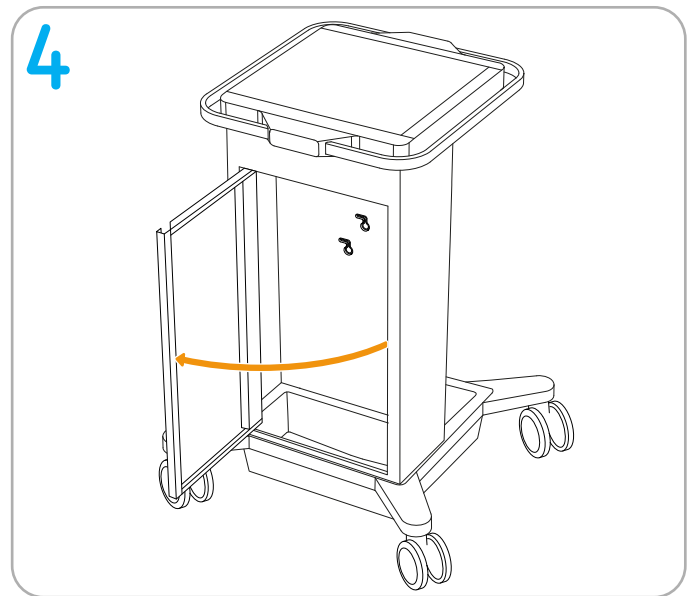
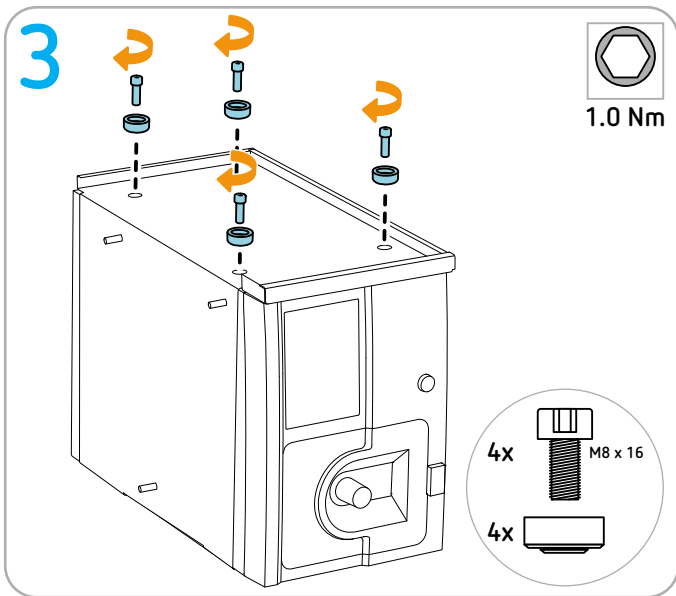
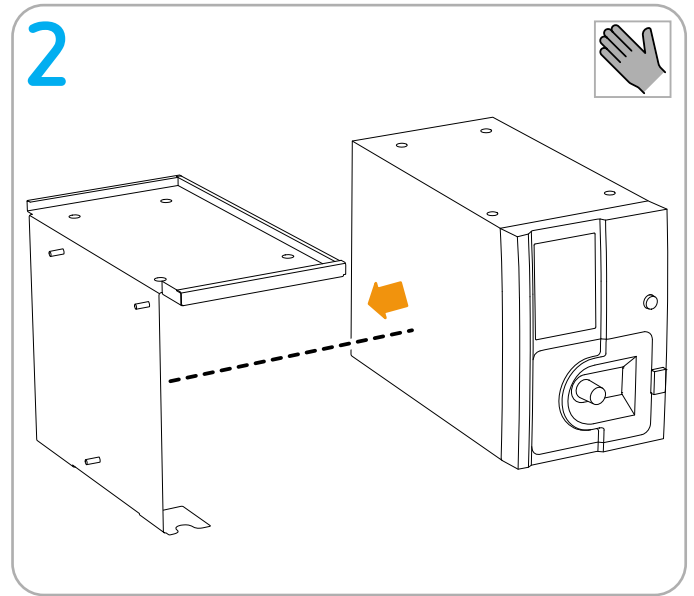
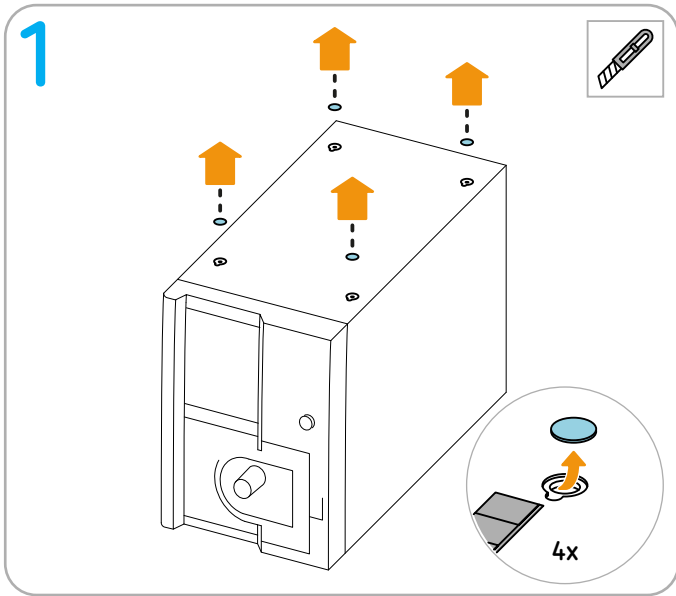
Torque screwdriver (4.9 Nm) open-end wrench 10




Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

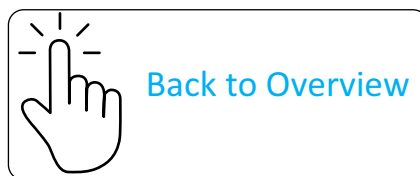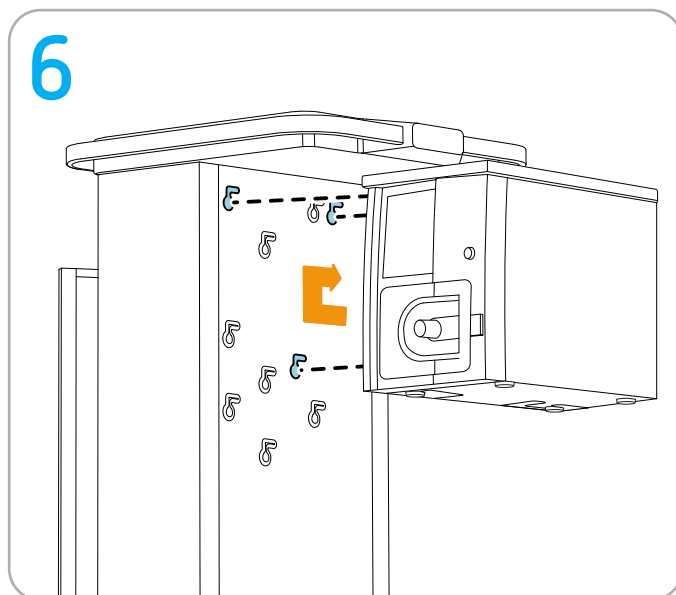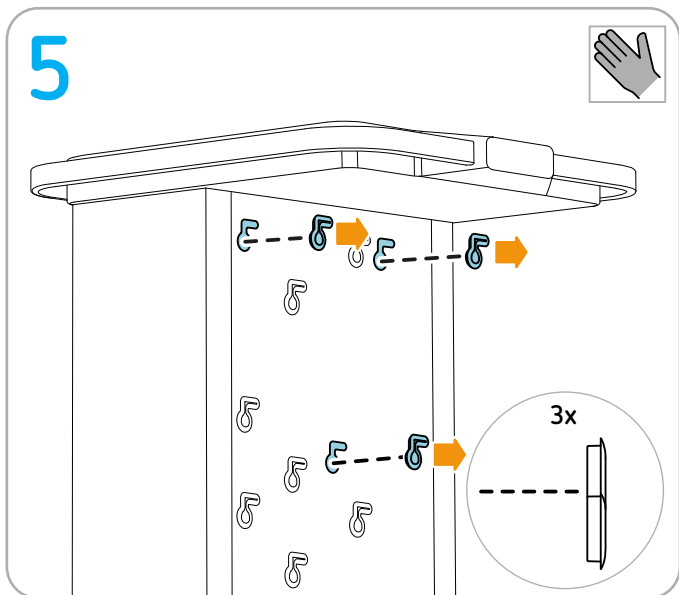Installing ESM 2

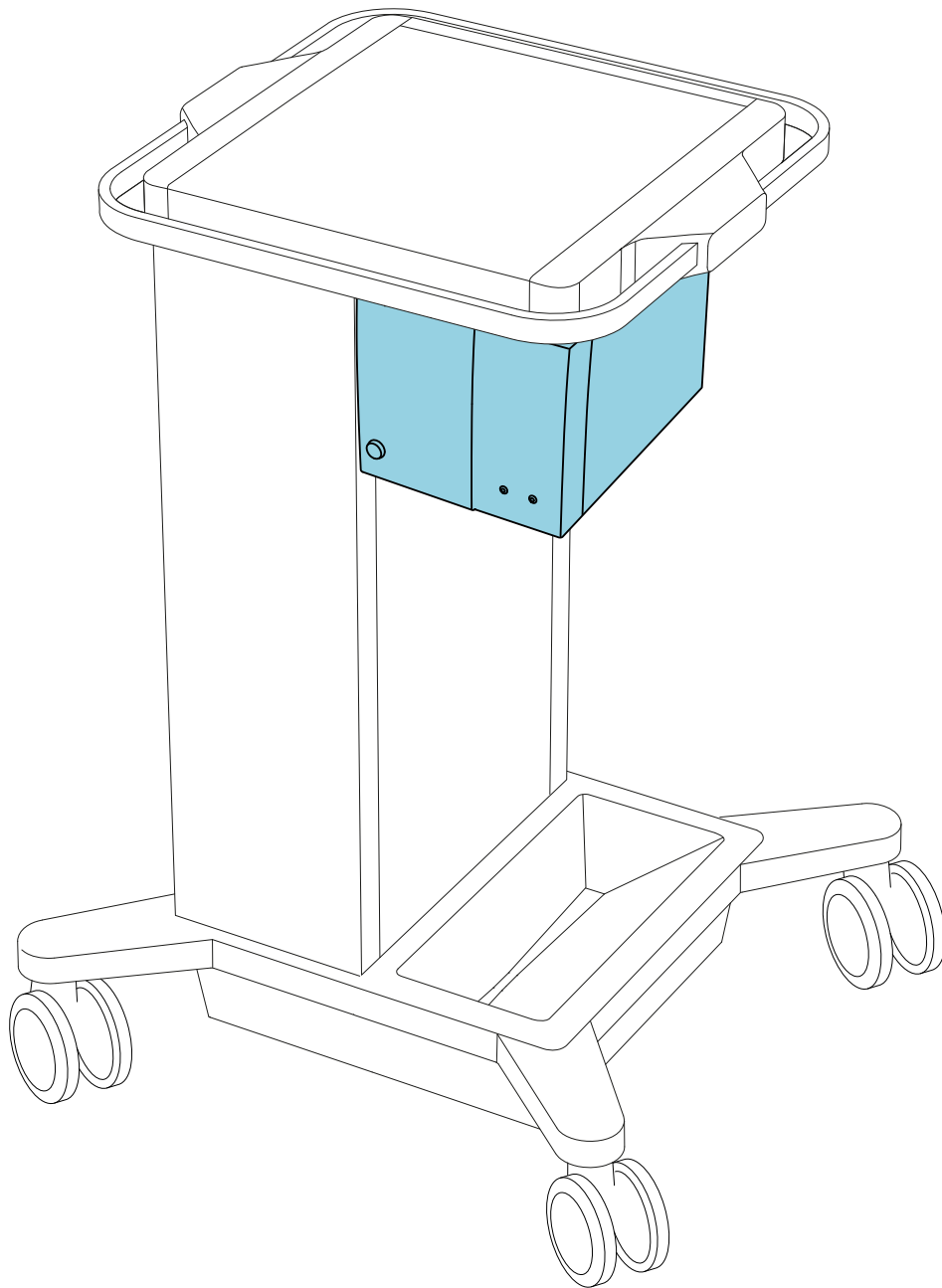
Permitted positioning

Erbe fastening set

IMPORTANT! Only use original Erbe accessories.

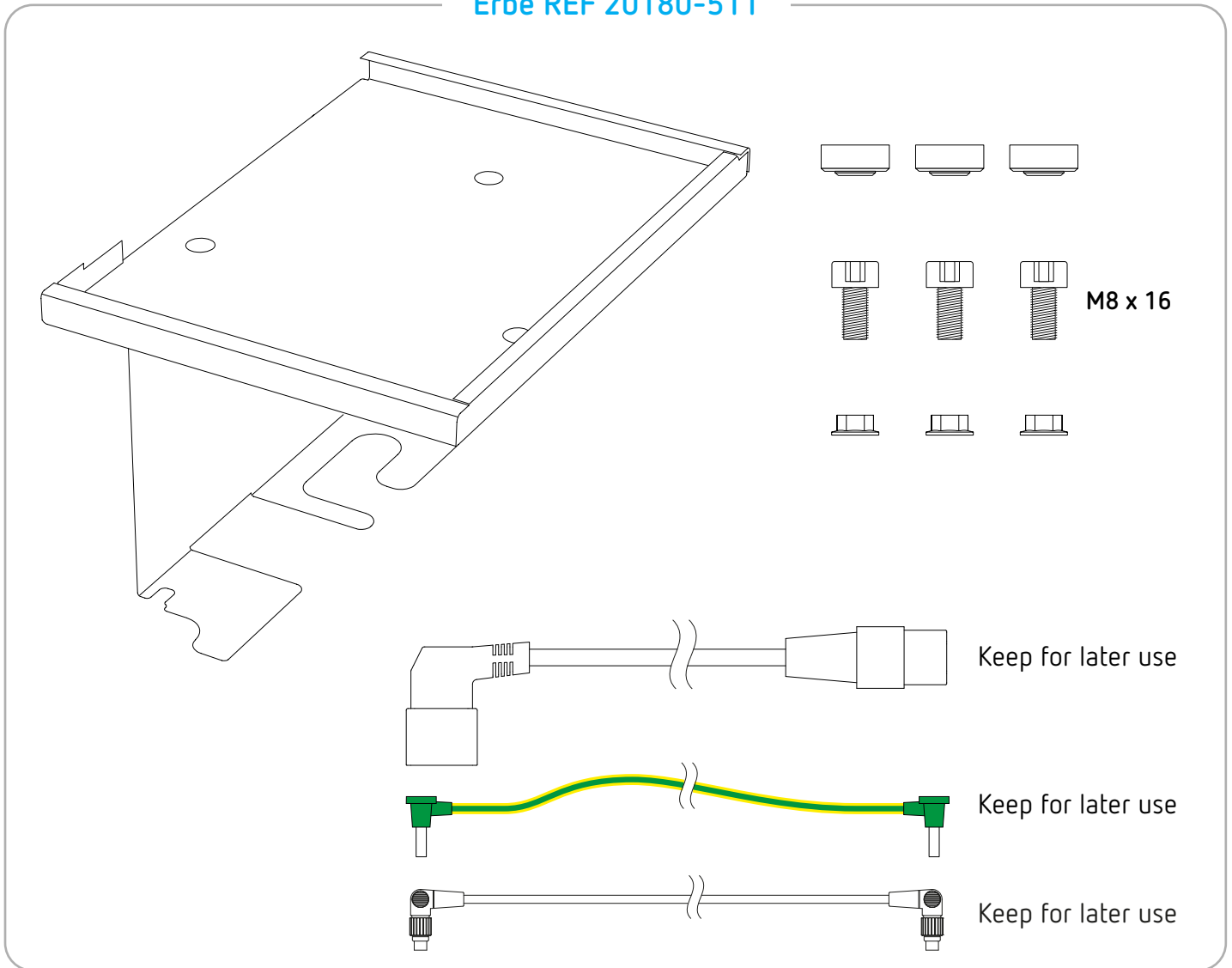Erbe REF 20180-511

Tools required

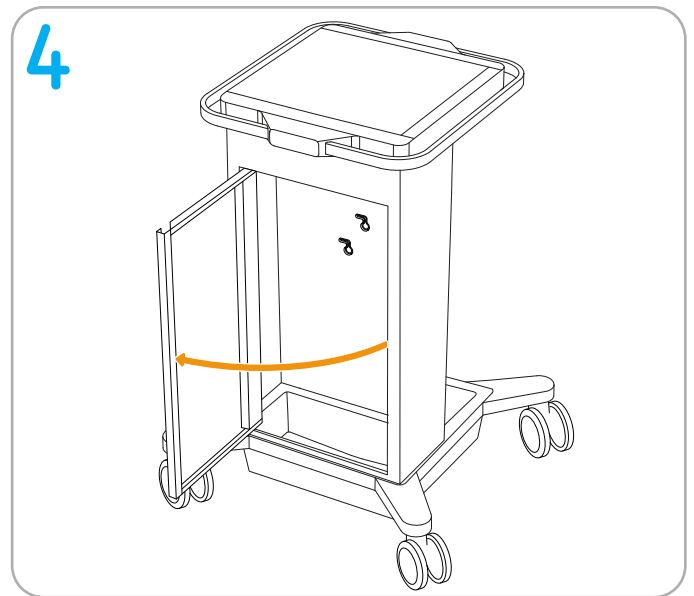
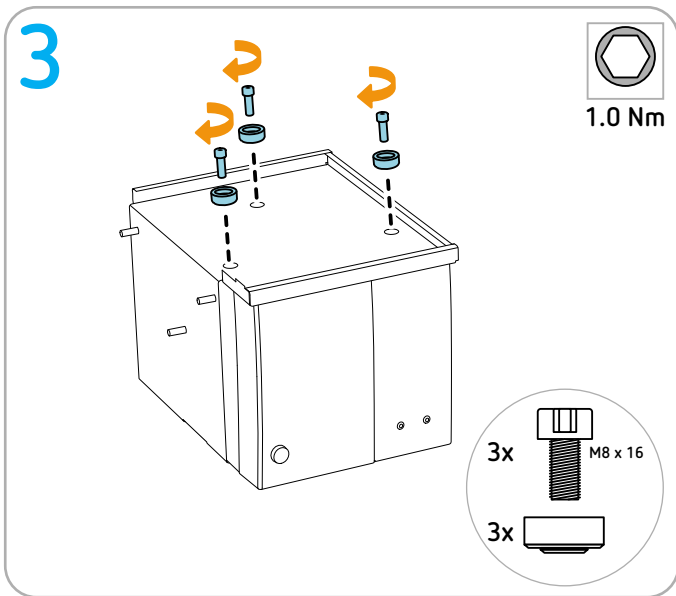
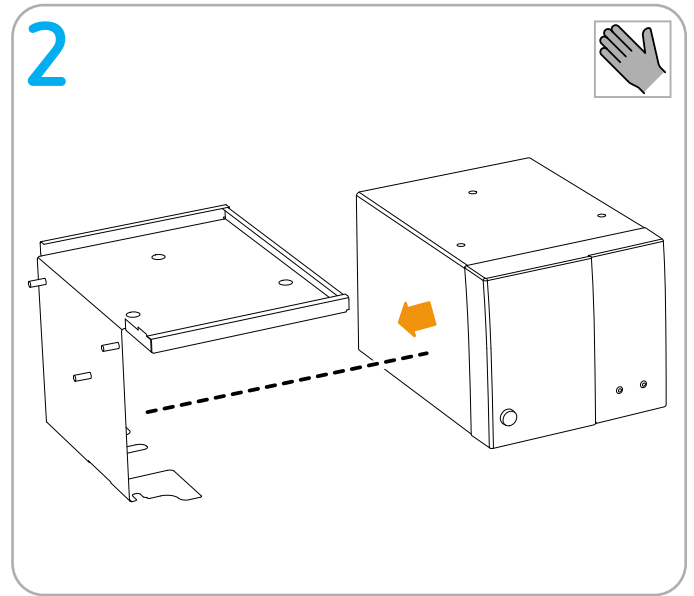
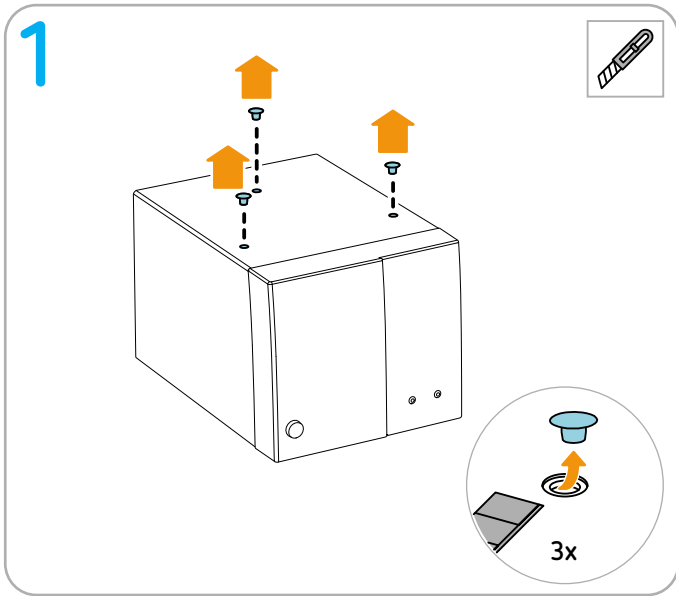
 Torque screwdriver (1 Nm) hex socket 6

 Torque screwdriver (4.9 Nm) open-end wrench 10





 Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

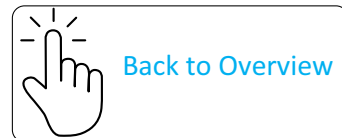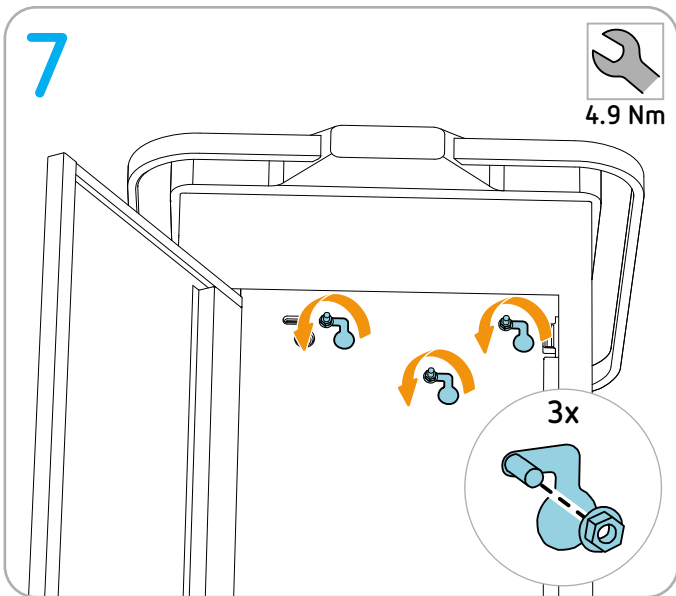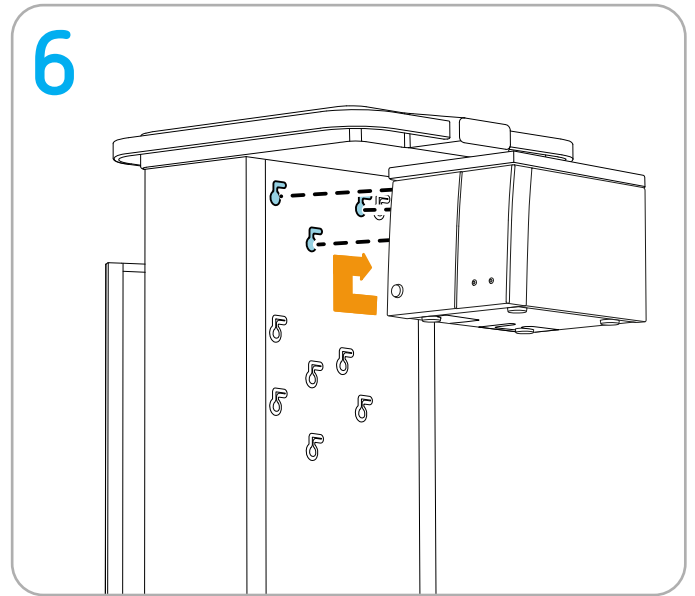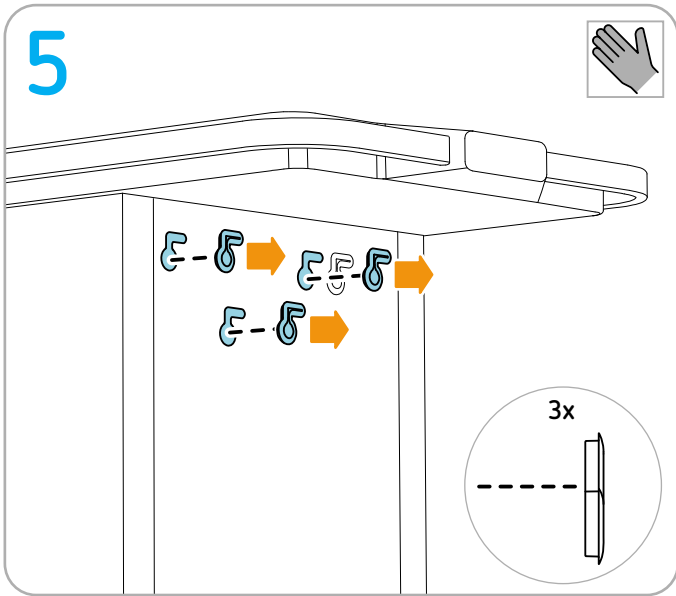Installing EIP 2

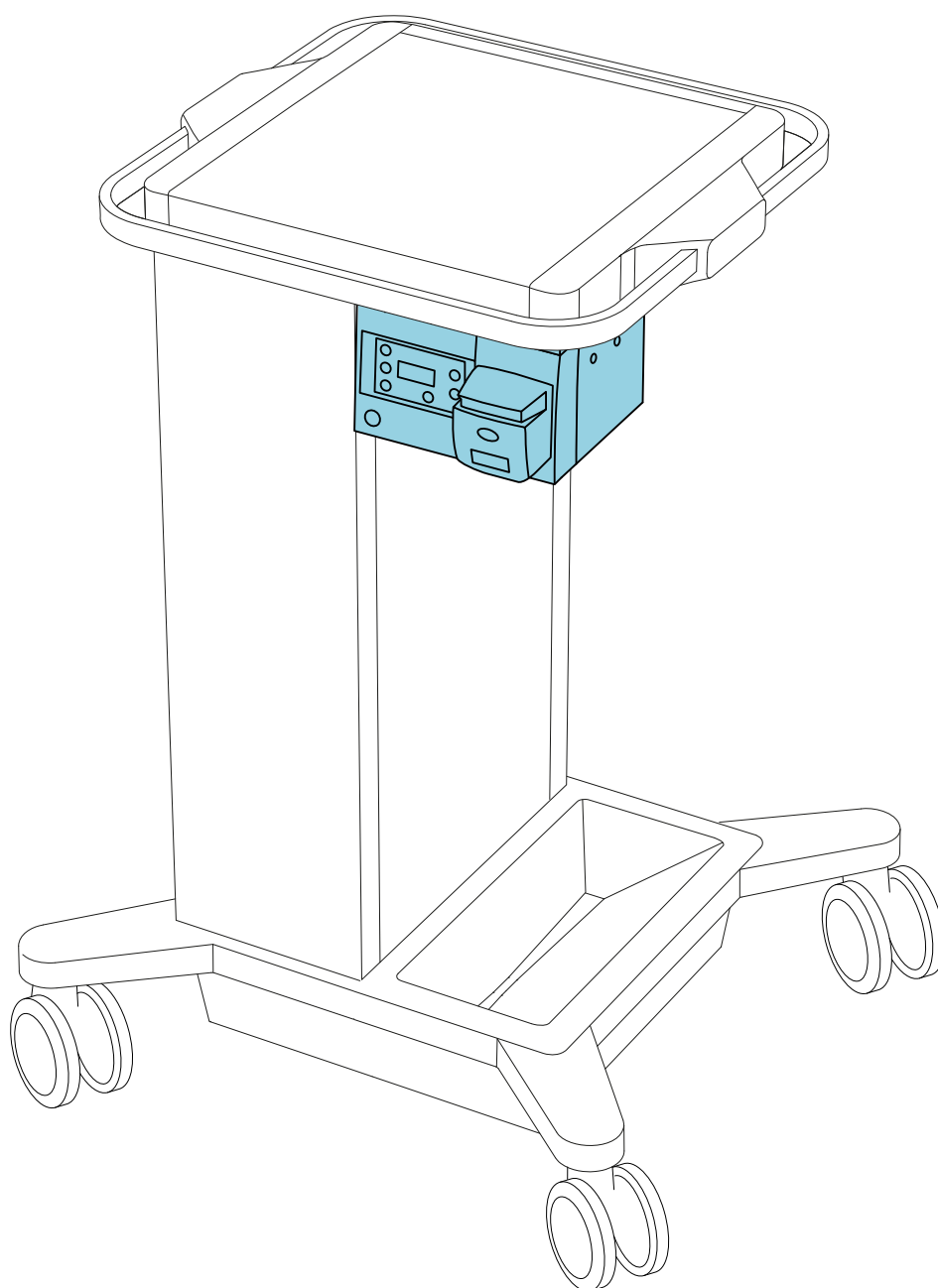
Permitted positioning

Erbe fastening set

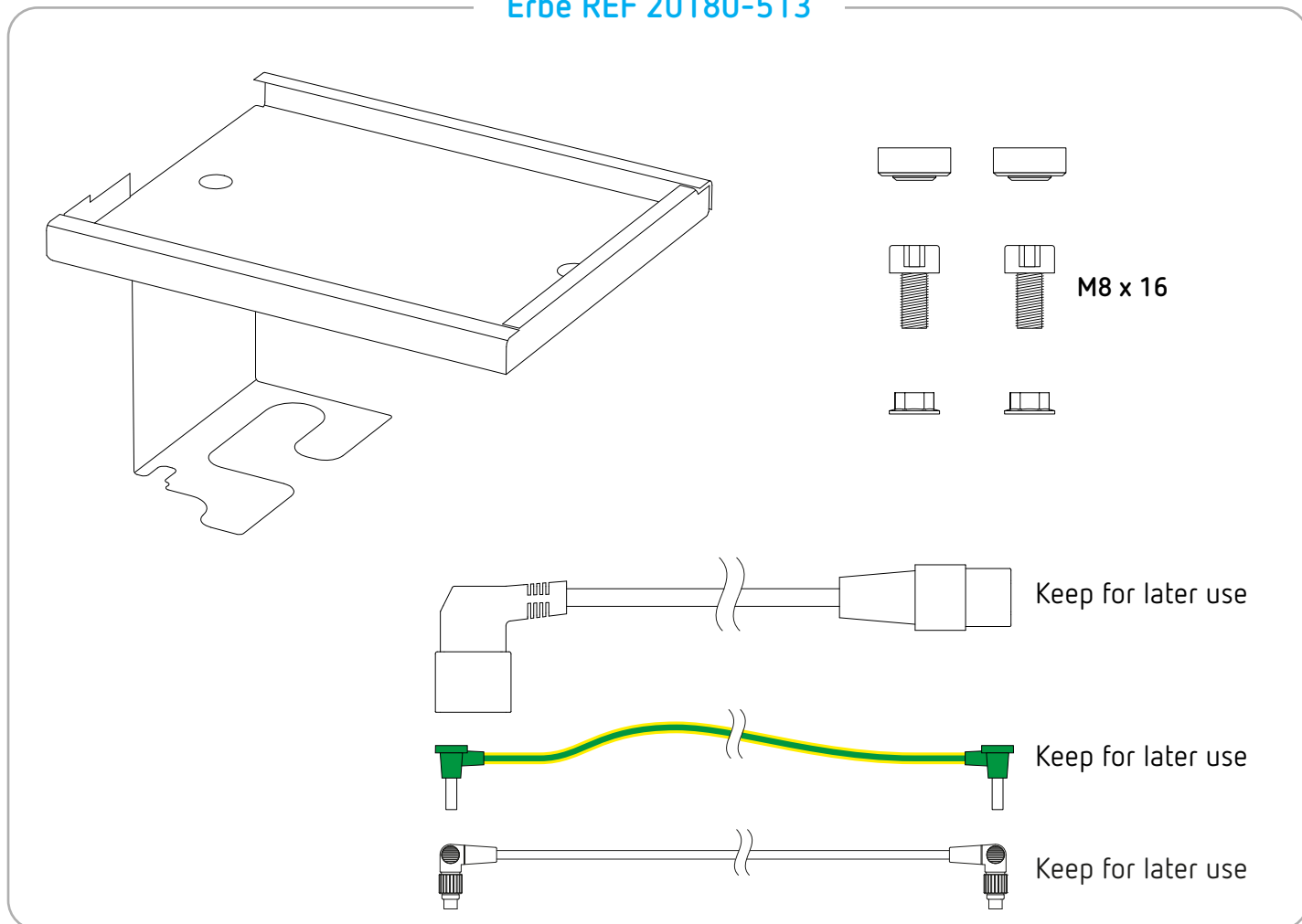IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-513

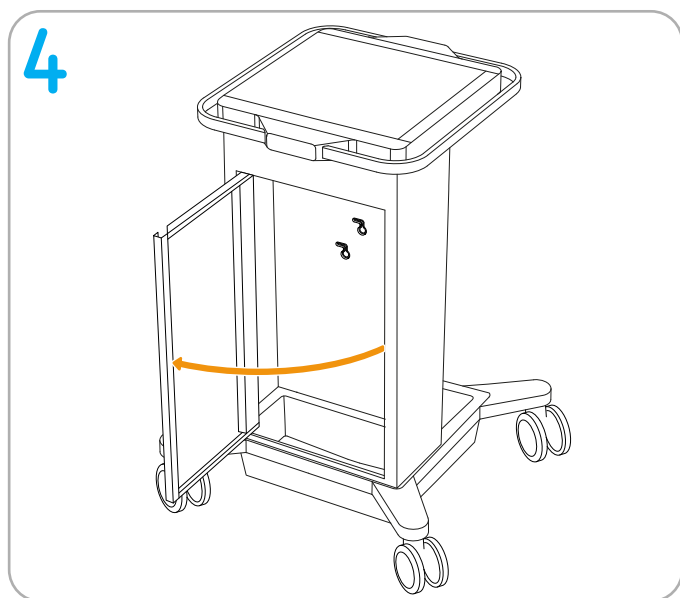
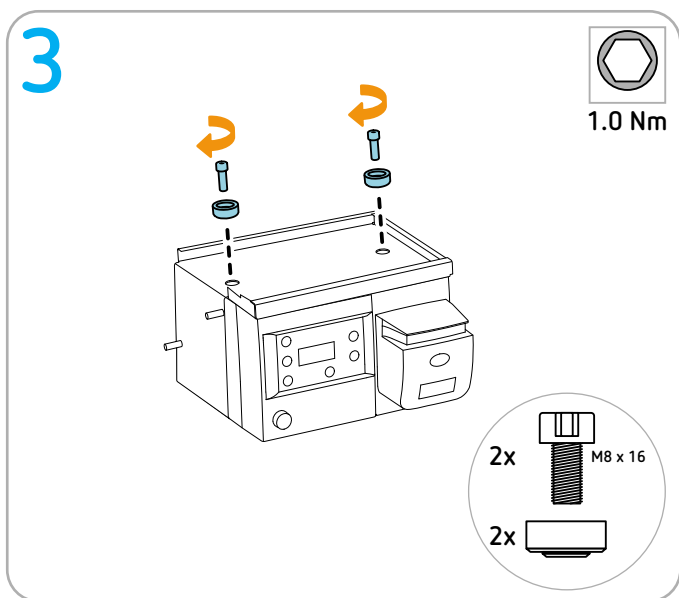
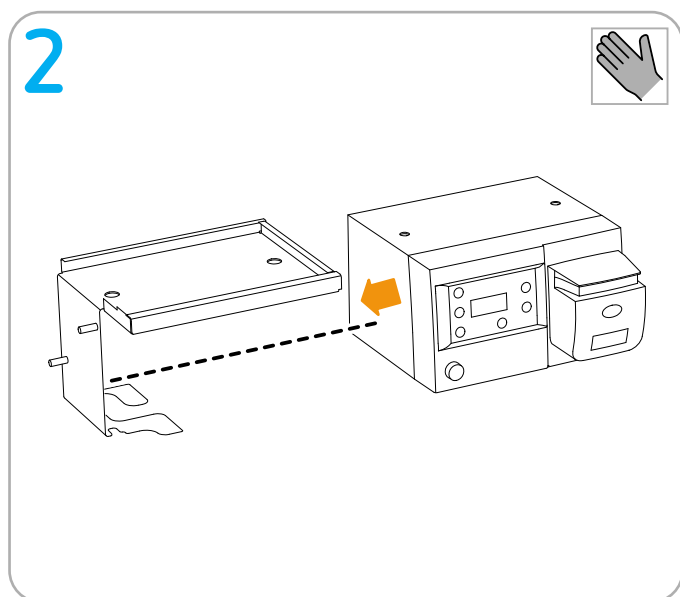
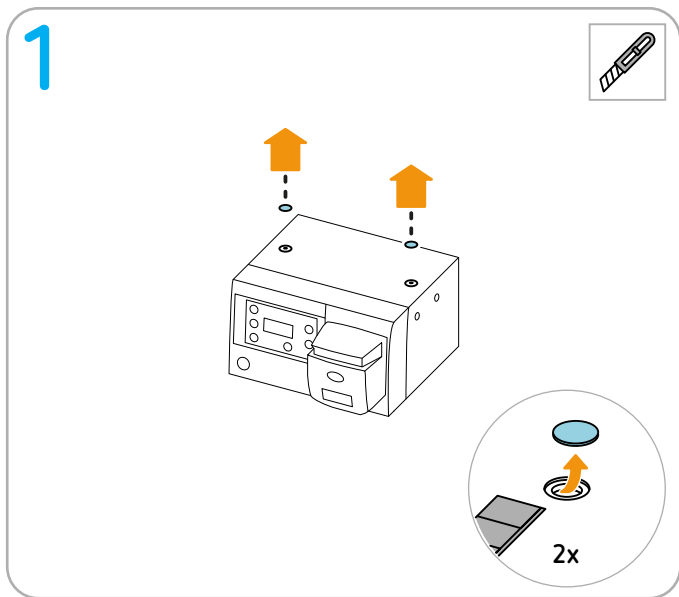
Tools required




-  Torque screwdriver (1 Nm) hex socket 6
-  Torque screwdriver (4.9 Nm) open-end wrench 10
-  Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

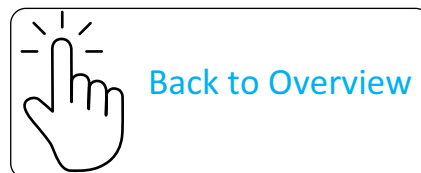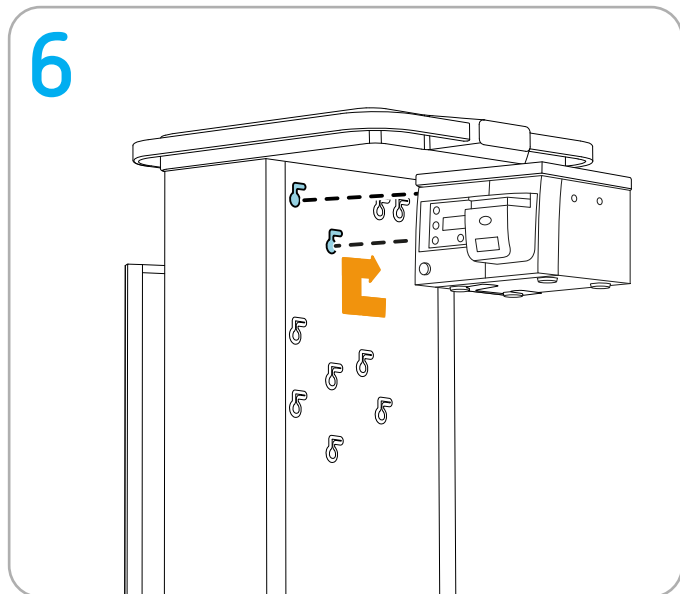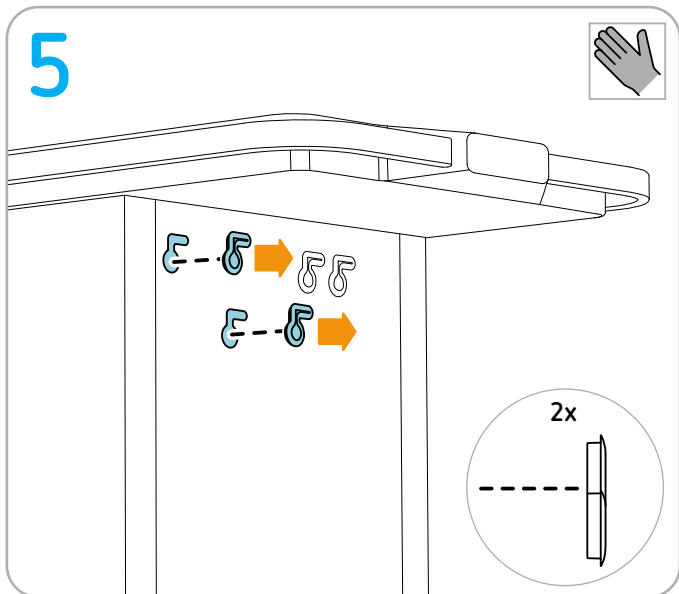Installing IES 3 and ESM 2

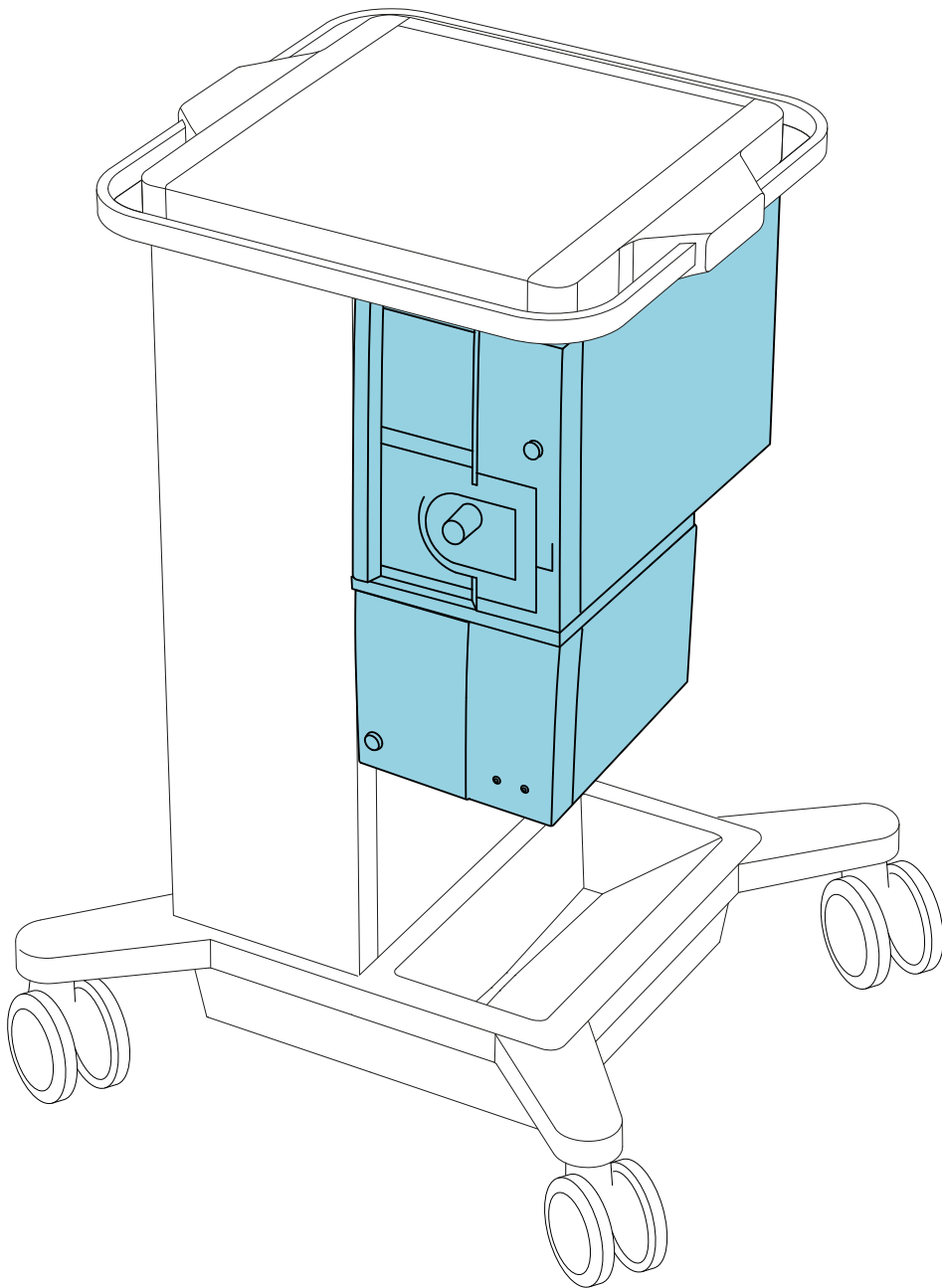
Permitted positioning

Erbe fastening set

IMPORTANT! Only use original Erbe accessories.

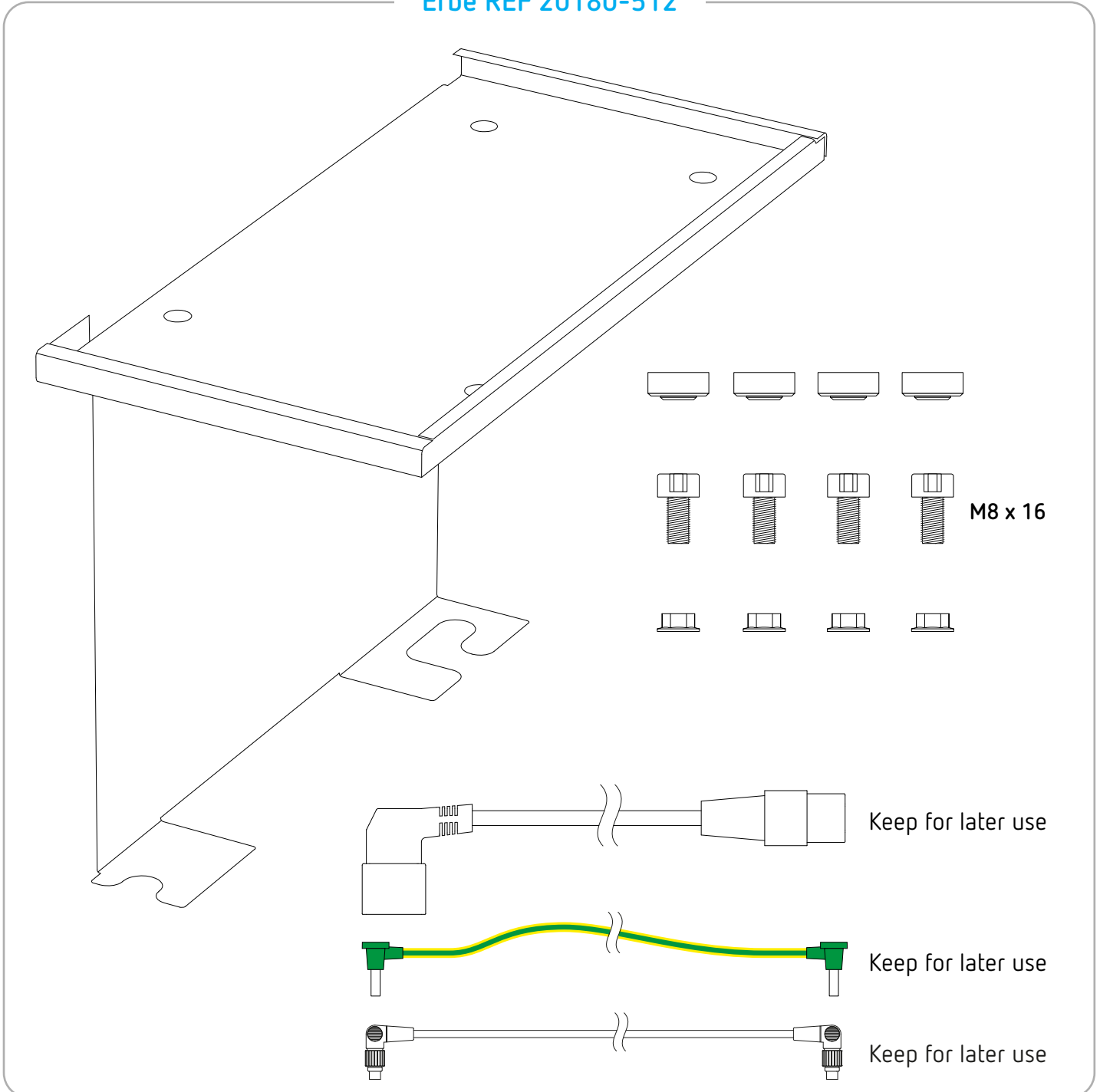Erbe REF 20180-512

Tools required

Torque screwdriver (1 Nm) hex socket 6

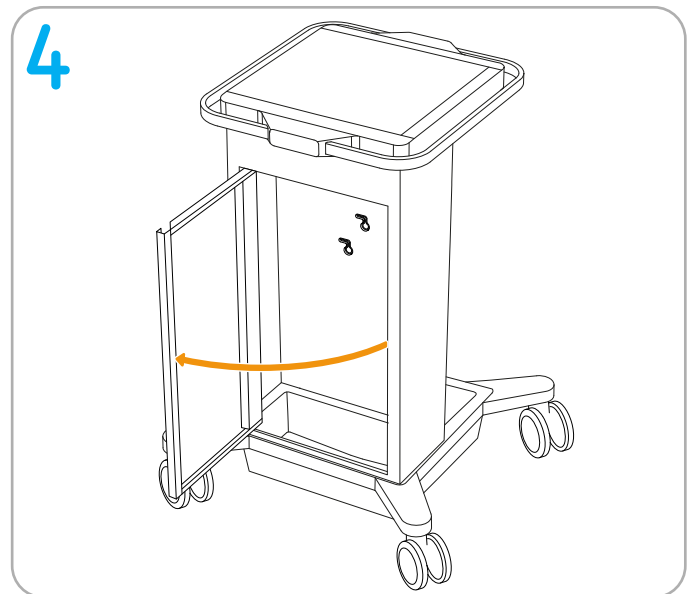
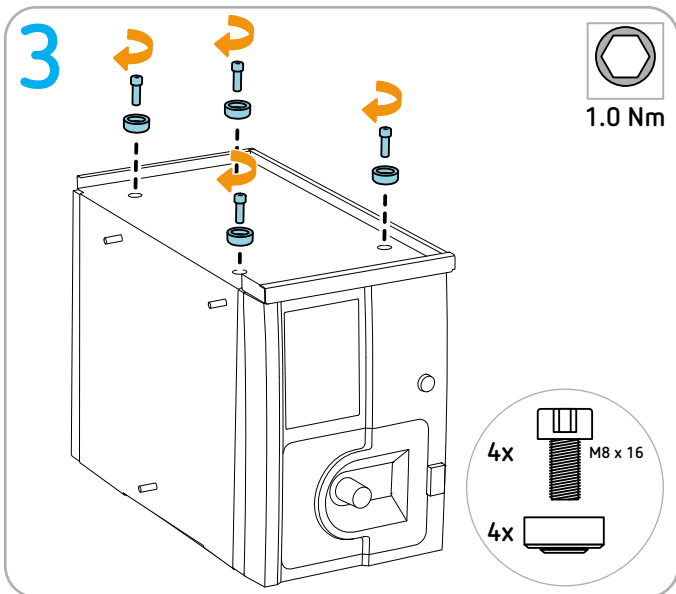
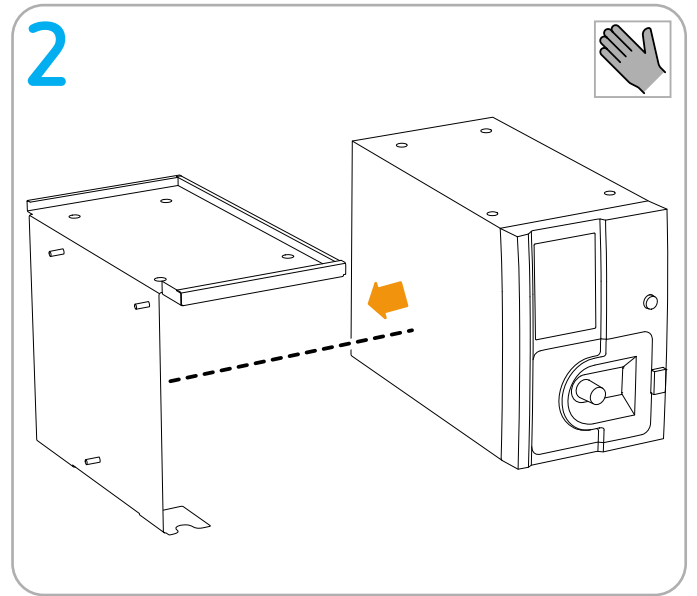
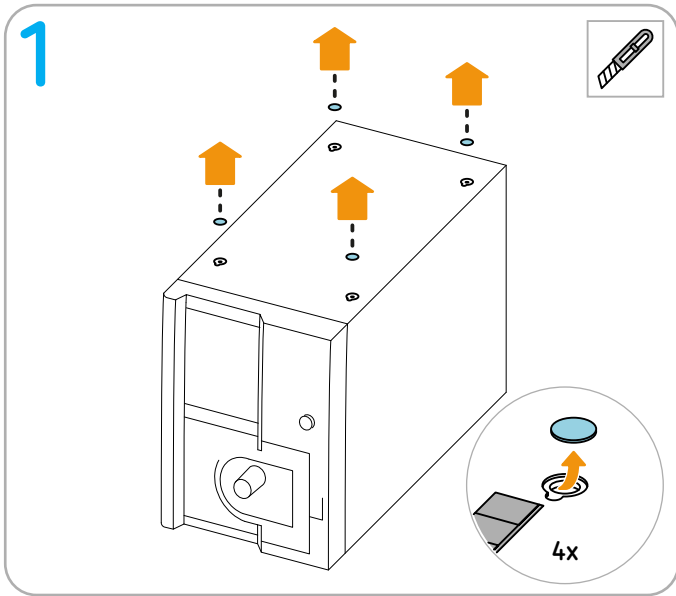
Torque screwdriver (4.9 Nm) open-end wrench 10




Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

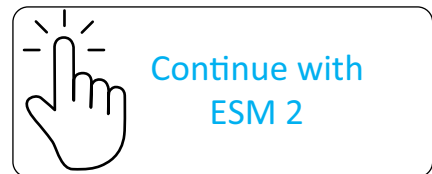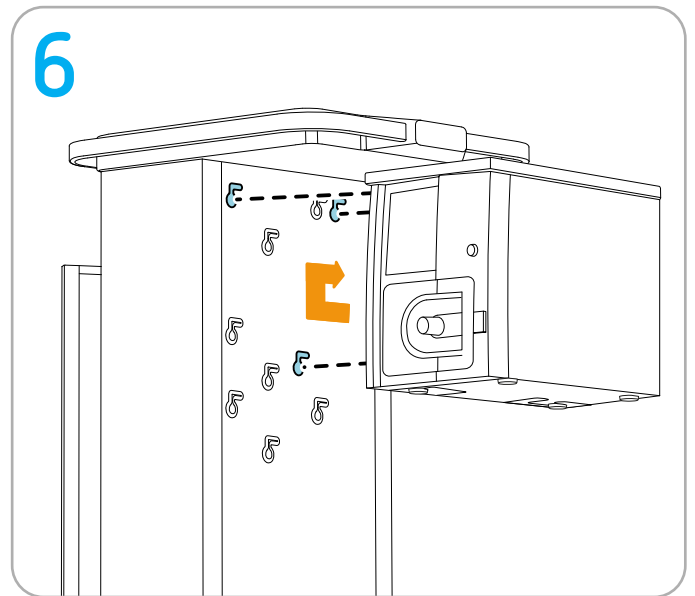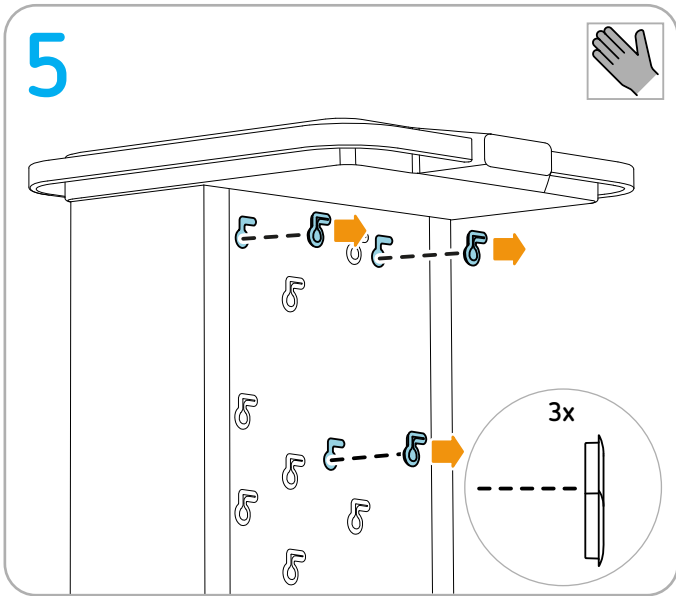Erbe fastening set

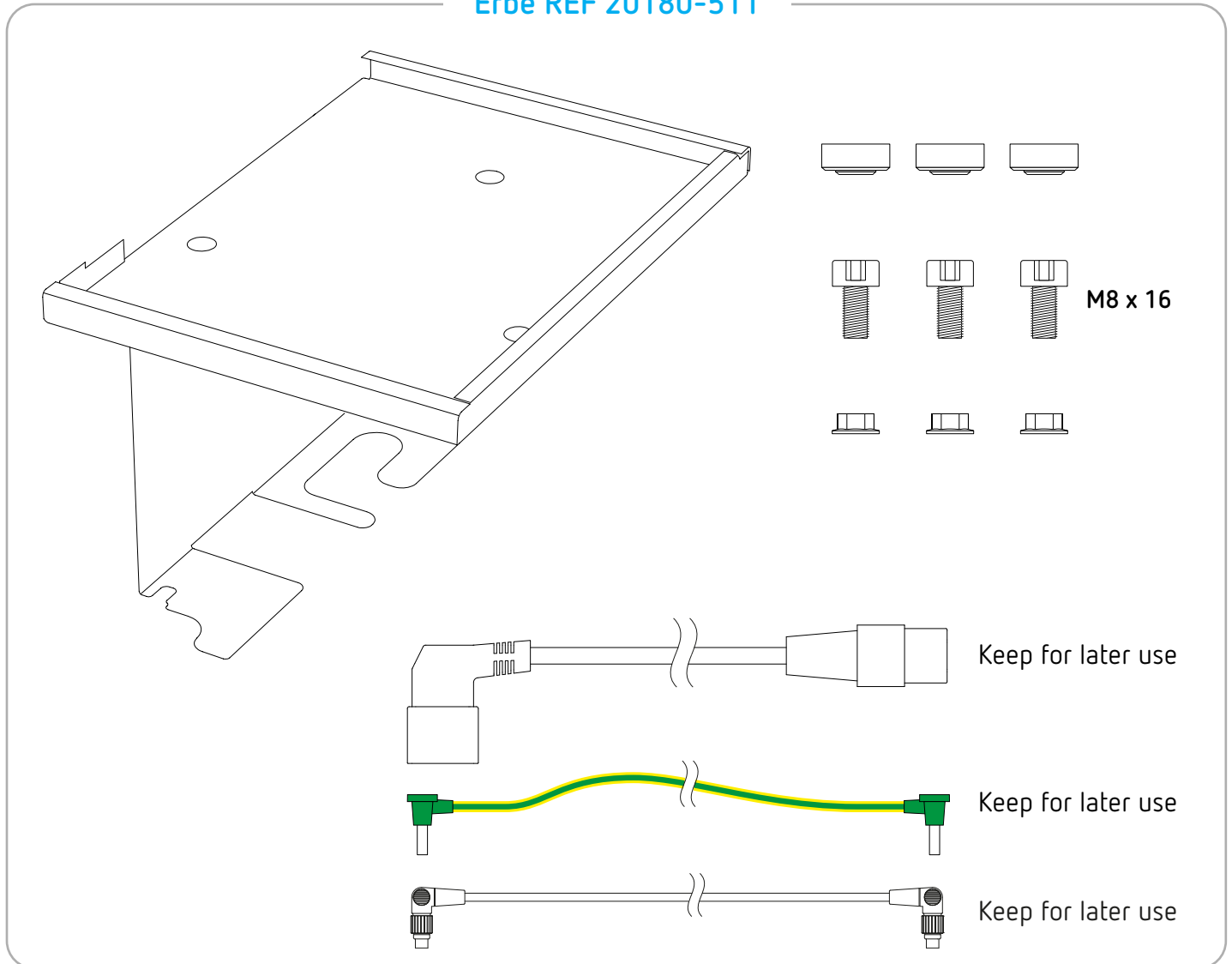IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-511

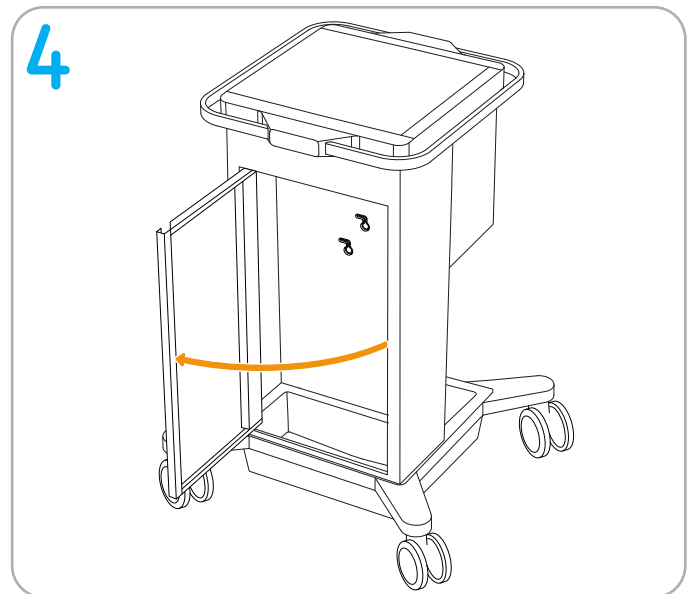
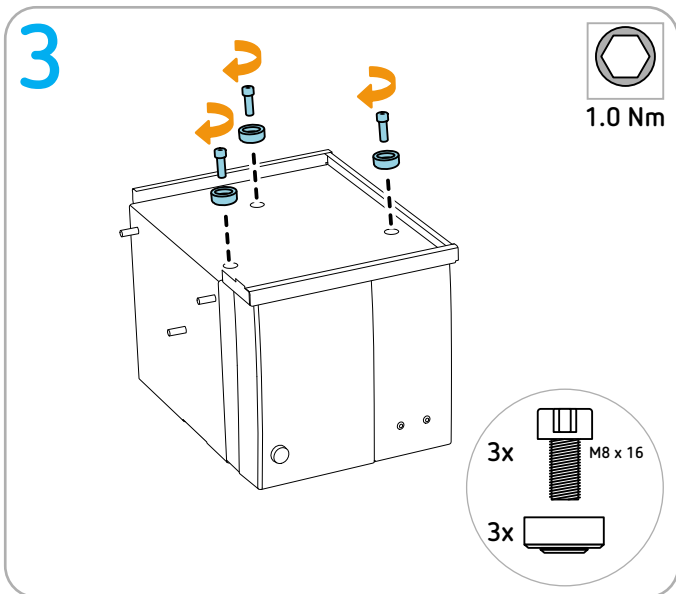
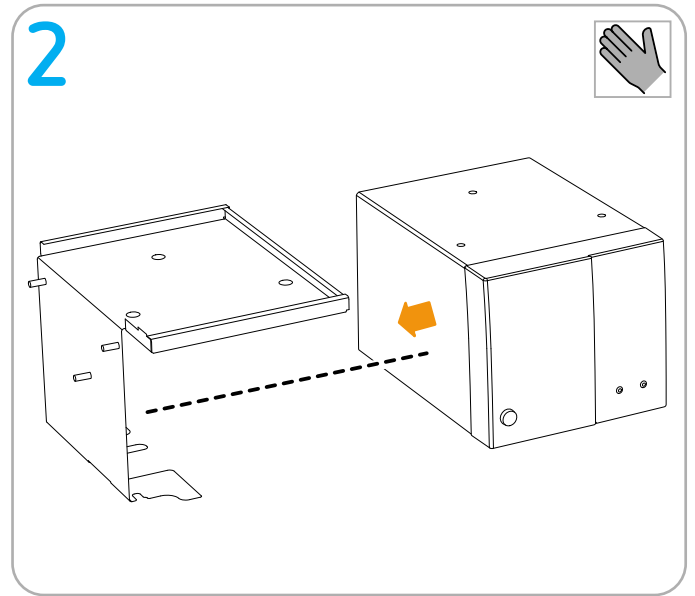
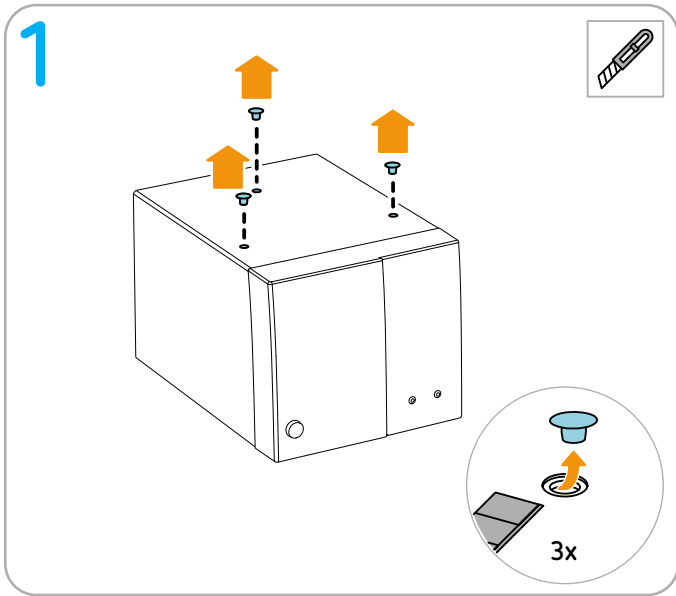
Tools required




-  Torque screwdriver (1 Nm) hex socket 6
-  Torque screwdriver (4.9 Nm) open-end wrench 10
-  Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

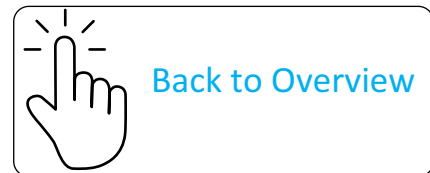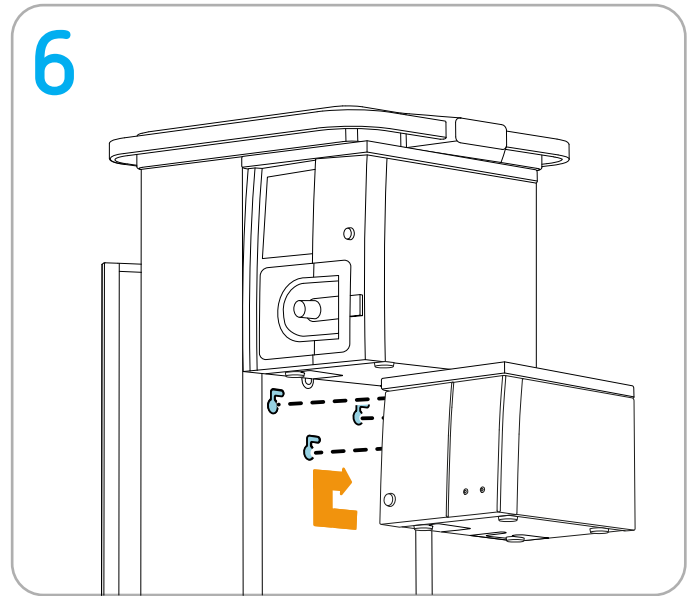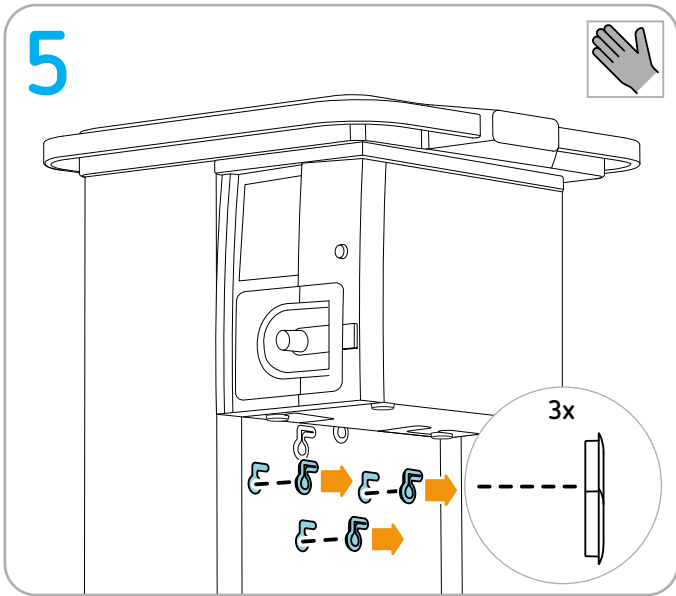Installing IES 3 and EIP 2

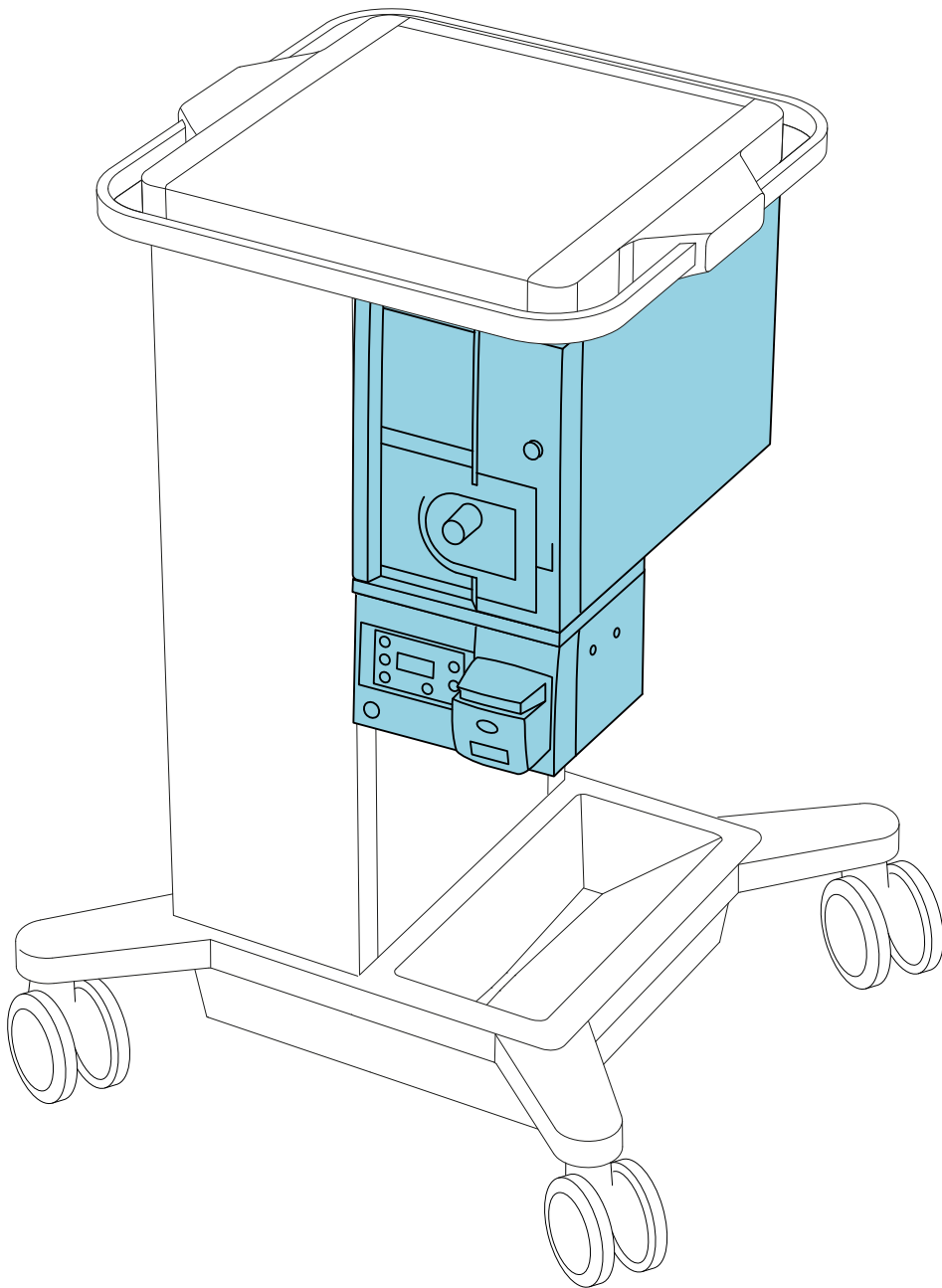
Permitted positioning

Erbe fastening set

IMPORTANT! Only use original Erbe accessories.

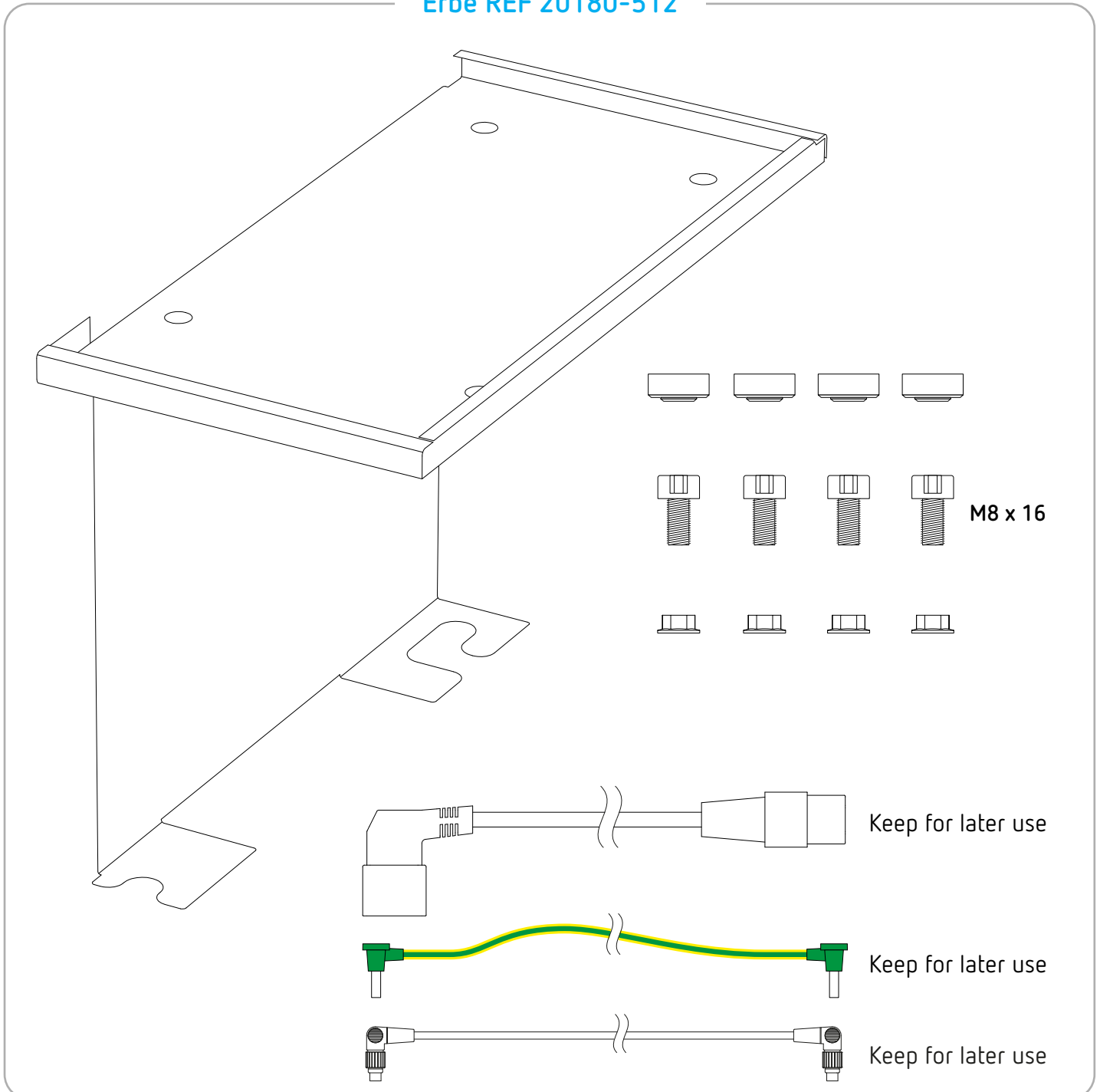Erbe REF 20180-512

Tools required

Torque screwdriver (1 Nm) hex socket 6

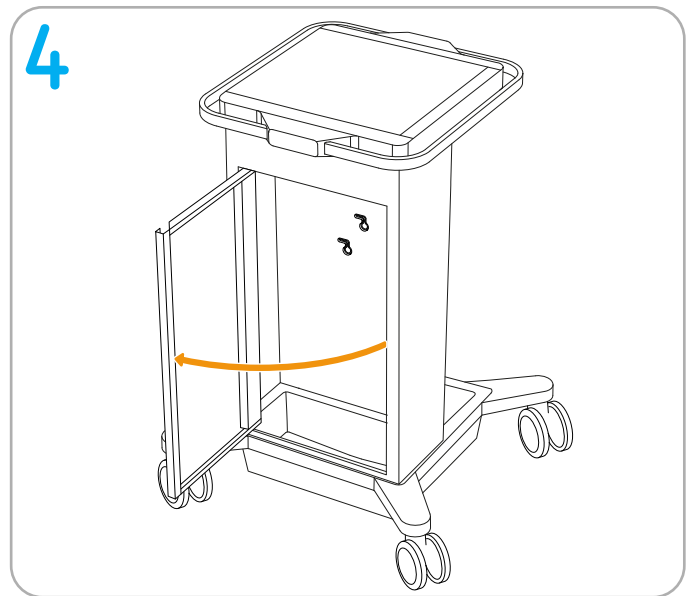
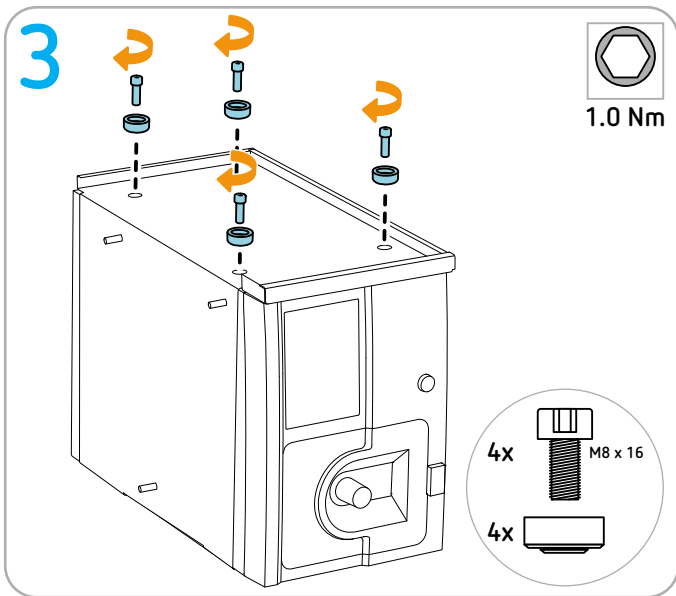
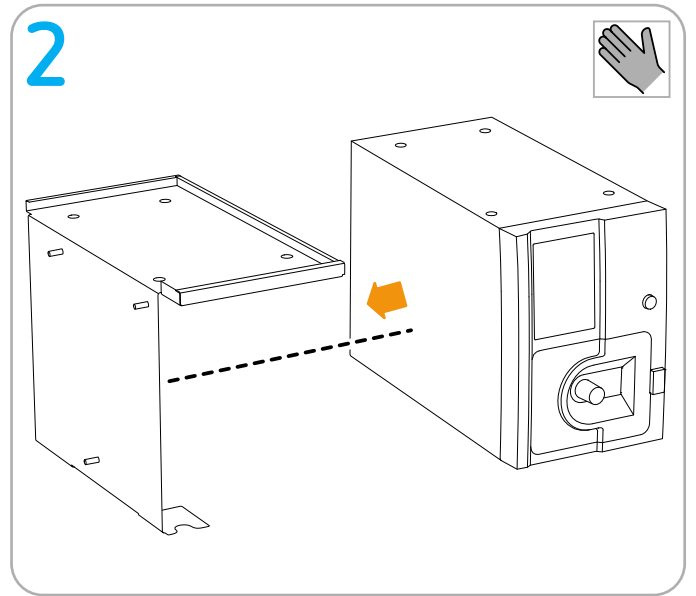
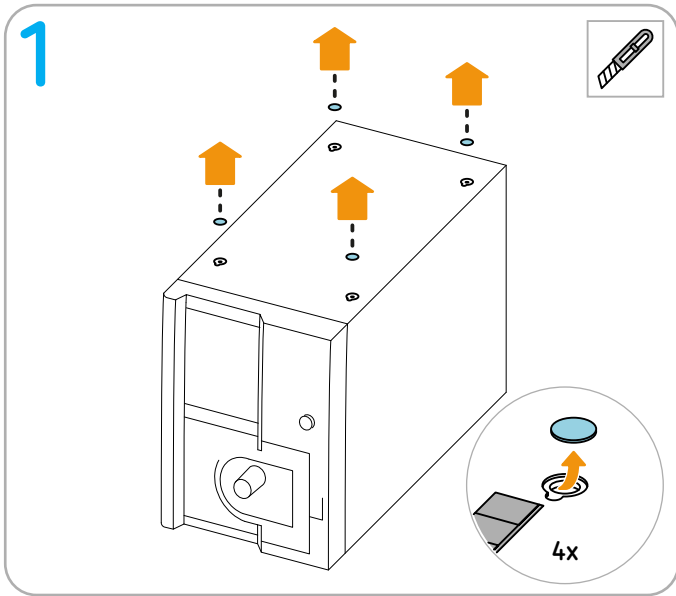
Torque screwdriver (4.9 Nm) open-end wrench 10



Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

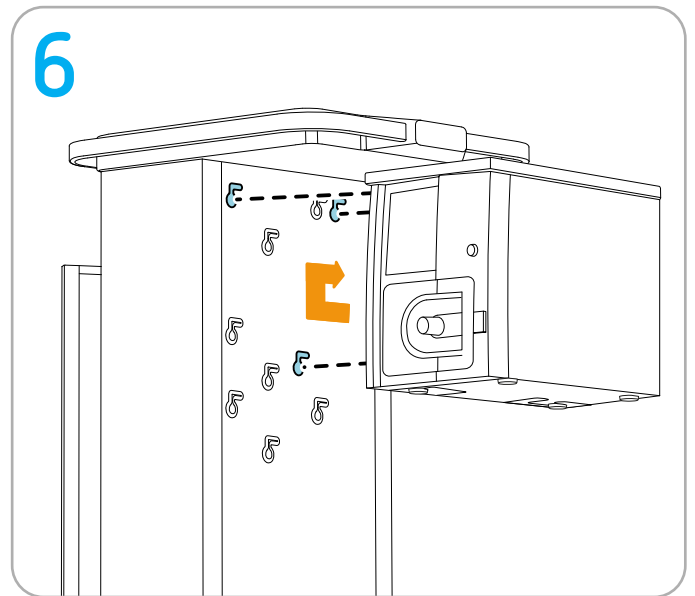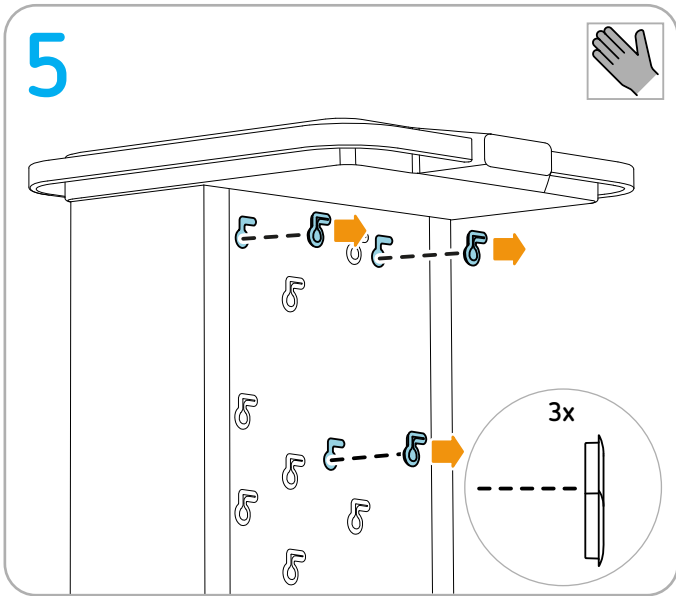Erbe fastening set

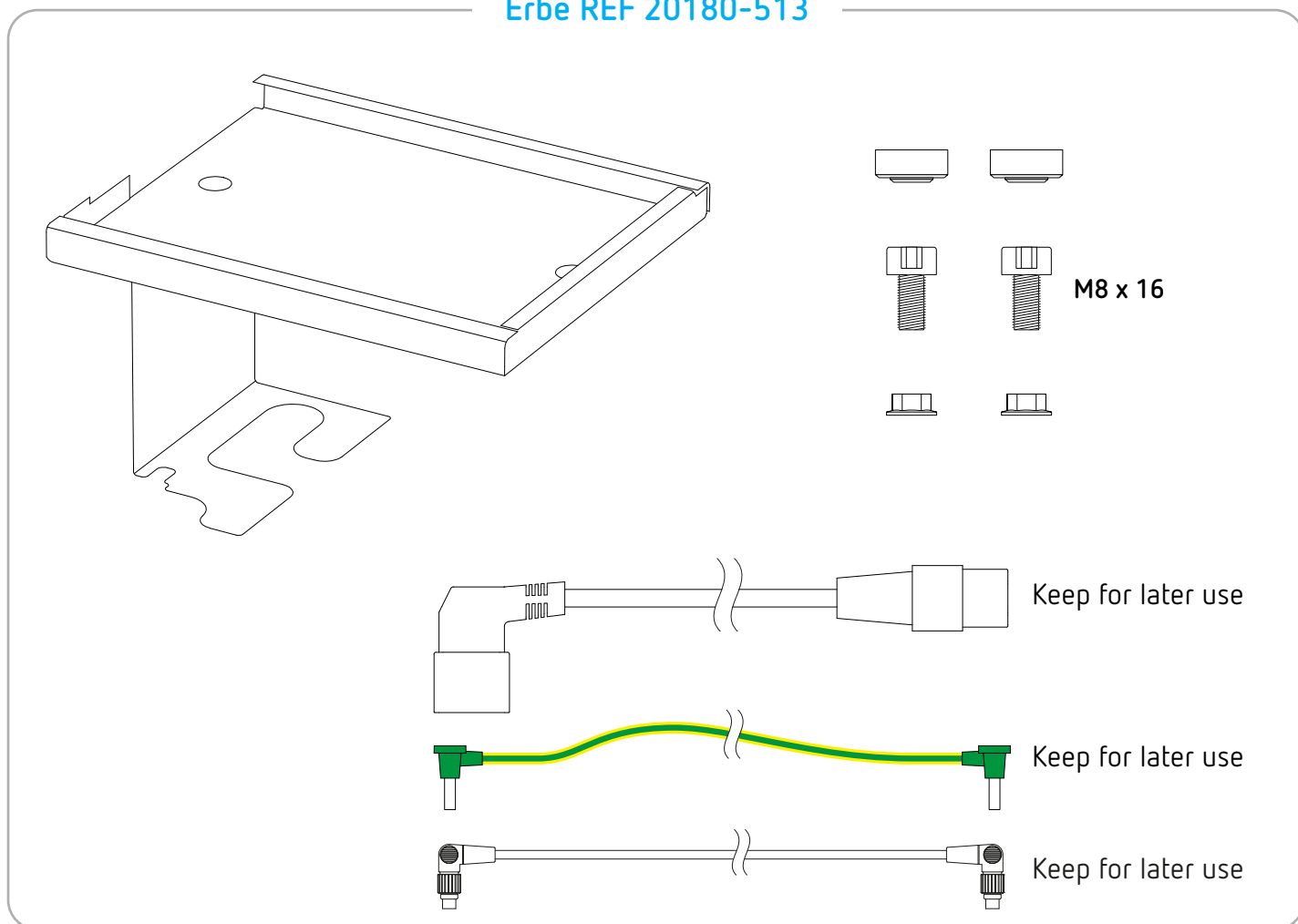IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-513

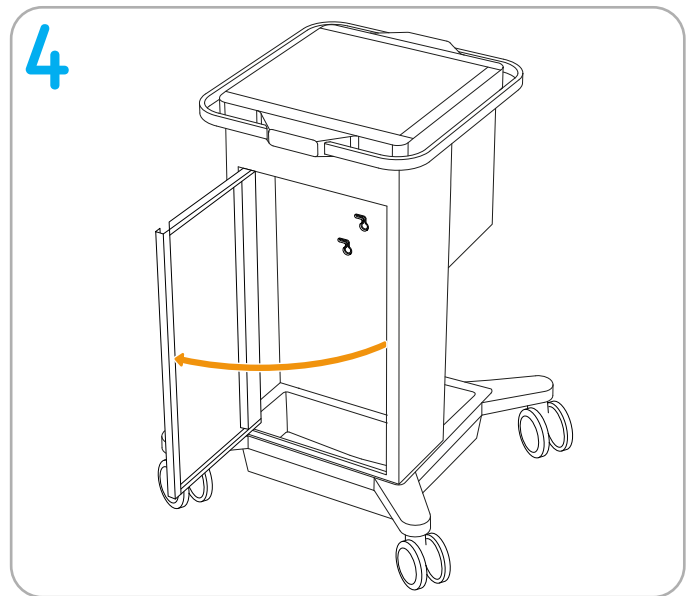
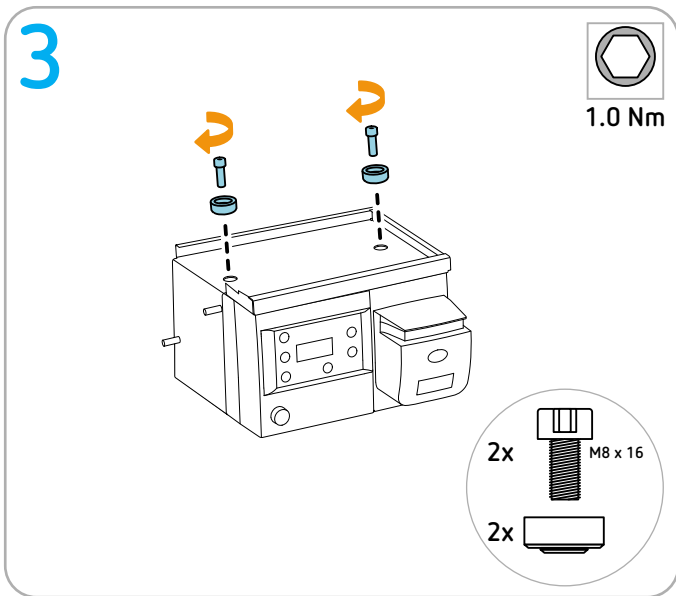
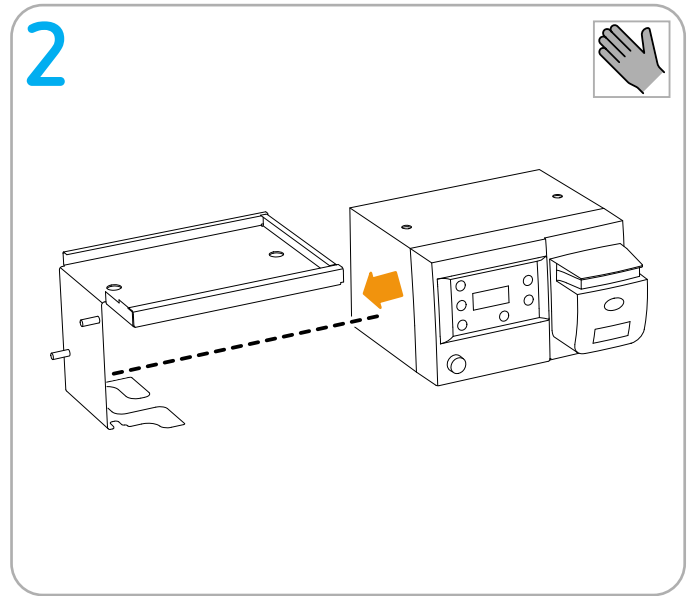
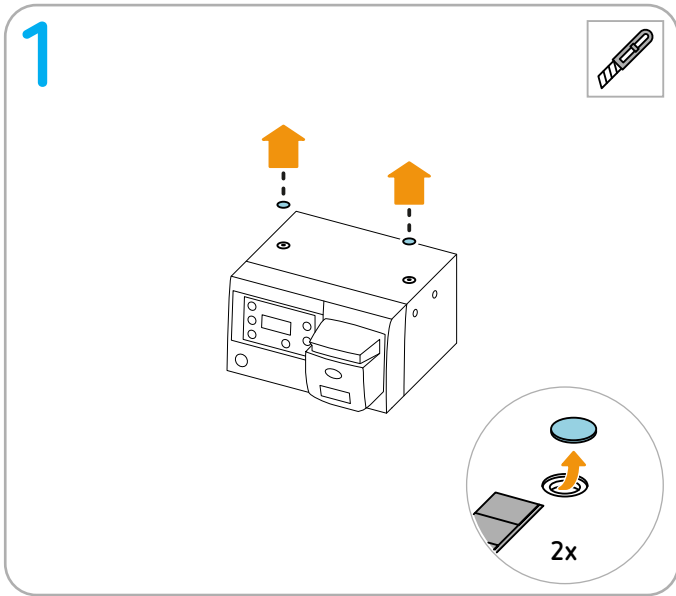
Tools required




-  Torque screwdriver (1 Nm) hex socket 6
-  Torque screwdriver (4.9 Nm) open-end wrench 10
-  Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

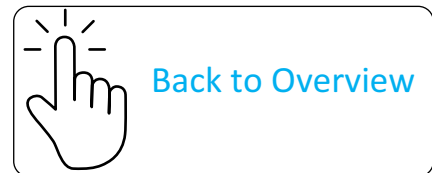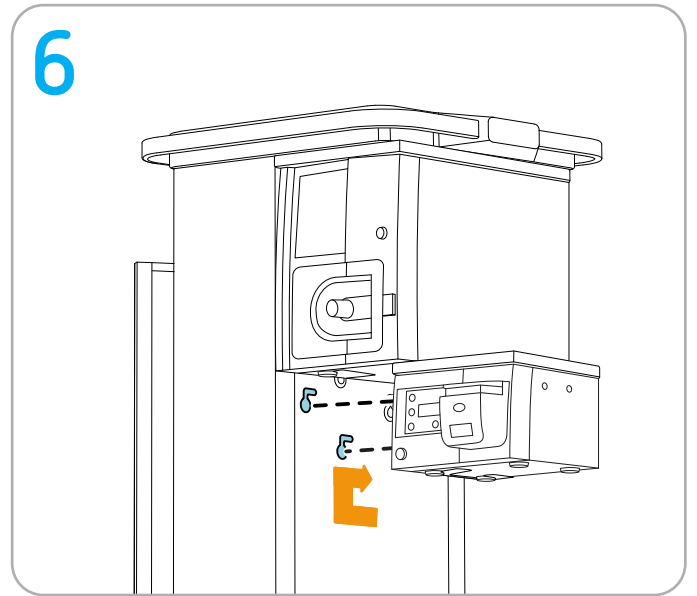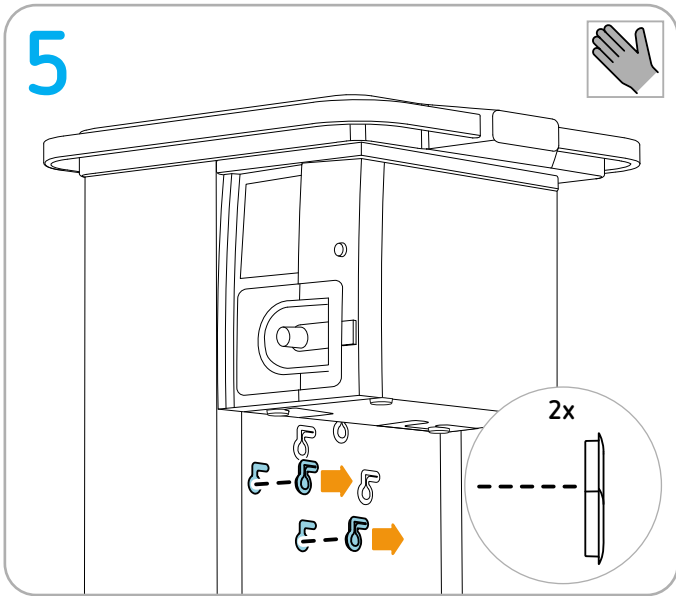Installing ESM 2 and EIP 2

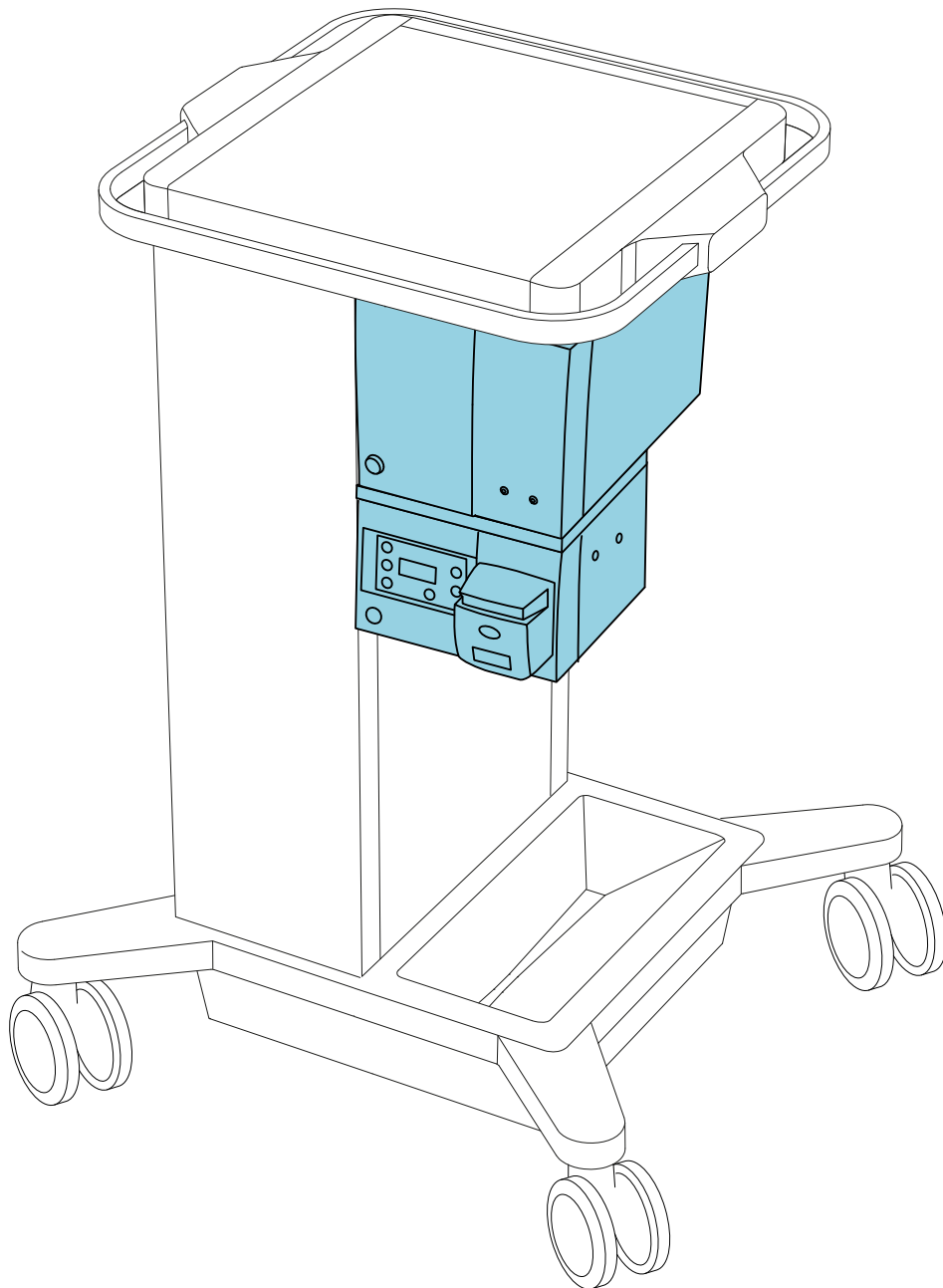
Permitted positioning

Erbe fastening set

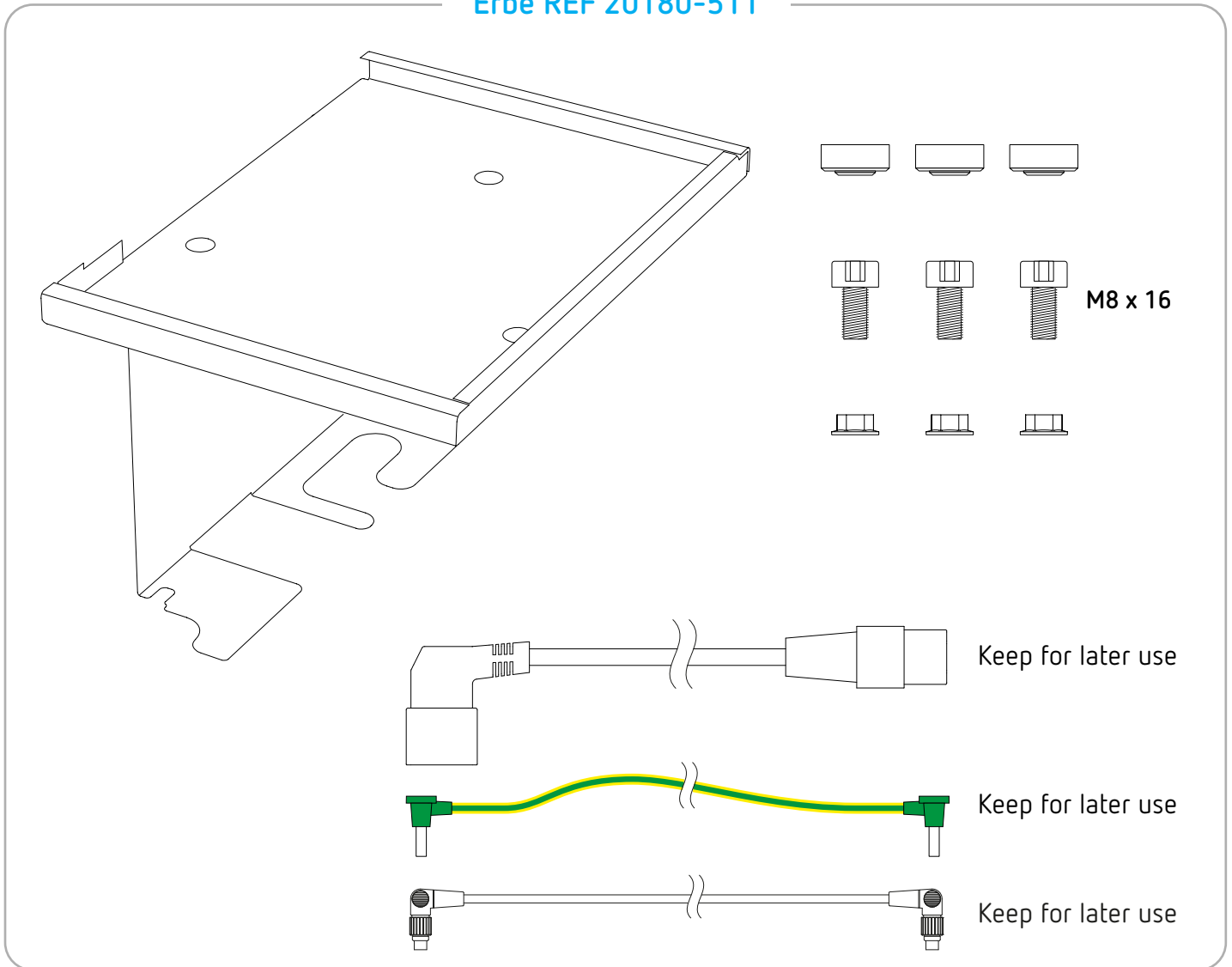IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-511

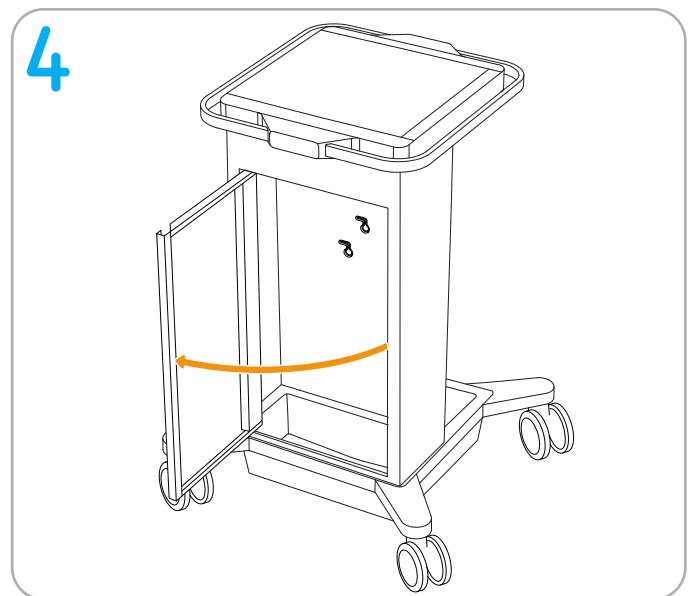
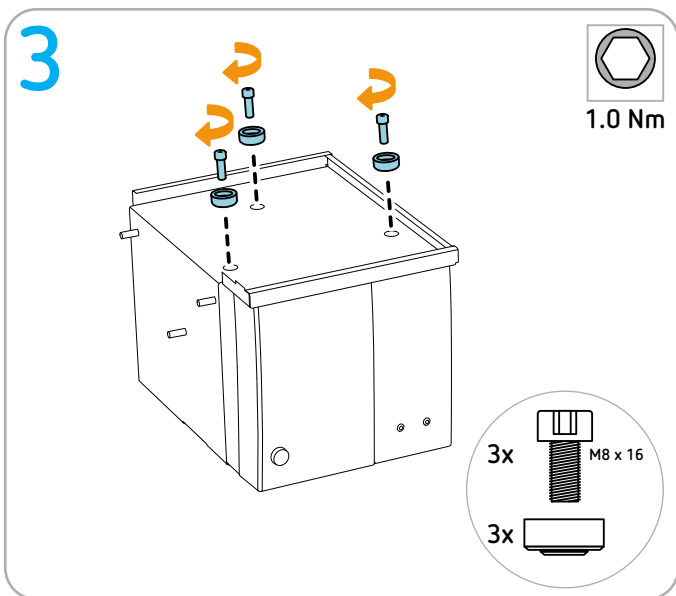
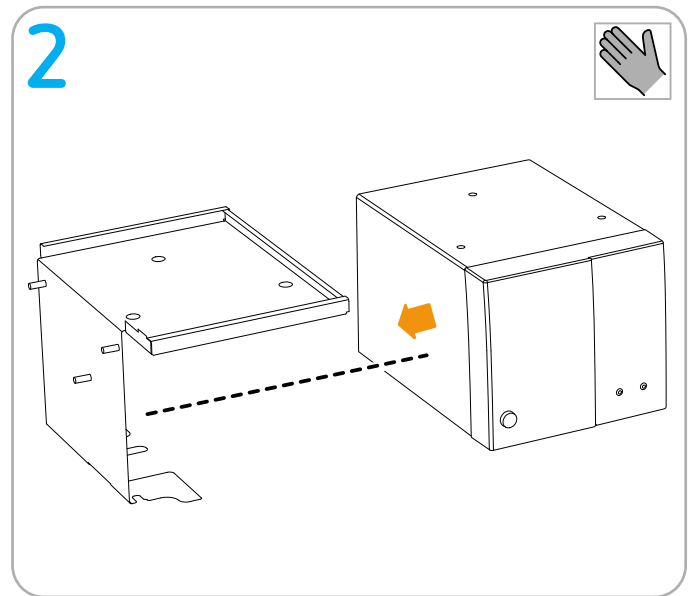
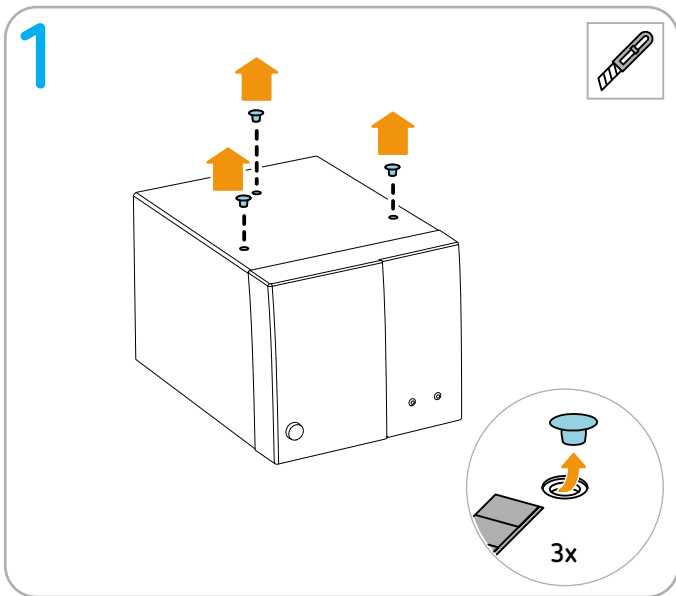
Tools required



-  Torque screwdriver (1 Nm) hex socket 6
-  Torque screwdriver (4.9 Nm) open-end wrench 10
-  Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

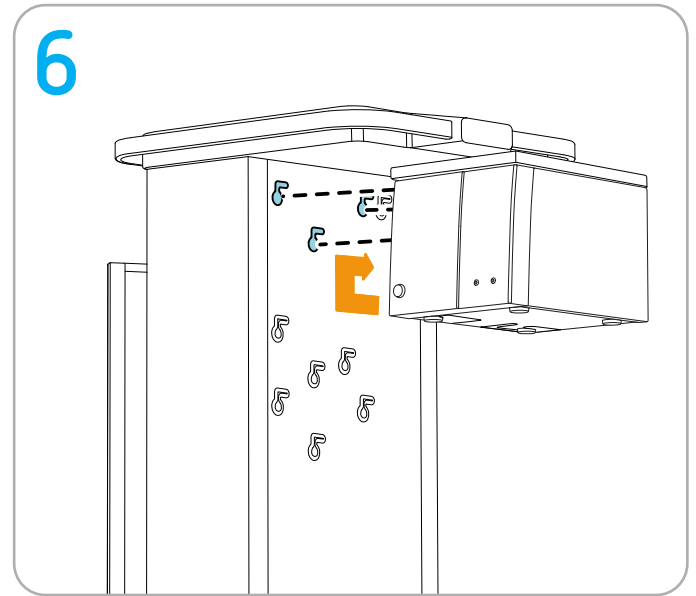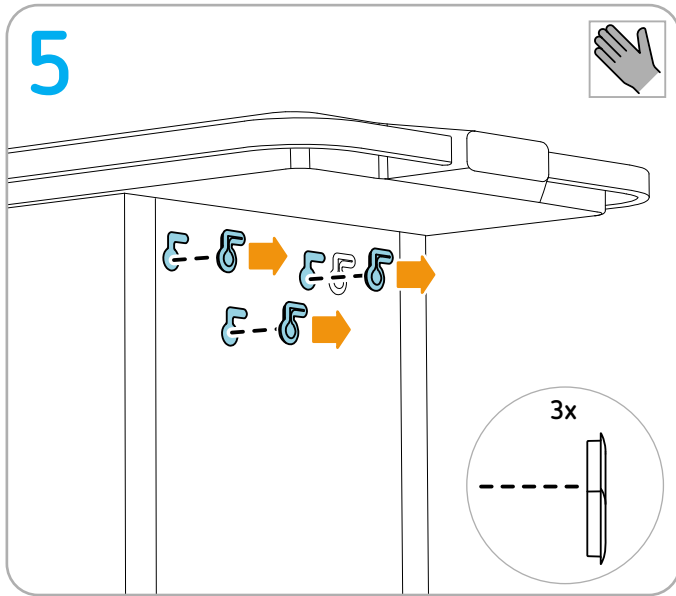Erbe fastening set

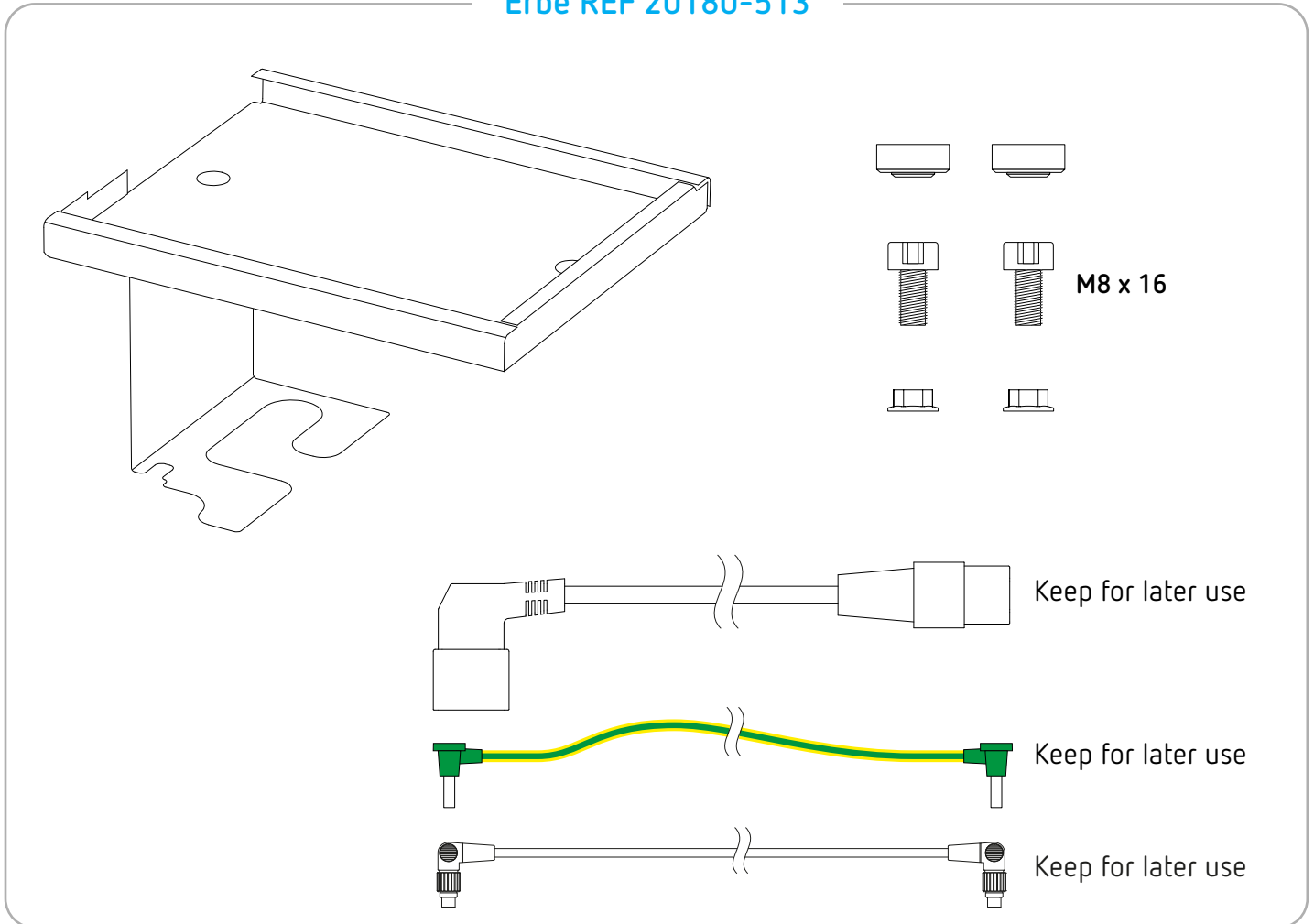IMPORTANT! Only use original Erbe accessories.

Erbe REF 20180-513

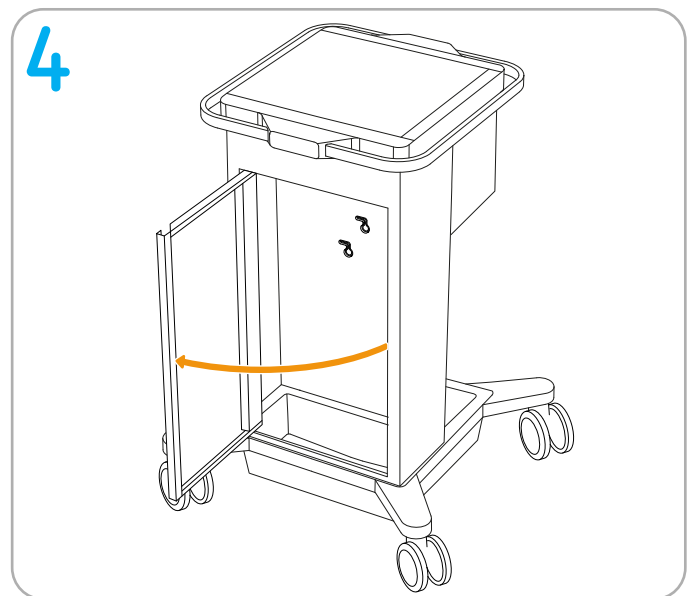
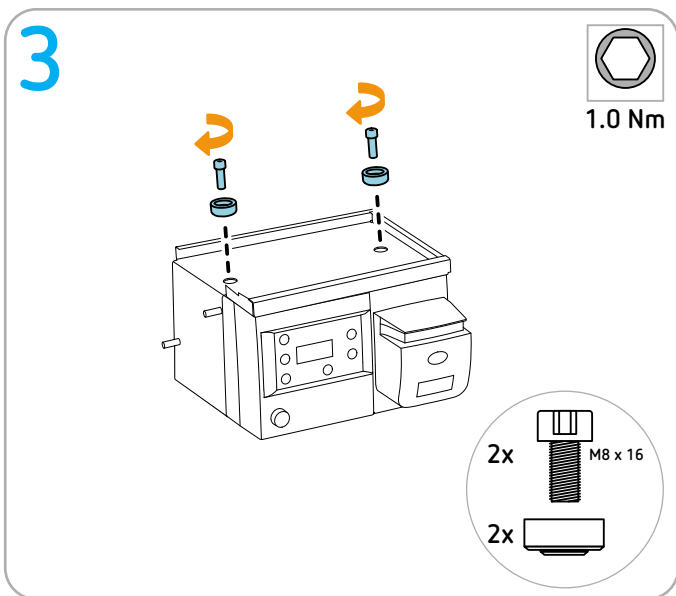
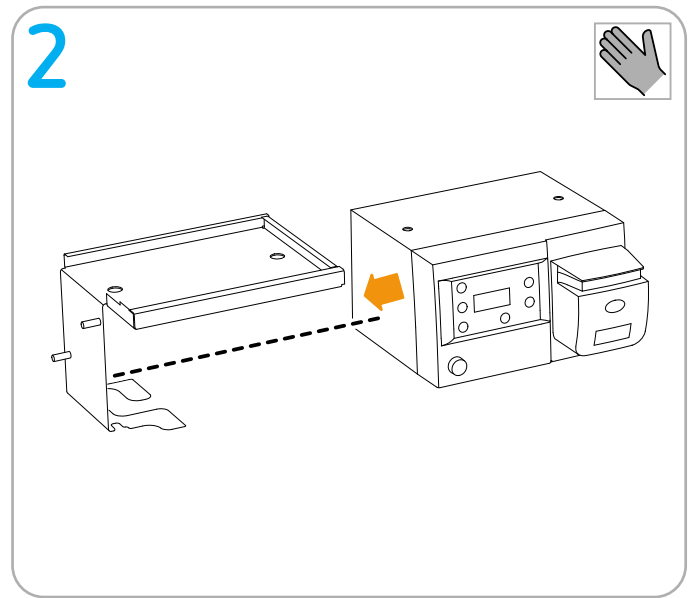
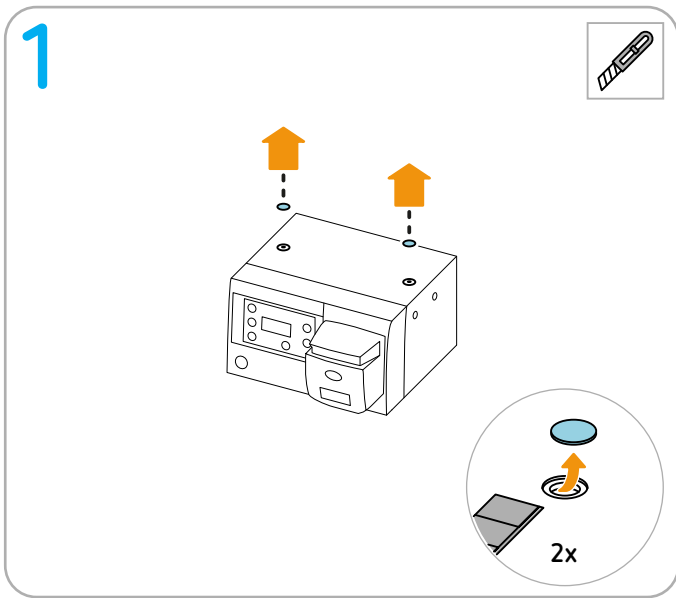
Tools required




-  Torque screwdriver (1 Nm) hex socket 6
-  Torque screwdriver (4.9 Nm) open-end wrench 10
-  Cutter

Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

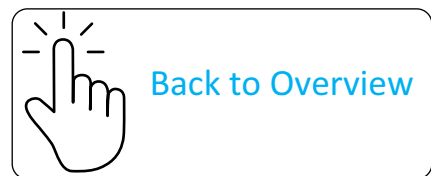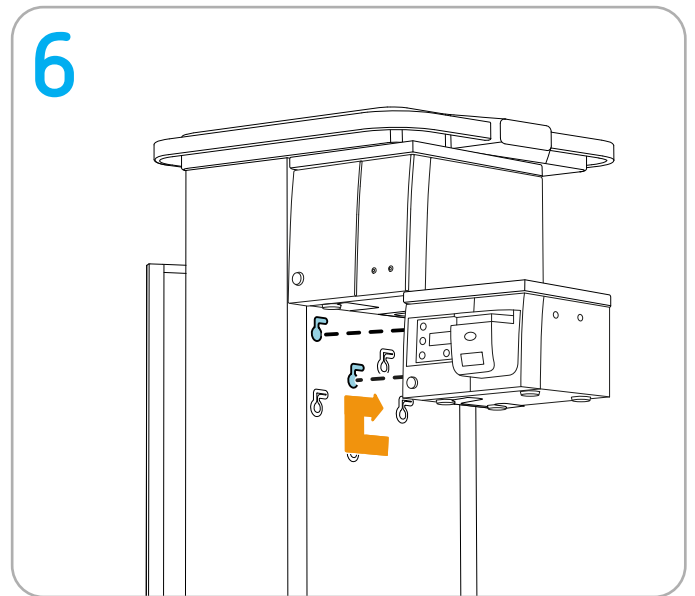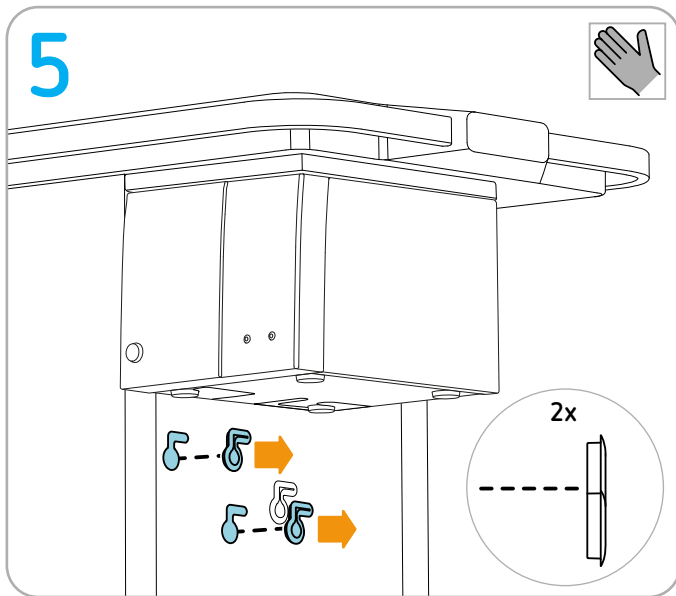
Installing different VIO series

Permitted positioning

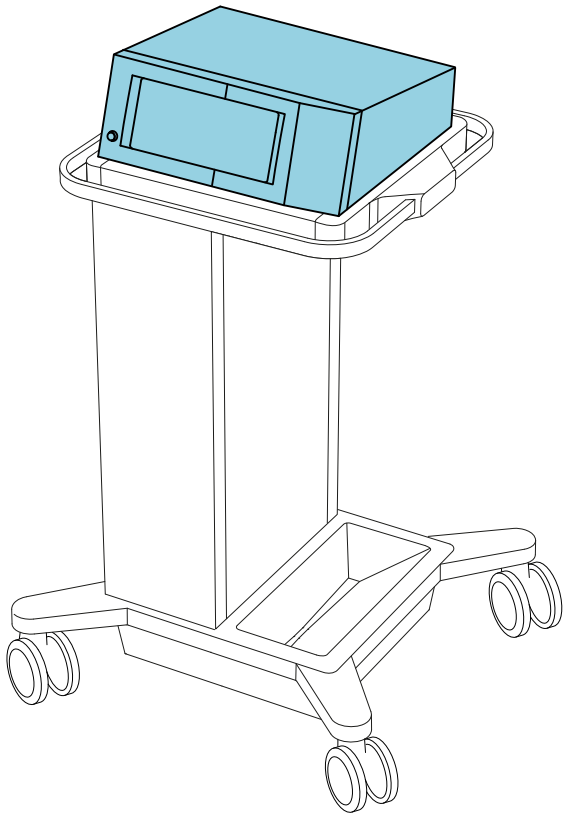VIO S series and
VIO D series

VIO 3 series

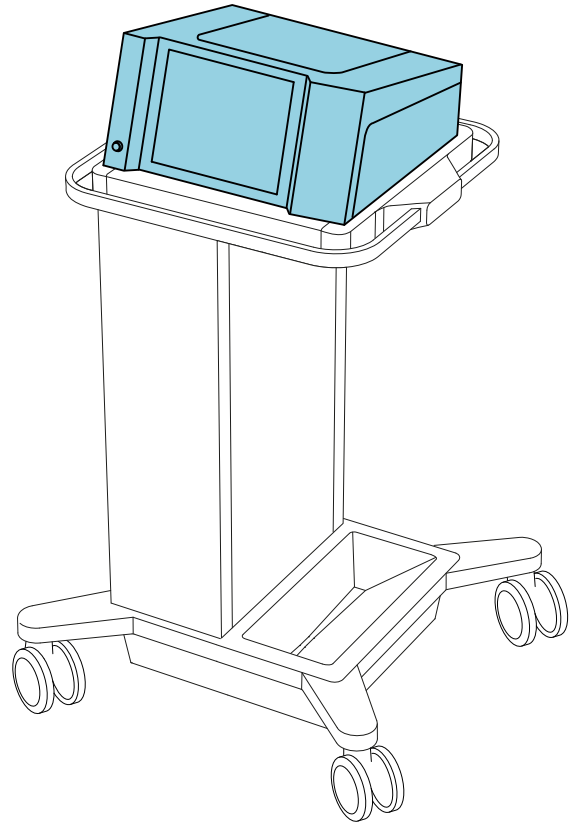Erbe fastening set

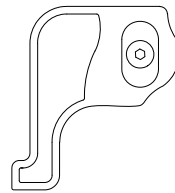
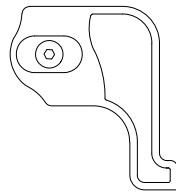
IMPORTANT! Only use original Erbe accessories.

IMPORTANT! Order the protective caps for the via of the units (see Step 1) separately as needed; they are not part of this fastening set 20180-150. For order numbers for the matching protective caps, see the repair instructions for the respective VIO unit.


Erbe REF 20180-150

M4 x 8

M4 x 16 screws included

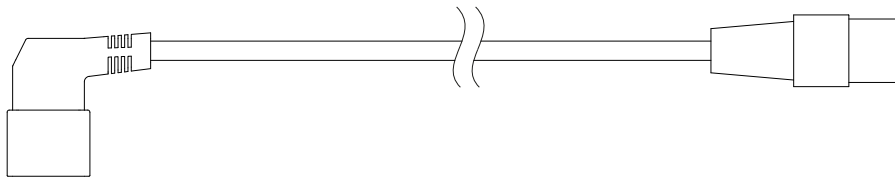Keep for later use

Keep for later use

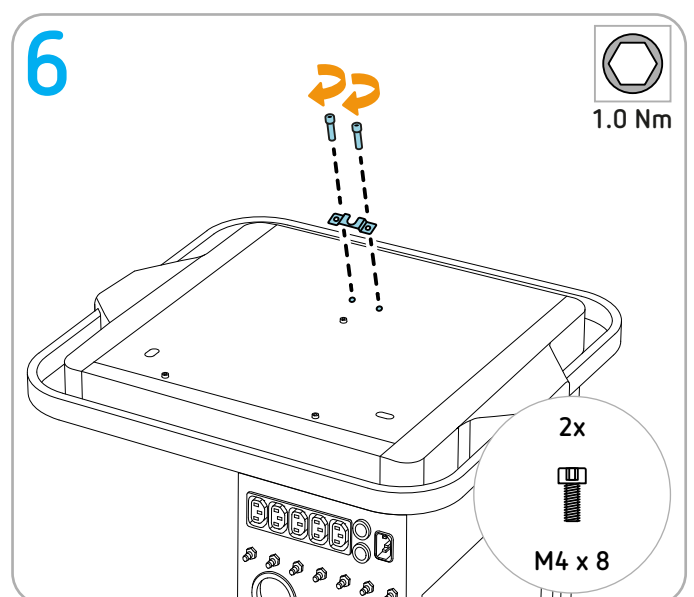
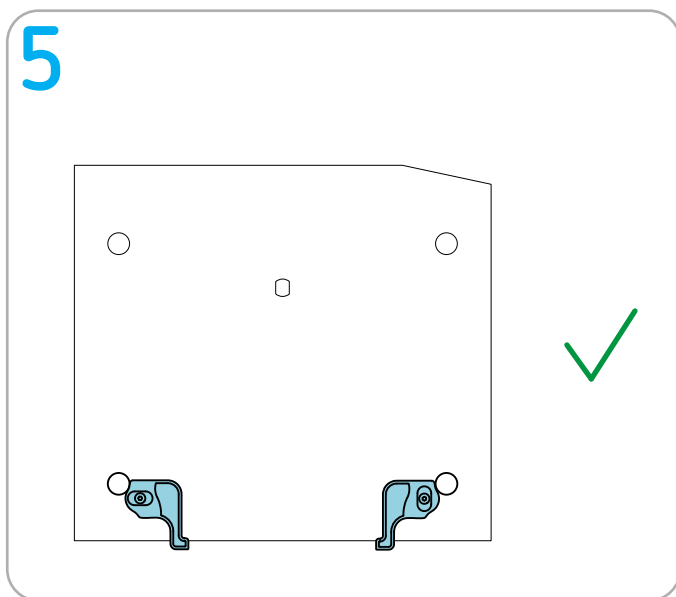
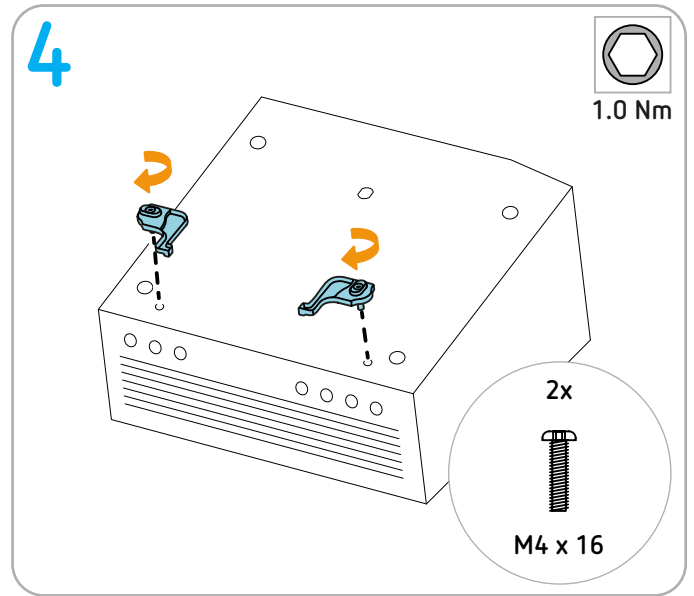
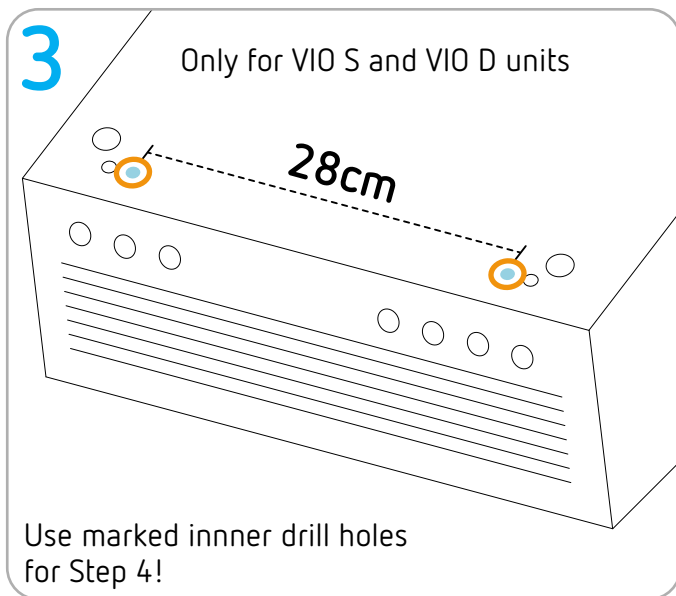
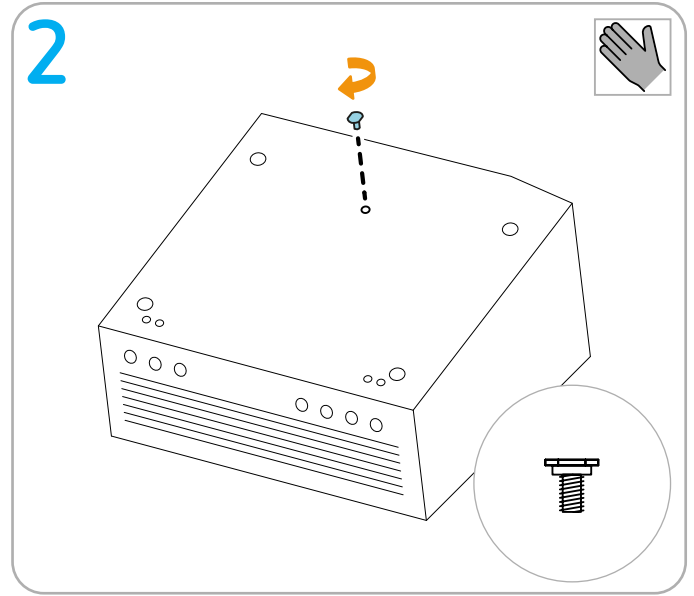
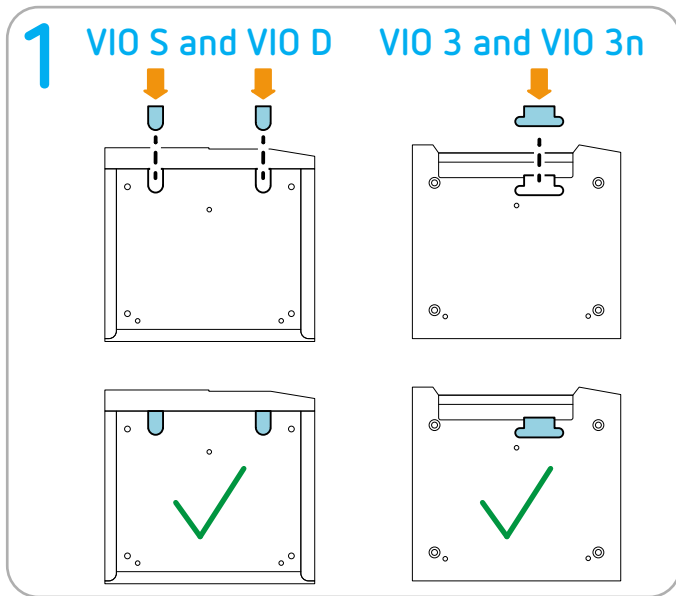
Tools required


Torque screwdriver (1 Nm) hex socket 2.5

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the system carrier

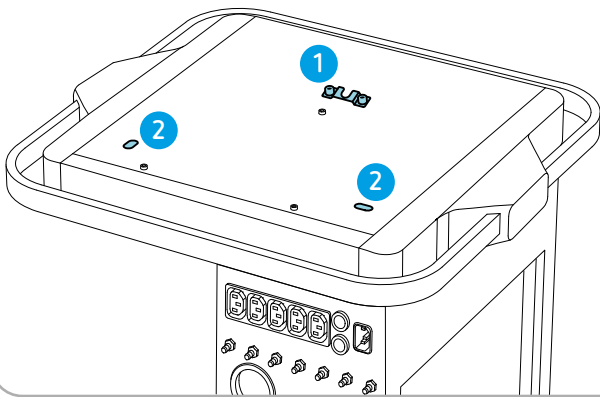
8 Attachment points on the unit


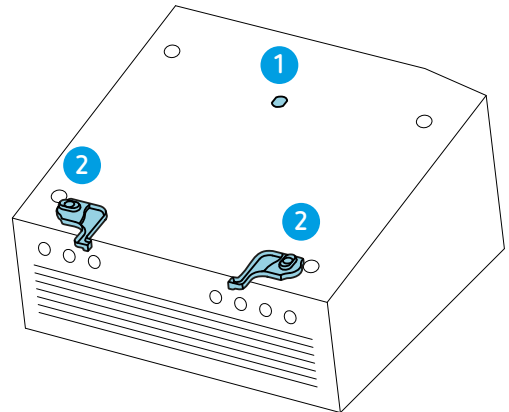
9

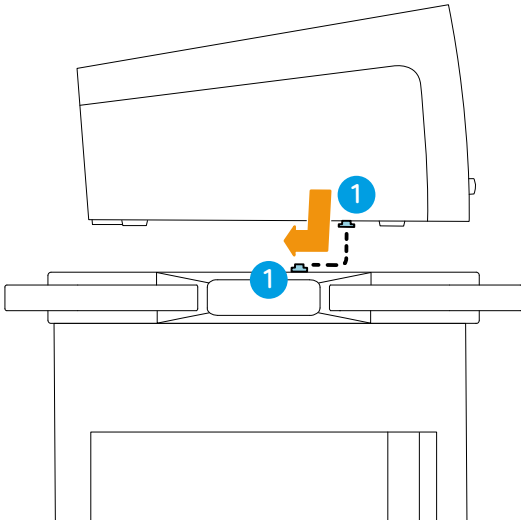10

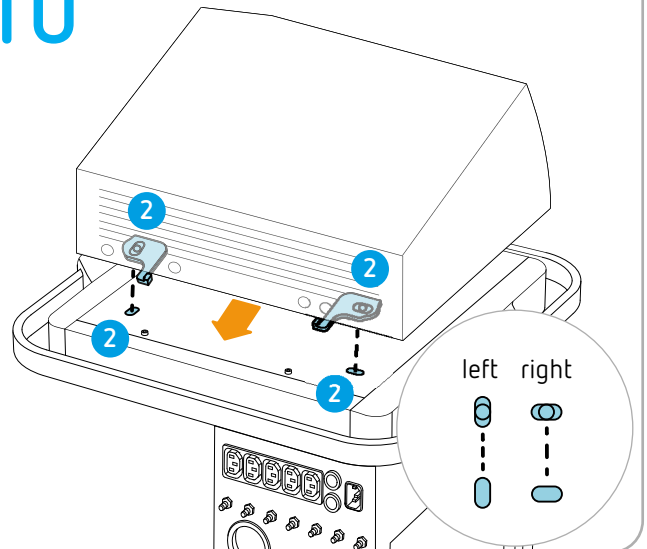
11

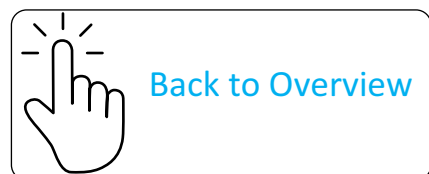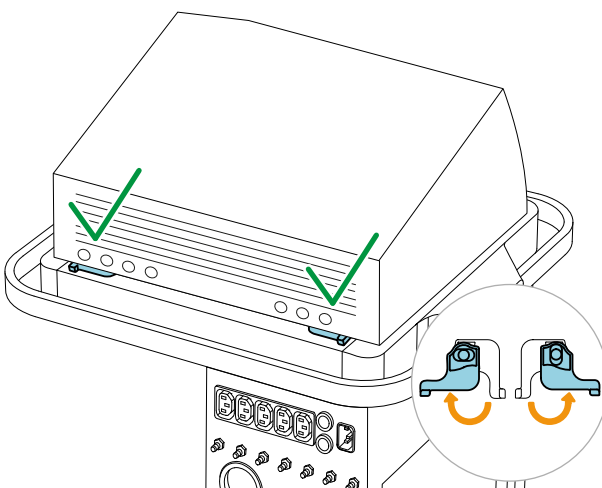
Installing ERBEJET 2 or ERBECRYO 2

Permitted positioning

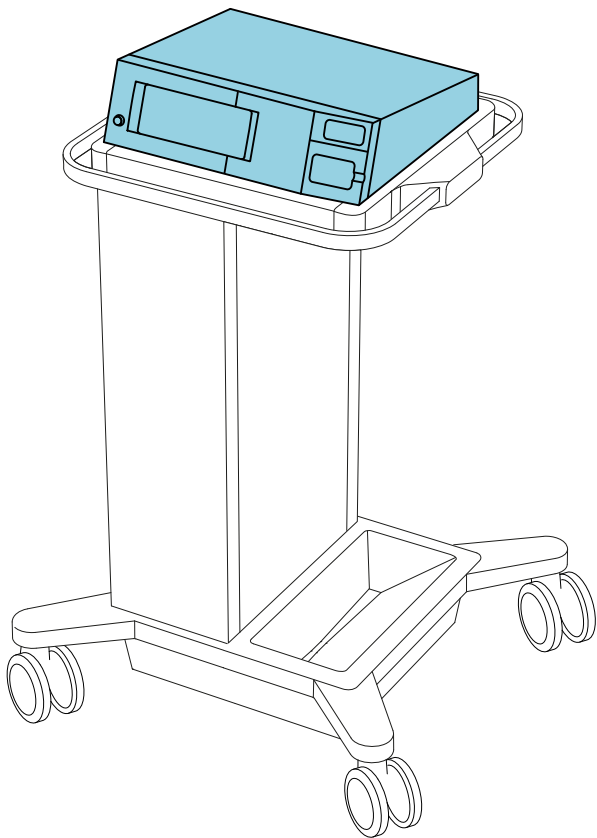ERBEJET 2

ERBECRYO 2

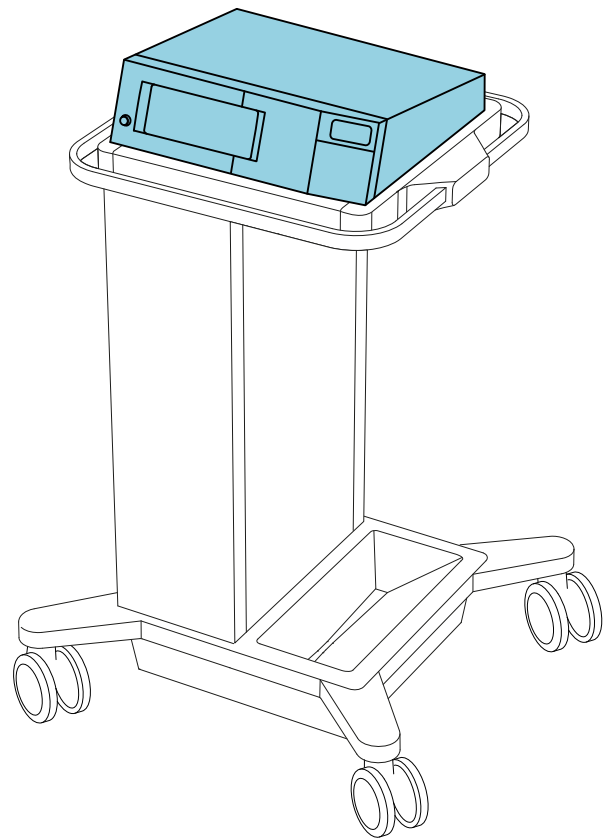
Erbe fastening set

IMPORTANT! Only use original Erbe accessories.


Erbe REF 20180-150

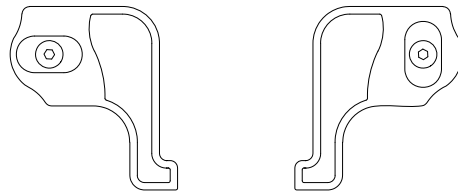M4 x 8

M4 x 16 screws included

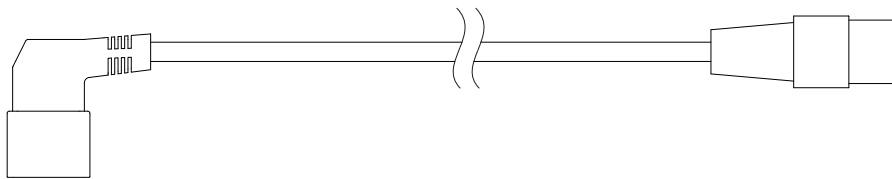Keep for later use

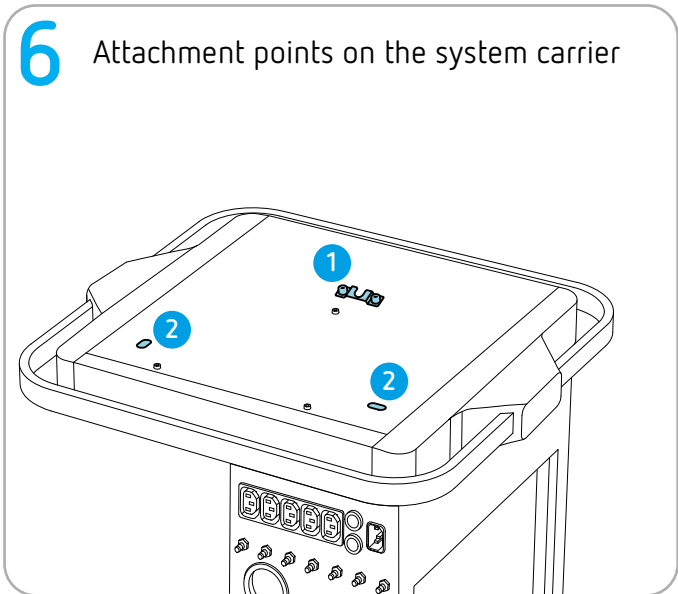
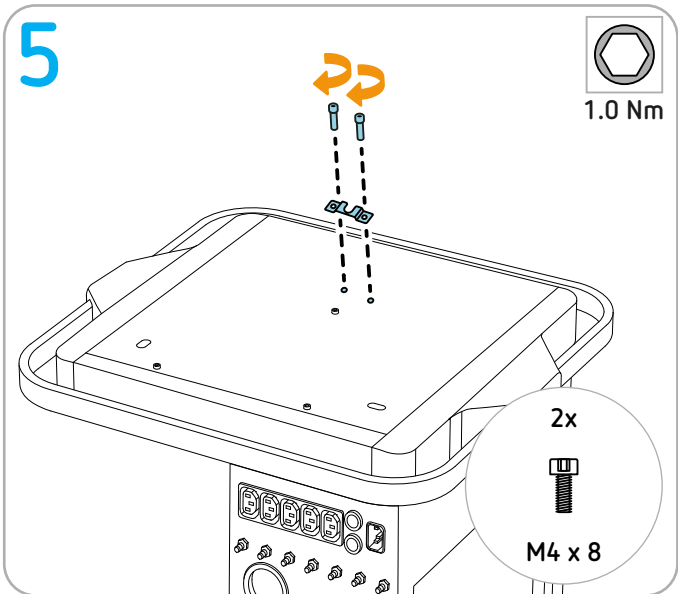
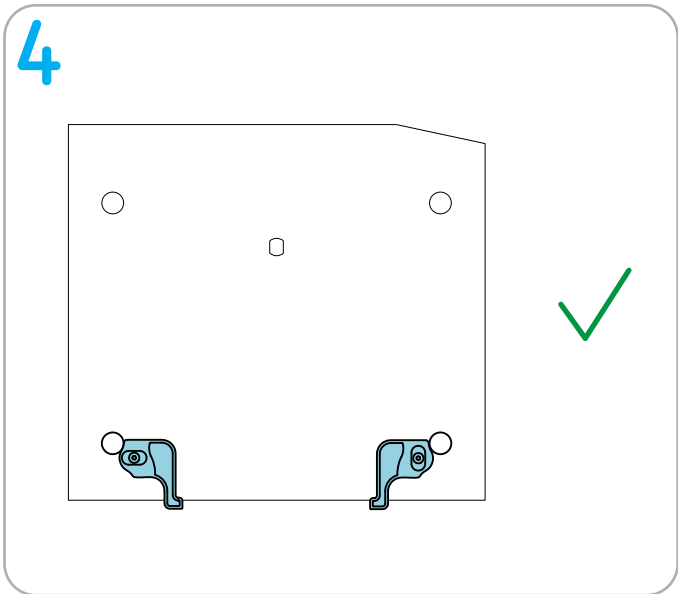
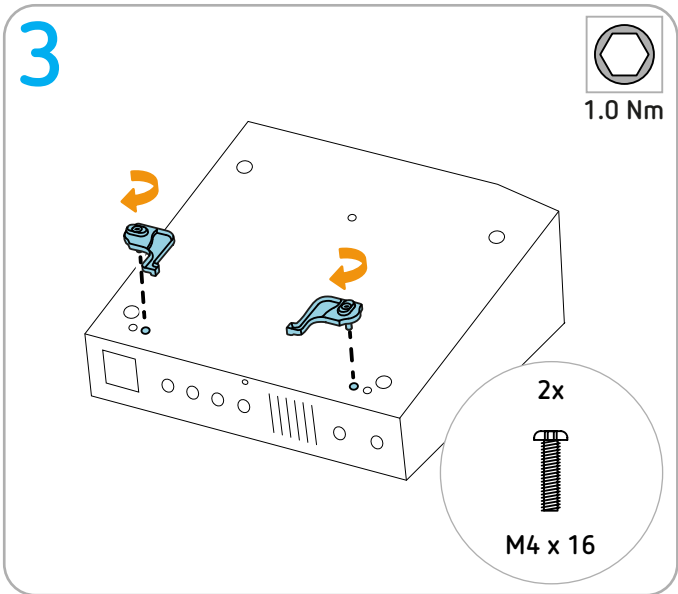
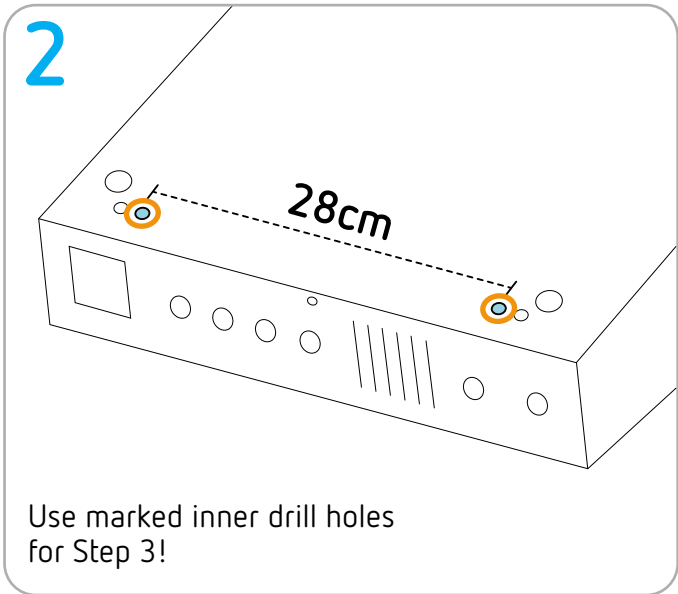
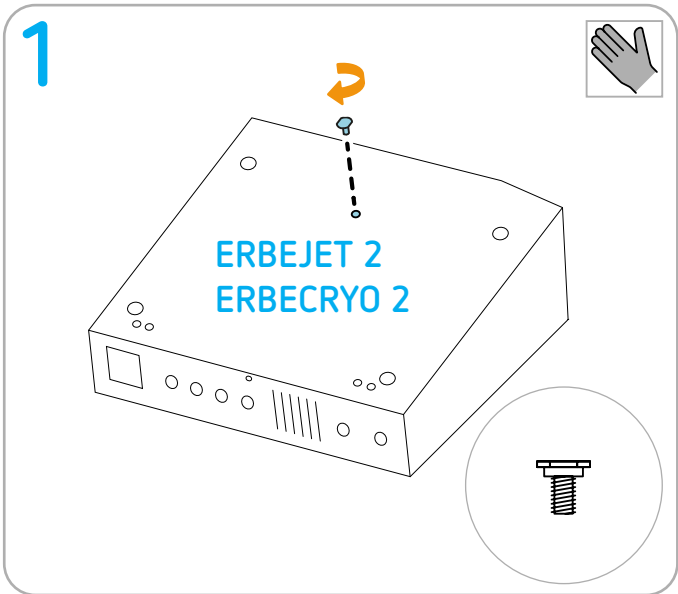
Keep for later use

Tools required


 Torque screwdriver (1 Nm) hex socket 2.5

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the unit

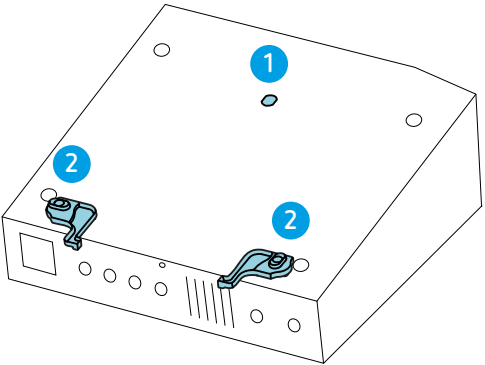
8

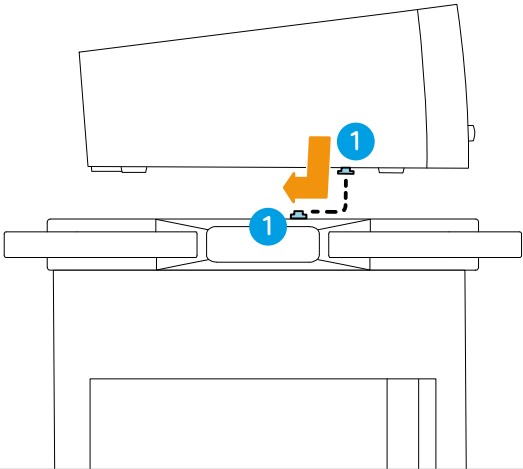9

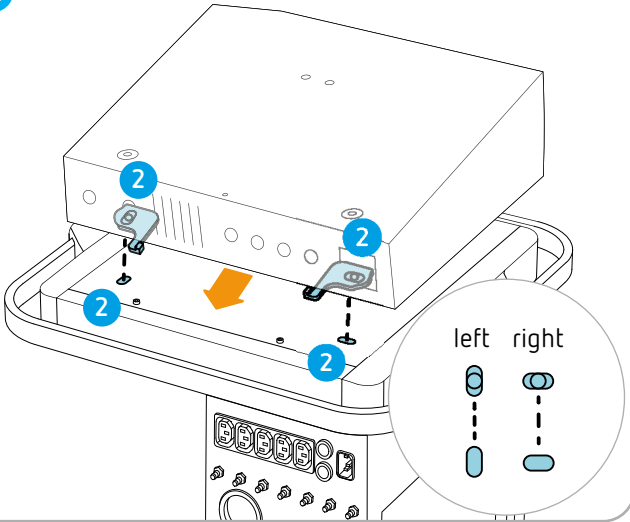10

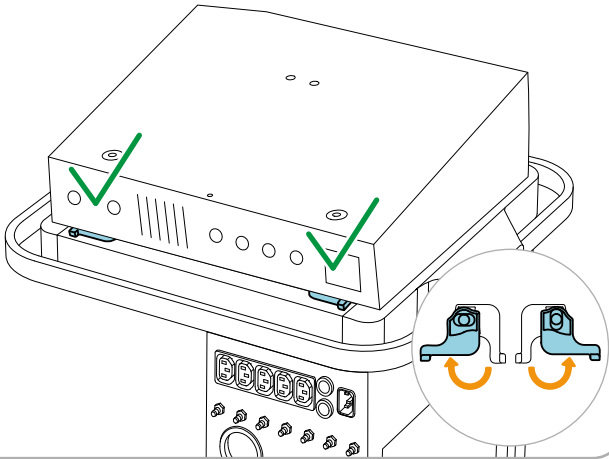
 [Back to Overview](#)

Installing APC 2 and VIO S or VIO D series

Permitted positioning

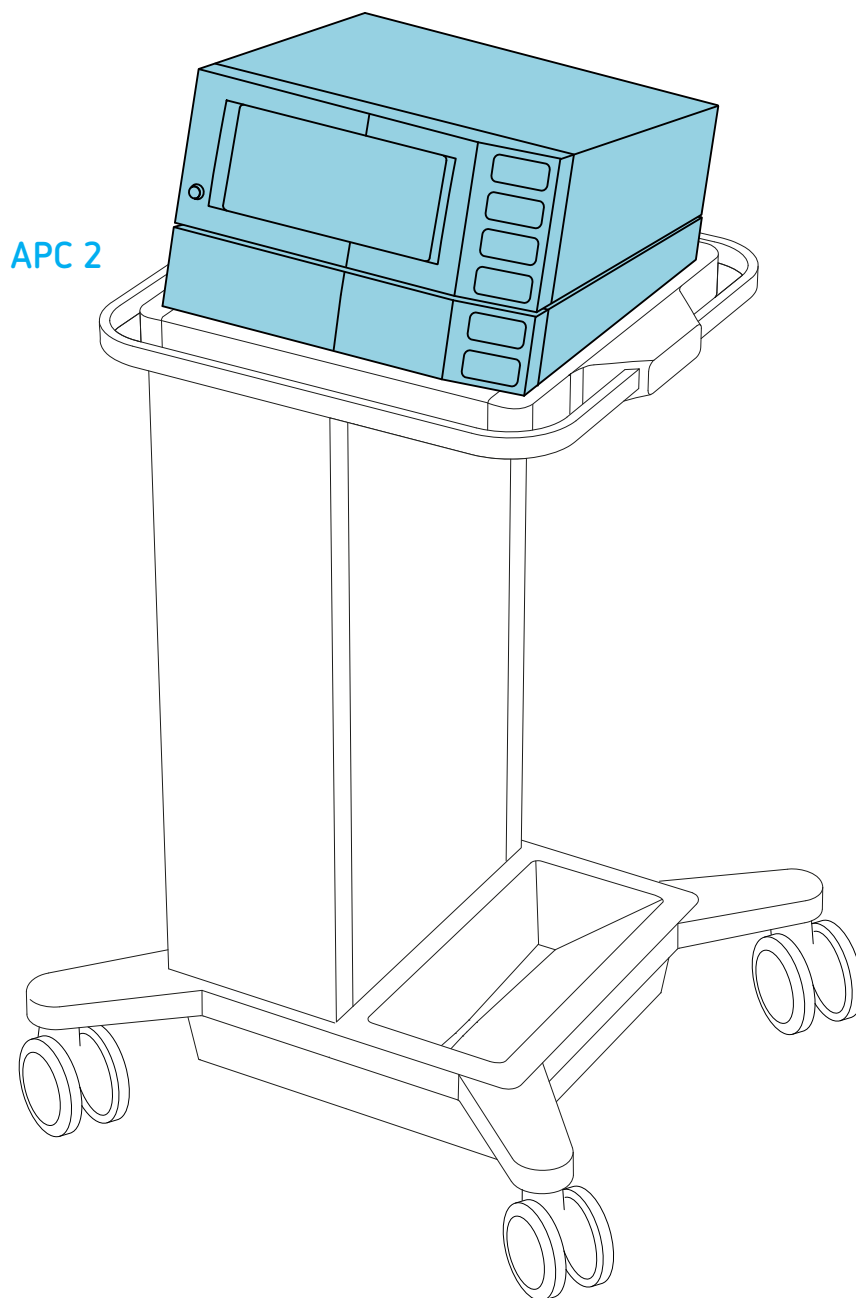VIO S series and VIO D series


Erbe fastening sets

IMPORTANT! Only use original Erbe accessories.


Fastening set for APC on system carrier Erbe REF 20180-150

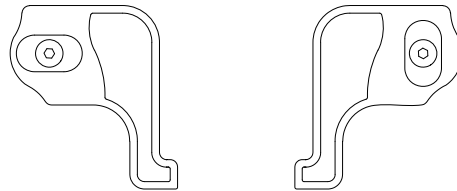M4 x 8

M4 x 16 screws included

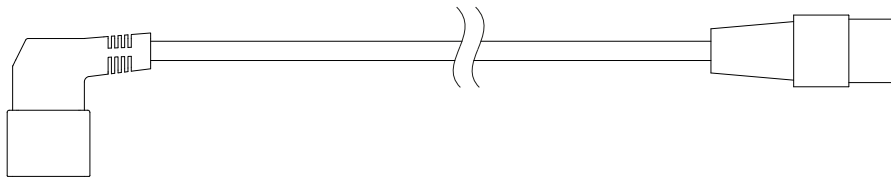
Keep for later use

Keep for later use

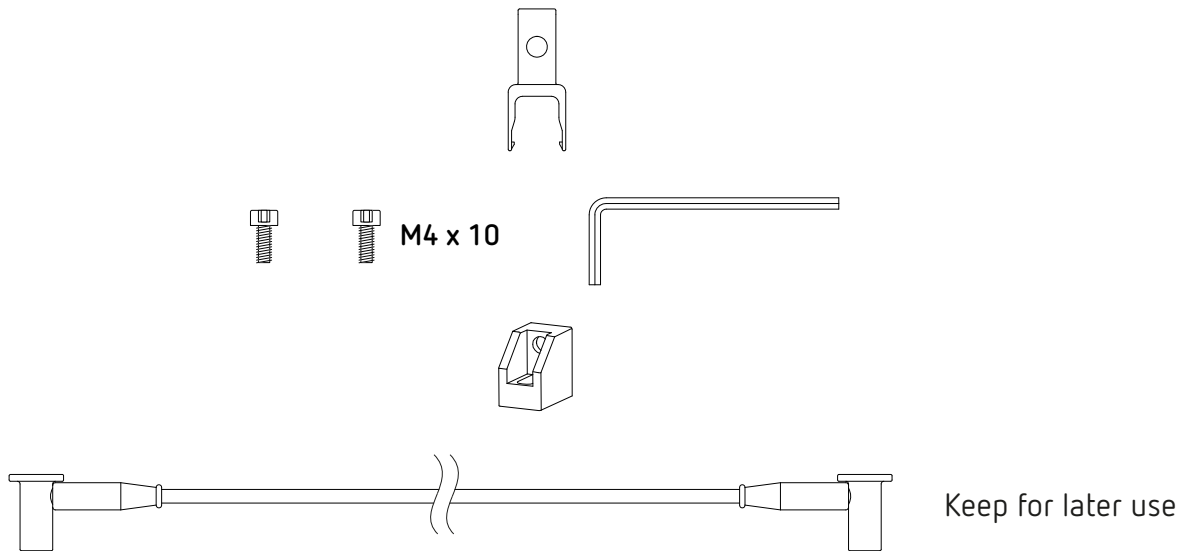Fastening set for VIO on APC Erbe REF 20180-134







Tools required


-  Torque screwdriver (1 Nm) hex socket 2.5

Preparation

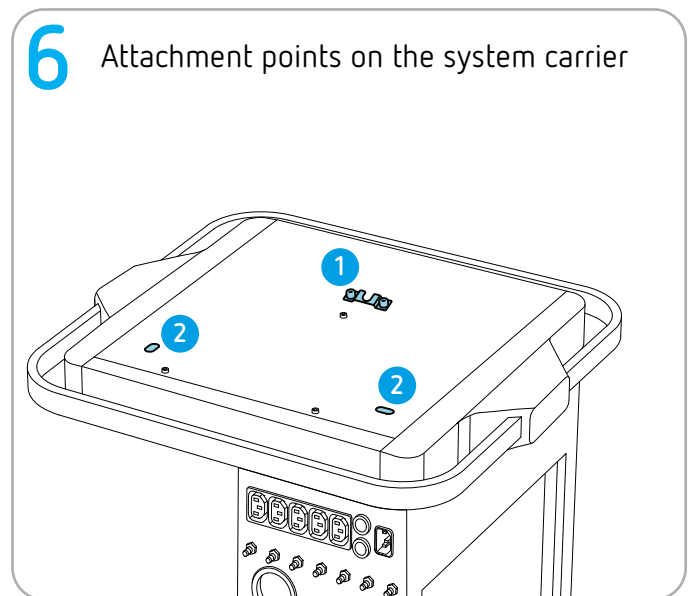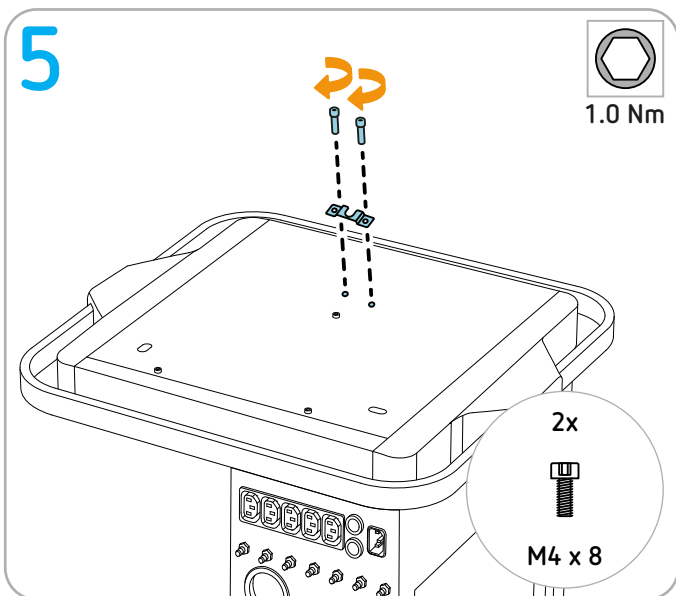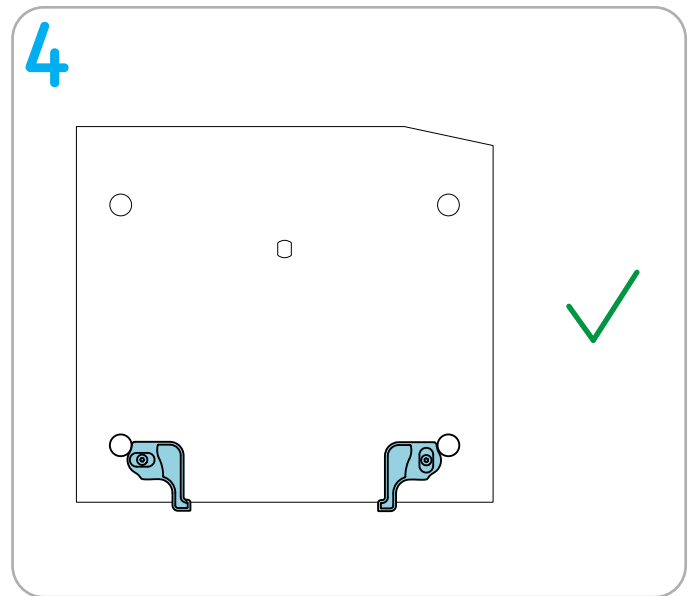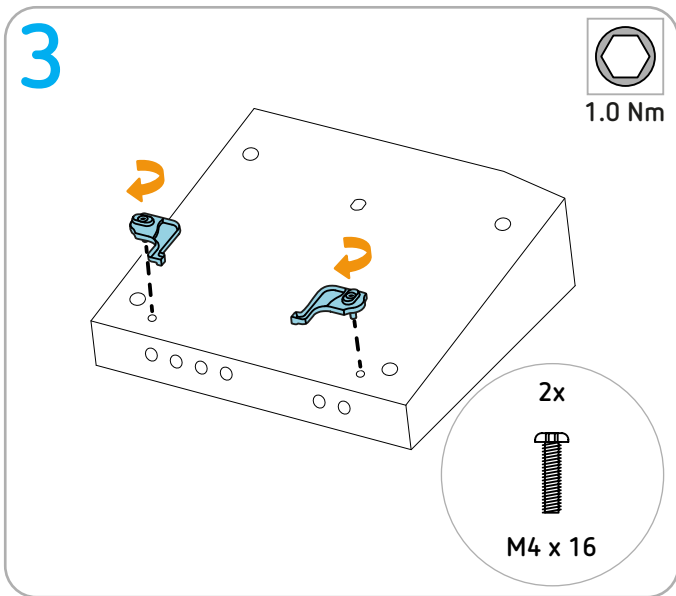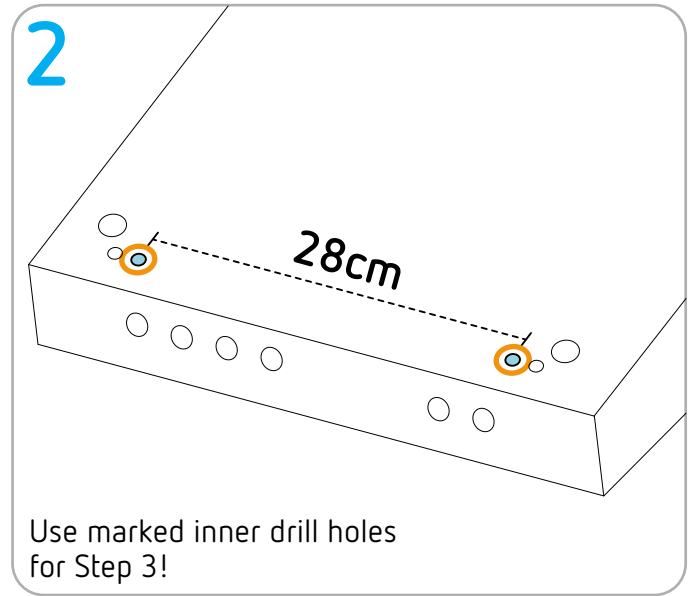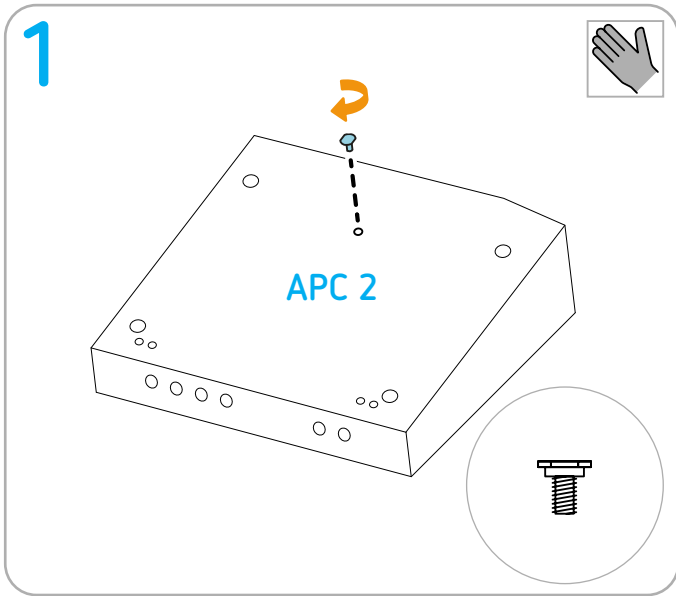
IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the unit

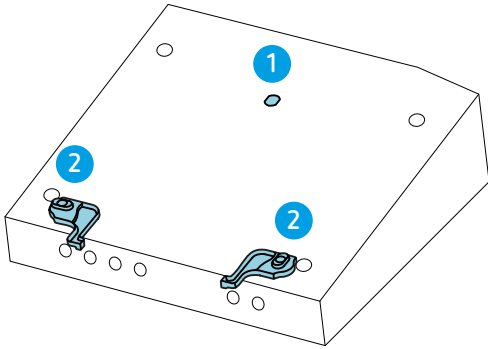
8




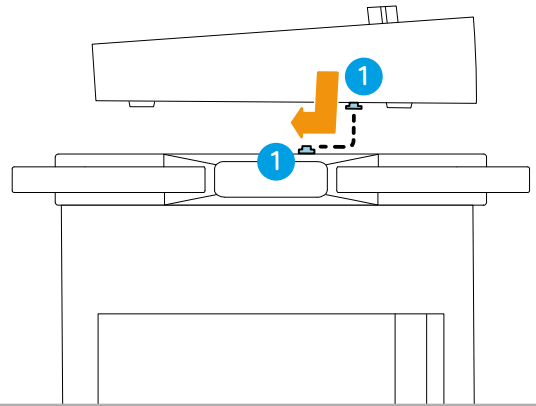
9

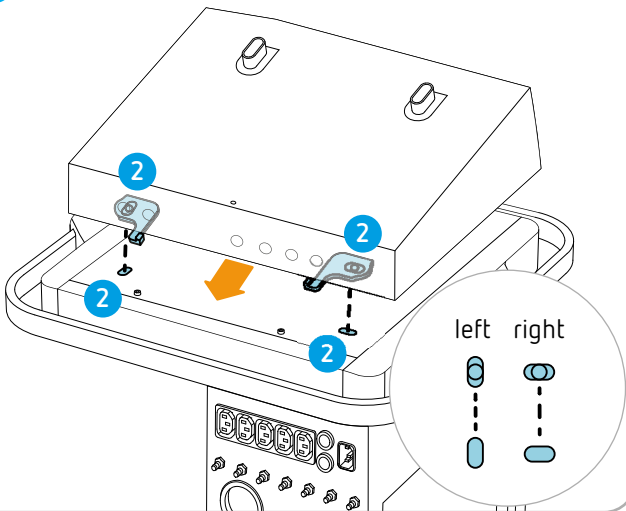10

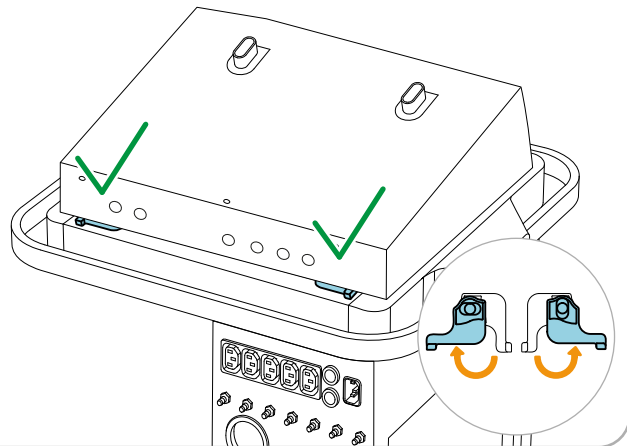
Continue to next page

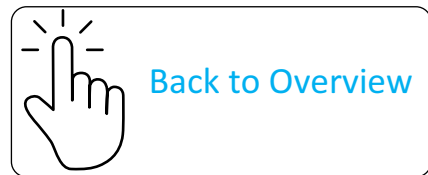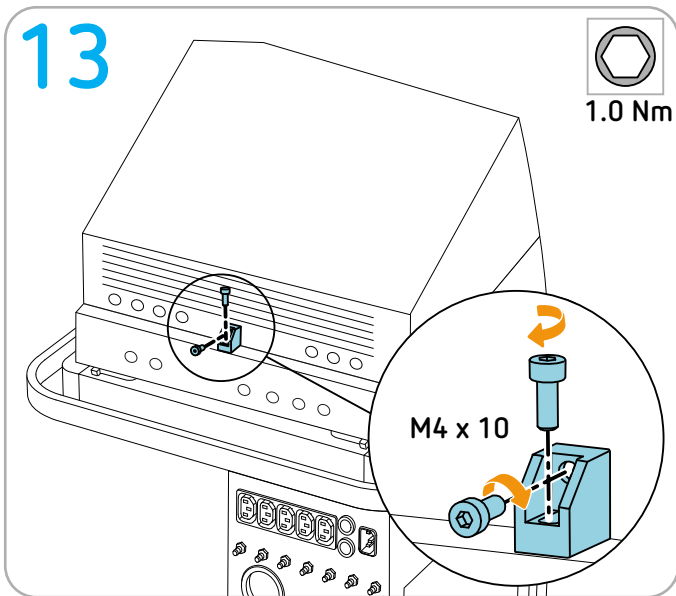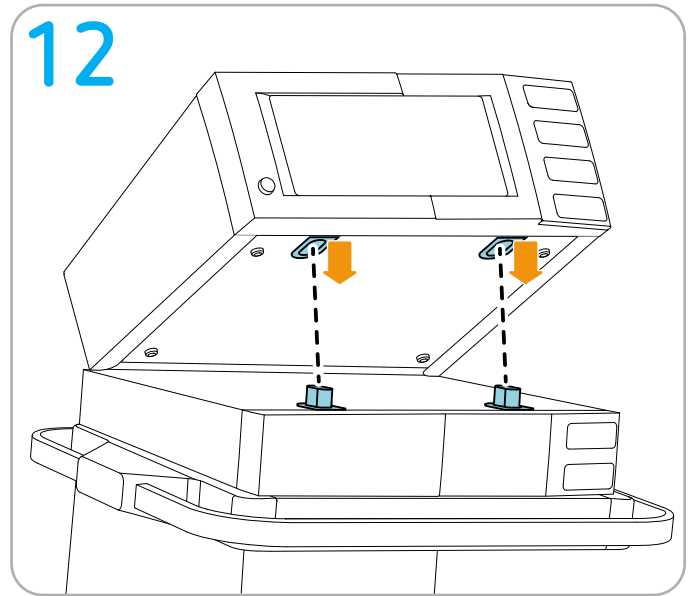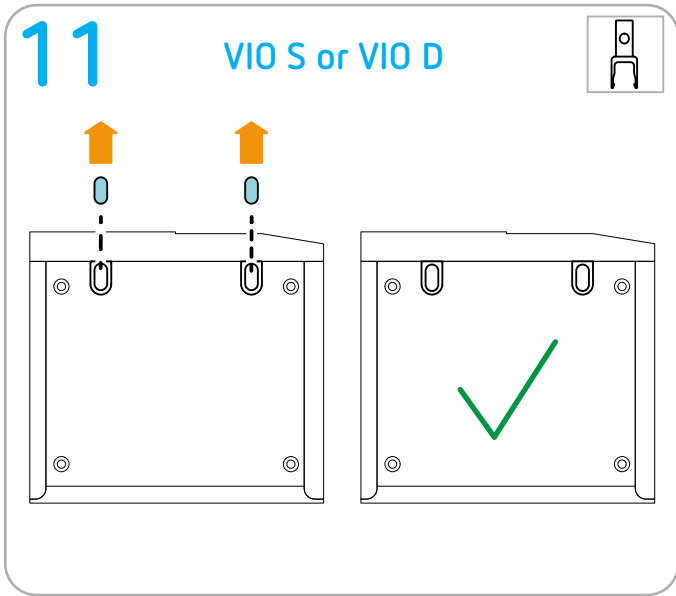Installing APC 3 and VIO 3 series

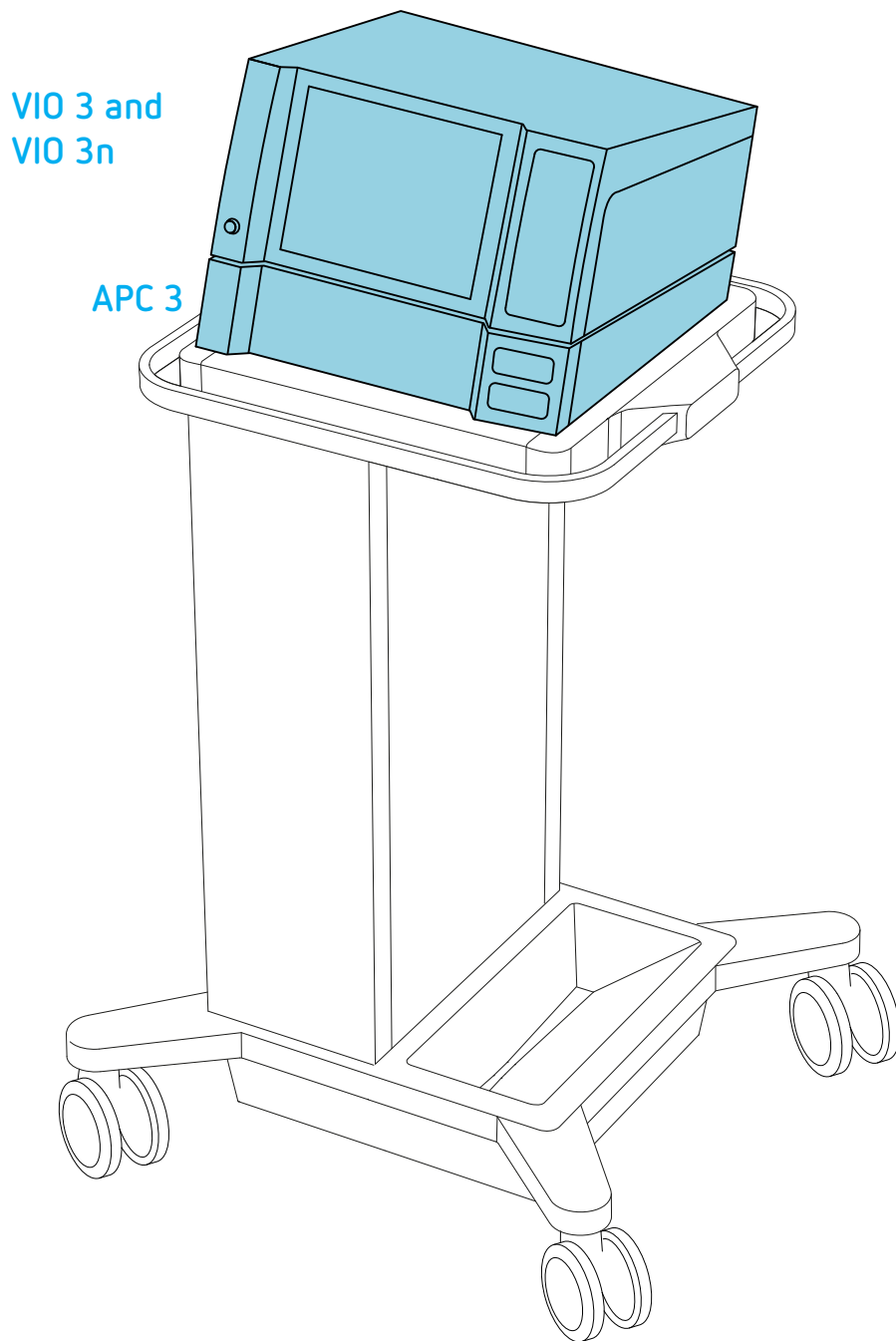Permitted positioning


Erbe fastening sets

IMPORTANT! Only use original Erbe accessories.


Fastening set for APC on system carrier Erbe REF 20180-150

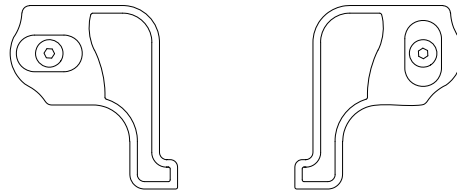M4 x 8

M4 x 16 screws included

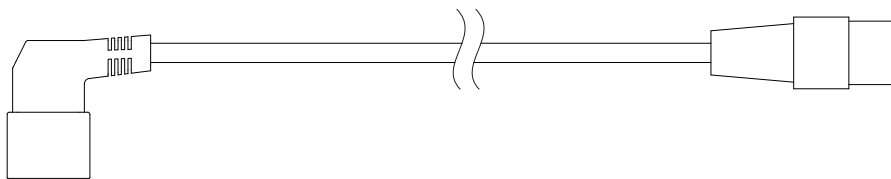
Keep for later use

Keep for later use

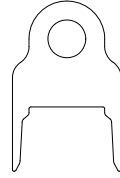Fastening set for VIO on APC: Erbe REF 30135-030 Bracket: Erbe REF 40160-065


M4 x 8


M4 x 16

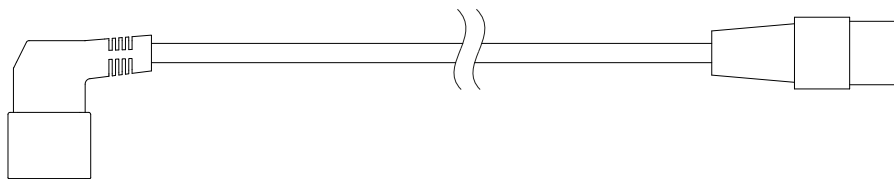Keep for later use

Keep for later use

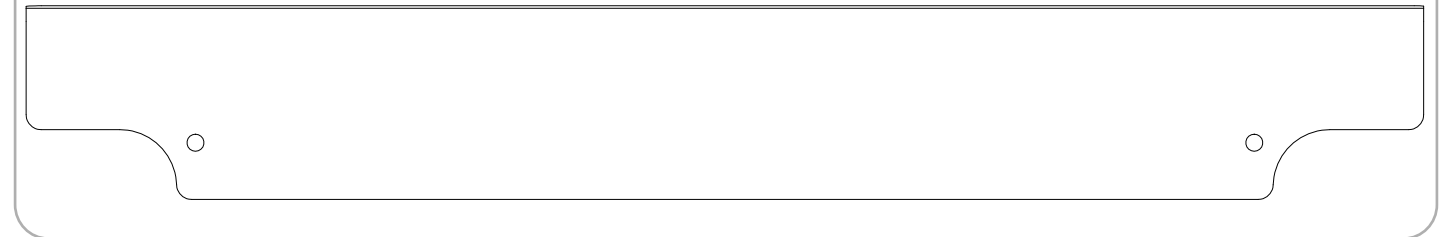Tools required

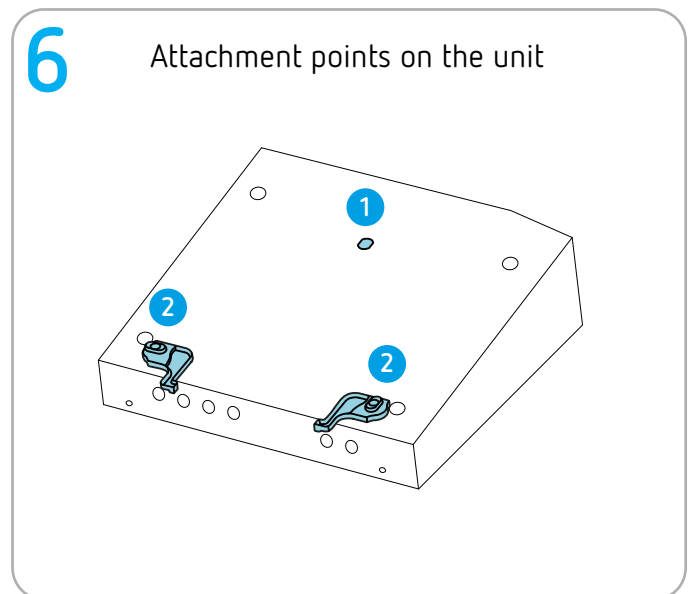
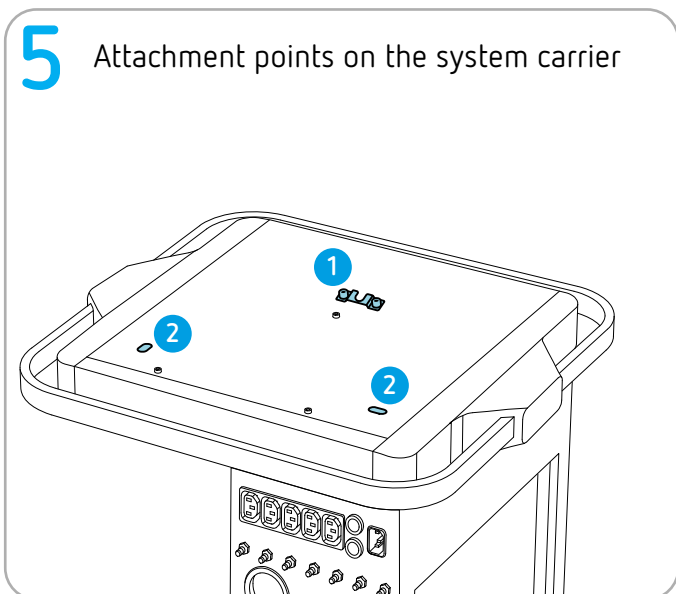
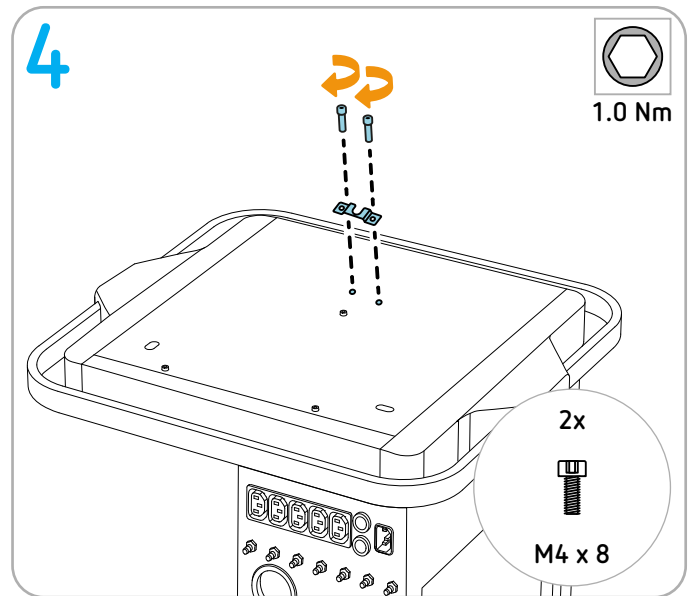
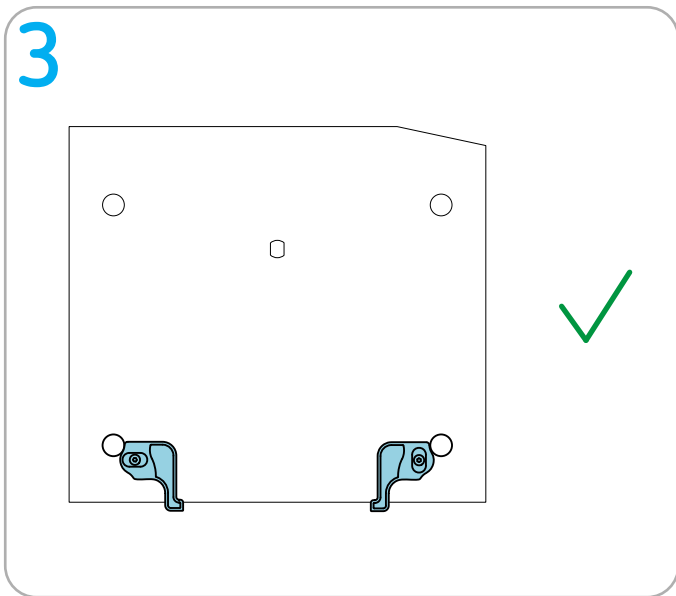
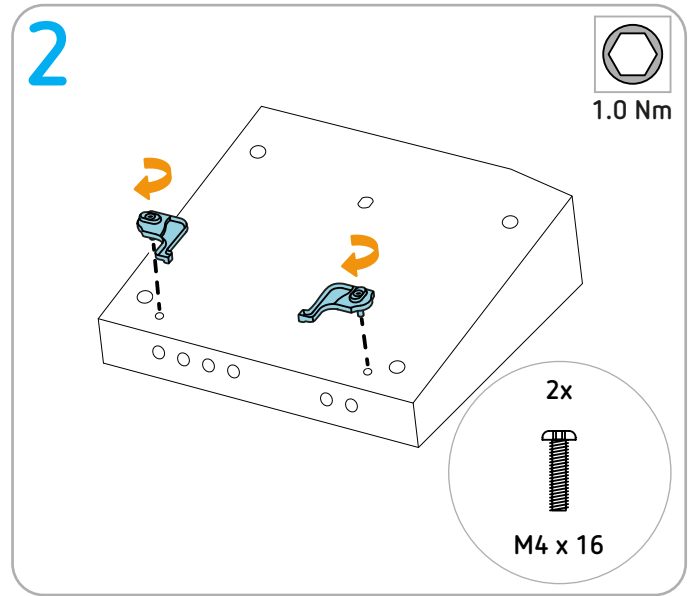
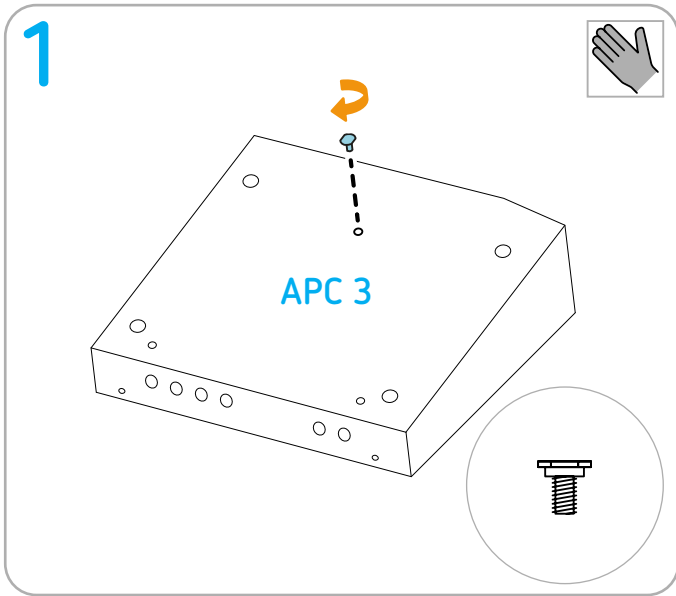
Torque screwdriver (1 Nm) hex socket 2.5

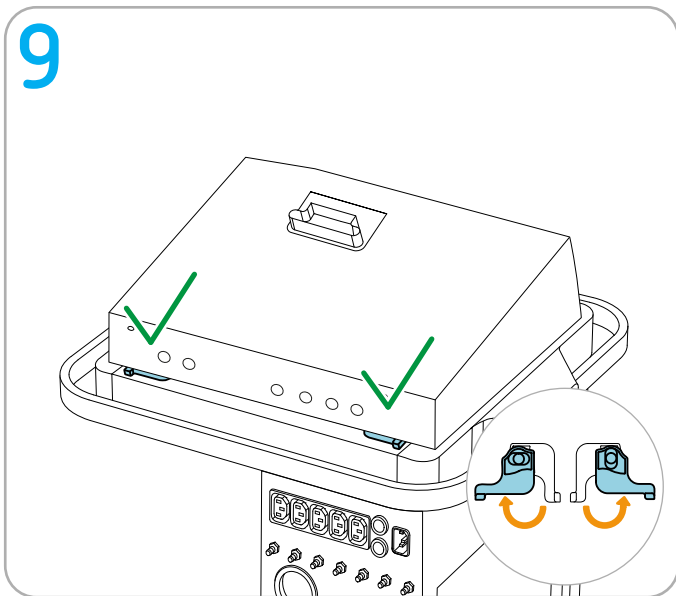
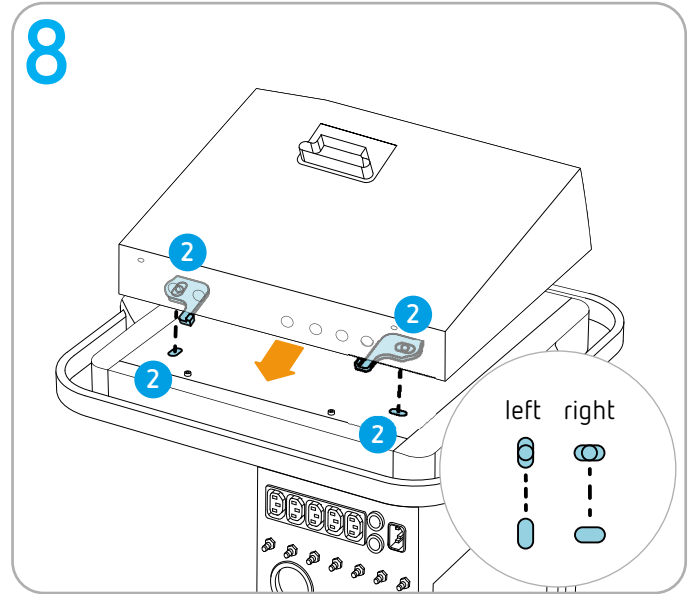
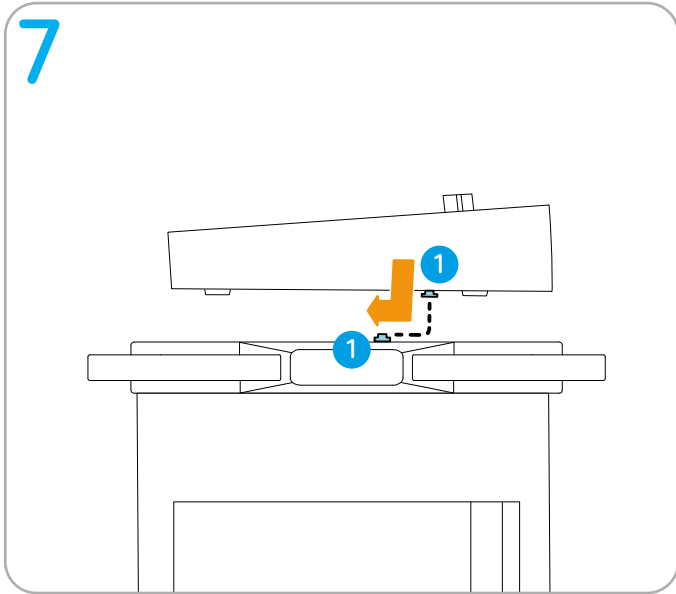
Torque screwdriver (1 Nm) hex socket 3






Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

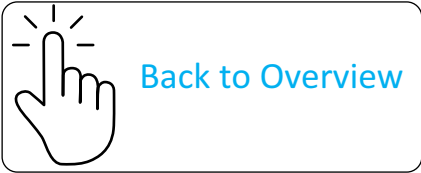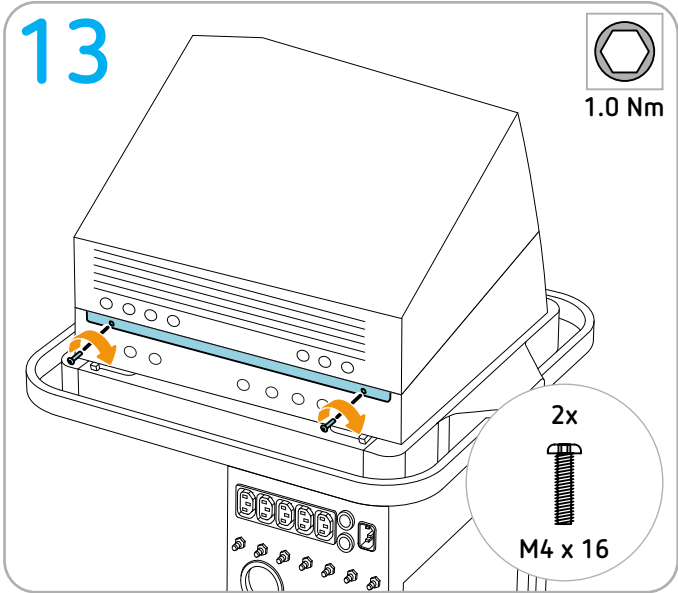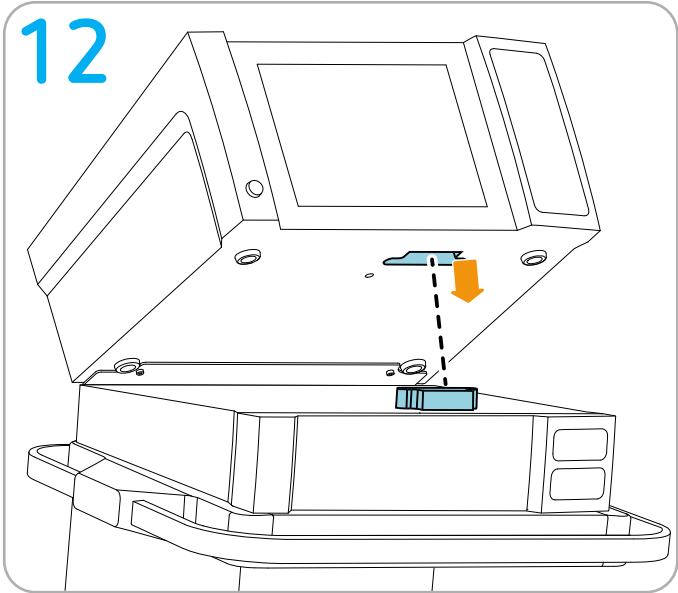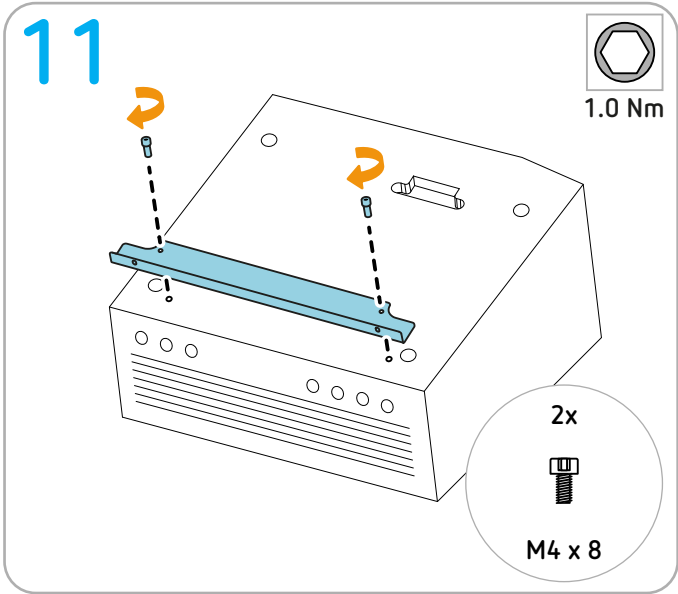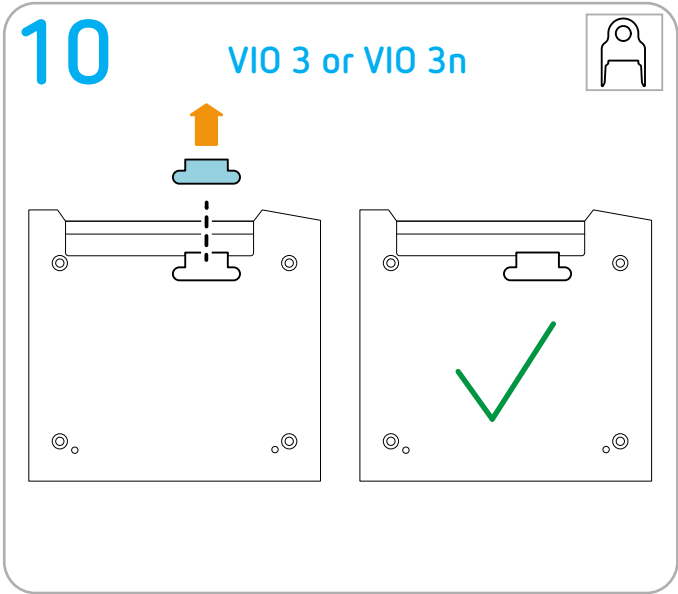Installing VIO seal and VIO 3n

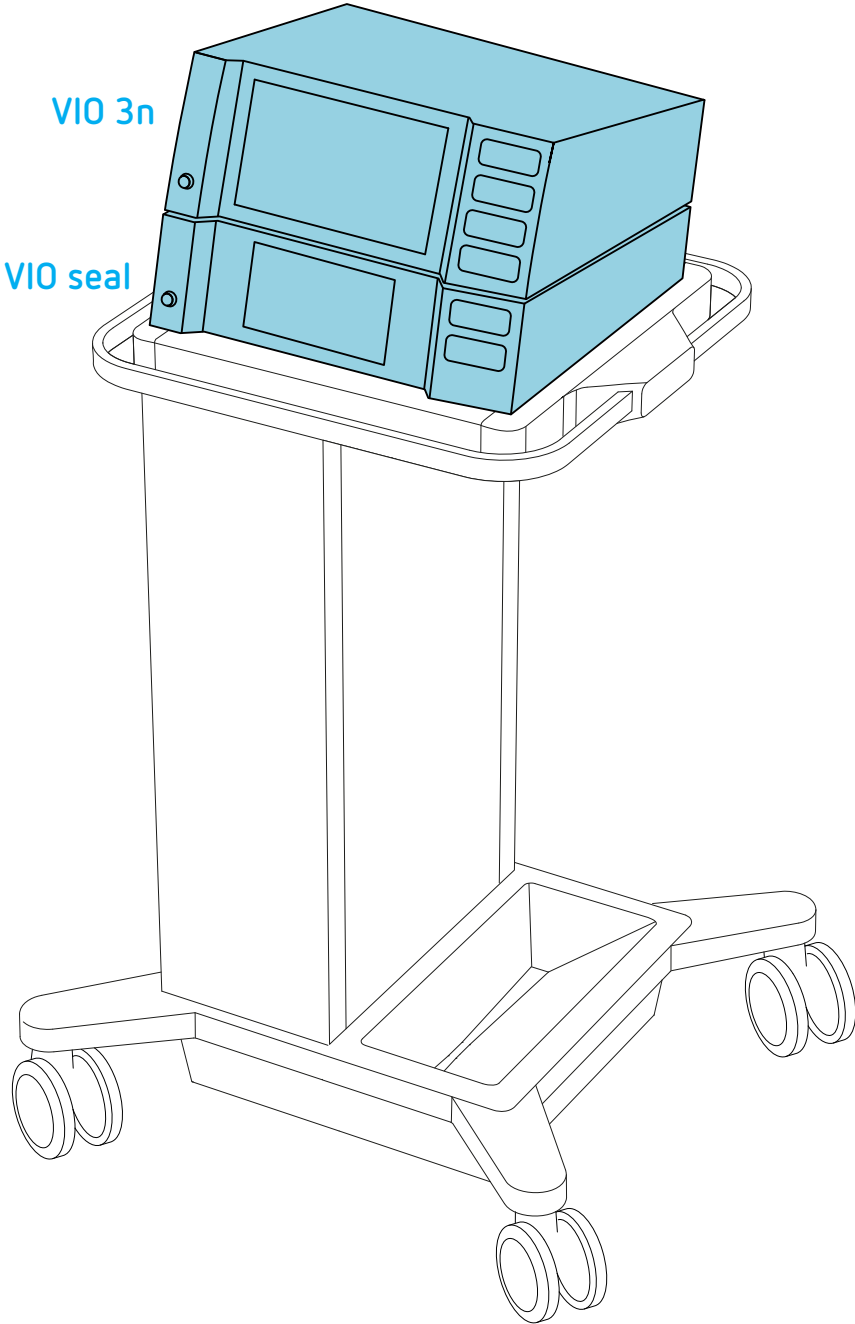Permitted positioning

Erbe fastening set

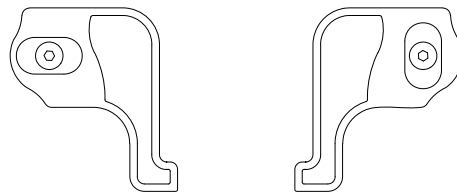
IMPORTANT! Only use original Erbe accessories. For the following assembly sequence, the Erbe fastening set shown below must be ordered in quantity 2 (1 for mounting the unit on the other unit, 1 for mounting the unit on the system carrier).

IMPORTANT! Order the protective cap for the via of the VIO 3n (see Step 10) separately as needed; it is not part of this fastening set 20180-150. Erbe order number 40160-131: Protective cap for VIO 3n Fire, Metal or Water: Erbe REF No. 40160-112. Protective cap for VIO 3n Stone or Timber: Erbe REF No. 40160-131.


Erbe REF 20180-150

M4 x 8

M4 x 16 screws included

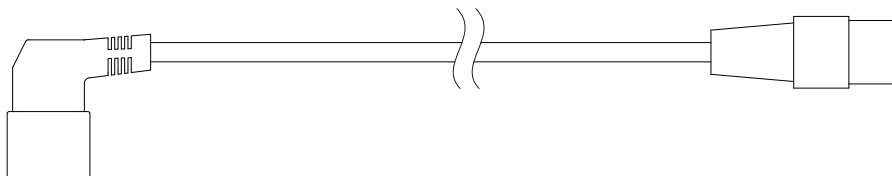Keep for later use

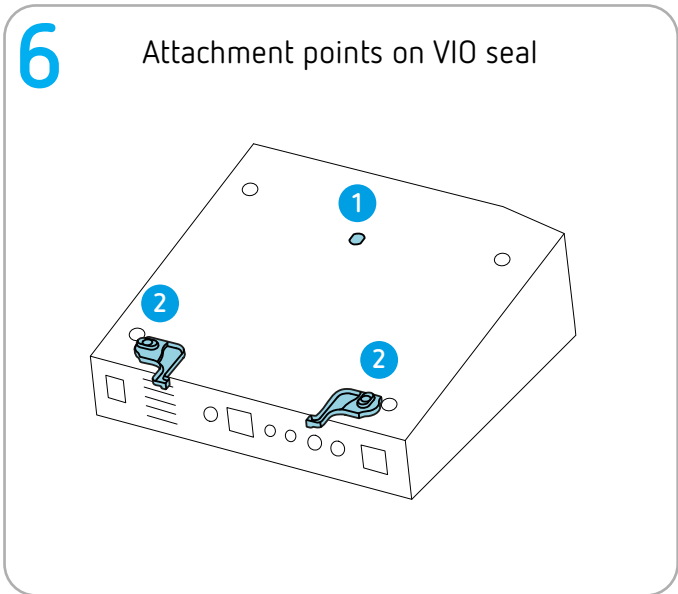
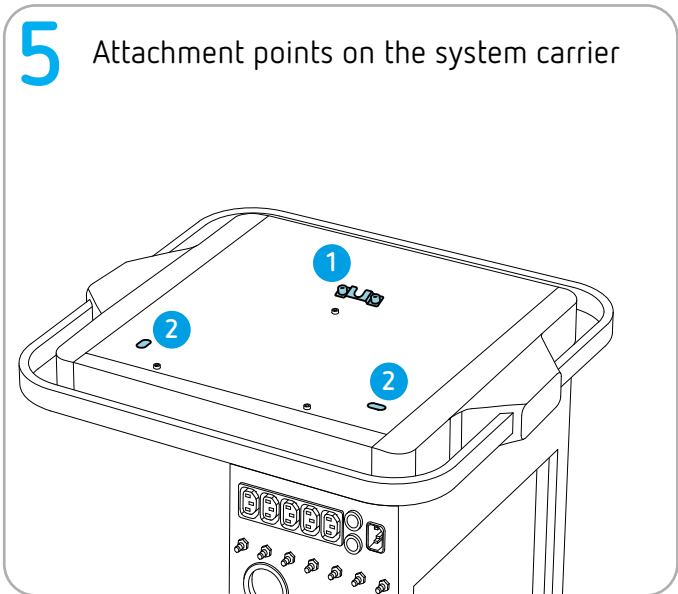
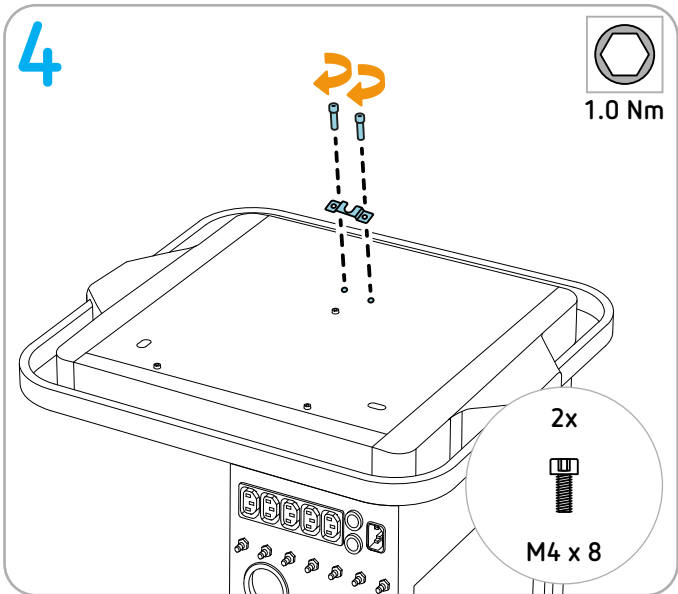
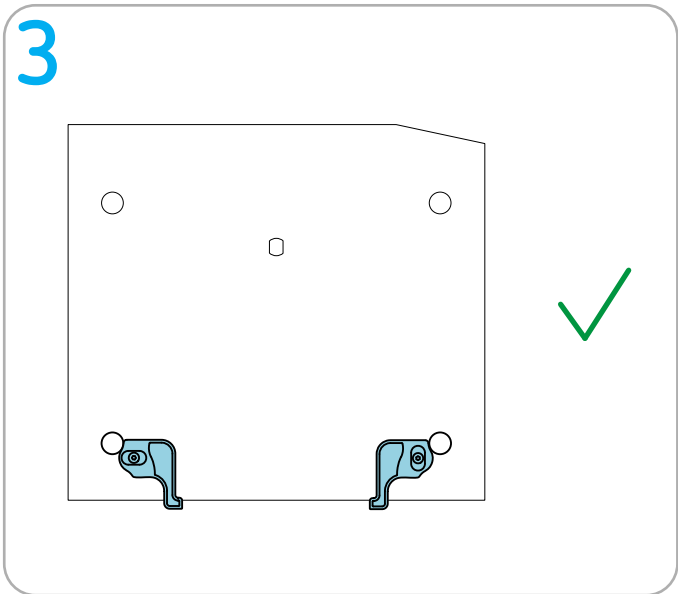
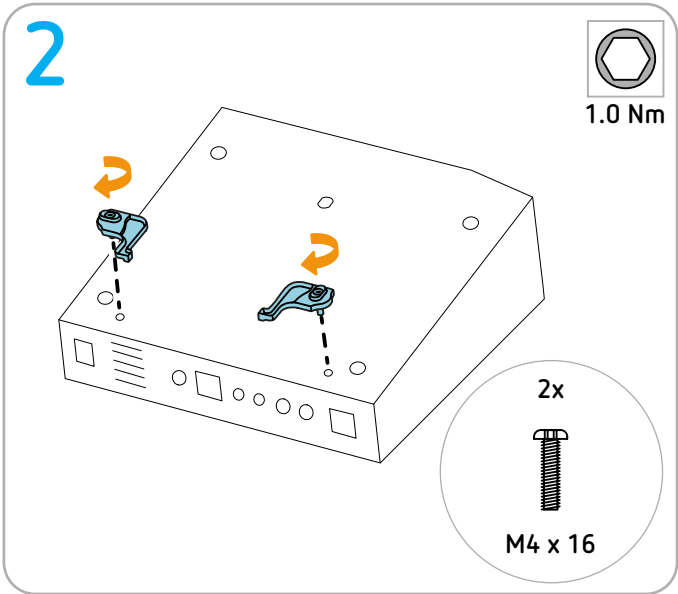
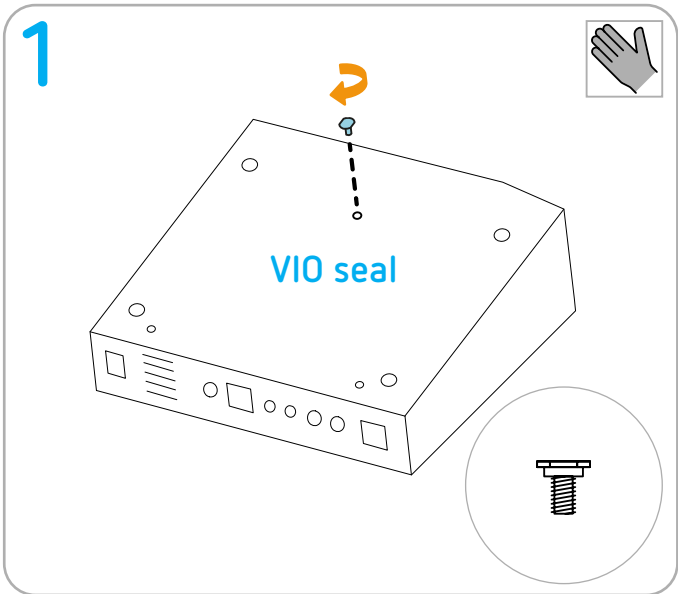
Keep for later use

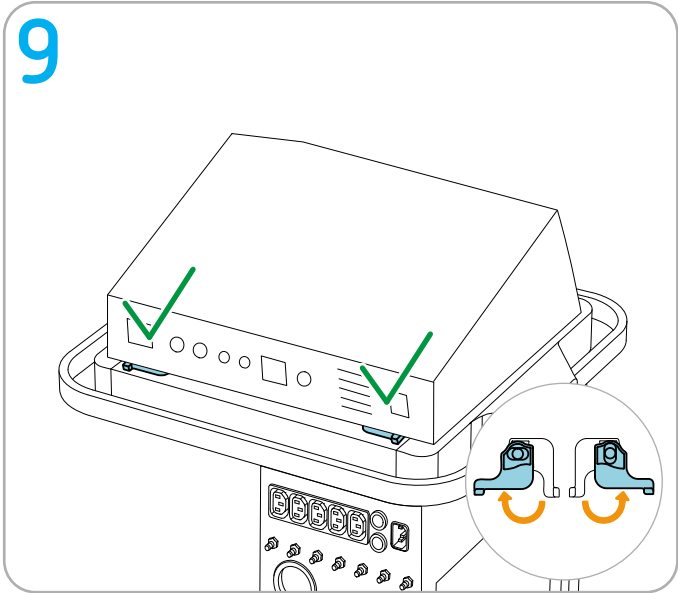
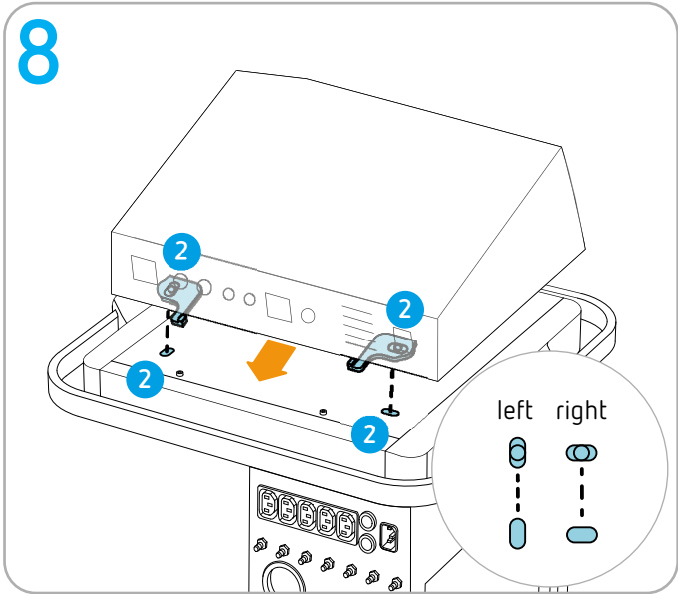
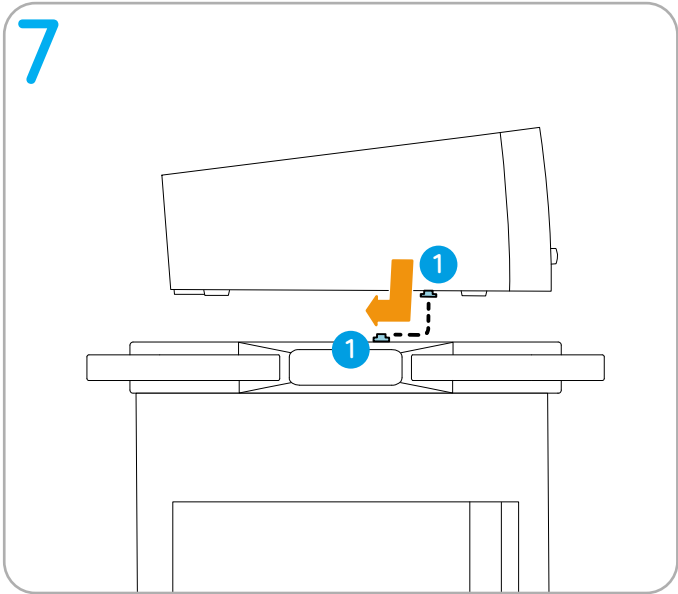
Tools required

 Torque screwdriver (1 Nm) hex socket 2.5






Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


Continue to next page

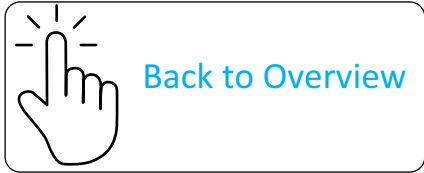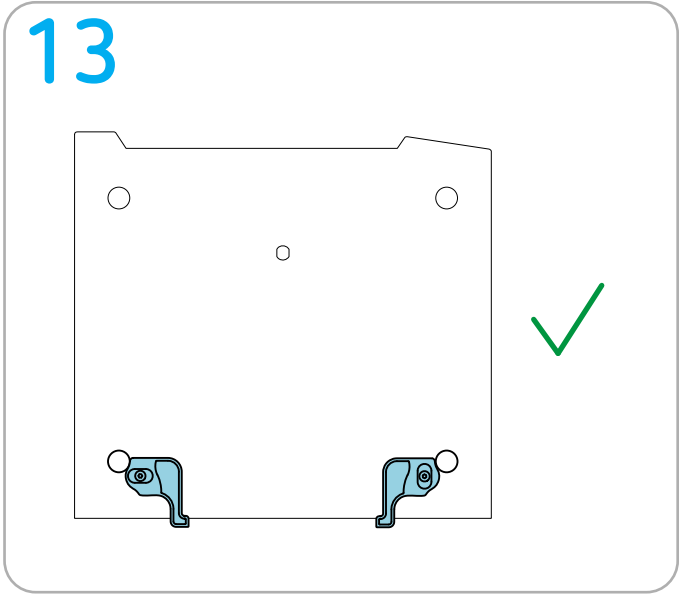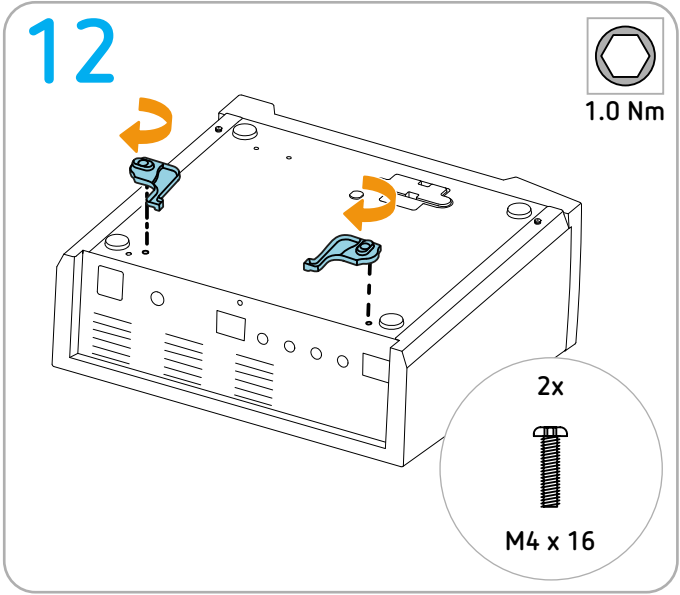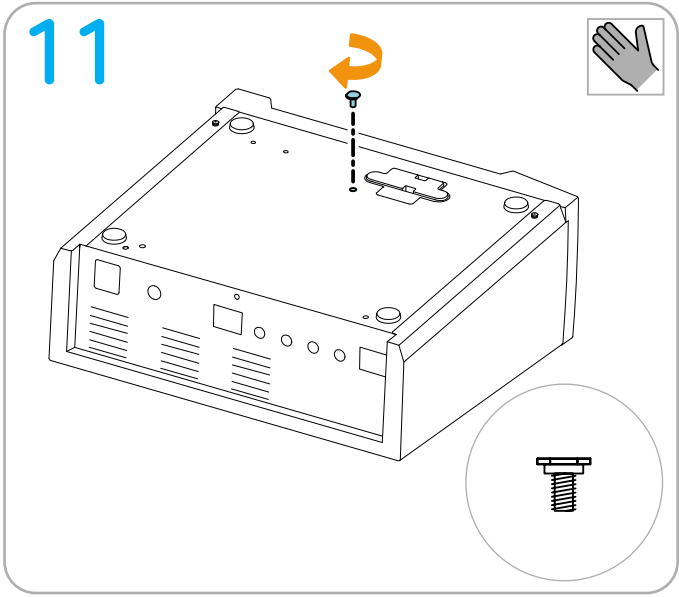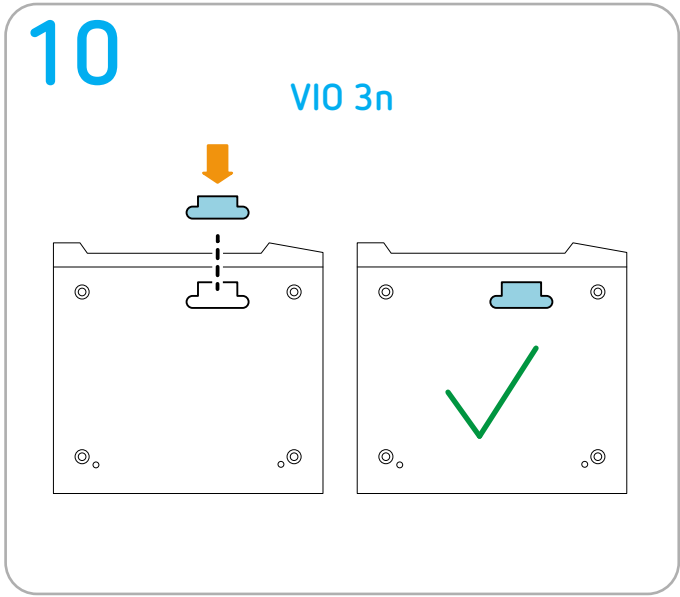
Installing ERBEJET 2 or ERBECRYO 2 + APC 2 + VIO S or VIO D series

Permitted positioning

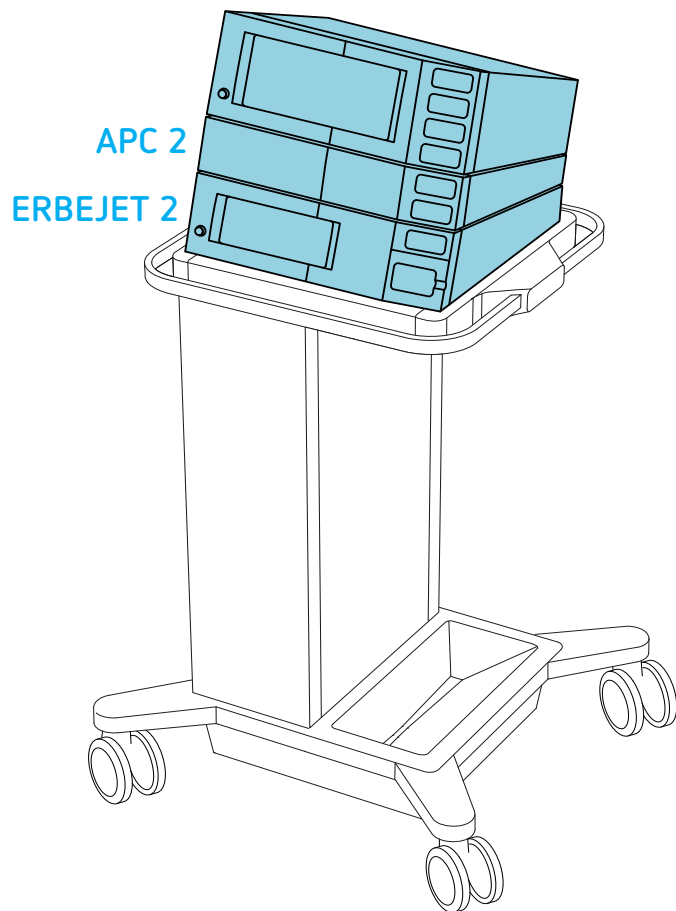VIO S series or VIO D series

VIO S series or VIO D series



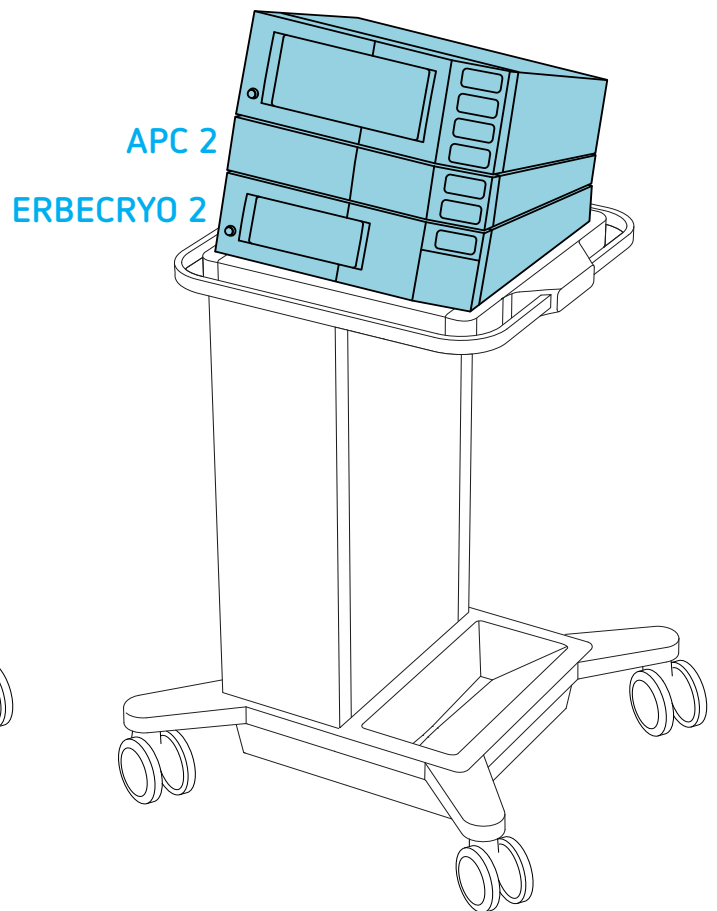
Erbe fastening set

IMPORTANT! Only use original Erbe accessories.


Erbe REF 20180-150

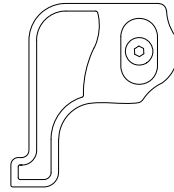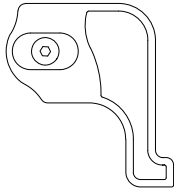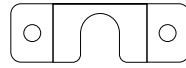M4 x 8

M4 x 16 screws included


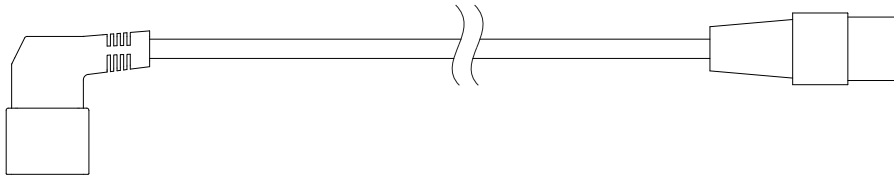
Keep for later use

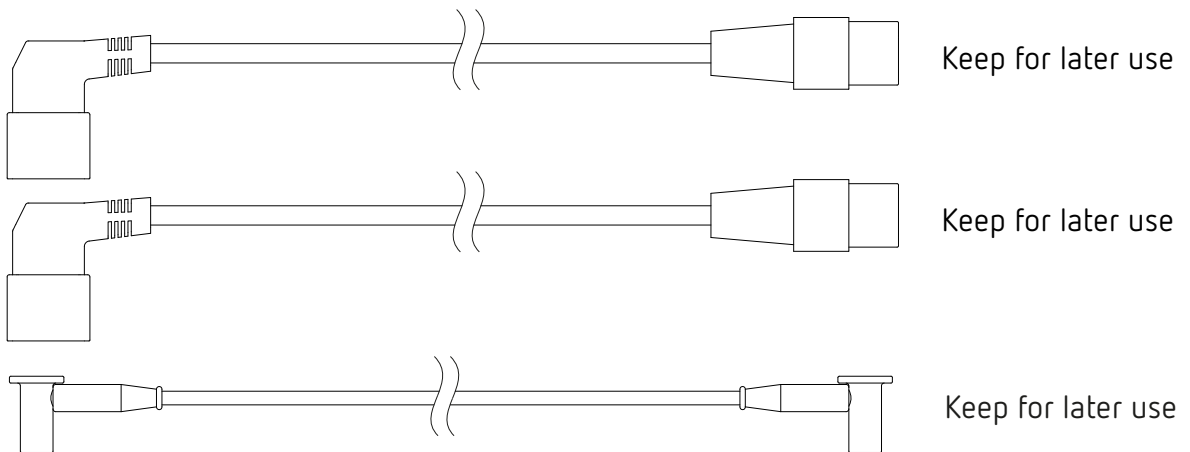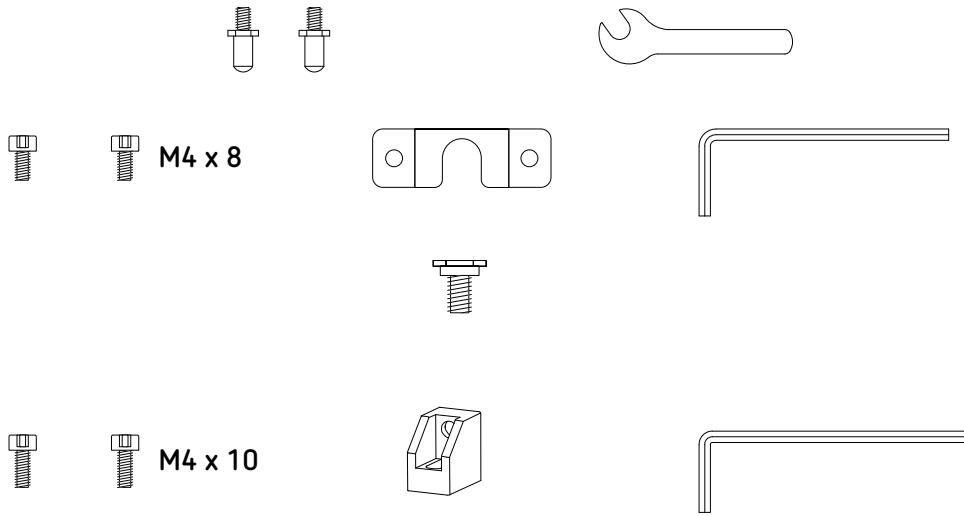
Keep for later use

Erbe REF 20150-050

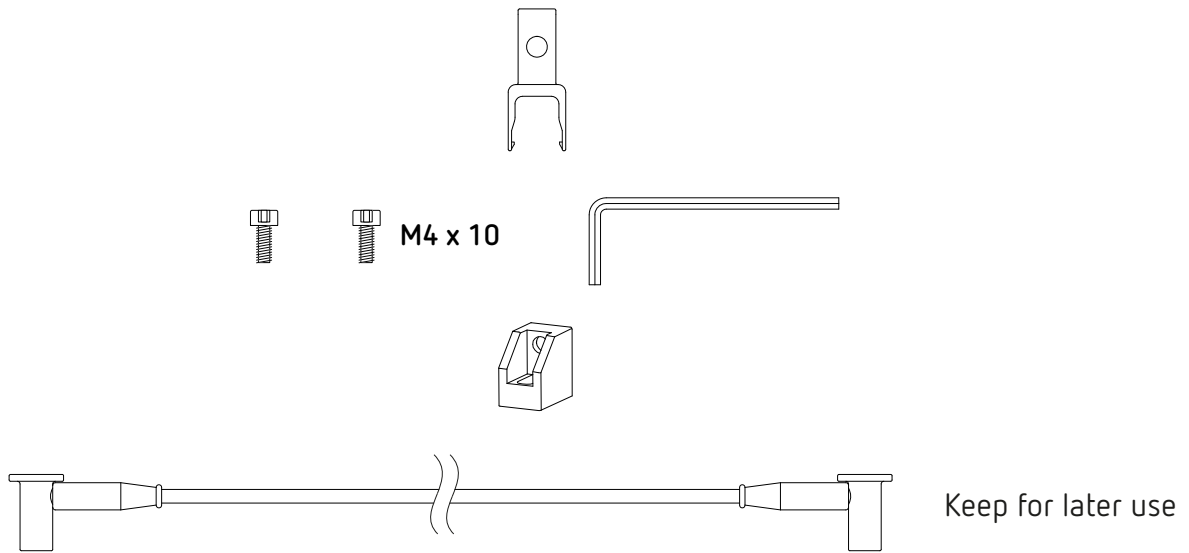Fastening set for VIO on APC Erbe REF 20180-134

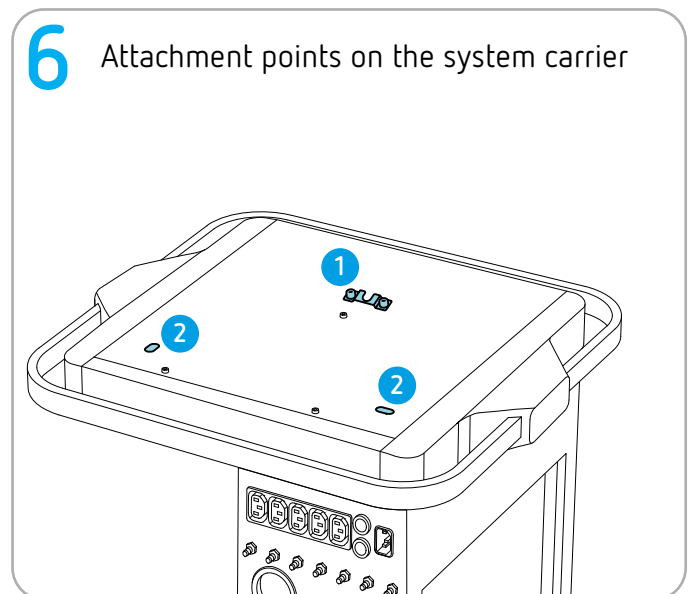
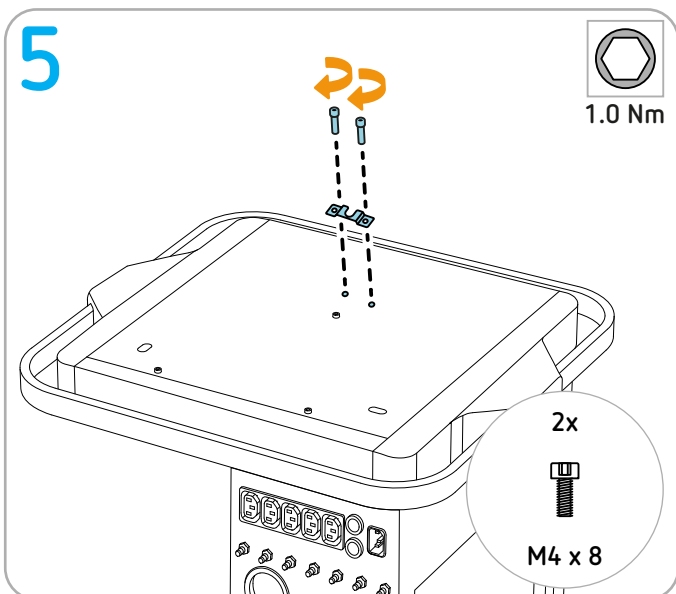
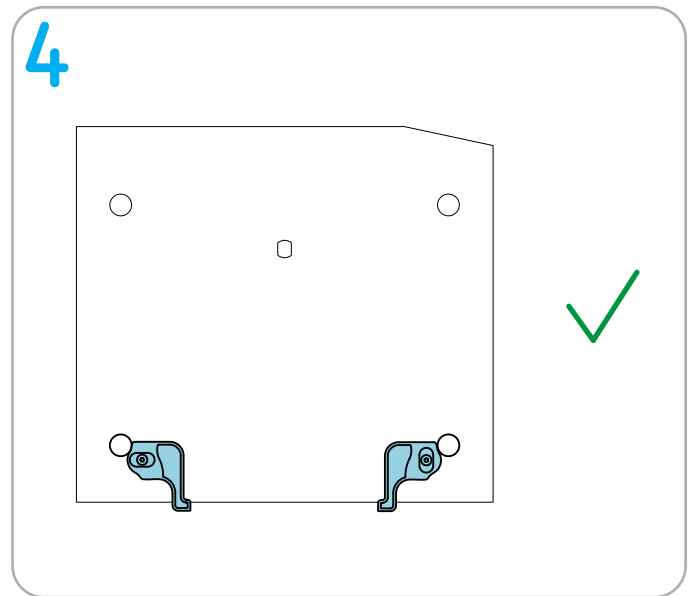
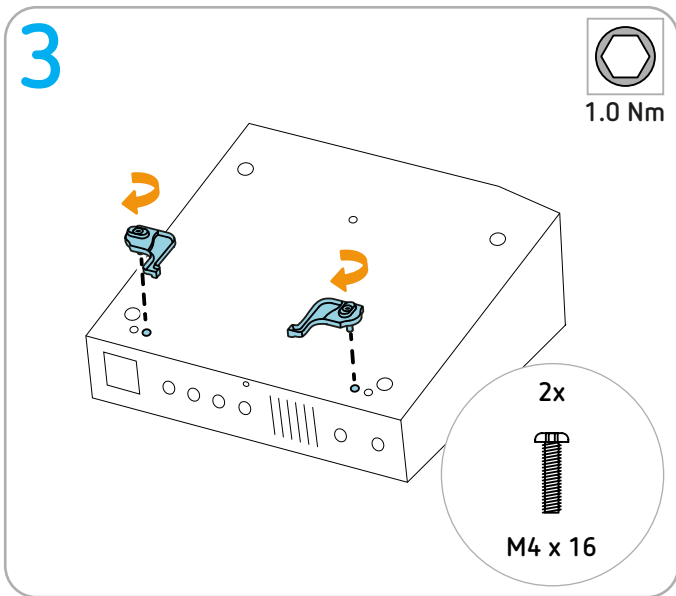
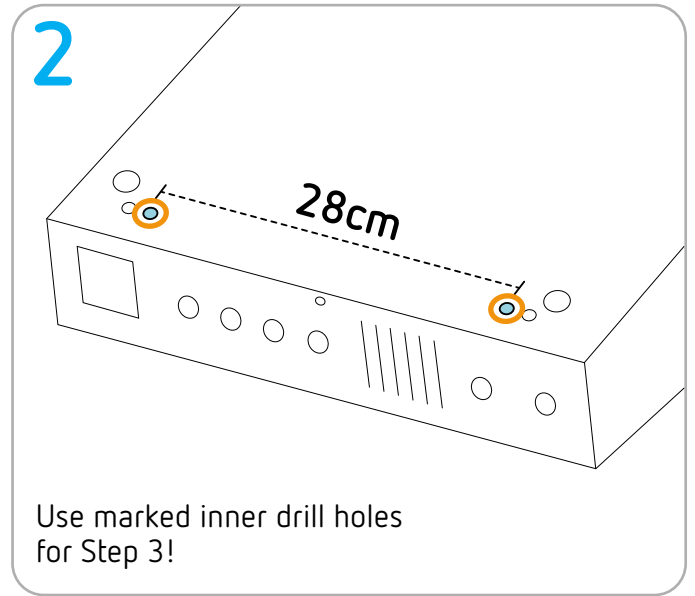
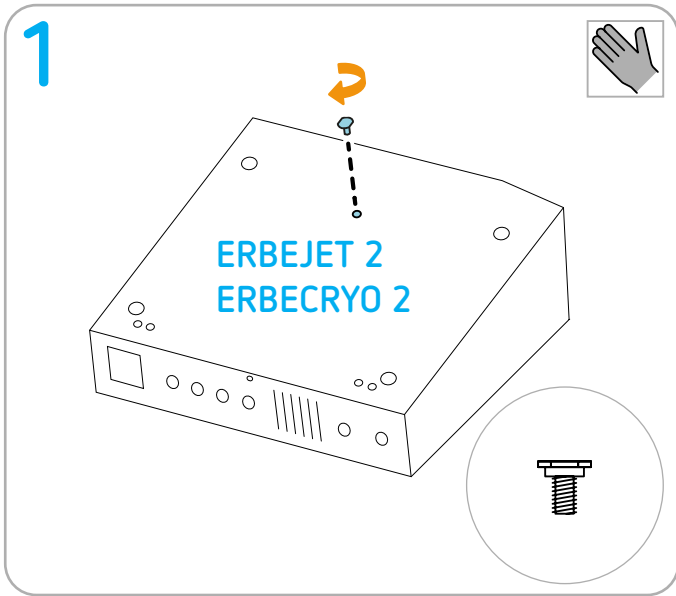
Tools required


-  Torque screwdriver (1 Nm) hex socket 2.5
-  Torque screwdriver (1 Nm) hex socket 3
-  Torque screwdriver (1 Nm) open-end wrench 7
-  Cutter

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the unit

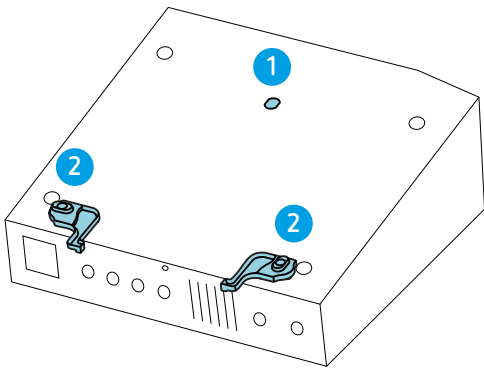
8

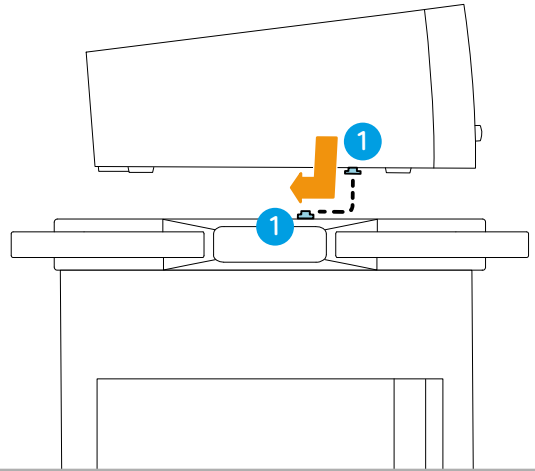
9

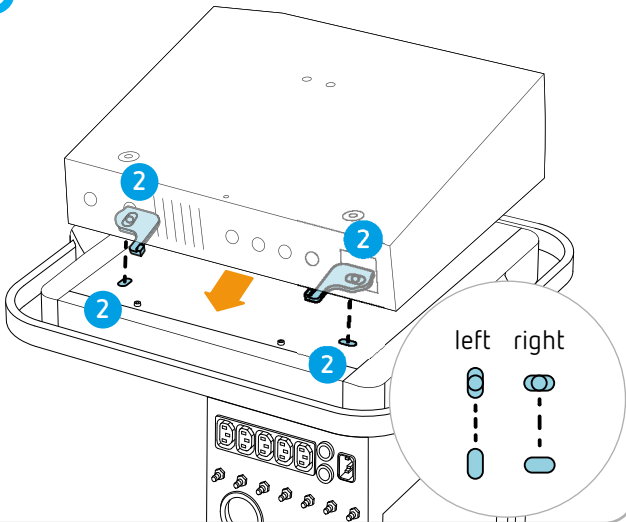
10











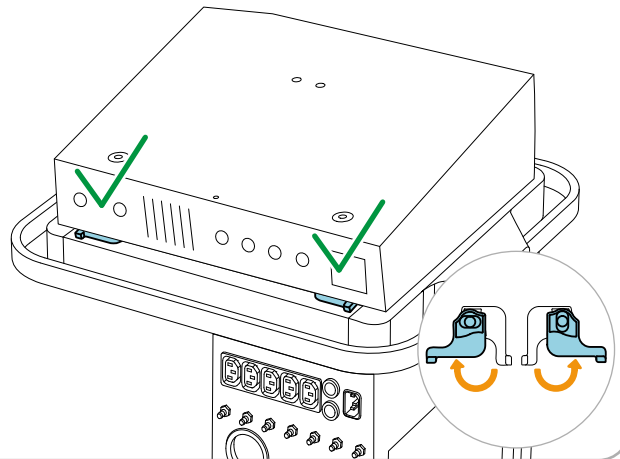
11

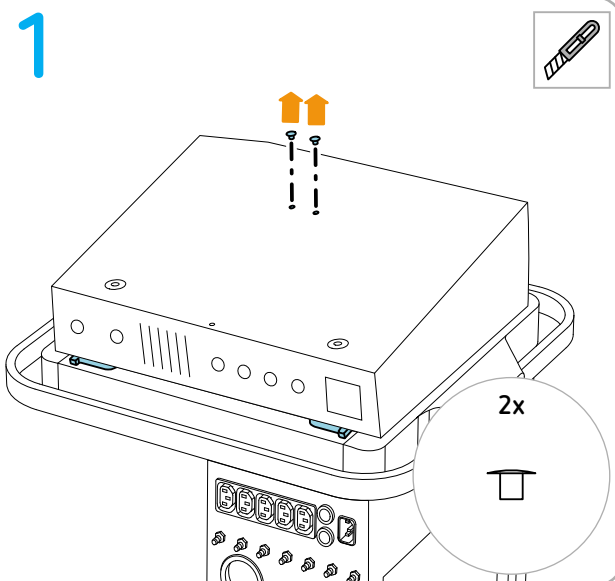12

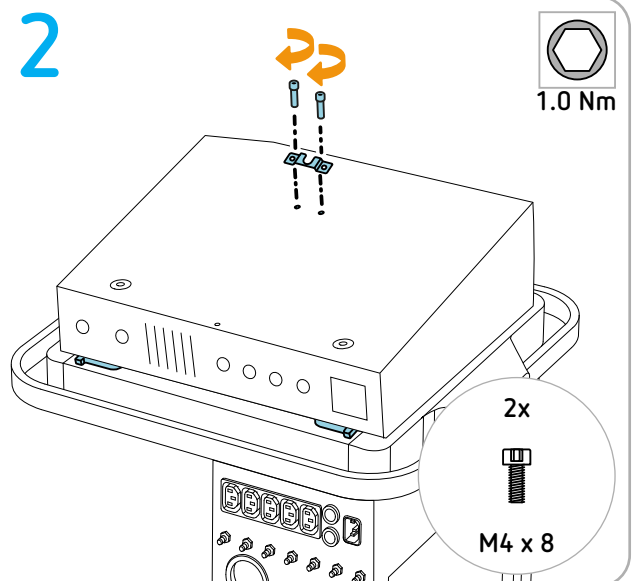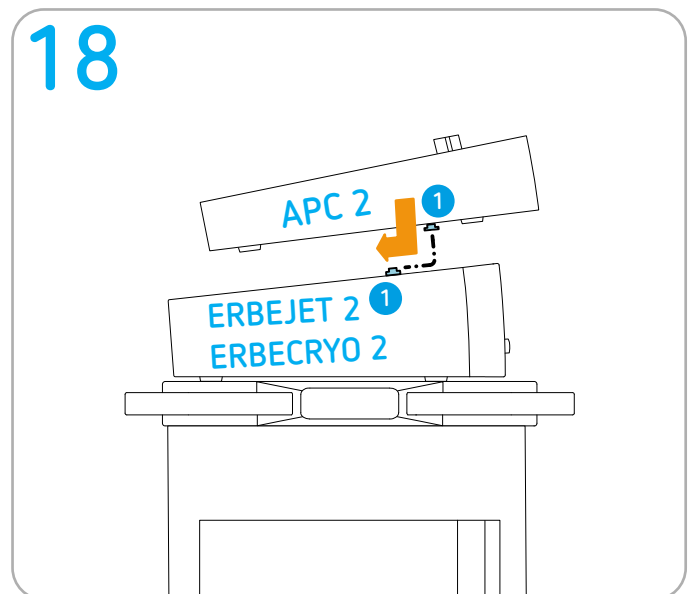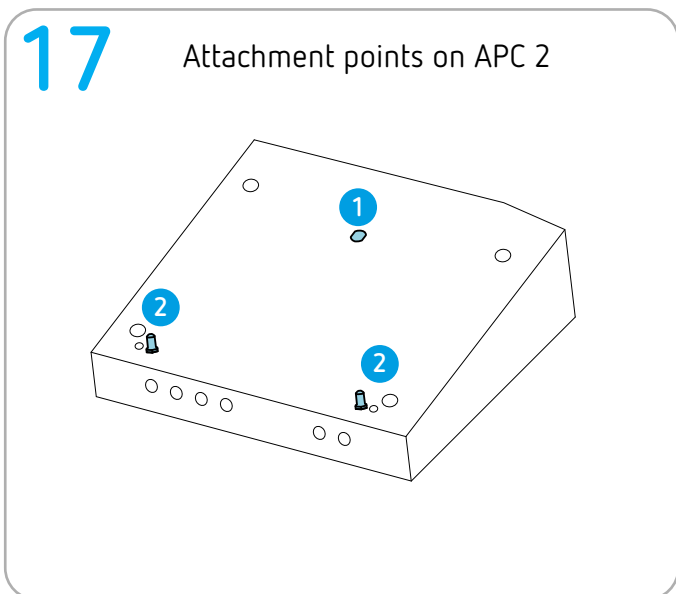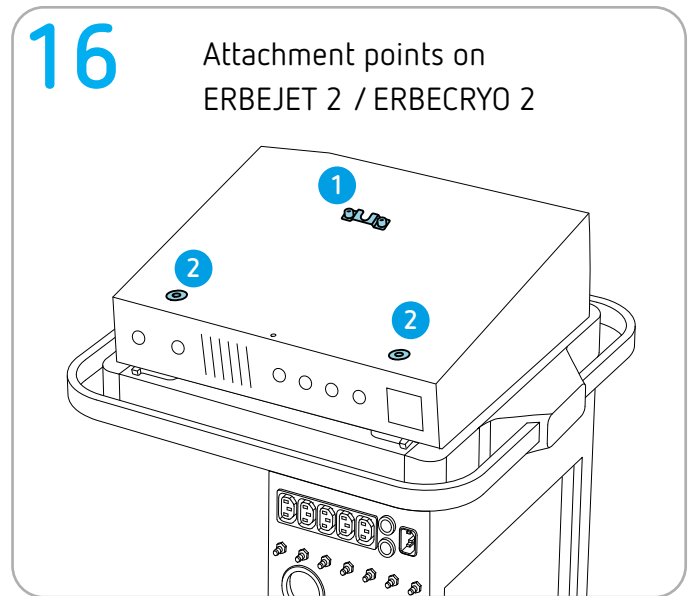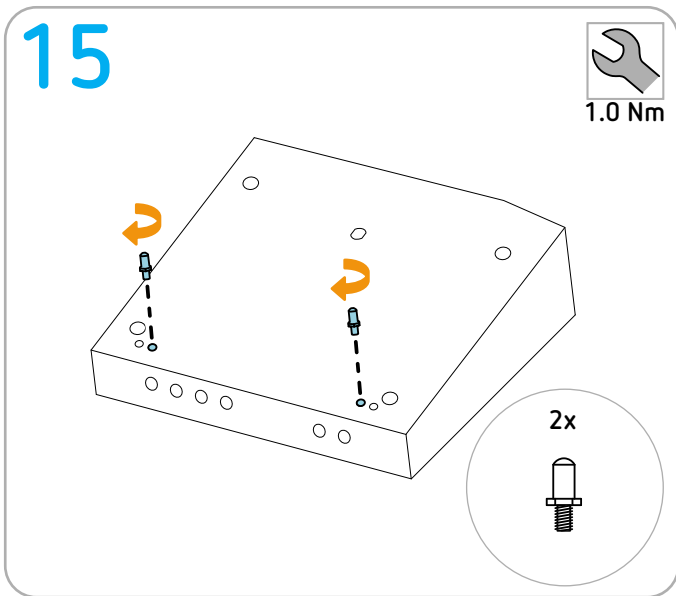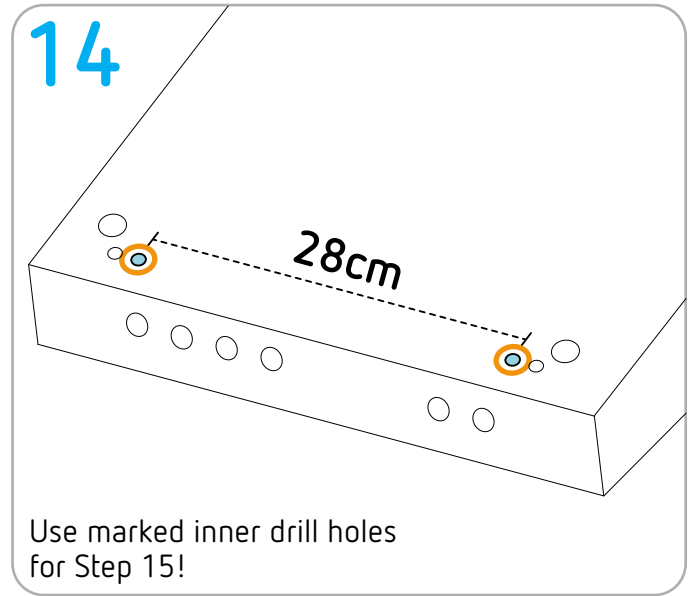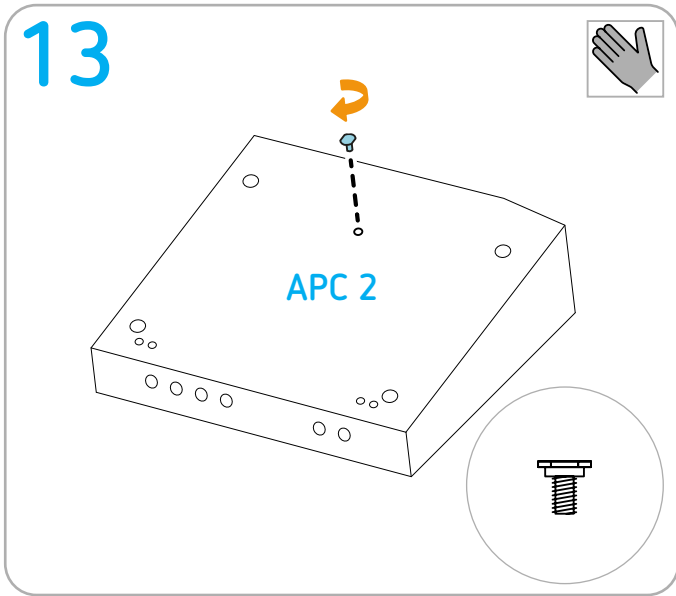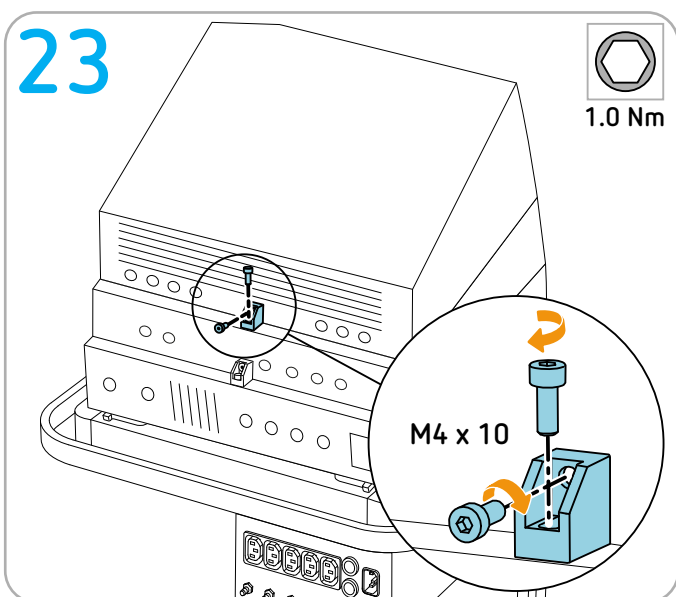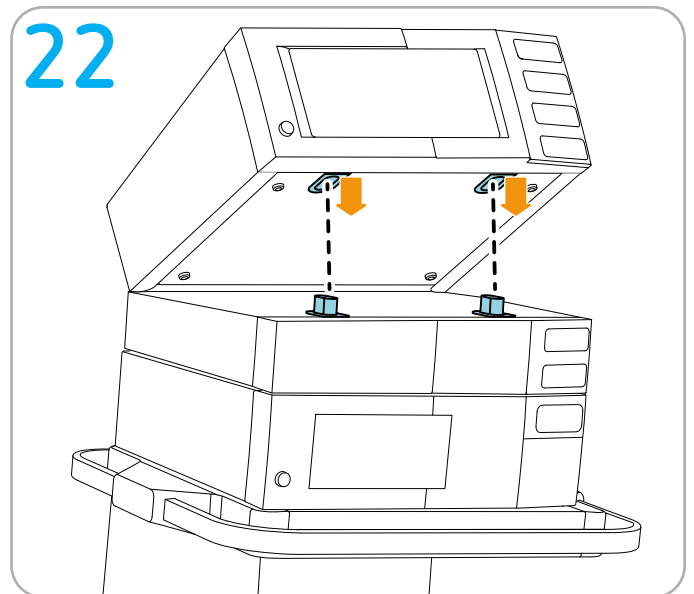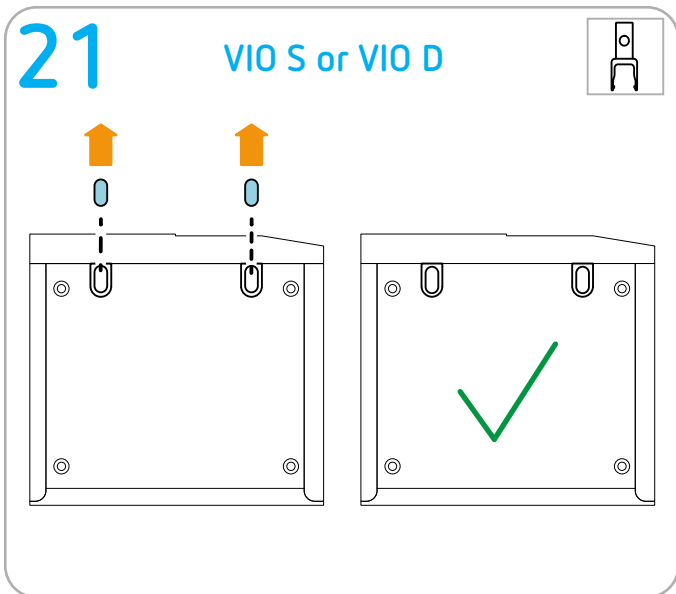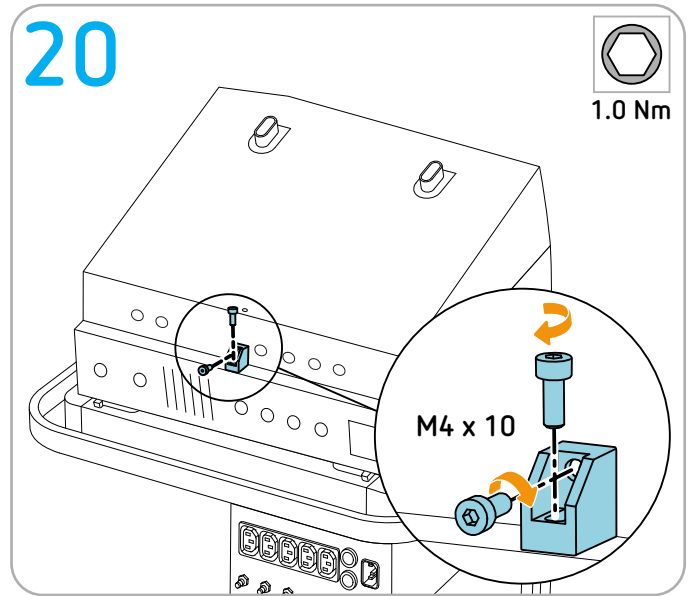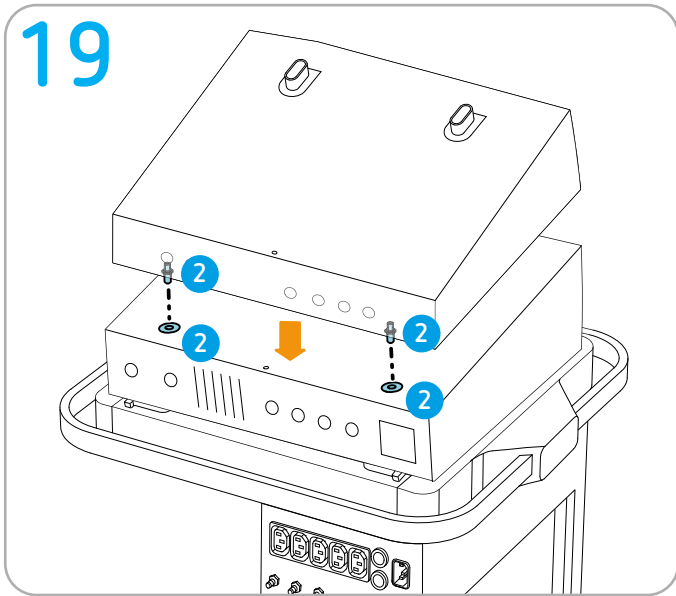
Installing VIO seal + APC 3 + VIO 3 series

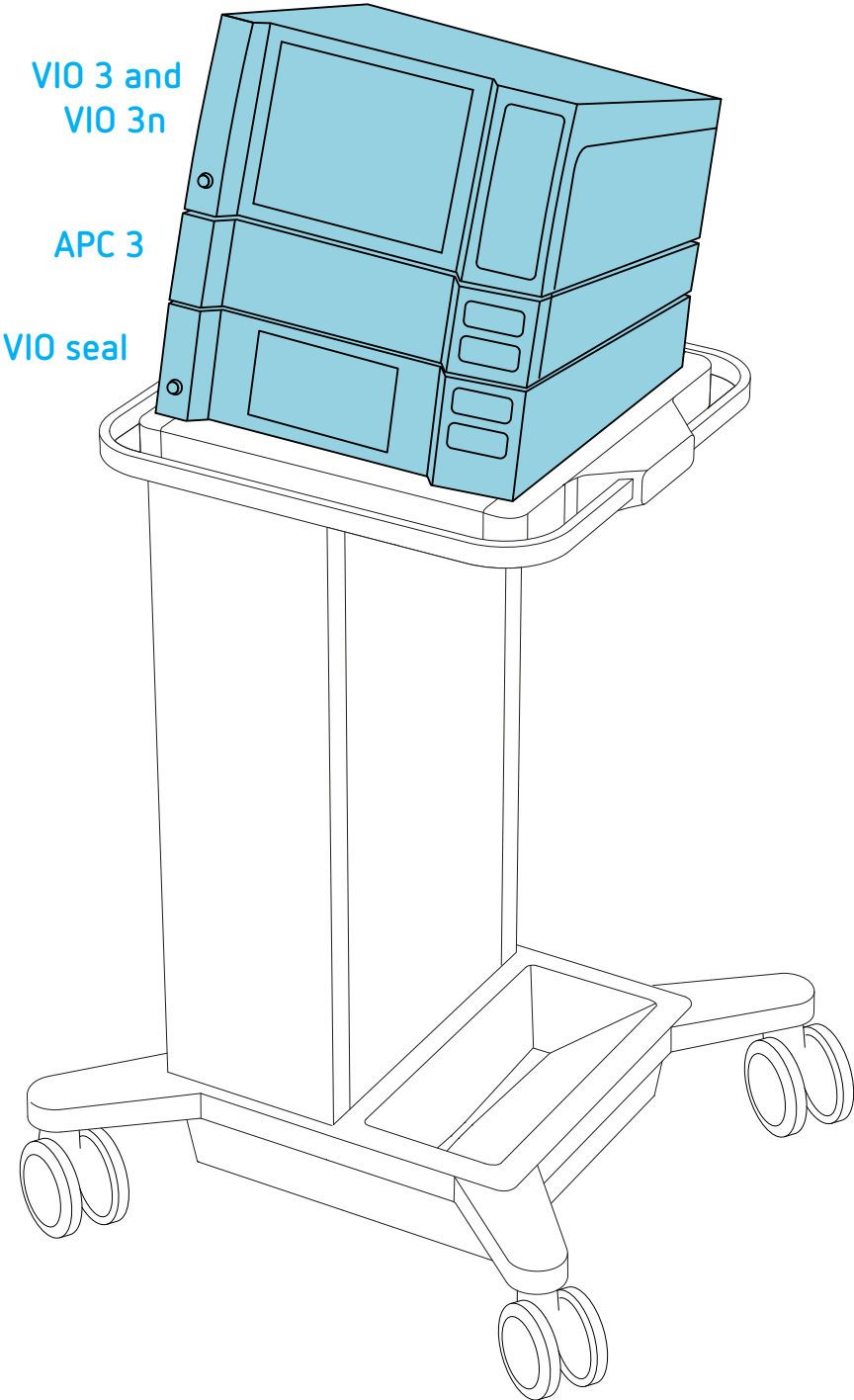Permitted positioning



Erbe fastening set


IMPORTANT! Only use original Erbe accessories. For this assembly sequence you will need the following Erbe fastening sets:

- 1 x Erbe fastening set 20180-150 VIO seal on system carrier
- 1 x Erbe fastening set 20180-150 APC 3 on VIO seal
- 1 x 30135-030 + 40160-065 VIO 3 and VIO 3n series on APC 3

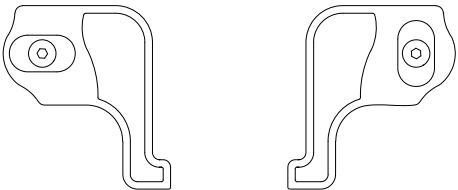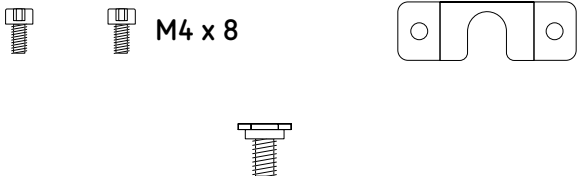Erbe REF 20180-150


M4 x 16 screws included

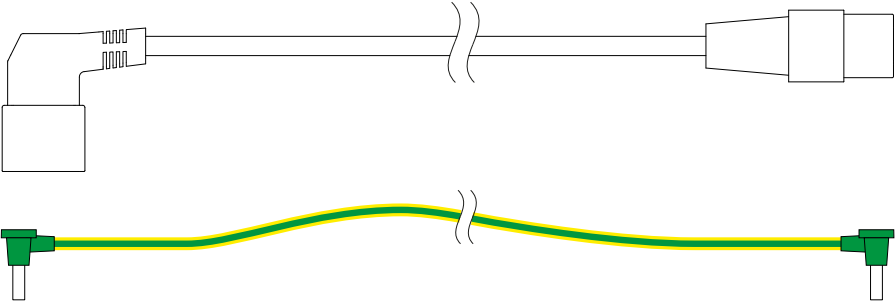Keep for later use

Keep for later use

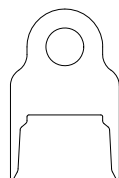Fastening set for VIO on APC: Erbe REF 30135-030 Bracket: Erbe REF 40160-065


M4 x 8

M4 x 16

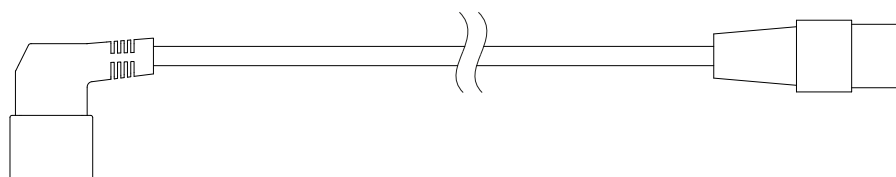Keep for later use

Keep for later use

Tools required

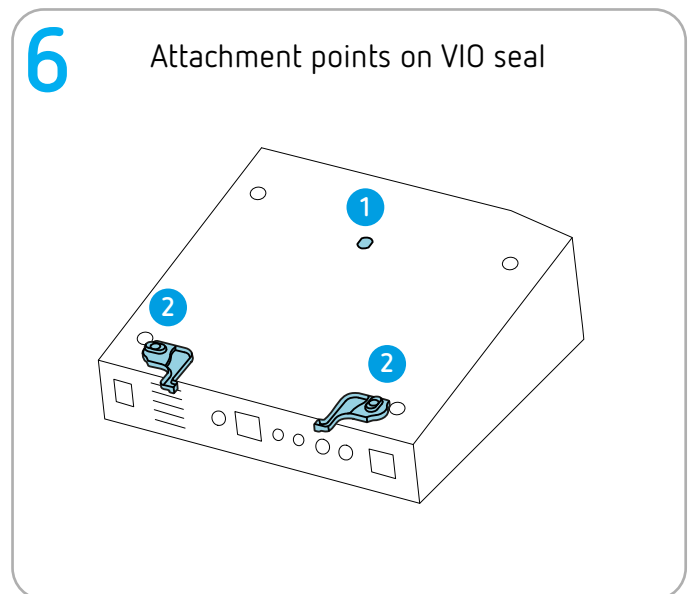
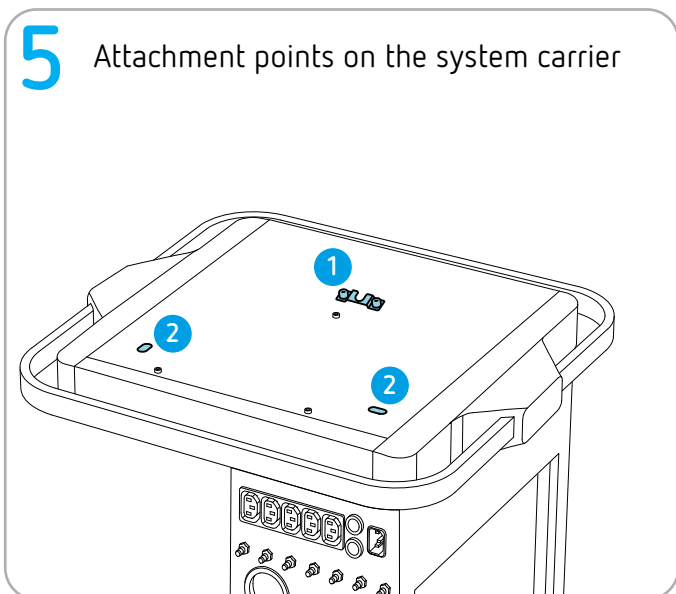
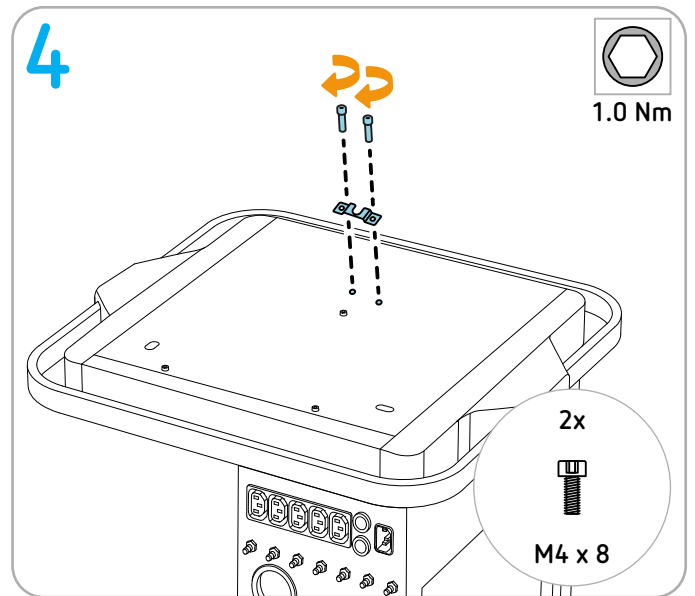
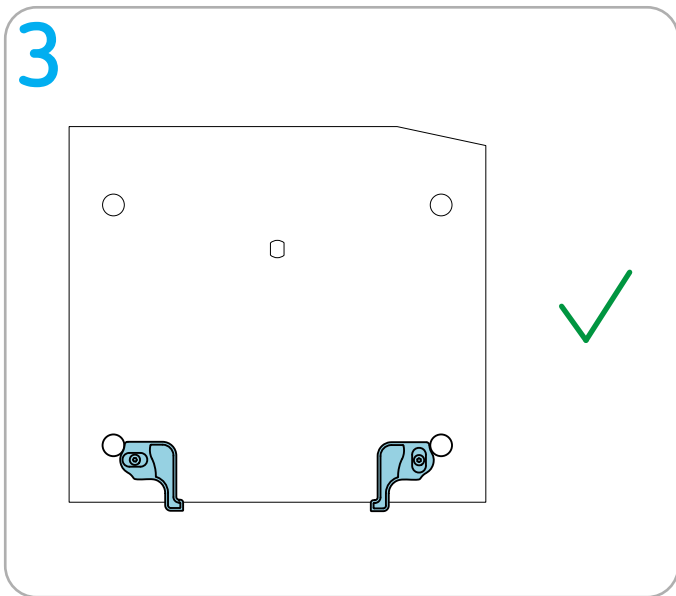
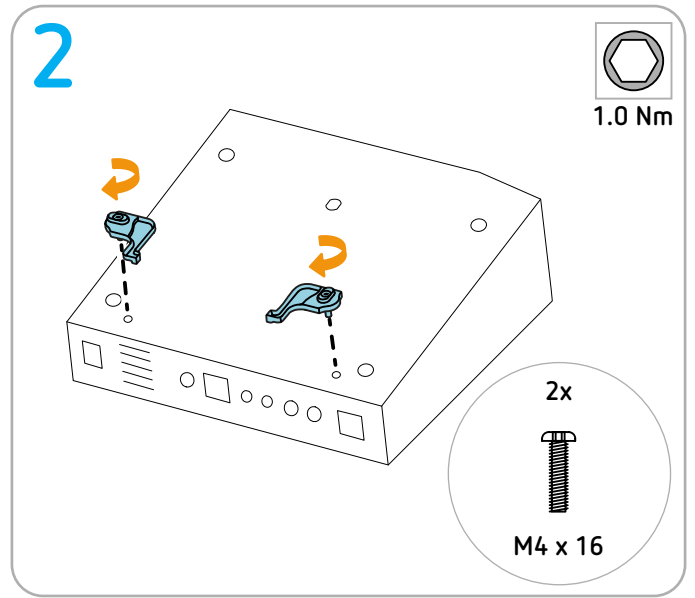
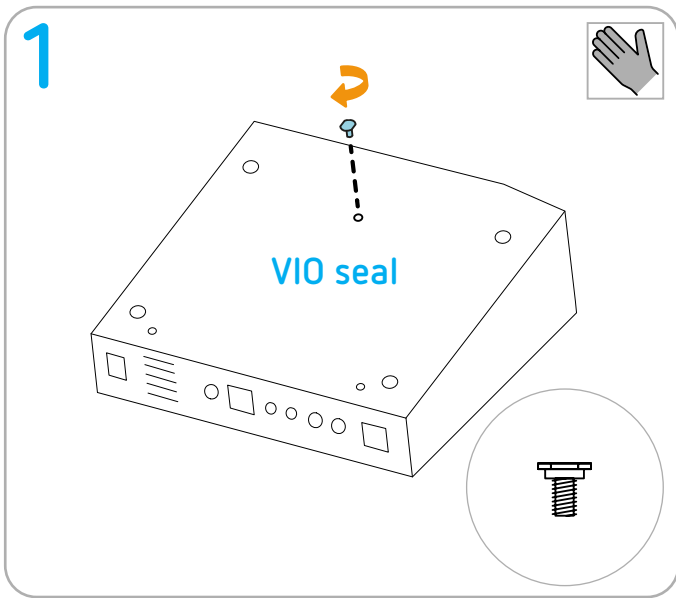
 Torque screwdriver (1 Nm) hex socket 2.5

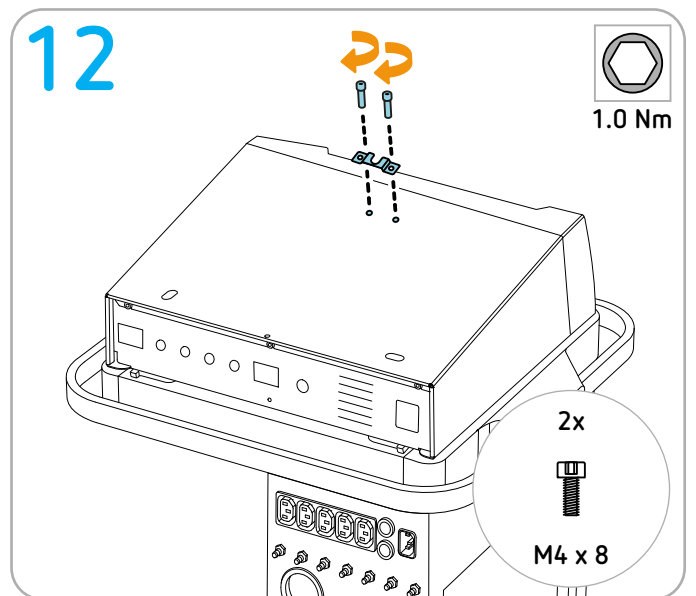
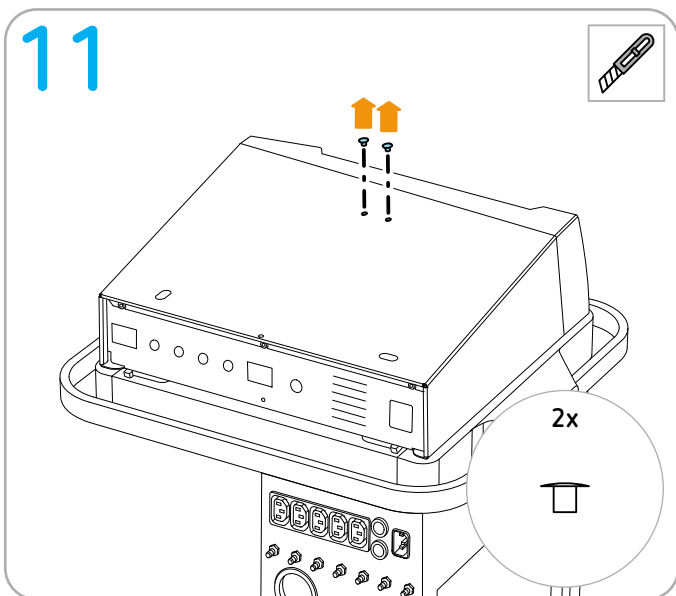
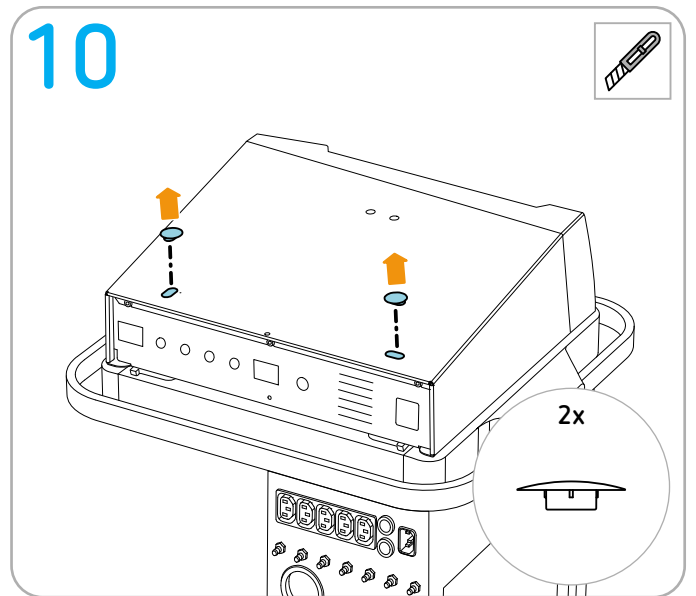
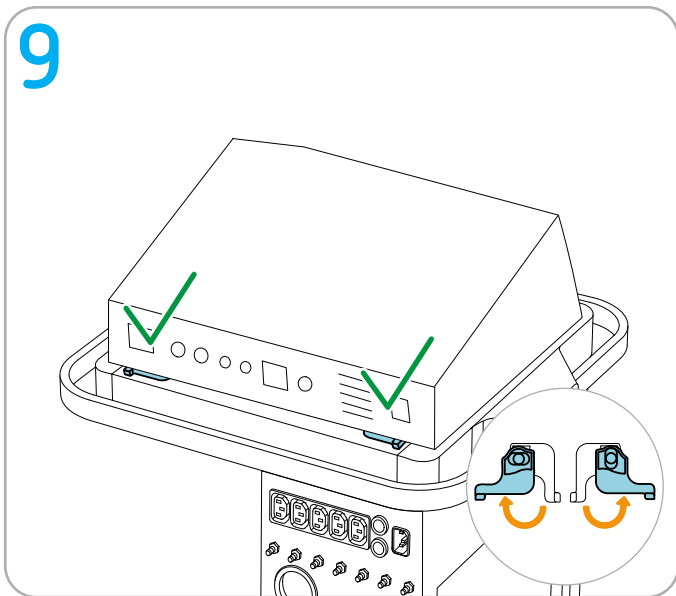
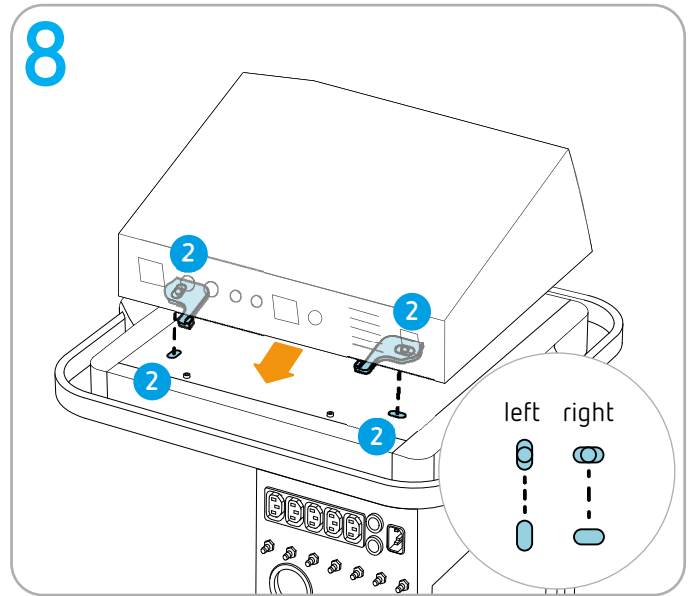
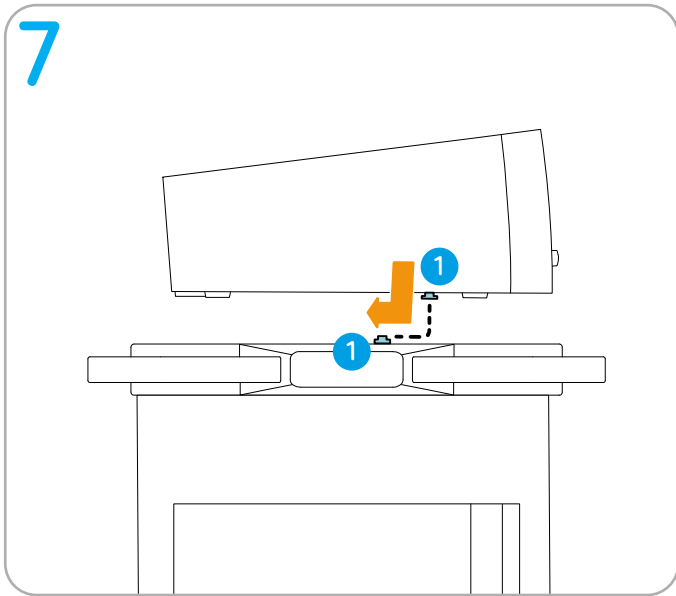
 Torque screwdriver (1 Nm) hex socket 3

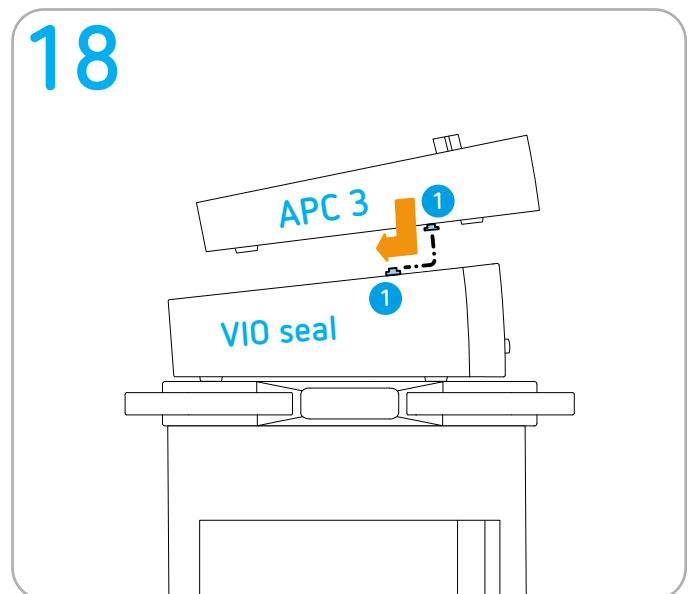
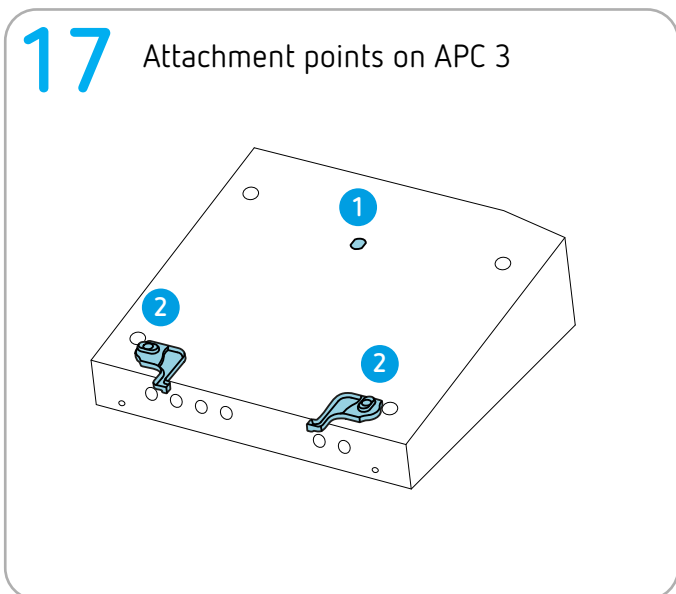
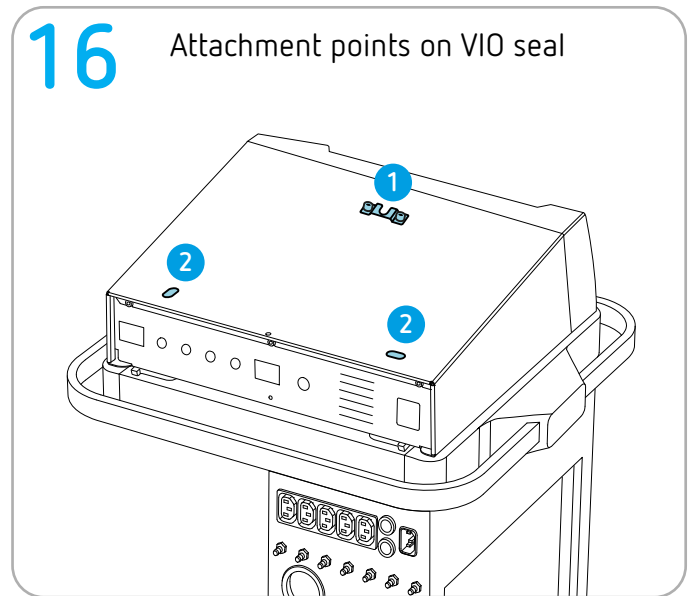
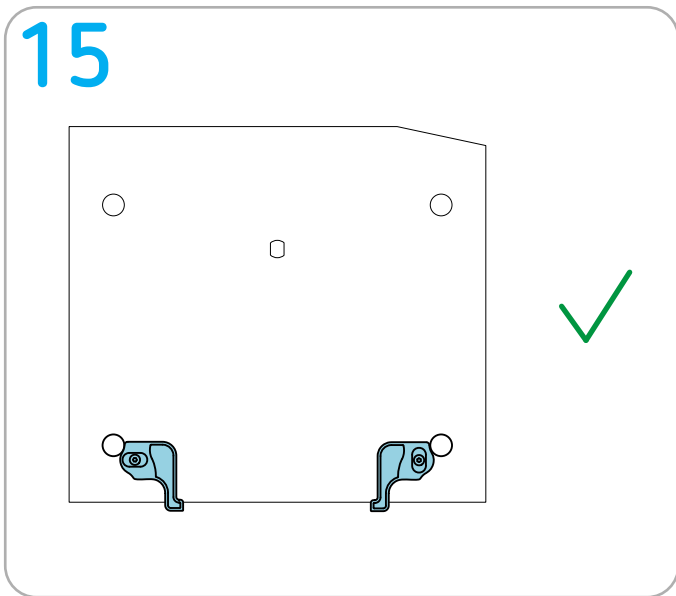
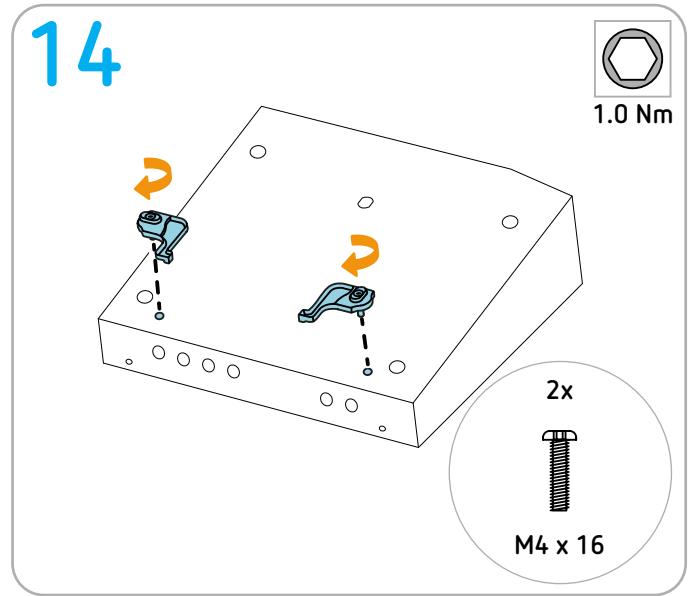
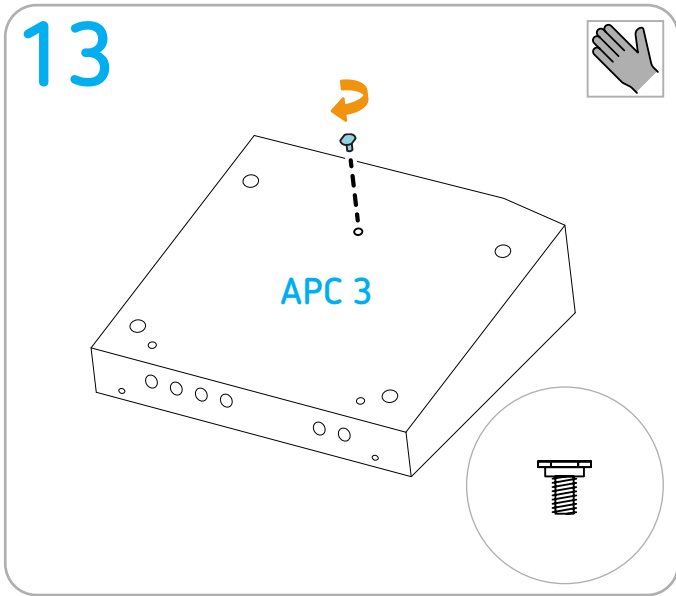
 Cutter

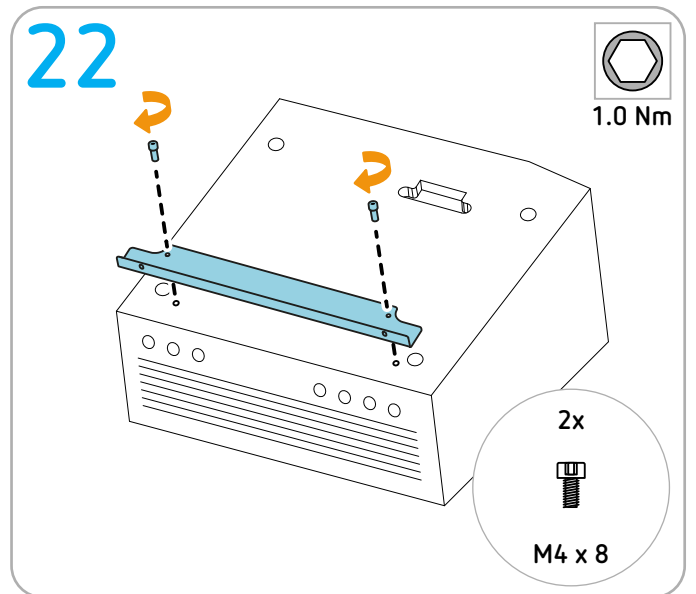
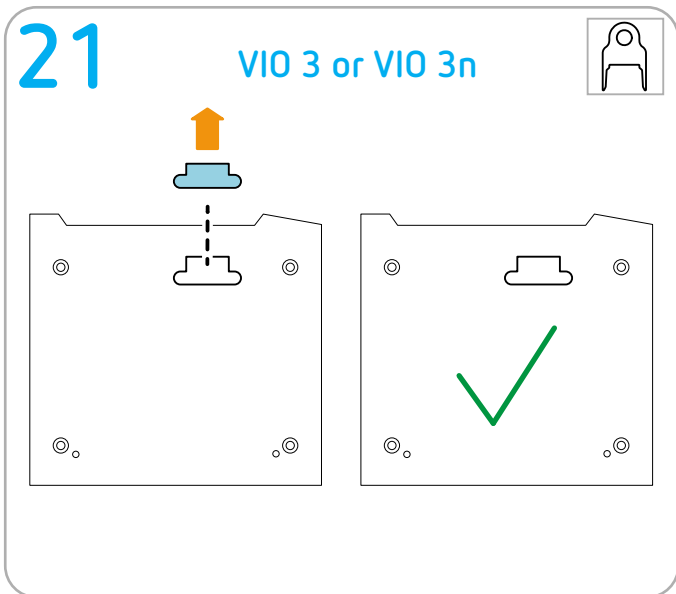
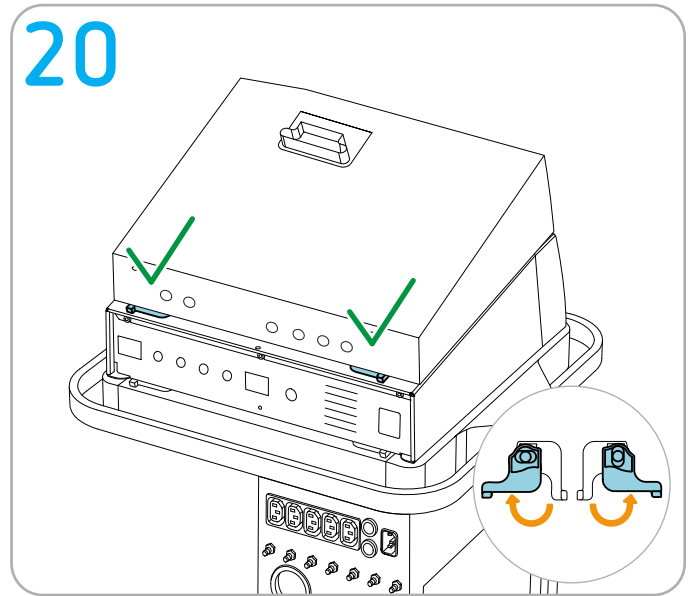
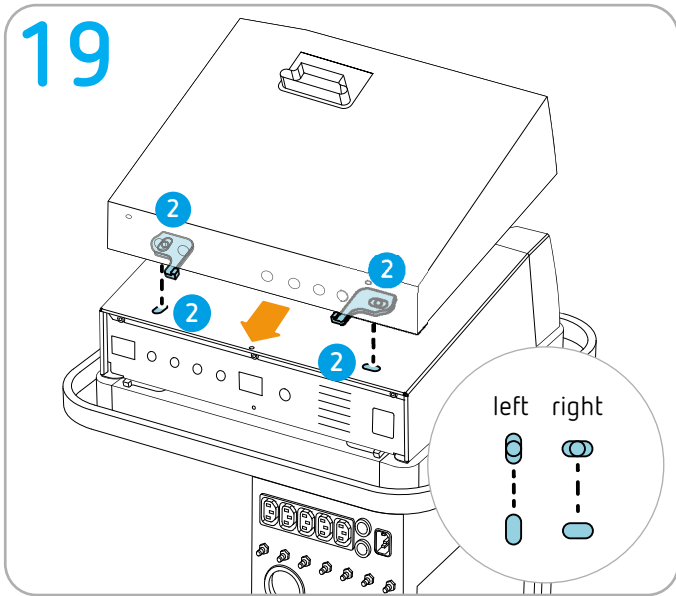
Preparation

IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.






Continue to next page

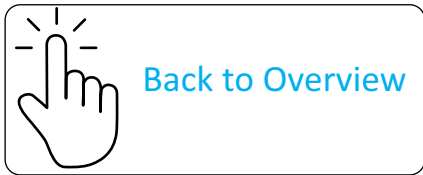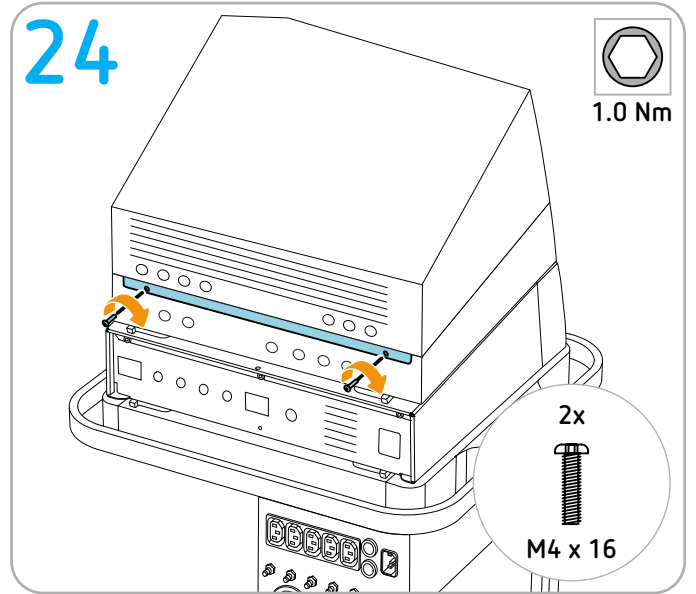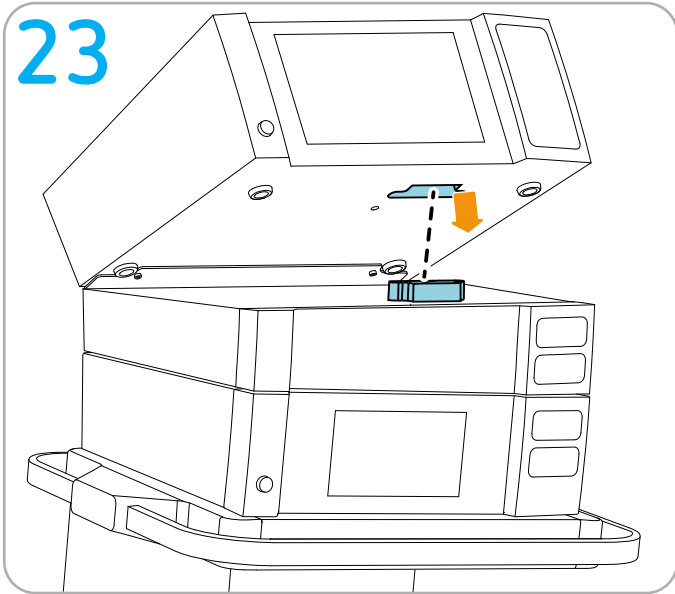Installing ERBEJET 2 or ERBECRYO 2 + APC 3 + VIO 3

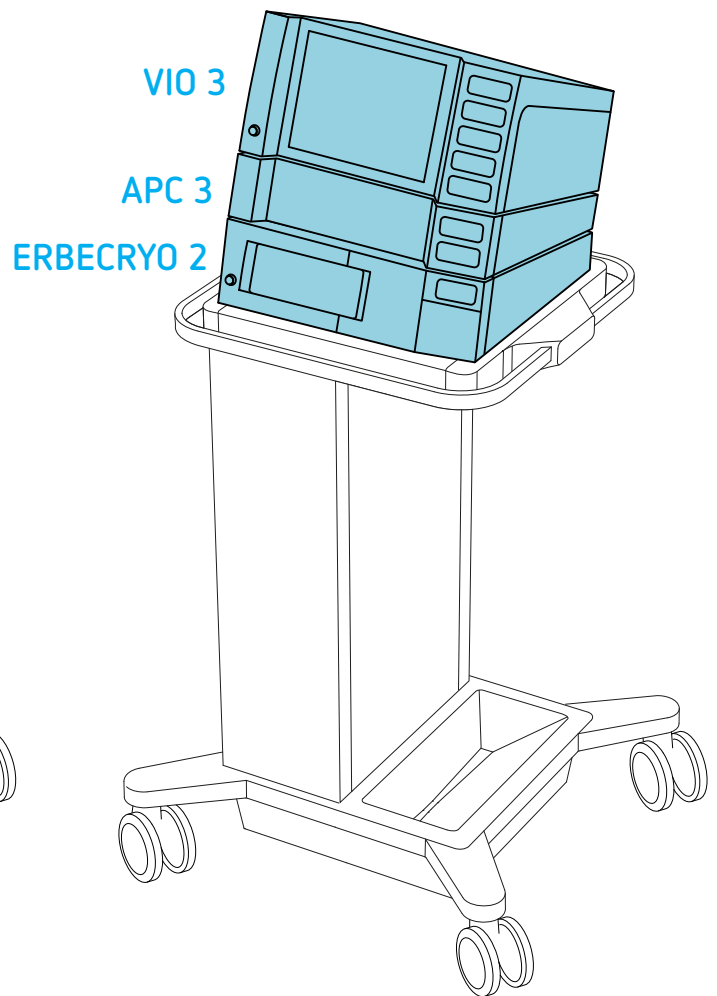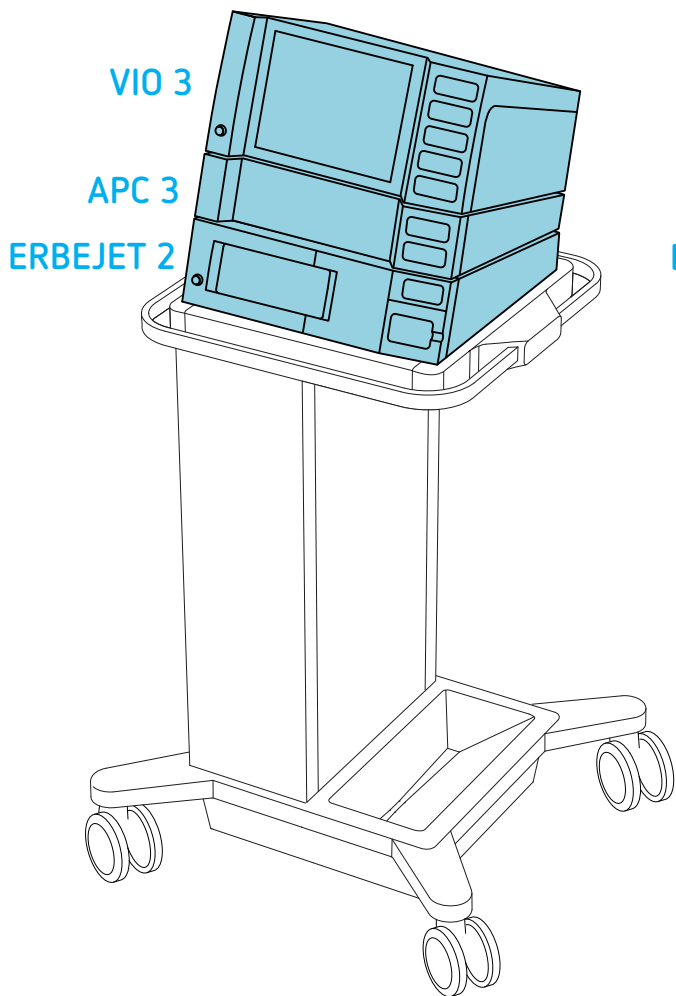Permitted positioning



Erbe fastening set

IMPORTANT! Only use original Erbe accessories.


Erbe REF 20180-150

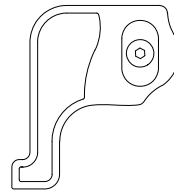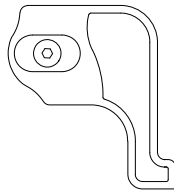M4 x 8

M4 x 16 screws included


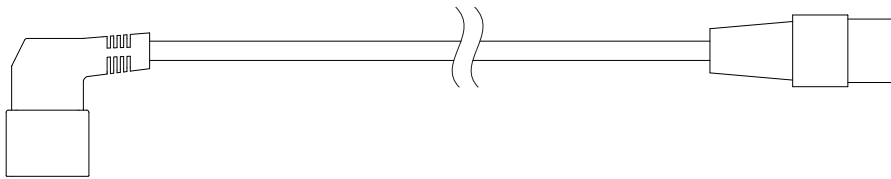
Keep for later use


Keep for later use

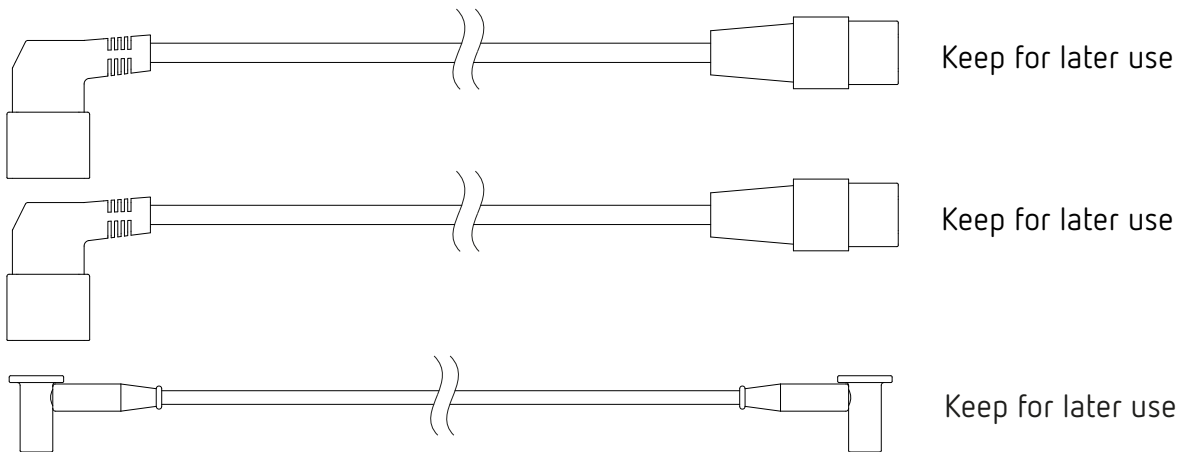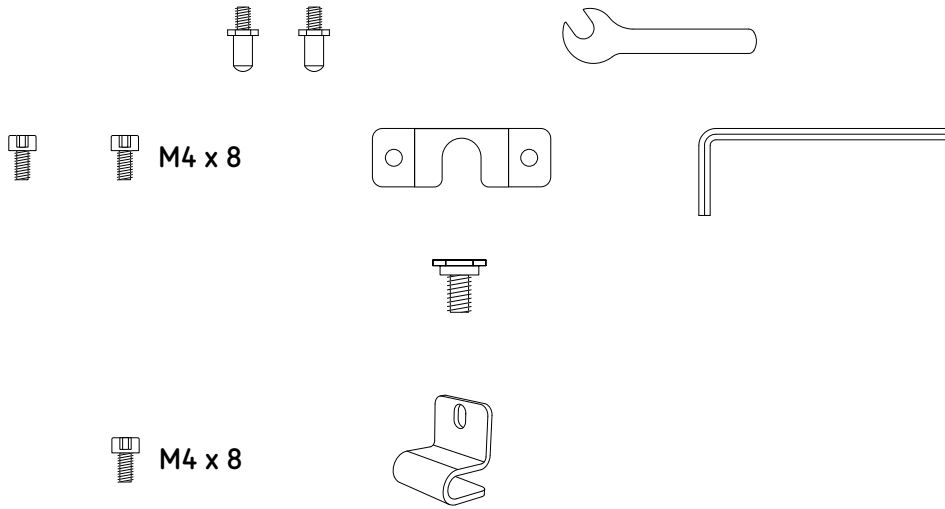Erbe REF 20180-144

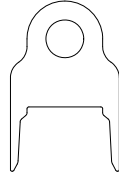Fastening set for VIO on APC: Erbe REF 30135-030
Bracket: Erbe REF 40160-065


M4 x 8

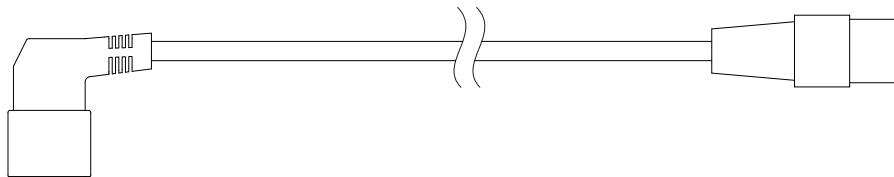M4 x 16

Keep for later use

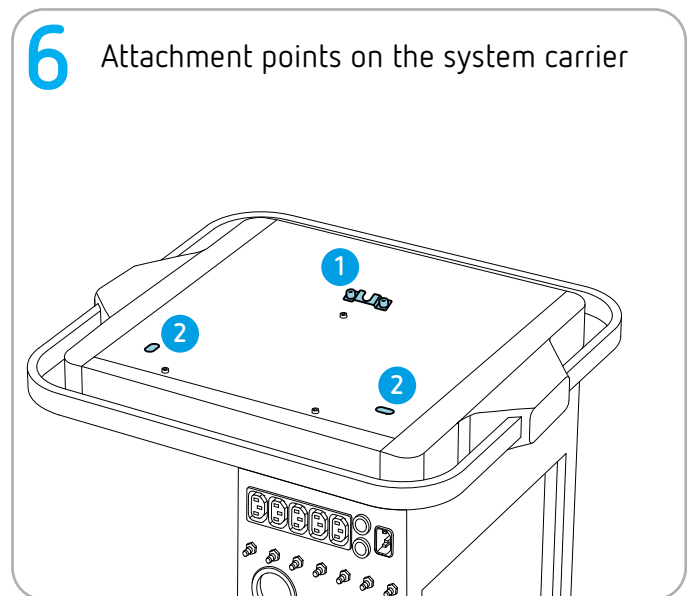
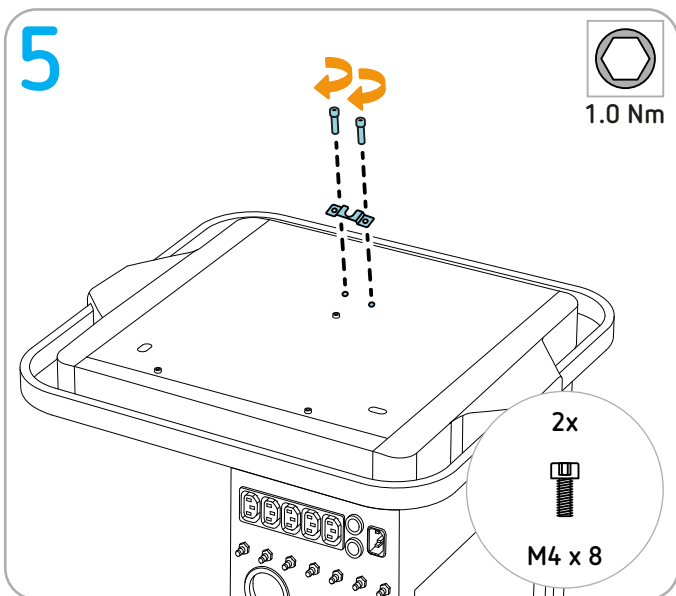
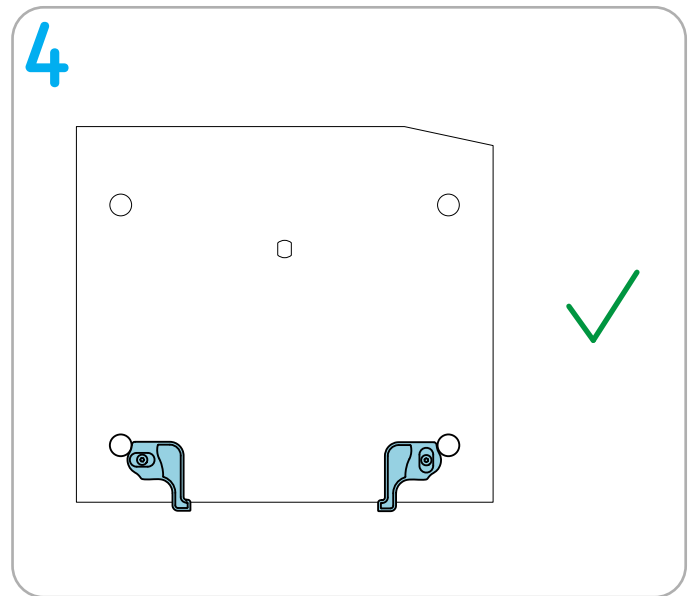
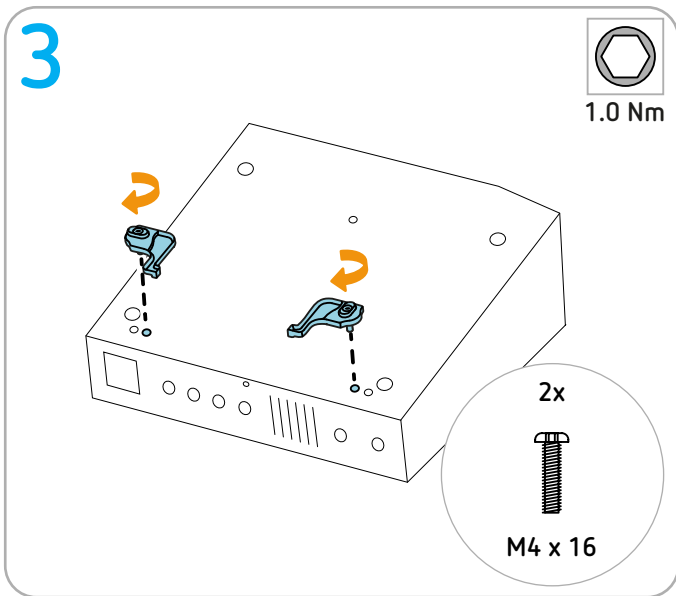
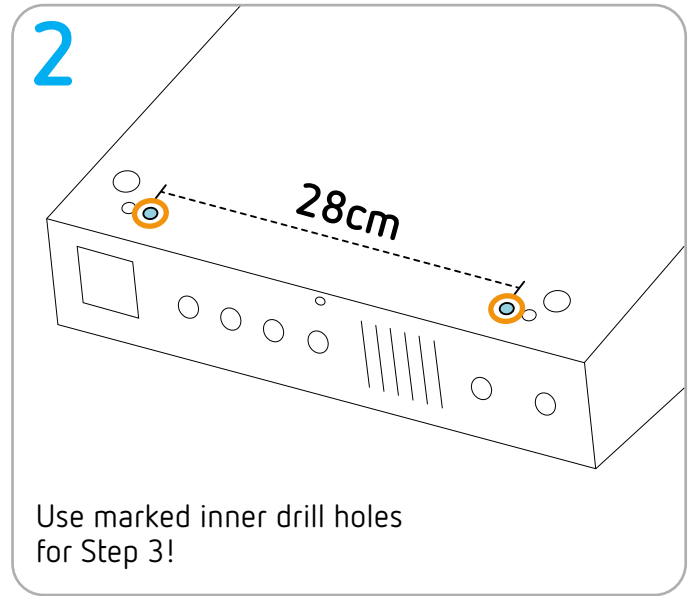
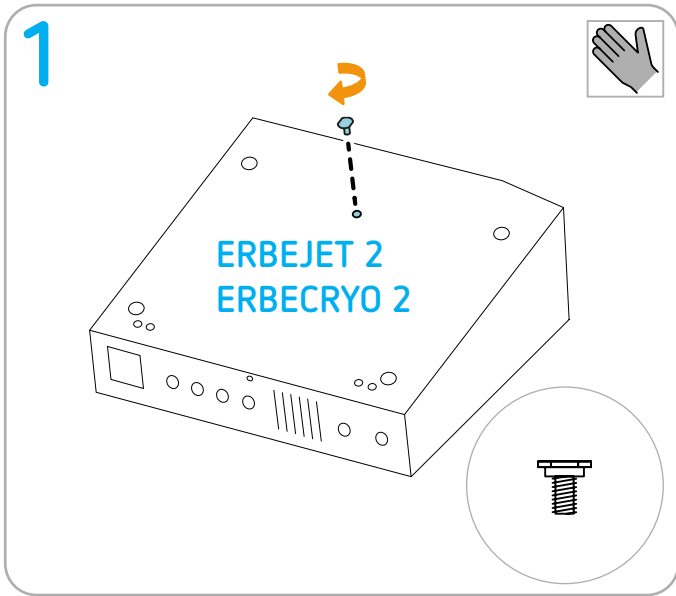
Keep for later use

Tools required


-  Torque screwdriver (1 Nm) hex socket 2.5
-  Torque screwdriver (1 Nm) hex socket 3
-  Torque screwdriver (1 Nm) open-end wrench 7
-  Cutter

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the unit

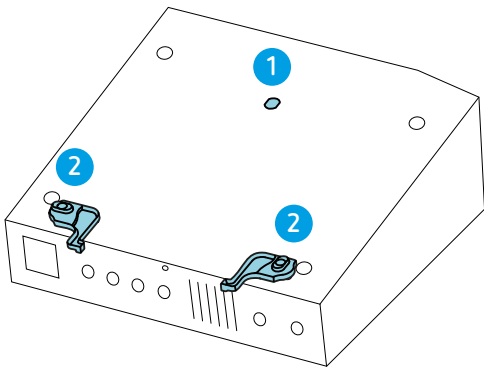
8

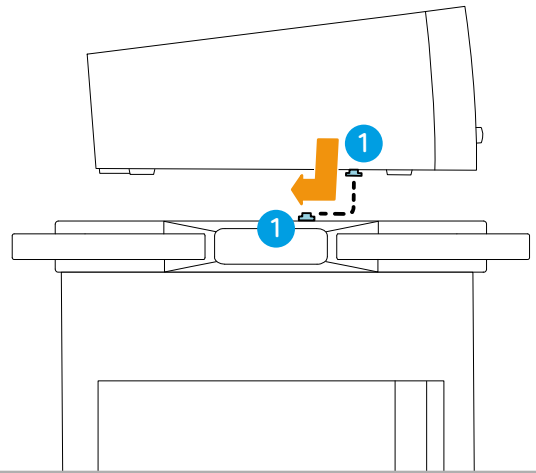
9

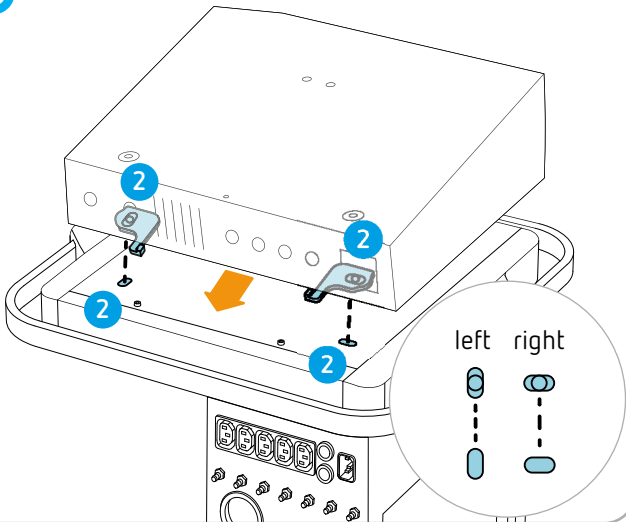
10












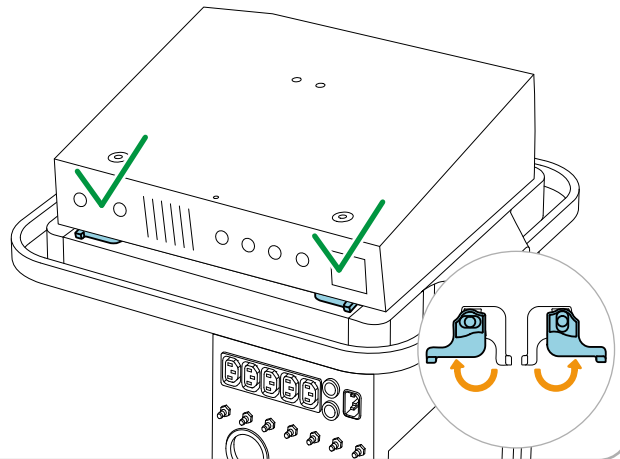
11

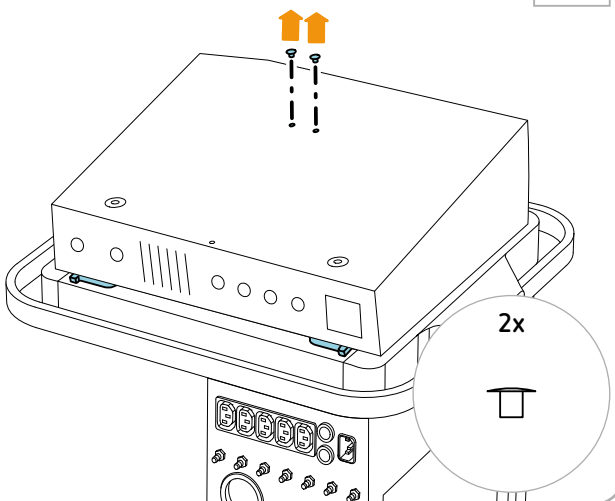12

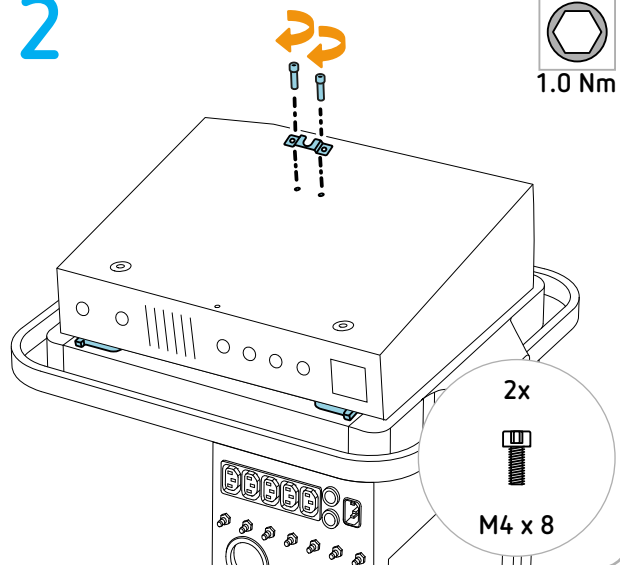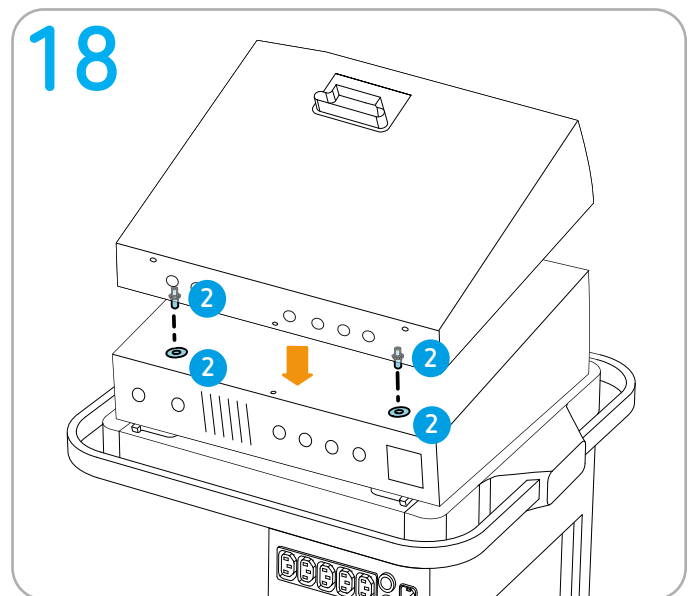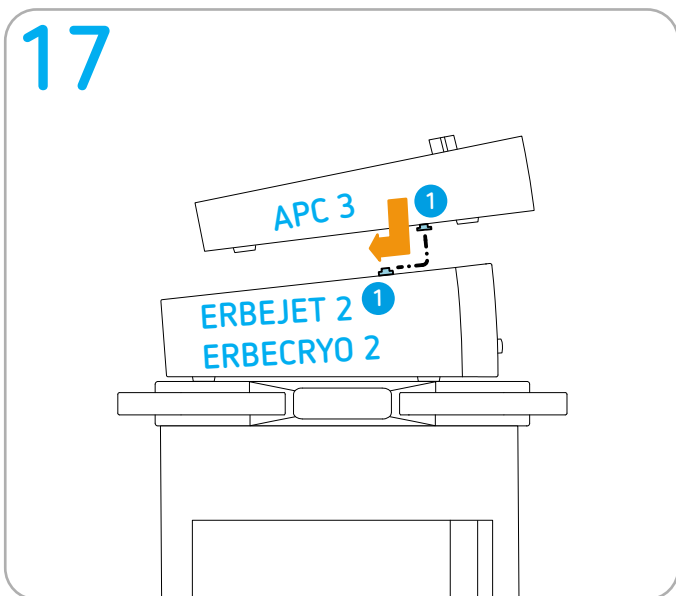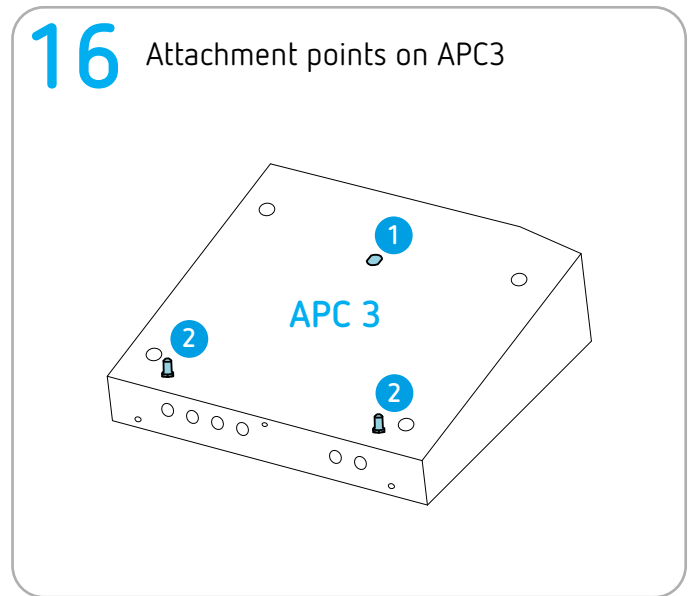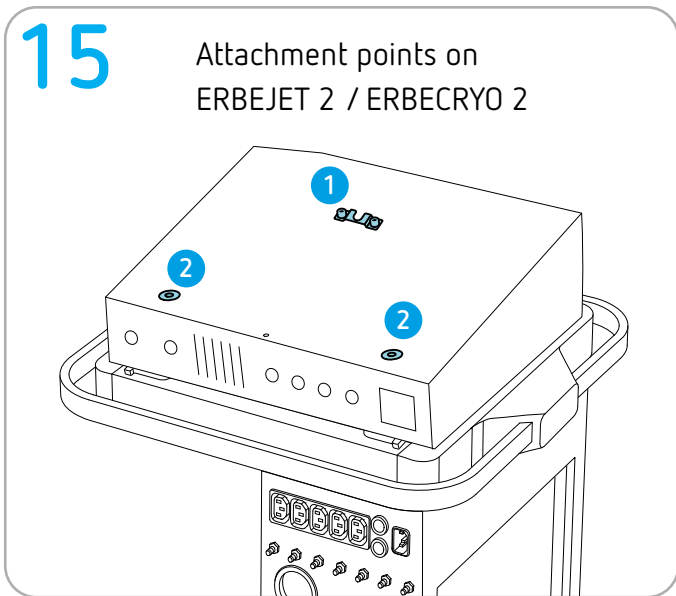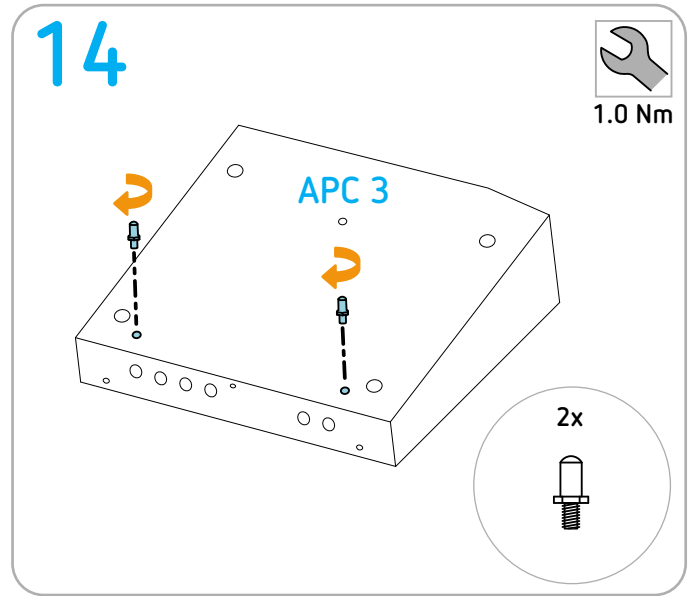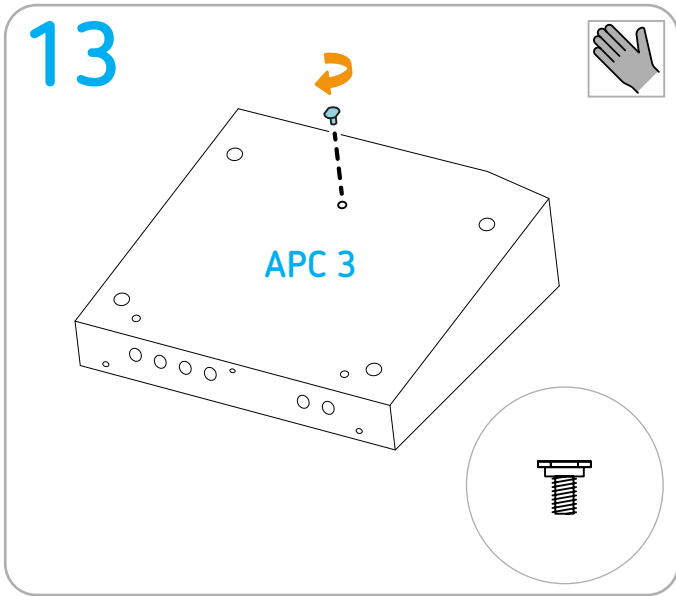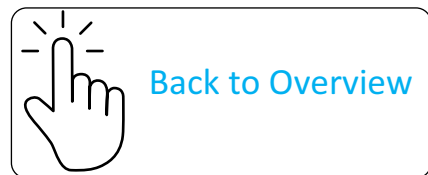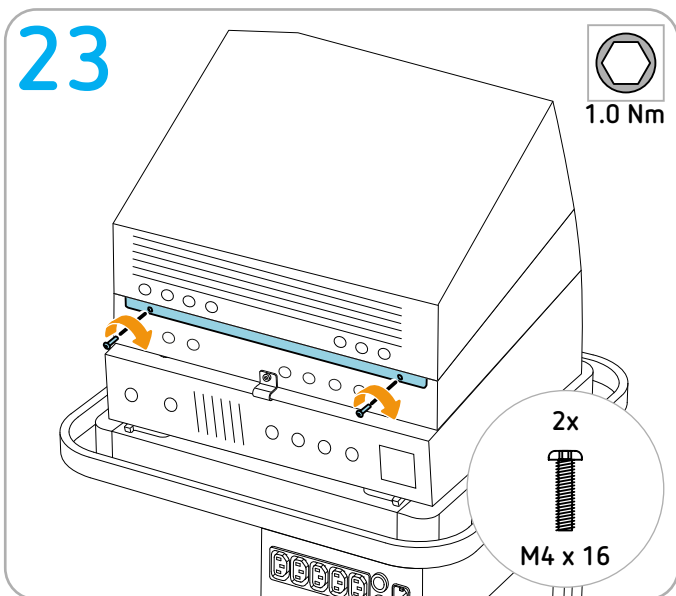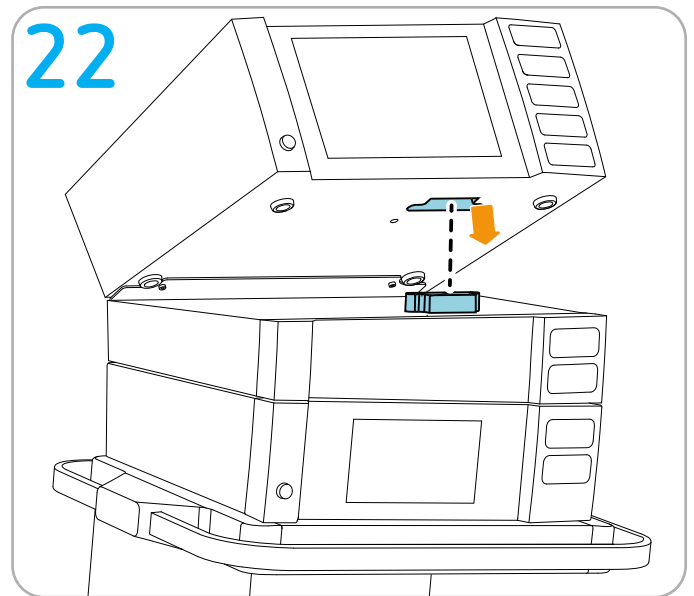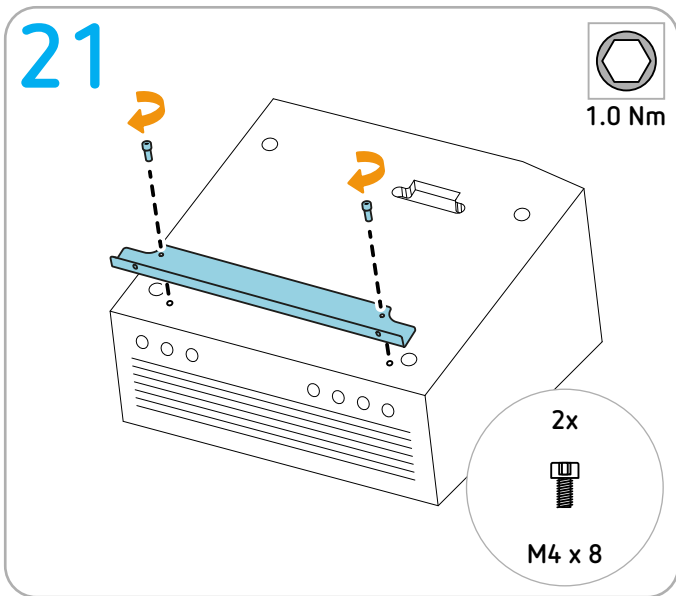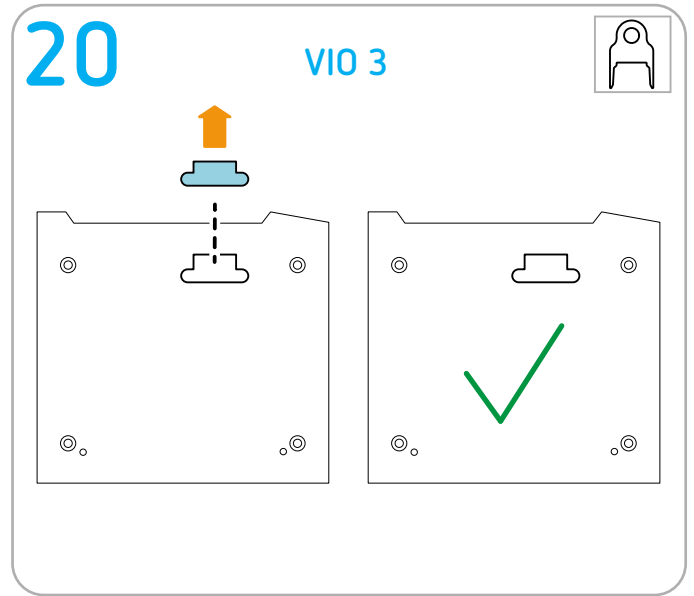
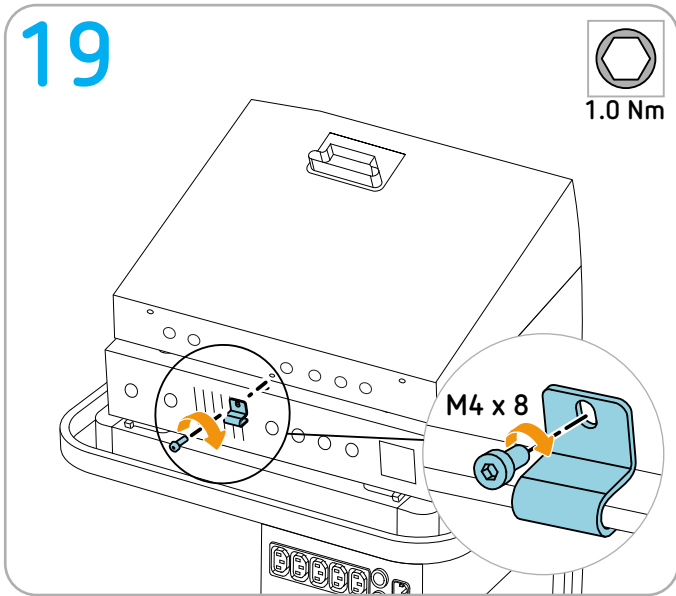
Installing ERBEJET 2 or ERBECRYO 2 + VIO seal + APC 3 + VIO 3n

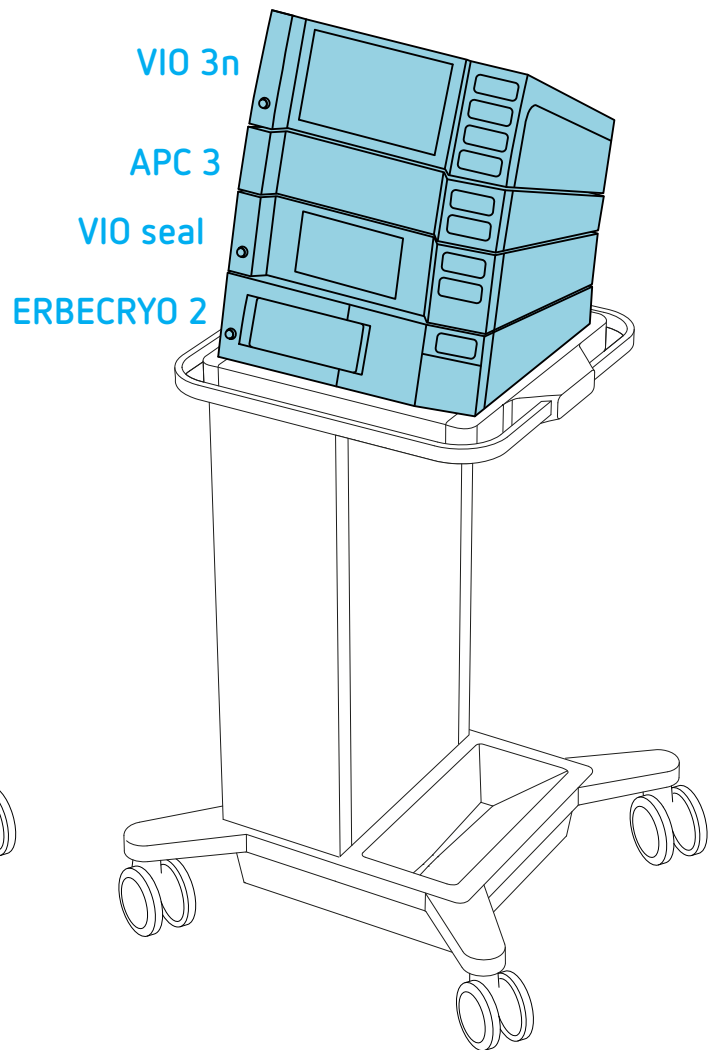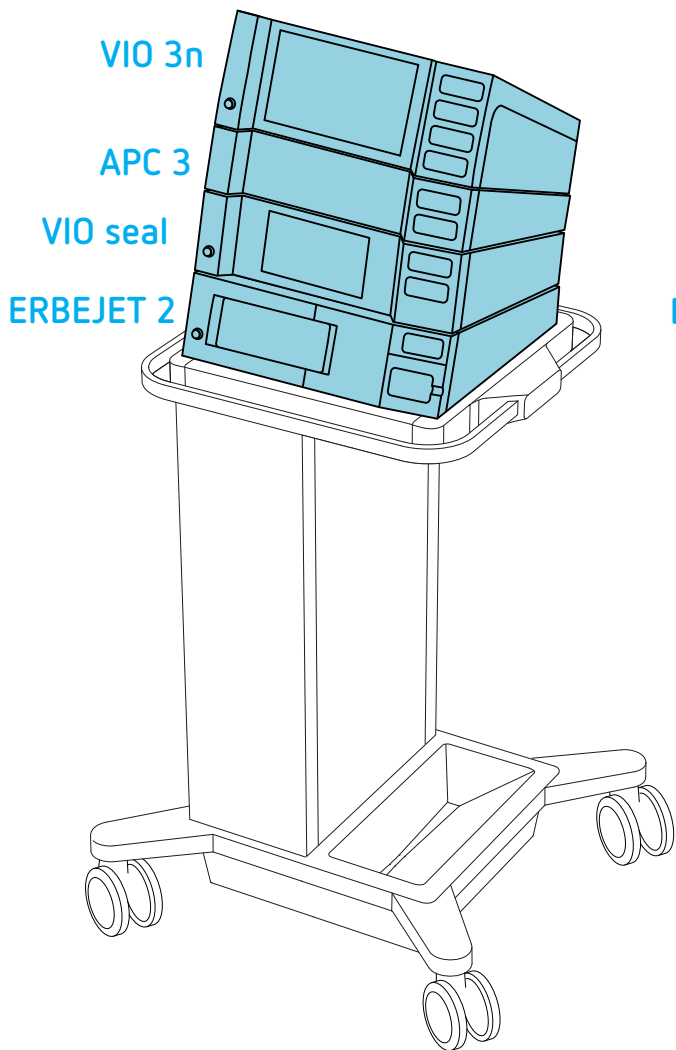Permitted positioning



Erbe fastening set


IMPORTANT! Only use original Erbe accessories. For this assembly sequence you will need the following Erbe fastening sets:

- 1 x Erbe fastening set 20180-150 ERBEJET 2 / ERBECRYO 2 on system carrier
- 1 x Erbe fastening set 20180-144 VIO seal on ERBEJET 2 / ERBECRYO 2
- 1 x Erbe fastening set 20180-150 APC 3 on VIO seal
- 1 x Erbe fastening set 30135-030 + 40160-065 VIO 3n on APC 3

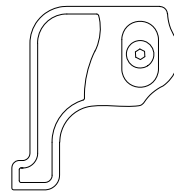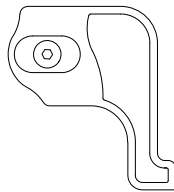Erbe REF 20180-150

M4 x 16 screws included


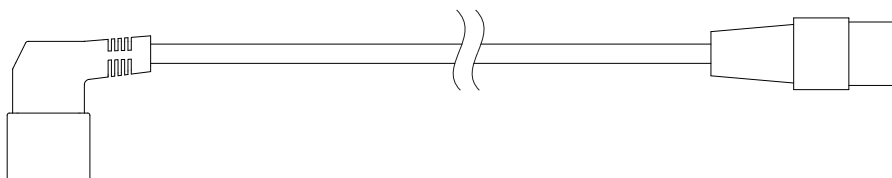
Keep for later use


Keep for later use

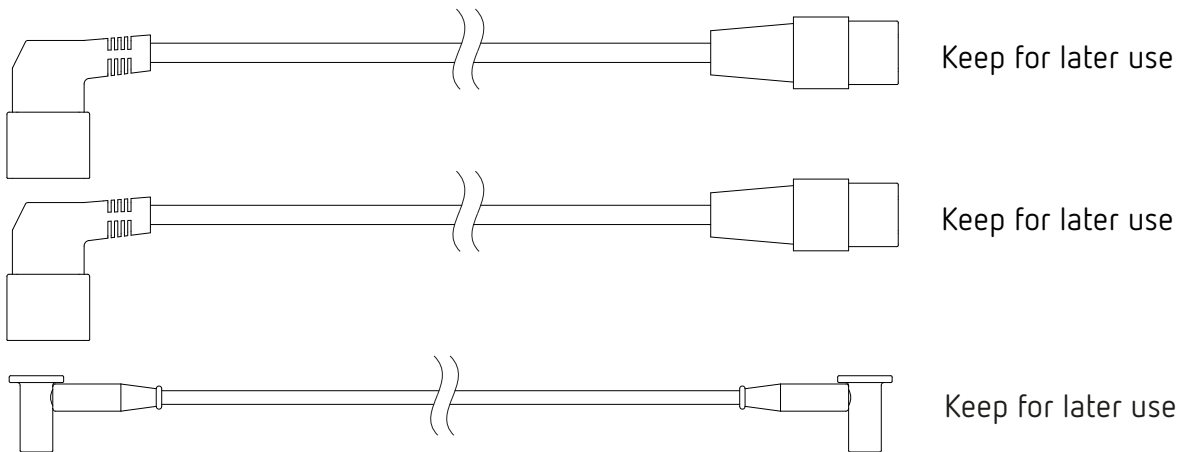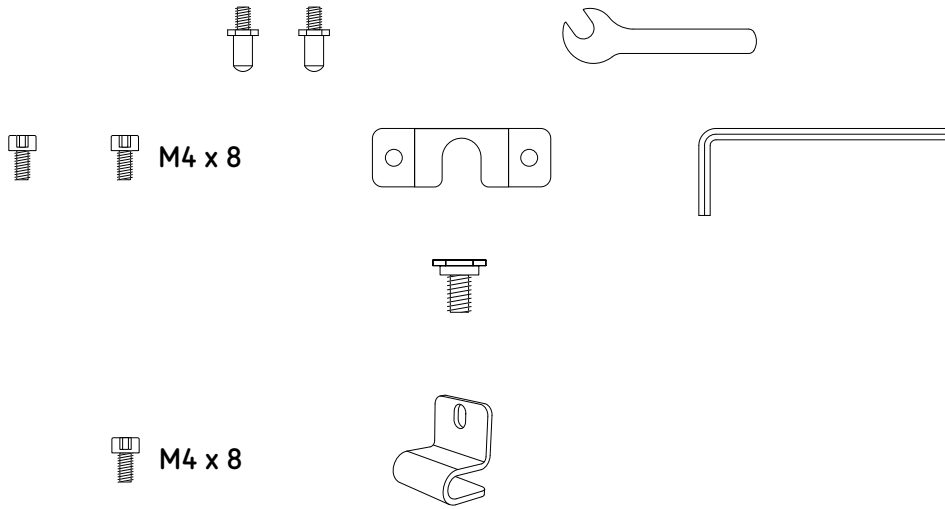Erbe REF 20180-144

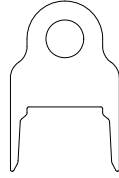Fastening set for VIO on APC: Erbe REF 30135-030
Bracket: Erbe REF 40160-065


M4 x 8

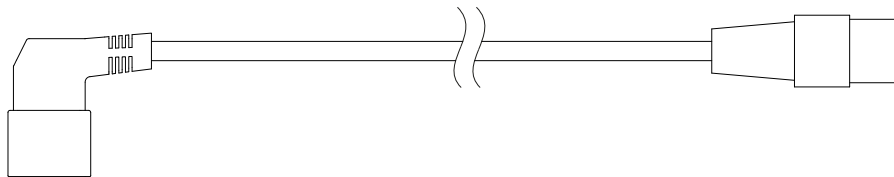
M4 x 16

Keep for later use

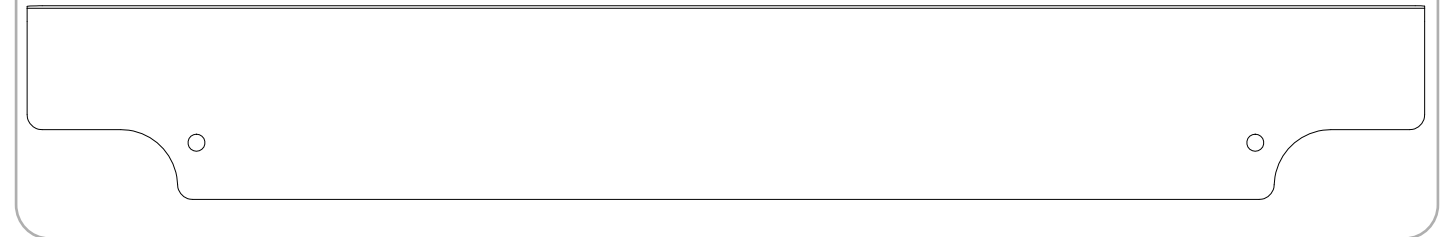Keep for later use

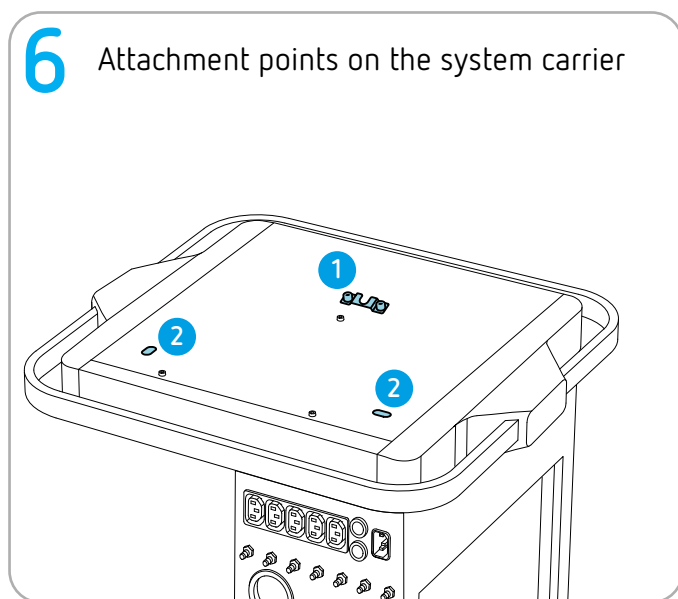
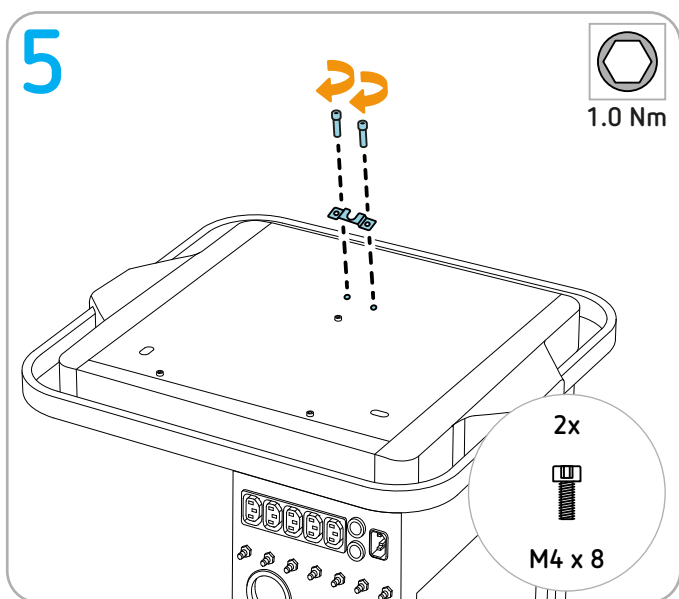
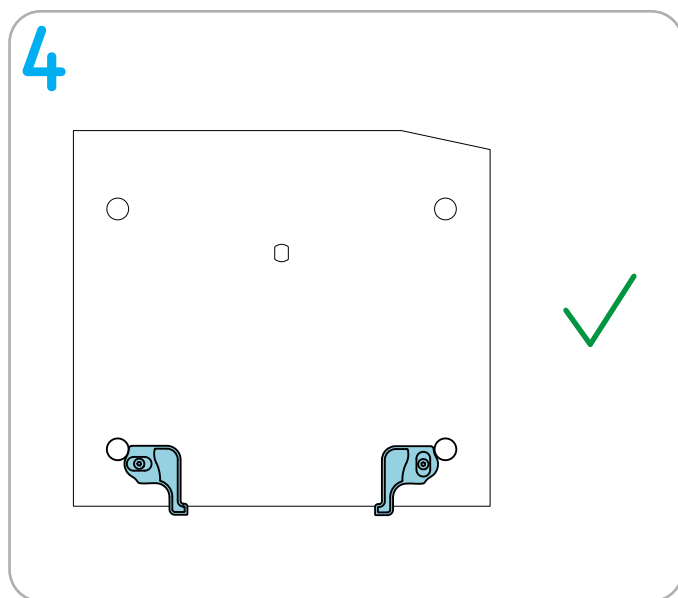
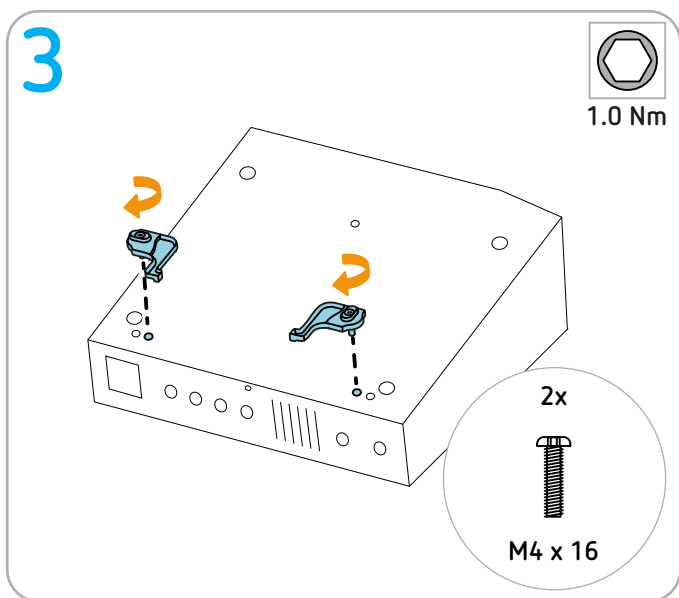
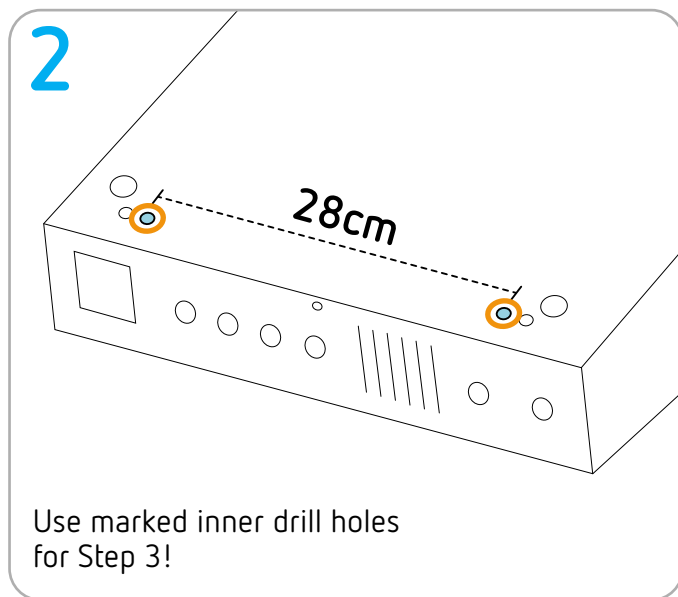
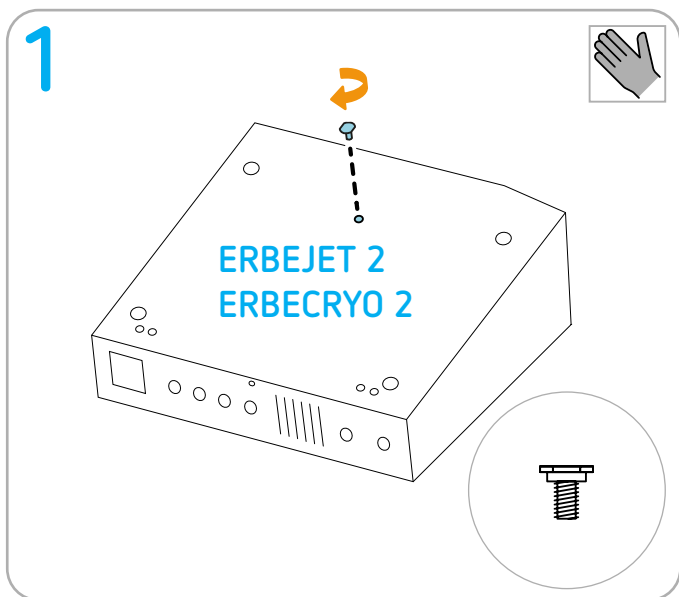
Tools required


-  Torque screwdriver (1 Nm) hex socket 2.5
-  Torque screwdriver (1 Nm) hex socket 3
-  Torque screwdriver (1 Nm) open-end wrench 7
-  Cutter

Preparation


IMPORTANT! Before installing the units on the system carrier, remove all cables, tubing, and instruments.


7 Attachment points on the unit

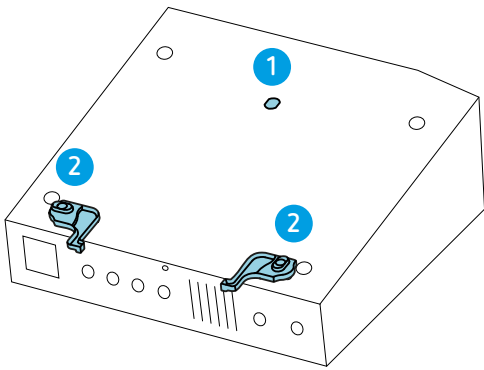
8

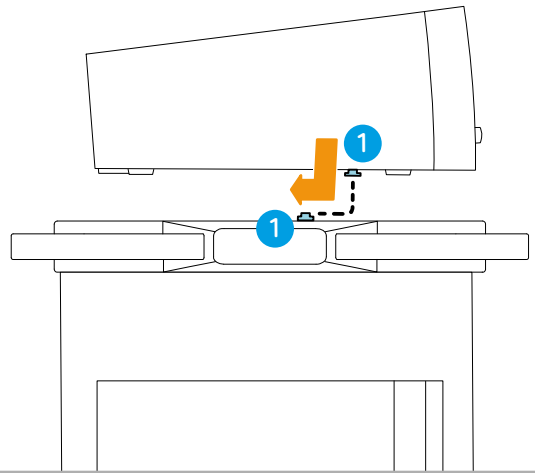
9

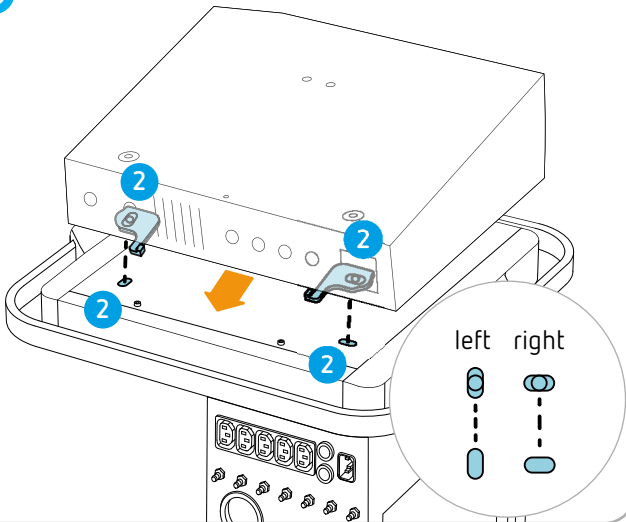
10












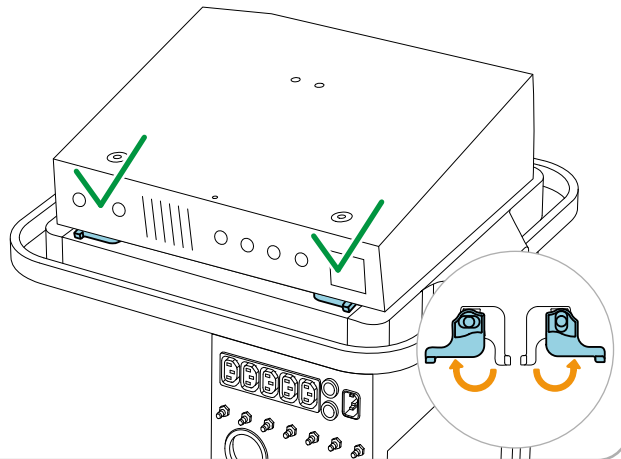
11

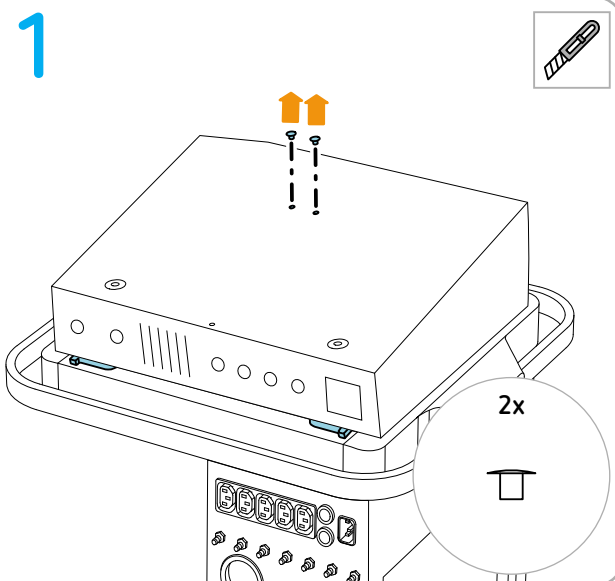
12

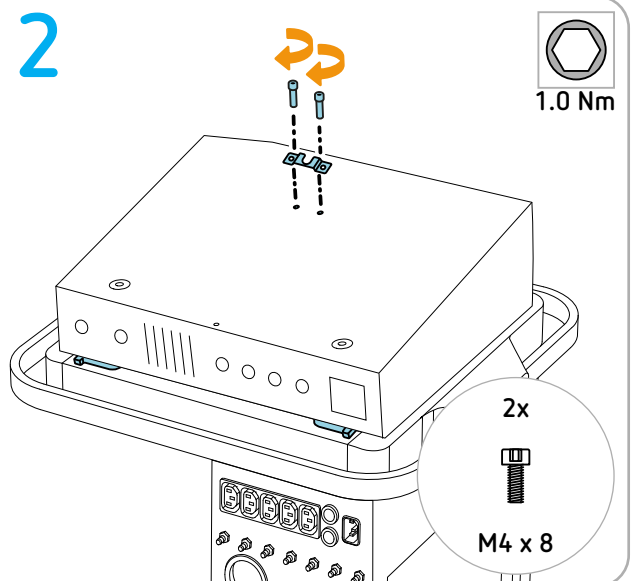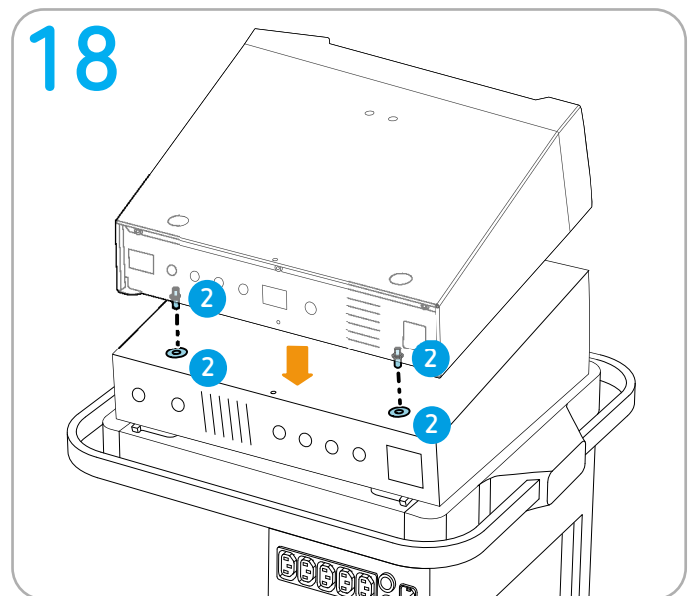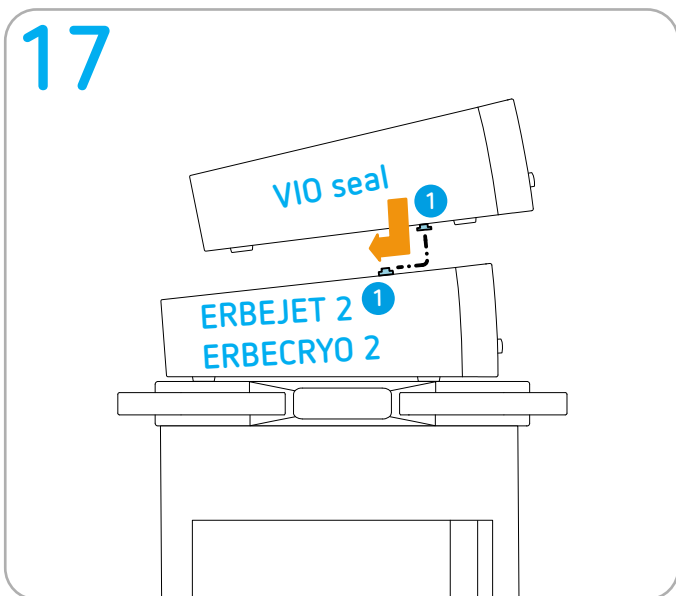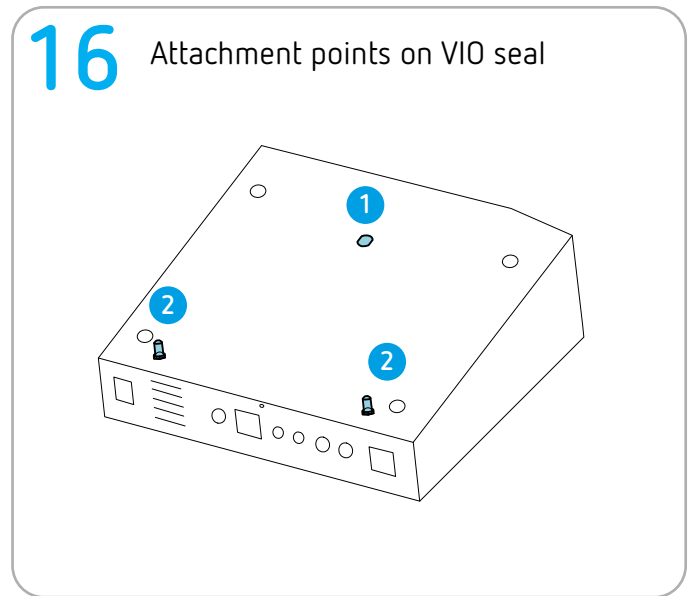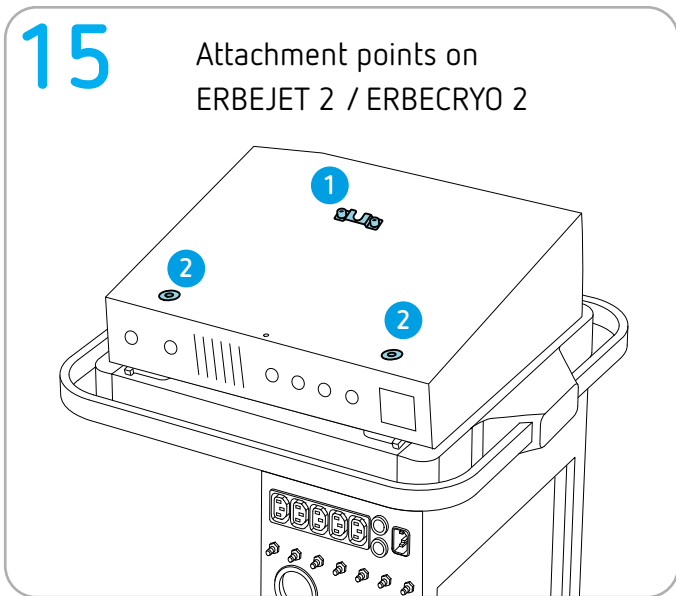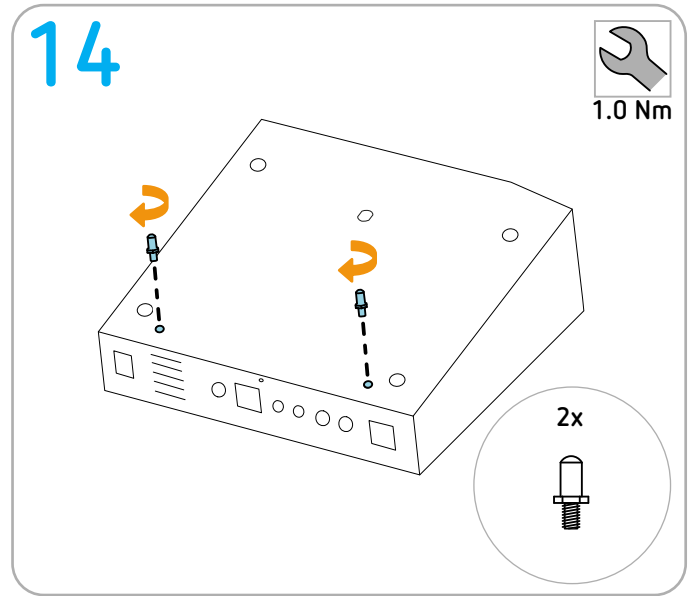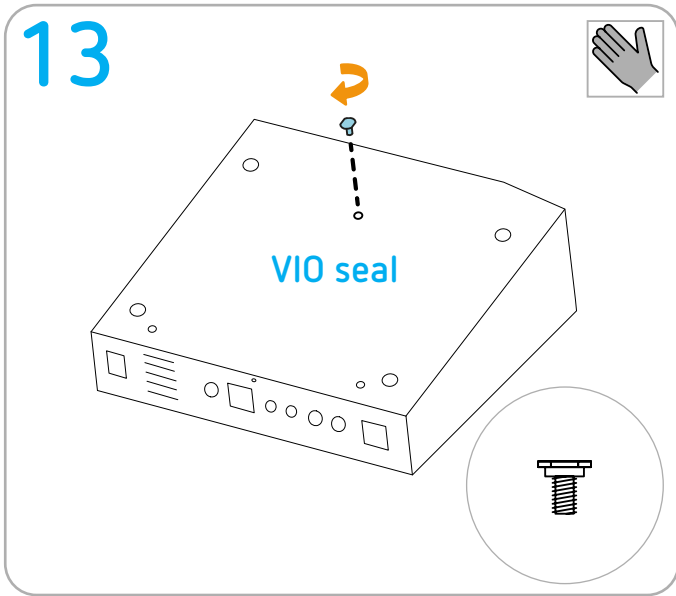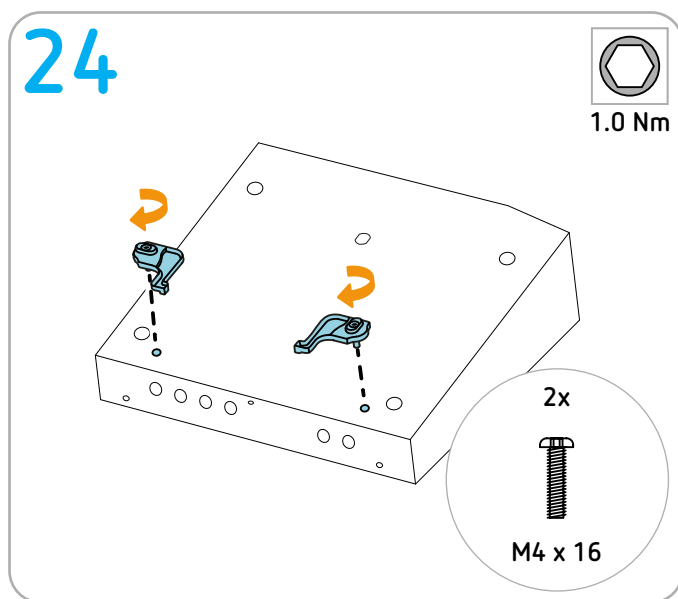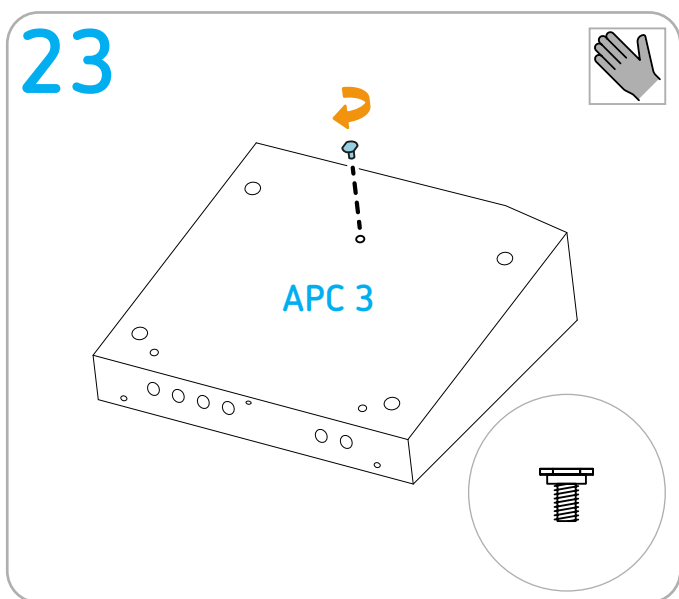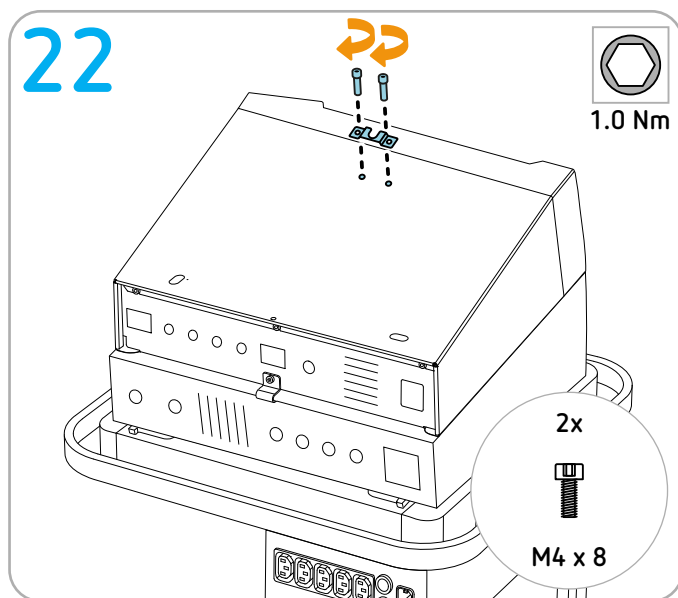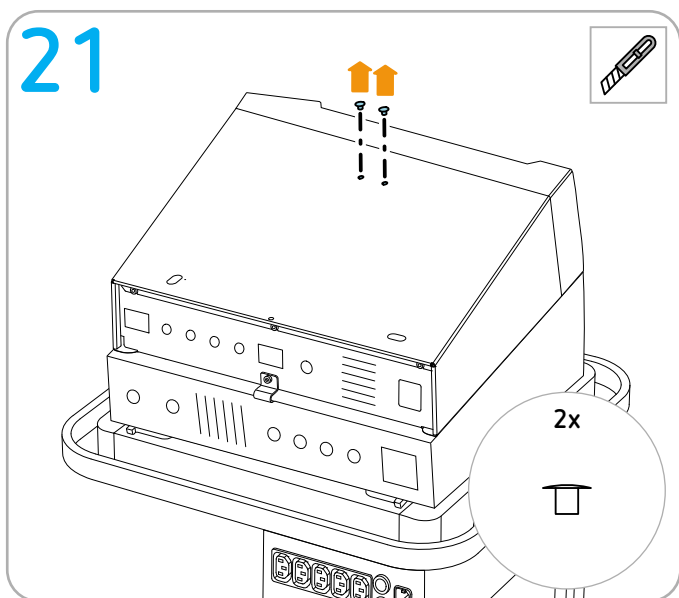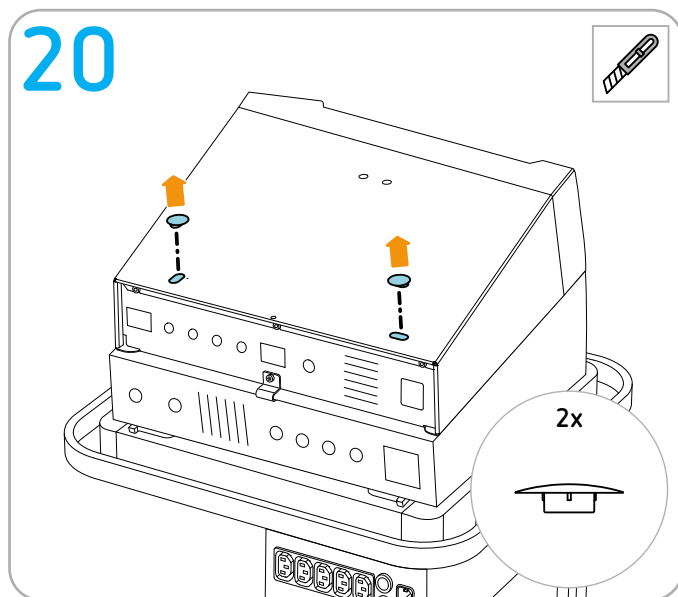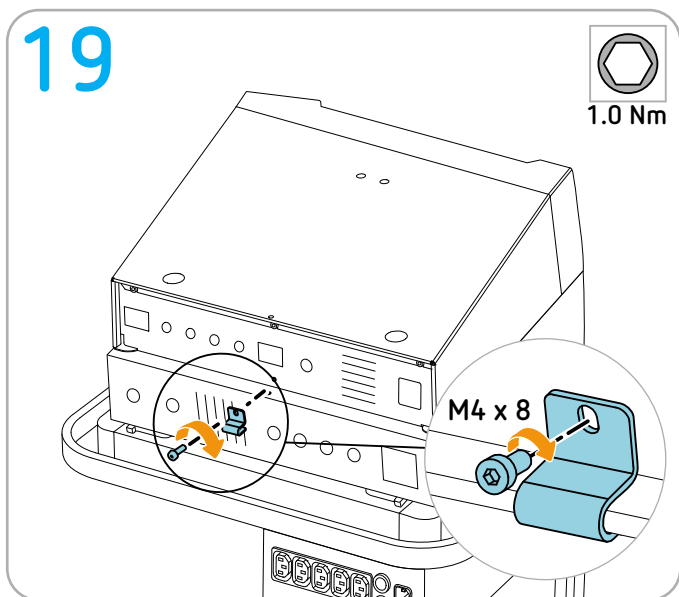
25

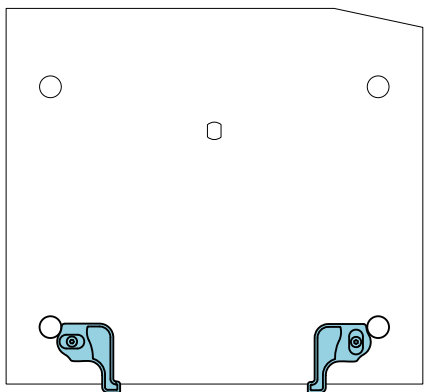
26

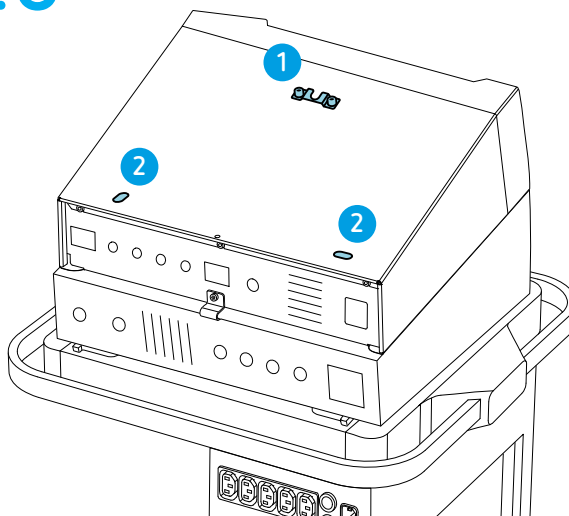Attachment points on VIO seal


27

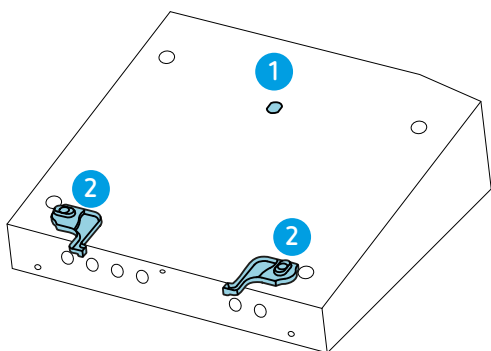
Attachment points on APC 3


28





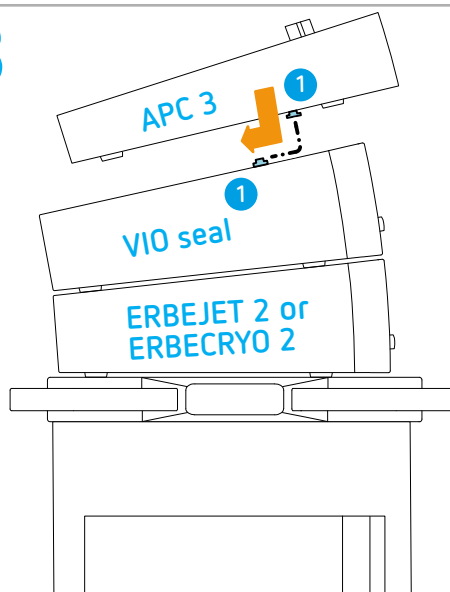
29

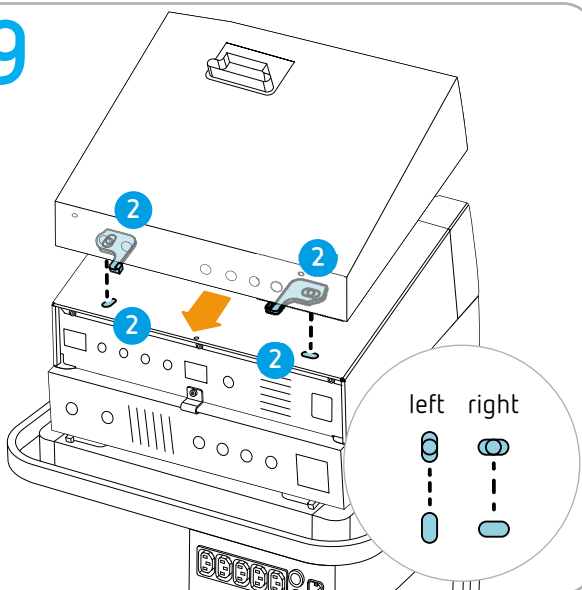30

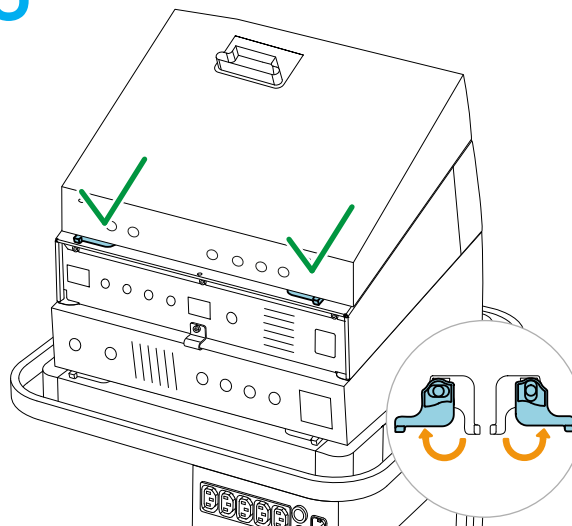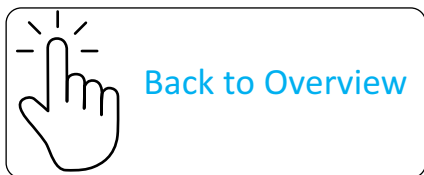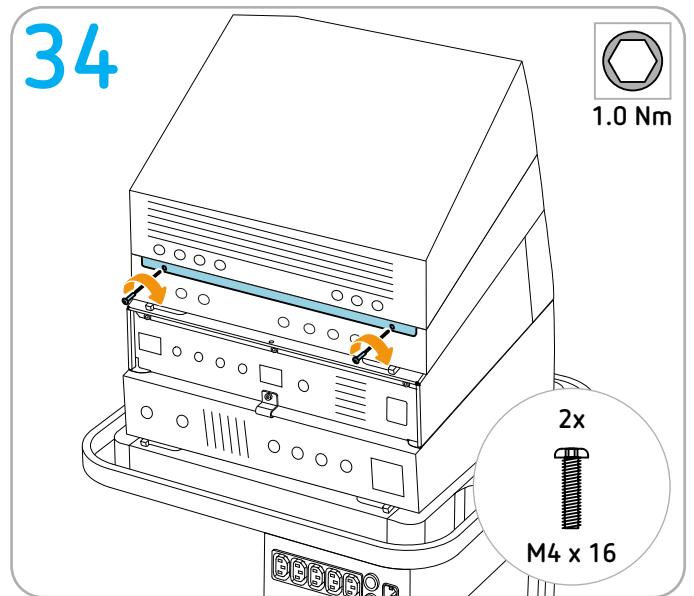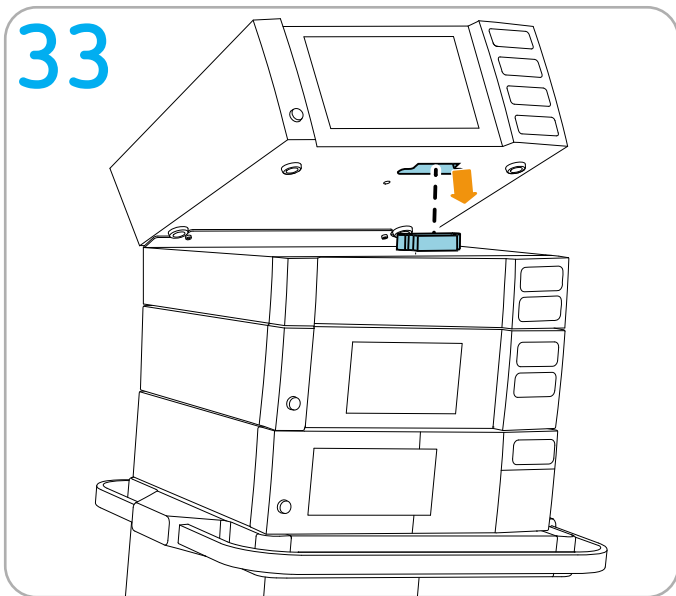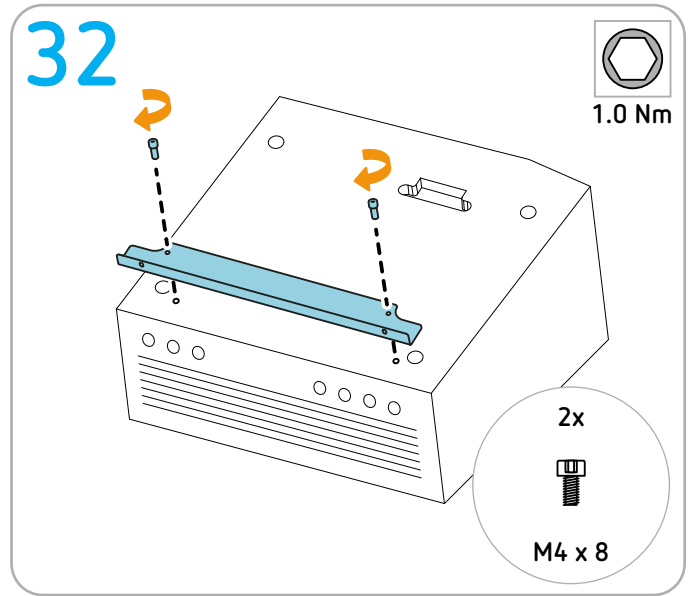
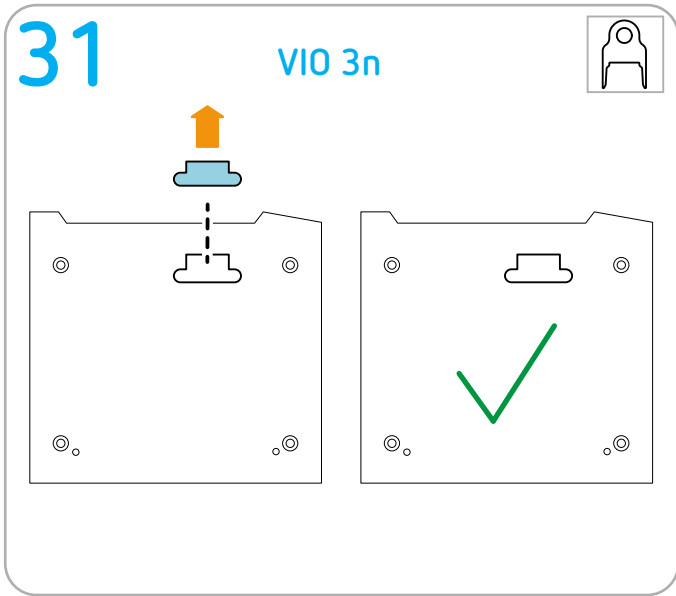
Stowing gas bottles

Permitted positioning

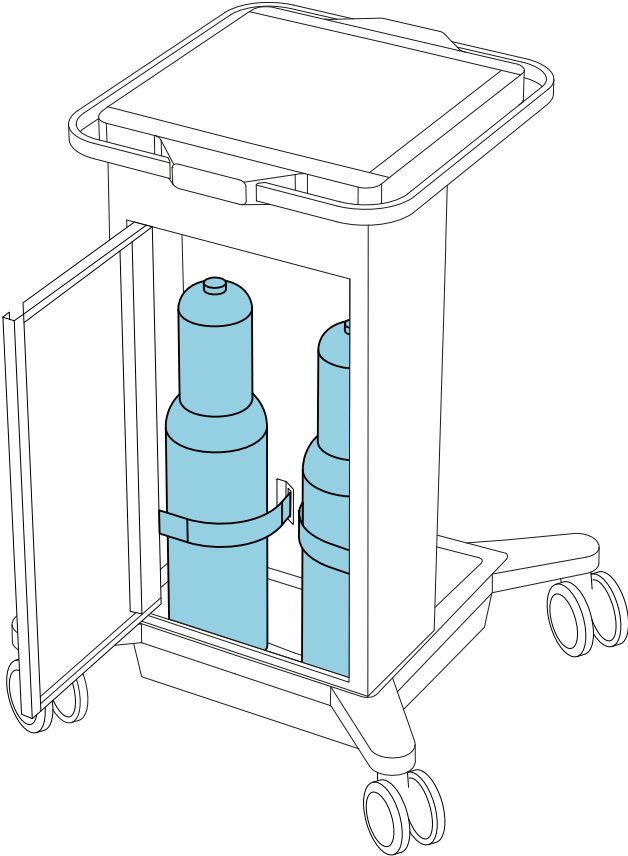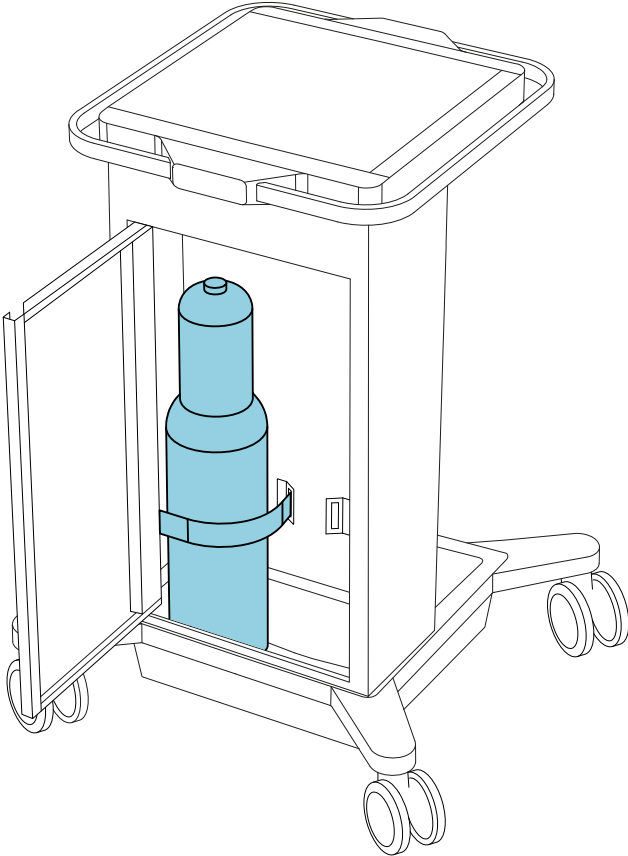Maximum loading:
2 item





Tools required

No tool needed.

Preparation

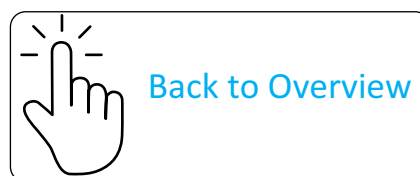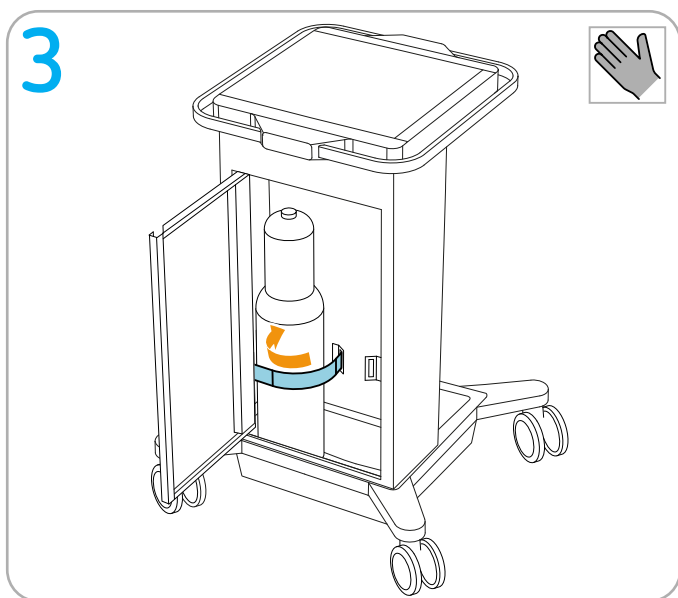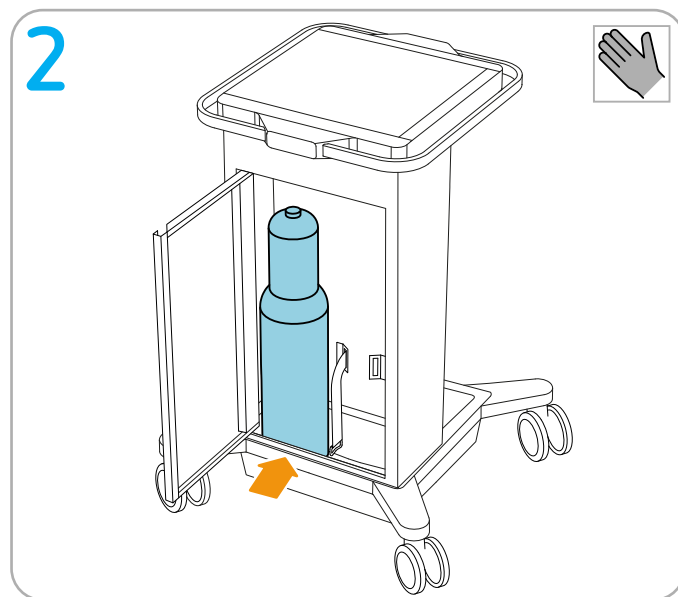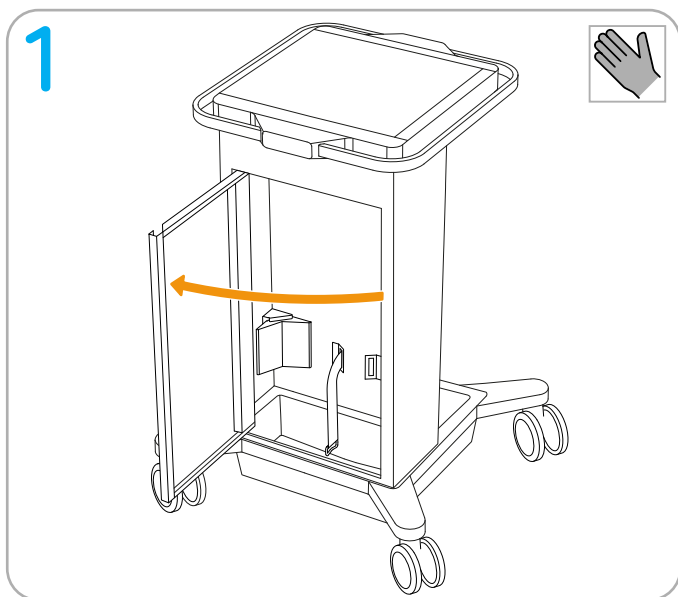
IMPORTANT! Remove the power cord from the system carrier.

Making connections


Potential equalization and mains connection

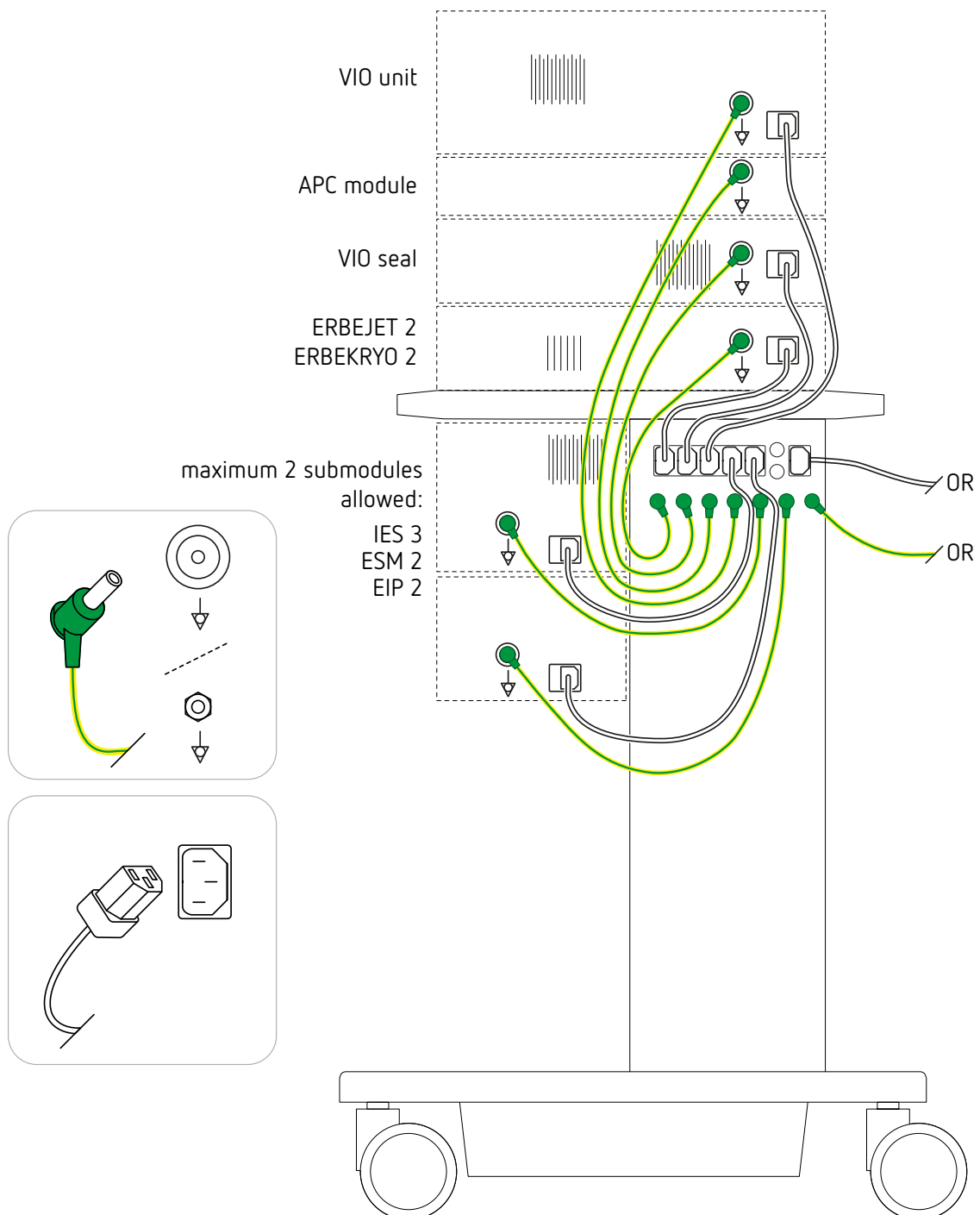IMPORTANT! Only use original Erbe cables.


ECB connection (optional, for communication between the units)

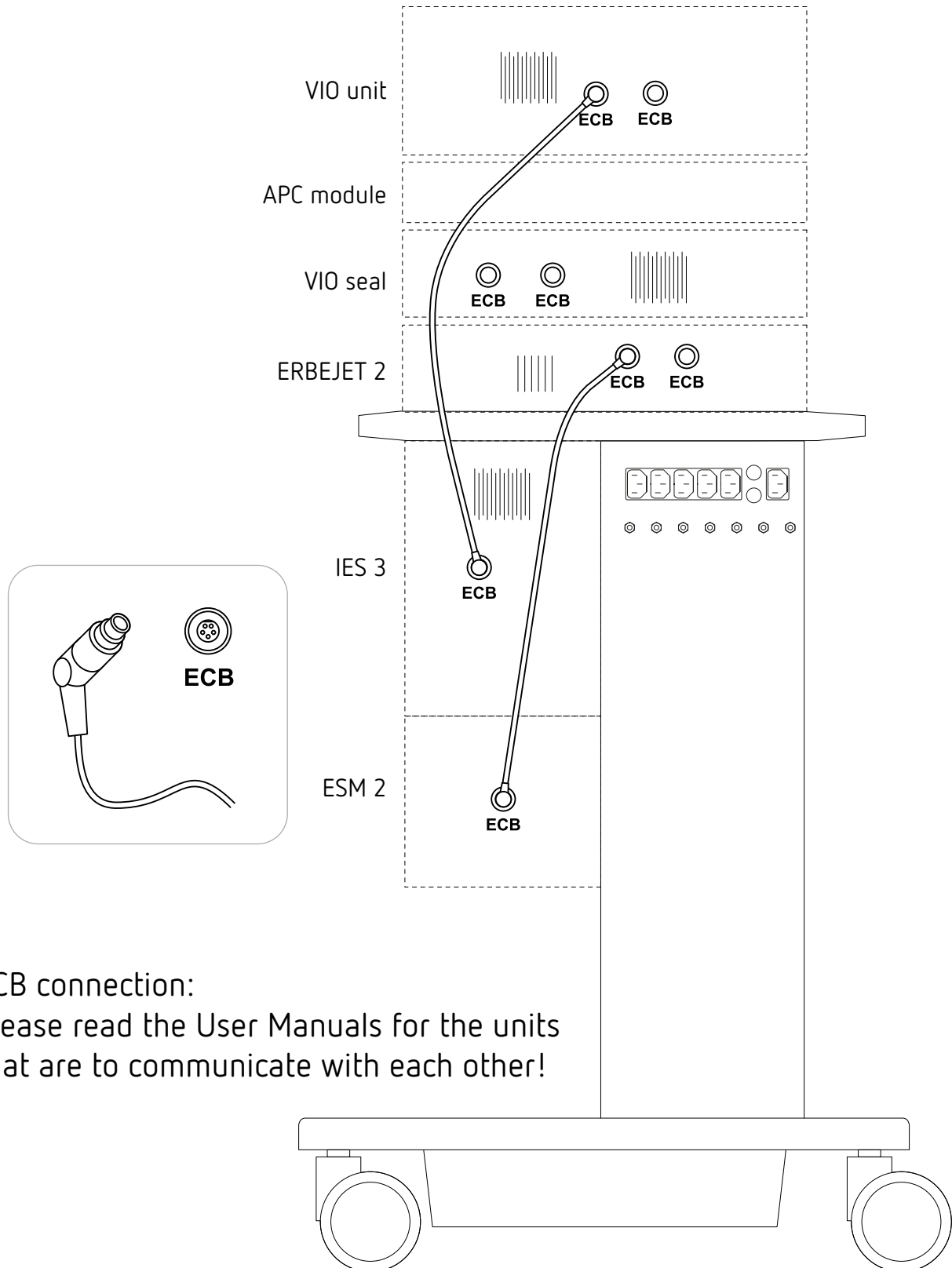IMPORTANT! Only use original Erbe cables.

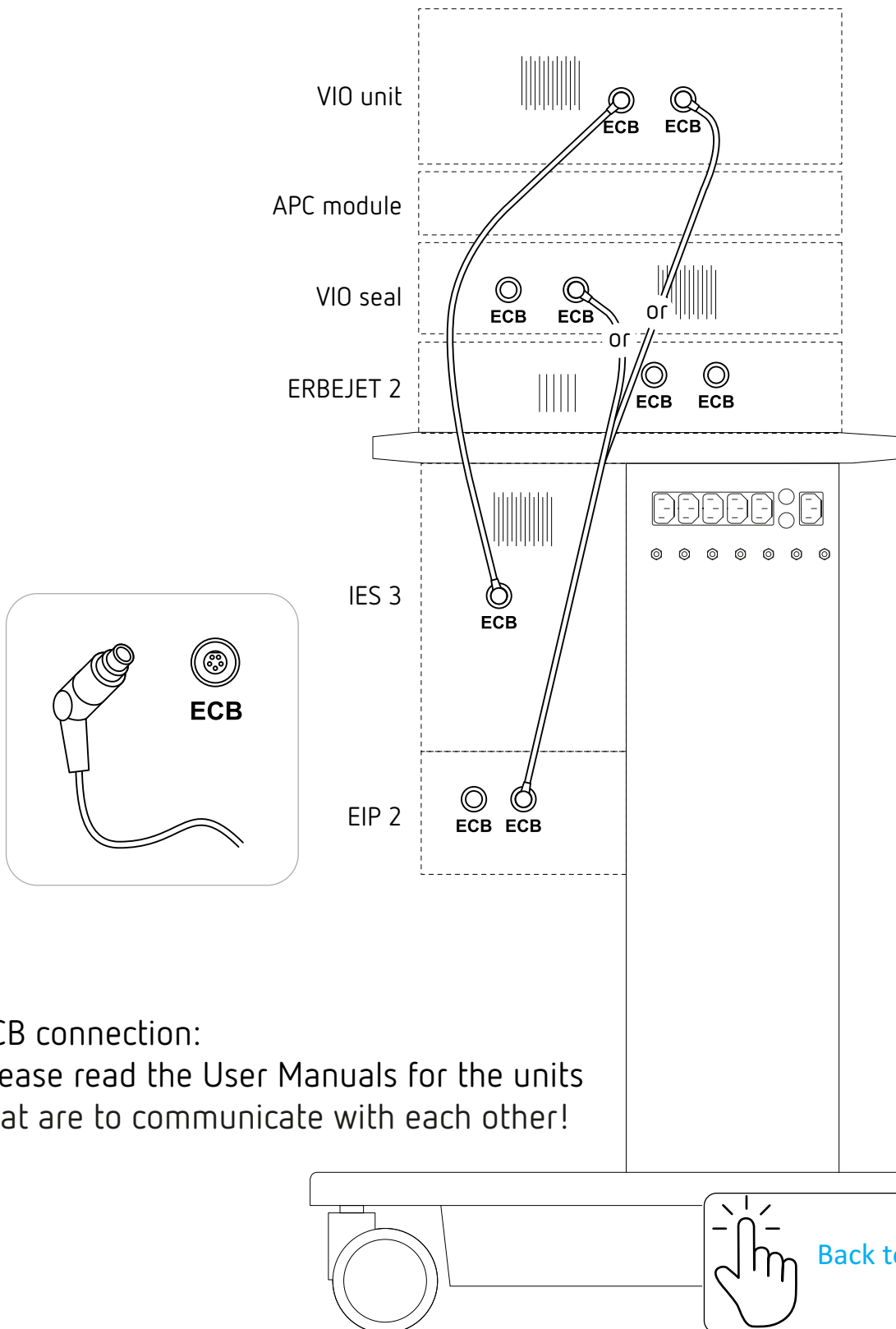
ECB connection:
Please read the User Manuals for the units
that are to communicate with each other!

ECB connection:
Please read the User Manuals for the units
that are to communicate with each other!


APC connection

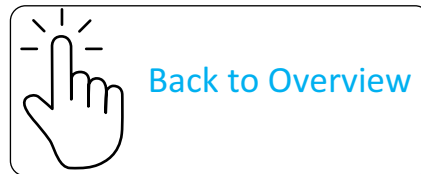Please read the User Manual for the APC 2 or the User Manual for the APC 3.

CO₂ connection

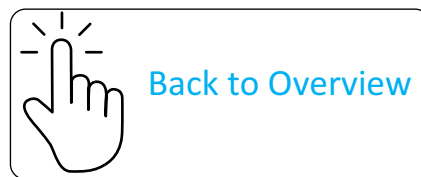Please read the User Manual for the ERBECRYO 2.

 **Manufacturer**
Erbe Elektromedizin GmbH
Waldhörnlestr. 17
72072 Tübingen, Germany
erbe-med.com

Service Hotline
Tel +49 7071 755 437
Fax +49 7071 755 189
techservice@erbe-med.com

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Copyright © Erbe Elektromedizin GmbH, Tübingen 2024